M.4[®]s OA comfort

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M.4[®]s OA comfort

Donning instructions

4-point knee orthosis for varus or valgus pressure relief and stabilisation

Note: It is best if you sit on the edge of a chair or a stool when putting on the brace.

① Open all straps and attach the ends of hook and loop fastener straps to the belt to avoid damage.

Bend the affected leg to an approx. 70 degree angle and slip the brace over it (image 1).

② Position the brace so that the hinge centre is located at the level of the upper edge of the kneecap (see diagram).

Make sure that the brace is not twisted when it is fitted to the leg (image 2).

③ Close straps 1 and 2. Then check again that the hinge fits correctly.

The lateral hinges should be positioned in a slightly backwards direction.

Then close straps 3 to 6 in ascending order (image 3).

④ Walk a few steps to check that the brace is properly fitted. If necessary, the straps can be adjusted by securing them in the same ascending order (image 4).

Digital therapy guide

Access instruction video and more information using the QR code or short URL medi.biz/m4s-oa-comfort-aza

Intended purpose: M.4s OA comfort is a rigid brace designed for relief and stabilisation of the knee joint.