# medi Rehab one

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medi

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## Donning instruction

medi

**Note: It is best if you sit on the edge of a chair or a stool when putting on the compression stocking.**

These donning instructions show you how to put on the stocking with the open toe. Please talk to your doctor about what you should pay attention to when donning, for example how far you can bend, stretch or strain your leg.


**\* For the closed toe variant, steps 1, 9, 10 and 11 are not required.**


Slip your foot into the medi donning aid.\*


Grasp the heel and from there turn the stocking to the left.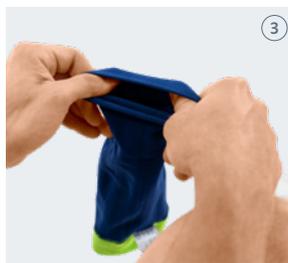


The foot section (heel to toe) remains turned to the right.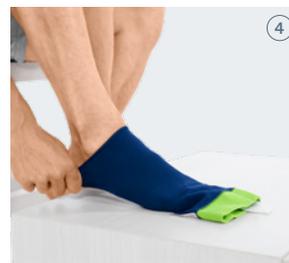

Pull the stocking over the foot to the heel.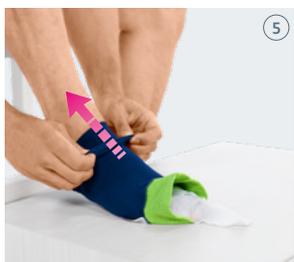

Now grasp the top layer of the stocking as in Fig. (5) and place it on the lower leg (Fig. (6)).

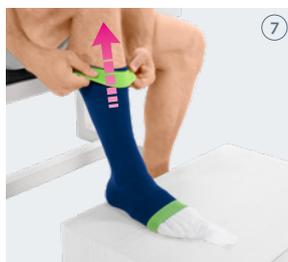Repeat the process step by step until the stocking is completely in place.

Make sure that the stocking fits properly, without twists or folds. The green border should end approx. 2 cm below the hollow of the knee.

To remove the donning aid, pull the stocking back over the heel from the top.\*

Remove the donning aid.\*

Guide the forefoot part over the heel and foot again. Spread it out evenly to the top of your toes.\*

### Donning videos

Access to donning video and product information using the QR-Code or Short-URL [medi.biz/rehab-one](https://medi.biz/rehab-one)

Intended purpose: Round-knit medical compression stocking for compressive care of the lower limbs, mainly for prevention and treatment of postoperative and post-traumatic oedema and general swelling.