# medi Shoulder action

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medi

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## Donning instruction

medi

These donning instructions show you a way to put on the shoulder support in just a few steps. The instructions apply equally to both the left and right arms.

Please consult your doctor about what you should pay attention to when donning – for example, how far you can lift or strain your arm.


① Insert the affected arm through the sleeve of the support (Fig. 1).


② Close the two ends of the chest strap on the front, just below the chest (Fig. 2).


③ The elastic band strap that is fastened to the lower end of the sleeve should run along the inner side of the affected arm. (Fig.3)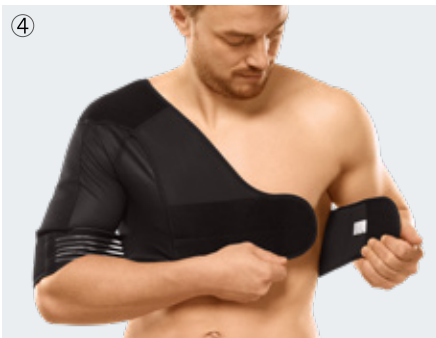

④ Fasten the small strap on the inner side of the upper arm (Fig. 4).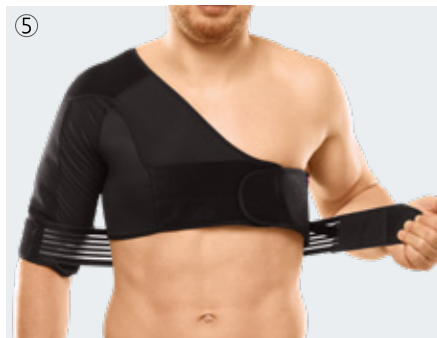

⑤ Now tighten the elastic strap running across the back and attach it to the chest strap as well. You can further restrict the freedom of movement by tugging lightly (Fig.5).

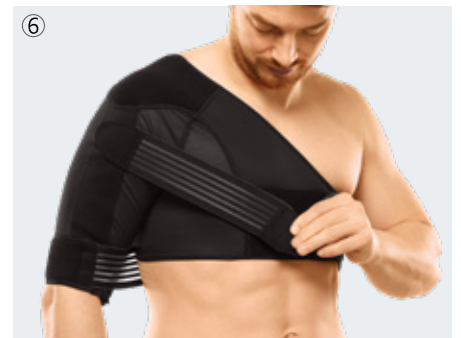⑥ Take the second additional strap and attach it to the front of the upper arm. Slide it diagonally over the chest area while tugging lightly and attach it to the chest strap (Fig.6).

### medi tip

Check the correct fit of the entire shoulder joint brace in front of the mirror.

### Donning video

You can watch the donning videos via QR code or short URL [medi.biz/shoulder-action](https://medi.biz/shoulder-action)

Intended purpose: medi Shoulder action is a shoulder joint brace with defined limitation of the range of motion.