



medi

## medi Rehab<sup>®</sup> one

Effective oedema therapy after  
orthopaedic procedures and injuries  
to the musculoskeletal system

medi. I feel better.

# medi Rehab<sup>®</sup> one

## Specially designed for posttraumatic and postoperative oedema therapy


medi Rehab one is the first medical compression stocking specially developed by medi to treat posttraumatic and postoperative oedema in the field of orthopaedics.

It combines tried-and-tested medical compression with a new knit technology and smart details. This makes using the stockings easier, safer and more convenient for your patients.

The medi Rehab one provides support for oedema therapy


after many orthopaedic procedures and injuries, for example after ligament injuries in the knee or ankle, for fractures and also in the rehabilitation phase after arthroplasty procedures.

The medi Rehab one is impressive thanks to the focus on detail and sets new standards in posttraumatic and postoperative compression therapy for oedema reduction and prevention.

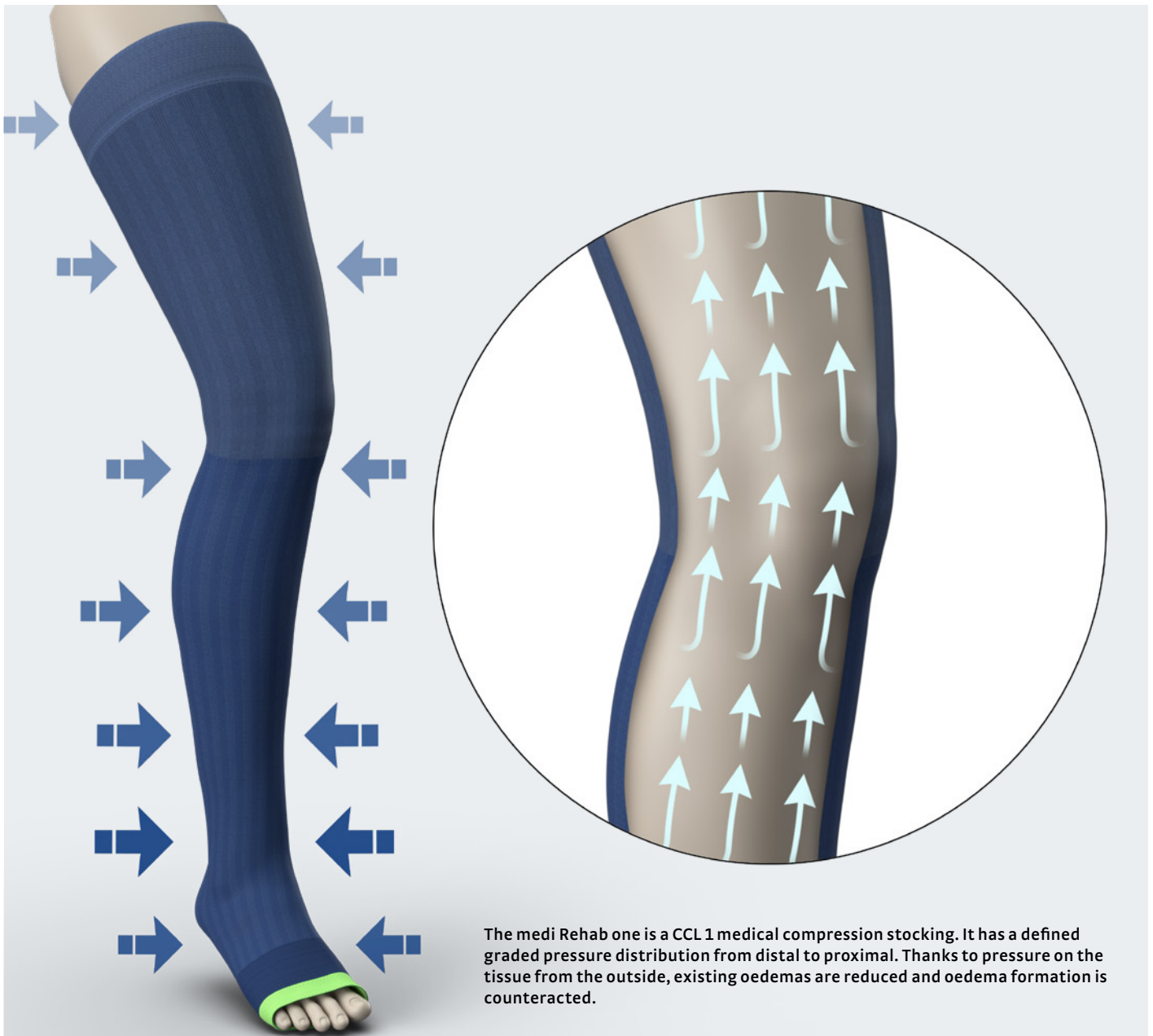

# medi Rehab® one AG

Compression stocking for posttraumatic and postoperative oedema therapy

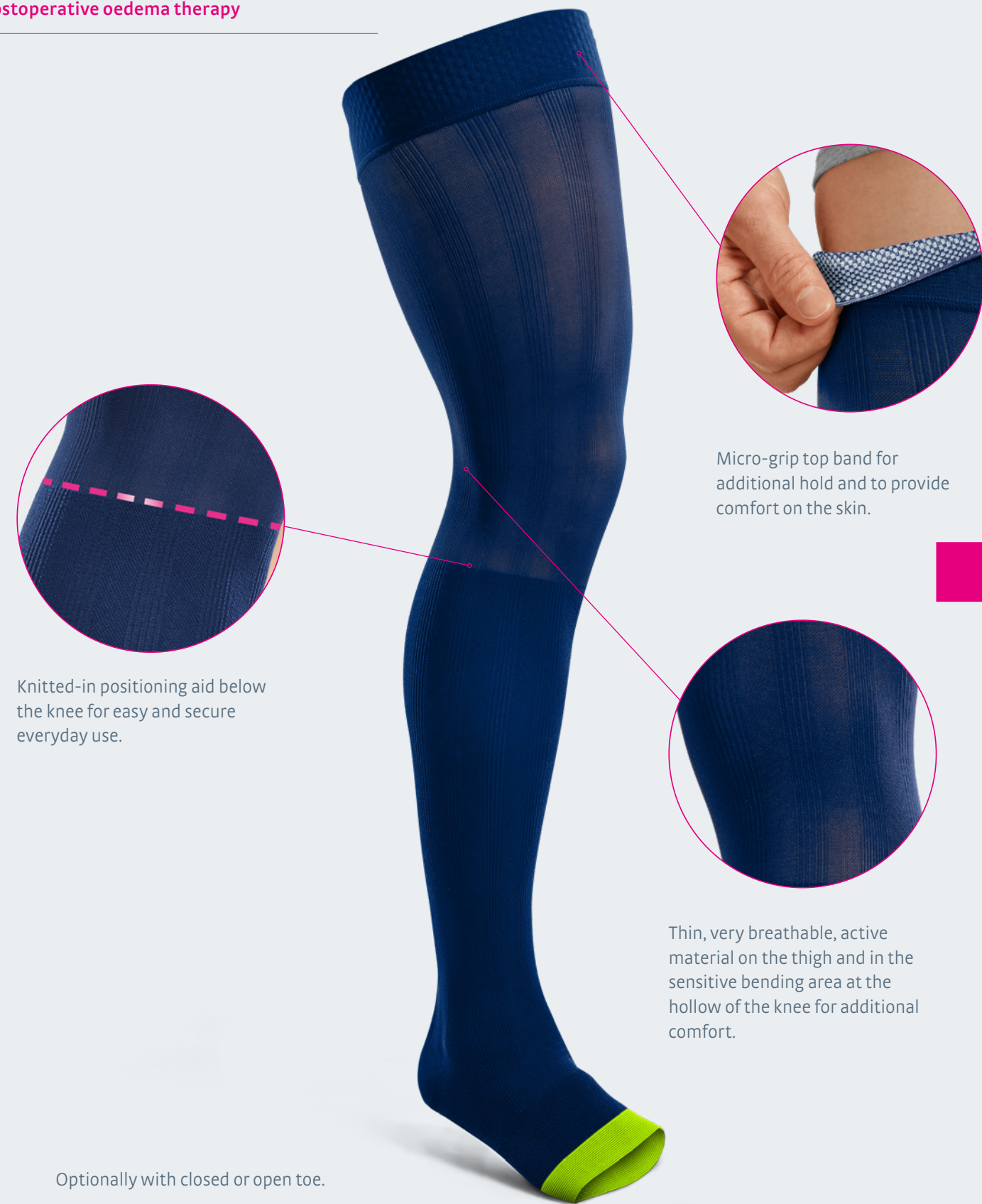

Optionally with closed or open toe.

**Intended purpose:** round-knit medical compression stocking for compressive care of the lower limbs, mainly for prevention and treatment of postoperative and posttraumatic oedema and general swelling.

## Compression and Stabilisation

The ideal combination for therapy of knee injuries

### Rigid braces



M.4s® Comfort*	M.4s PCL dynamic*	M.4 X-lock**	Collamed®/Collamed® long
Rigid brace designed to guide and stabilise the knee joint using limitations for extension / flexion. <sup>1</sup>	Rigid brace designed to guide and stabilise the knee joint using limitations for extension / flexion. <sup>1</sup>	Rigid brace designed to guide and stabilise the knee joint using limitations for extension / flexion. <sup>1</sup>	Knee support brace which function based on the 4-point principle using limitation of extension / flexion. <sup>1</sup>

### Soft braces

### Immobilisation and post-op braces



medi ROM	medi Classic	Classic air	medi PTS®
Knee brace for mobilisation. <sup>1</sup>	Knee brace for immobilisation. <sup>1</sup>	Knee brace for immobilisation. <sup>1</sup>	Knee brace for immobilisation and load relief. <sup>1</sup>

\* High quality cooling compress is included to the product.

<sup>1</sup>Intended purpose

## Thigh-length stocking medi Rehab® one AG

### Indications

- Oedema
- Posttraumatic oedema
- Postoperative oedema

Thigh-length stocking (AG)	ℓ = length c = circumference	size	II	III	IV	V	VI
			Topband normal	45 – 52	49 – 56	53 – 60	56 – 64
cG	cG	Topband wide	53 – 62	57 – 67	61 – 72	65 – 77	69 – 81
		circumference cG (cm)	45 – 62	49 – 67	53 – 72	56 – 77	60 – 81
cF	cF	circumference cF (cm)	41 – 56	44 – 60	47 – 65	50 – 69	53 – 73
		circumference cE (cm)	33 – 40	35 – 43	37 – 45	39 – 48	41 – 51
cD	cD	circumference cD (cm)	29 – 36	32 – 39	34 – 42	36 – 45	38 – 48
		circumference cC (cm)	30 – 37	33 – 40	35 – 43	37 – 46	39 – 49
cB1	cB1	circumference cB1 (cm)	24 – 29	26 – 32	29 – 35	31 – 37	33 – 39
		circumference cB (cm)	20 – 22	22 – 24	24 – 26	26 – 28	28 – 30
cY	cY	circumference cY (cm)	28 – 33	29 – 35	31 – 37	32 – 38	33 – 40
		circumference cA (cm)	19 – 24	21 – 26	23 – 29	25 – 32	27 – 34
cA	cA	article number	R.A.Δ◊.402	R.A.Δ◊.403	R.A.Δ◊.404	R.A.Δ◊.405	R.A.Δ◊.406

Δ = fill in for length: 6 = normal normal (72 – 83 cm), 5 = short (62 – 71 cm)  
◊ = fill in variant: 1 = normal Silicone micro dot topband sensitive, 5 = normal Silicone micro dot topband sensitive-W

# Knee stocking medi Rehab® one AD

## Indications

Oedema

- Posttraumatic oedema
- Postoperative oedema

Wadenstrumpf (AD)	ℓ = length	c = circumference	size	size				
				II	III	IV	V	VI
ℓD		cD	circumference cD (cm)	29–36	32–39	34–42	36–45	38–48
ℓC		cC	circumference cC (cm)	30–37	33–40	35–43	37–46	39–49
ℓB1		cB1	circumference cB1 (cm)	24–29	26–32	29–35	31–37	33–39
ℓB		cB	circumference cB (cm)	20–22	22–24	24–26	26–28	28–30
ℓA		cY	circumference cY (cm)	28–33	29–35	31–37	32–38	33–40
ℓA		cA	circumference cA (cm)	19–24	21–26	23–29	25–32	27–34
		article number		R.A4Δ.402	R.A4Δ.403	R.A4Δ.404	R.A4Δ.405	R.A4Δ.406

Δ = fill in for length: 0 = normal (39–44 cm), 7 = short (34–38 cm)

## medi Rehab® one AD

Compression stocking for posttraumatic and postoperative oedema therapy



3D lengthwise ribbing facilitates correct placement and provides secure handling.

Broad knee stocking band for additional hold.

Pre-formed heel for easy positioning and improved fit.

Forefoot is sized individually to provide improved fit.

Intended purpose: round-knit medical compression stocking for compressive care of the lower limbs, mainly for prevention and treatment of postoperative and posttraumatic oedema and general swelling.

## An overview of the advantages

New **knit technology with micro-plush**, which makes the stockings easy to put on and provides a secure fit

**Positioning aid and 3D longitudinal ribbing** for secure use and easy donning

**Thin and especially breathable** in the sensitive knee bend area

**Thermoregulating merino plush** in the midfoot and forefoot area for increased user comfort

Tried and tested **CCL 1 medical compression**. Additionally: Comfortable compression at the midfoot area

Ideal supplement in therapy for knee and ankle injuries in combination with medi or thoses

## Further information about the product

Use the QR code or the short-url [medi.biz/rehab-one](https://medi.biz/rehab-one) to access further information and to watch the donning videos.



# Compression and Stabilisation

The ideal combination for therapy of foot injuries

## Ankle braces



**medi® Ankle RTS®**

Adjustable ankle brace for stabilisation in one plane.<sup>1</sup>

**Levamed® stabili-tri**

Ankle brace for mobilisation in a defined position, parts can be removed.<sup>1</sup>

**M.step®**

Ankle brace for stabilisation in one plane.<sup>1</sup>

**medi® Ankle sport brace**

Adjustable ankle brace for stabilisation in one plane.<sup>1</sup>

## Walker



**medi® ROM Walker**

Lower-leg foot brace for mobilisation with an adjustable range of motion.<sup>1</sup>

**protect.CAT Walker**

Lower-leg foot brace for immobilisation in a specified position.<sup>1</sup>

**protect.CAT Walker short**

Lower-leg foot brace for immobilisation in a specified position.<sup>1</sup>

<sup>1</sup>Intended purpose

Headquarters  
medi GmbH & Co. KG  
Medicusstraße 1  
95448 Bayreuth  
Germany  
T +49 921 912-333  
ortho@medi.de  
www.medi.de



0090121/12.2025