



medi

# Therapy Concept Lymphology

medi product catalogue  
Valid from January 2026



medi. I feel better.









## Driving the future in compression therapy

**A strong brand. A wide-ranging product range.  
Convincing concepts.**

---

medi World of Compression: signpost to the future with 75 years' competence in the field of compression. Your partner for the highest demands on quality, innovation and design. Let's shape the future together and create positive brand experiences – for you and your clients!

## Convincing treatment concepts

**New potential in lymphoedema therapy  
arises from combining the decongestion,  
the transition and the maintenance  
phases with innovative products.**

---

Position yourself as the specialist in the treatment of lymphatic diseases in your region and gain new satisfied customers that you can accompany through their therapy journey.





# medi World of Compression

medi provides comprehensive marketing solutions – always keeping our focus on the market, its demands and variables, and the medi vision.




To introduce you to the medi World of Compression we created indication related catalogues. This enables you to find the correct product for all the different indications and situations in no time. Please find a short explanation of the different literature below:



## Venous Health

mediven round-knit compression stockings are the cornerstone of the conservative treatment for patients with venous disorders. Discover more about the numerous styles and variants of mediven and circaid products for venous health.

Art. No.: 0097D67

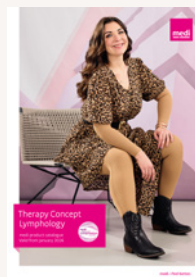

## Therapy Concept Lymphology

The medi Therapy Concept Lymphology shows the new approach in the treatment of lymphoedema. It combines the decongestion phase, the transition phase and the maintenance phase and provides the right products for every stage.

The circaid and the mediven flat knit products make the treatment of lymphoedema more individual than ever before.

Art. No.: 0098D50



## Therapy Concept Leg Ulcer

The medi Therapy Concept Leg Ulcer accompanies the patient during the complete treatment of leg ulcer by connecting the acute phase to the prevention of recurrence phase.

Available as PDF only



## Clinical Compression

The world of Clinical Compression is more than just thrombosis prophylaxis in the clinical environment. Learn more about the multiple application fields of medi's hospital products and benefit from over 40 years of experience with Clinical Compression.

Art. No.: 0099580



# We have the direct connection

To communicate. To co-operate. To be there for each other. The medi service team respects and values this requirement seriously.

Therefore, our office not only offers consultants, but also a state-of-the-art communications system and logistics.

Just-in-time: in consultancy, in service, in ordering and delivery.

We are always here for you.

This is your medi export service team.

## Hotline

for consultancy, ordering, problem solutions

## Administration

T +49 921 912 -0

F +49 921 912 -781

You can visit our website:

[www.medi.de/en](http://www.medi.de/en)

or contact us by email:

[export@medi.de](mailto:export@medi.de)

or order by email:


[orders@medi.de](mailto:orders@medi.de)



## 8

### General

- 8 More than just a manufacturer
- 9 Quality Certificates

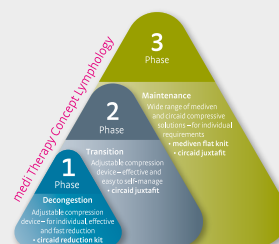

## 10

### Therapy Concept Lymphology

- 12 Indication overview lymphoedema
- 16 Excursus: Indication overview lipoedema
- 18 circaid – adjustable compression
- 22 mediven – medical compression





1  
Phase

24

### Therapy Concept Lymphology – Leg Decongestion Phase

- 29 circaid reduction kit leg
- 32 circaid reduction kit  
foot options & accessories



1  
Phase

84

### Therapy Concept Lymphology – Arm Decongestion Phase

- 89 circaid reduction kit arm
- 91 circaid reduction kit hand options and  
accessories



122

### Commercial products

- 124 medi donning and doffing aids
- 128 medi skin care
- 132 medi textile care



2  
Phase

36

### Transition Phase

- 39 circaid juxtafit premium leg
- 43 circaid juxtafit essentials leg
- 48 circaid foot options
- 51 circaid accessories leg



2  
Phase

92

### Transition Phase

- 95 circaid juxtafit essentials arm
- 97 circaid hand options
- 98 circaid accessories arm



134

### Service

- 136 Overview delivery content circaid



3  
Phase

54

### Maintenance Phase mediven

- 56 mediven flat knit care compass leg
- 59 mediven 550 leg
- 63 mediven 550 toe cap
- 67 mediven 550 SL toe cap
- 69 mediven cosy
- 73 mediven cosy toe cap
- 77 mediven mondi
- 81 mediven mondi toe cap

### circaid

- 82 circaid juxtafit premium leg and  
circaid juxtafit essentials leg  
(see page 38 – 47)



3  
Phase

100

### Maintenance Phase mediven

- 102 mediven flatknit care compass arm
- 105 mediven 550 arm
- 109 mediven esprit
- 113 mediven esprit SL
- 115 mediven harmony

### circaid

- 121 circaid juxtafit essentials arm  
(see page 95 – 96)





## More than just a manufacturer

**medi produces medical aids for a variety of conditions and life situations. But we don't see ourselves solely as a manufacturer. On the contrary, we handle our own research and development, technology, production, marketing and sales in-house – always keeping our focus on the market, its demands and variables, and the medi vision.**

medi offers products in three business divisions: Compression, Orthopaedics and Footcare. These products are used and distributed by medical supply stores, hospitals and clinics, doctors' practices and therapists.

In other words, specialists who know which users need which aids to improve their state of health and quality of life.

What we're proud of and what makes our products so special is that they're free of negative side effects.

Our range of products primarily includes medical compression stockings and adjustable compression for people with vein disorders, lipoeidema and lymphoedema, supports and braces for the conservative and post-operative treatment of orthopaedic injuries and disorders, stockings for the prevention of thrombosis, anti-embolism stockings for use after vein surgery, compression garments for use after plastic and cosmetic surgery plus a variety of therapy-supporting products.

We support our partners in the sale and supply of our products and have developed intelligent concepts and services for this purpose.

# Quality Certificates

As a manufacturer of medical products, medi has a high regard and awareness for quality. Product and system certificates prove that all legal requirements are met.

## CE mark

Our medi medical devices meet all of the requirements of the European Directive 93 / 42 / EEC and of the German Medical Devices Act. In accordance with regulations, our products are subject to the conformity assessment procedures and are granted the CE mark. As a kind of “passport for industrial products”, the CE mark guarantees free trade within the European Economic Area (EEA).



## Quality label

Our medical compression hosiery is regularly checked by the Hohenstein Research Institute according to the quality and test provisions of the RAL-GZ 387. As confirmation that our products meet the quality conditions, you will find the quality label printed on the packaging.



## Textile trust quality seal

All compression stockings and garments, whether circular-knit or flat-knit, thrombosis prophylaxis stockings and supports are regularly examined and rated according to strict standards by independent test centres for any toxic substances potentially damaging to human health. Granting of the Oeko-Tex Standard 100 certificate guarantees: tested quality for the protection of your health.



## EN ISO 13485

medi maintains the quality management system according to the EN ISO 13485. Since 1996 we have demonstrated our quality status to our customers with a certificate. Since then, medi has enhanced the opportunities offered by this standard to permanently improve products from the business units Phlebology, Orthotics, Footcare, Woundcare and Hospital in order to organise safer and more efficient processes and procedures. We define customer satisfaction as a decisive measure of quality.






# medi Therapy Concept Lymphology

Effective lymphoedema treatment.

**Step 1:** Consider in which phase of the therapy the patient is.

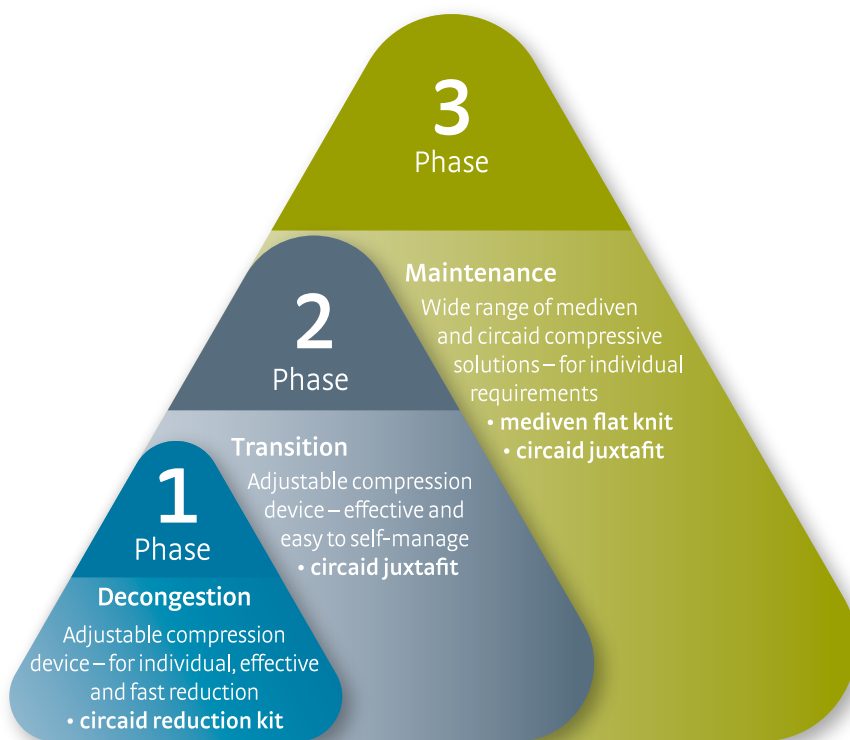

Adjustable and patient-individual compression for initial oedema reduction.

**circaid® reduction kit**

medi. I feel better.

2

## Transition Phase



Effective compression to optimise reduction results.

**circaid® juxtafit®**

3

## Maintenance Phase



A huge variety of compression solutions to maintain therapy results in the long term.

**mediven® flat knit**  
**circaid® juxtafit®**








# medi Therapy Concept Lymphology





## Indication overview lymphodema – lower extremities.

Step 2: Depending on the phase, choose the appropriate medi product

				
			<b>circaid reduction kit</b>	<b>circaid juxtafit premium and essentials leg</b>
			Adjustable compression device with the ability to reduce size of garment as the limb volume decreases to achieve optimal decongestion.	Adjustable compression device to optimise the therapy results achieved in the decongestion phase.
			for connective tissues • soft • normal • firm	for connective tissues • soft • normal • firm
			Tension 1 – 8**	20 – 50 mmHg**
Indication	Diagnosis	Characteristica	Phase 1 Decongestion	Phase 2 Transition
	<b>Stage 1</b>	Spontaneously reversible, oedema of soft consistency; elevation reduces swelling, pitting of the skin	•	•
	<b>Stage 2</b>	Spontaneously irreversible, hardened oedema; elevation does not eliminate swelling, skin does not pit	•	•
	<b>Stage 3</b>	Unformed swelling of extreme dimensions, severely hardened skin, formation of blisters and fistulae	•	•

### Explanation of compression classes

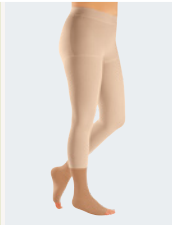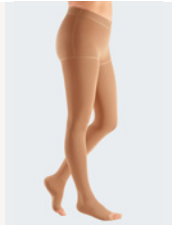CCL 1	Compression in mmHg 18.0 – 21.0	Compression in kPa 2.40 – 2.80
CCL 2	Compression in mmHg 23.0 – 32.0	Compression in kPa 3.10 – 4.30
CCL 3	Compression in mmHg 34.0 – 46.0	Compression in kPa 4.50 – 6.10
CCL 4	Compression in mmHg min. 49.0	Compression in kPa min. 6.50

			
<b>mediven 550 leg</b>	<b>mediven cosy*</b>	<b>mediven mondi*</b>	<b>circaid juxtafit premium and essentials leg</b>
Flat-knitted compression stocking with very high wall stability and a strong, firm fabric. For the therapy of lymphoedema in the maintenance phase – up to stage 3.	Flat-knitted compression stocking with medium wall stability and a particularly flexible fabric. For the therapy of lymphoedema in the maintenance phase – up to stage 2.	Flat-knitted compression stocking with light wall stability and a particularly soft fabric. For the therapy of lymphoedema in the maintenance phase – up to stage 2.	Easy to apply adjustable compression device for those patients who have challenges with flat-knitted compression stockings.
for connective tissues <ul style="list-style-type: none"> <li>• very soft, soft</li> <li>• normal</li> <li>• firm, hardened</li> </ul>	for connective tissues <ul style="list-style-type: none"> <li>• soft</li> <li>• normal</li> <li>• firm</li> </ul>	for connective tissues <ul style="list-style-type: none"> <li>• normal</li> <li>• firm</li> </ul>	for connective tissues <ul style="list-style-type: none"> <li>• soft</li> <li>• normal</li> <li>• firm</li> </ul>
CCL 1 – 4**	CCL 1 – 3**	CCL 1 – 3**	20 – 50 mmHg**
<b>Phase 3 Maintenance***</b>			
•	•	•	•
•	•	•	•
•			•

#### Caution

Please consider the patient's individual baseline characteristics including comorbidities, connective tissue, weight, skin wrinkles, severity of the lymphoedema.

\*\*\* Discover the new mediven flat knit care compass on page 56. It helps you to choose the optimal flat knit stocking for your patient in the maintenance phase.

#### Explanation

\* Please remember that mediven cosy and mediven mondi is indicated for lymphoedema stage 1 – 2.

They do not have sufficient wall stability to model the tissues in patients with very lax / soft connective tissue. They cut into skin folds and cause constriction.




\*\* Please adjust (circaid) or respectively hand in (mediven) as prescribed



# medi Therapy Concept Lymphology


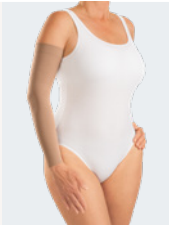


## Indication overview lymphodema – upper extremities.

Step 2: Depending on the phase, choose the appropriate medi product

				
			<b>circaid reduction kit</b>	<b>circaid juxtafit essentials arm</b>
			Adjustable compression device with the ability to reduce size of garment as the limb volume decreases to achieve optimal decongestion.	Adjustable compression device to optimise the therapy results achieved in the decongestion phase.
			for connective tissues • soft • normal • firm	for connective tissues • soft • normal • firm
			Tension 1 – 8**	10 – 40 mmHg**
Indication	Diagnosis	Characteristica	Phase 1 Decongestion	Phase 2 Transition
	<b>Stage 1</b>	Spontaneously reversible, oedema of soft consistency; elevation reduces swelling, pitting of the skin	•	•
	<b>Stage 2</b>	Spontaneously irreversible, hardened oedema; elevation does not eliminate swelling, skin does not pit	•	•
	<b>Stage 3</b>	Unformed swelling of extreme dimensions, severely hardened skin, formation of blisters and fistulae	•	•

### Explanation of compression classes

CCL 1	Compression in mmHg 18.0 – 21.0	Compression in kPa 2.40 – 2.80
CCL 2	Compression in mmHg 23.0 – 32.0	Compression in kPa 3.10 – 4.30
CCL 3	Compression in mmHg 34.0 – 46.0	Compression in kPa 4.50 – 6.10
CCL 4	Compression in mmHg min. 49.0	Compression in kPa min. 6.50

			
<b>mediven 550 arm</b>	<b>mediven esprit*</b>	<b>mediven harmony*</b>	<b>circaid juxtafit essentials arm</b>
Flat-knitted compression stocking with very high wall stability and a strong, firm fabric. For the therapy of arm lymphoedema in the maintenance phase – up to stage 3.	Flat-knitted compression stocking with light to medium wall stability and a particularly supple knit. For the therapy of arm lymphoedema in the maintenance phase – up to stage 2.	Round-knitted compression armsleeve for the therapy of arm lymphoedema in the maintenance phase – stage 1	Easy to apply adjustable compression device for those patients who have challenges with flat-knitted compression armsleeves.
for connective tissues • very soft, soft • normal • firm, hardened	for connective tissues • normal • firm	for connective tissues • normal	for connective tissues • soft • normal • firm
CCL 1 – 3**	CCL 1 – 3**	CCL 1 – 2**	10 – 40 mmHg**
<b>Phase 3 Maintenance***</b>			
•	•	•	•
•	•		•
•			•

#### Caution

Please consider the patient's individual baseline characteristics including comorbidities, connective tissue, weight, skin wrinkles, severity of the lymphoedema.

\*\*\* Discover the new mediven flat knit care compass on page 102. It helps you to choose the optimal flat knit stocking for your patient in the maintenance phase.

#### Explanation

\* Please remember that mediven esprit is indicated for lymphoedema stage 1 – 2 and mediven harmony for lymphoedema stage 1.

It does not have sufficient wall stability to model the tissues in patients with very lax connective tissue. It cuts into skin folds and causes constriction.


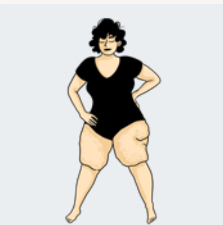
\*\* Please adjust (circaid) or respectively hand in (mediven) as prescribed






# Lipoedema

mediven compression stockings are also used for lipoedema. However, careful assessment of a patient with lipoedema is required to determine which type(s) of compression therapy are indicated.

## Indication overview lipoedema – lower extremities

			mediven compression
			<b>mediven 550 leg</b> for connective tissues <ul style="list-style-type: none"> <li>• very soft, soft</li> <li>• normal</li> <li>• firm, hardened</li> </ul>
Indication	Characteristica		CCL 1 – 4*
<b>Lipoedema</b> 	Painful fatty tissue distribution disorder that occurs disproportionately and symmetrically on the extremities	AT	<ul style="list-style-type: none"> <li>•</li> </ul>

## Indication overview lipoedema – upper extremities

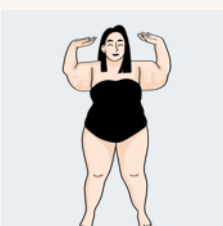			mediven compression
		 	<b>mediven 550 arm</b> for connective tissues <ul style="list-style-type: none"> <li>• very soft, soft</li> <li>• normal</li> <li>• firm, hardened</li> </ul>
Indication	Characteristics		CCL 1 – 3*
<b>Lipoedema</b> 	Painful fatty tissue distribution disorder that occurs disproportionately and symmetrically on the extremities	AG CG	<ul style="list-style-type: none"> <li>•</li> </ul>

### Explanation of compression classes

CCL 1	Compression in mmHg 18.0 – 21.0 Compression in kPa 2.40 – 2.80
CCL 2	Compression in mmHg 23.0 – 32.0 Compression in kPa 3.10 – 4.30
CCL 3	Compression in mmHg 34.0 – 46.0 Compression in kPa 4.50 – 6.10
CCL 4	Compression in mmHg min. 49.0 Compression in kPa min. 6.50

### Caution

Please consider the patient's individual baseline characteristics including comorbidities, connective tissue, weight, skin wrinkles, severity of the lipoedema.

Discover the new mediven flat knit care compass on page 56 and 102. It helps you to choose the optimal flat knit stocking for your patient.

mediven compression	
mediven cosy	mediven mondi
for connective tissues	for connective tissues
<ul style="list-style-type: none"> <li>• soft</li> <li>• normal</li> <li>• firm</li> </ul>	<ul style="list-style-type: none"> <li>• normal</li> <li>• firm</li> </ul>
CCL 1 – 3*	CCL 1 – 3*

mediven compression	
mediven esprit	
	for connective tissues
	<ul style="list-style-type: none"> <li>• normal</li> <li>• firm</li> </ul>
CCL 1 – 3*	

#### Explanation

They do not have sufficient wall stability to model the tissues in patients with very lax connective tissue. They cut into skin folds and cause constriction.

\* Please adjust as prescribed.



# circaid® adjustable compression devices

**Adjustable compression devices for the treatment of lymphoedema.**

circaid garments are an alternative to traditional compression bandaging in the treatment of lymphoedema. Thanks to the single juxtaposed bands an easy handling is ensured and offers patients to actively take part in their therapy. Therefore circaid garments are also a solution for patients who have challenges with established medical compression stockings.

The unique circaid Built-In-Pressure System offers adjustable, controllable and instantly re-adjustable compression. This enables consistent compression therapy and furthermore avoids sliding down of the garment.



# circaid<sup>®</sup> features

## Overview of circaid characteristics.



### Built-In-Pressure<sup>®</sup> System

- To set, control and re-adjust the correct therapeutic compression range



### Juxtaposed band system

- Optimum fit thanks to interlocking bands
- Each strap can be readjusted while on the limb




### Breathe-O-Prene

- Breathable material
- Odour-inhibiting effect



### Variety

- Wide range of care options
- Many sizes available

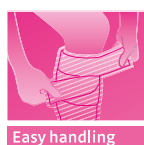


### Easy handling

- Simple donning and doffing
- Eases handling for the patient



### Inelastic

- Inelastic material for an effective therapy and pleasant wearing comfort
- Inspired by nature

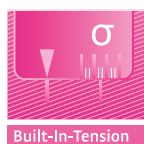

### Built-In-Tension<sup>®</sup> System

- To set and control the correct therapeutic level of tension
- For circaid reduction kit products only






# circaid® Built-In-Pressure® System (BPS)

circaid adjustable compression devices are engineered with the ability to be set to precise compression ranges with a graduated compression profile from the ankle up the leg or from the wrist to the upper arm.

The user-friendly design of the circaid Built-In-Pressure System (BPS) enables patients to achieve the necessary compression ranges needed for the treatment and management of lymphoedema. With the circaid Built-In-Pressure System (BPS) guide card patients are able to set their prescribed compression range while donning their garment and to readjust the compression range during the day as needed. Therefore, the appropriate compression range is ensured throughout the day. At the same time, it is very simple to verify the correct range of applied pressure.



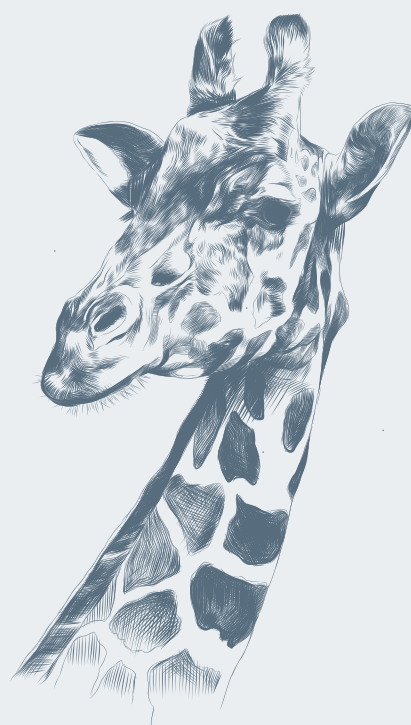

## Inspired by nature.

circaid products have an animal role model: the giraffe with its specific skin structure. Giraffes have venous pressures of more than 250 mmHg at their front and hind legs – three times more than humans. But they do not suffer from lymph-oedema or venous diseases, even though they are on their feet the whole day and some-times also sleep while standing. But why? In contrast to humans giraffes have inelastic skin, meaning it does not stretch.\*

The founder of circaid was inspired from this “natural compression” and designed the first patented inelastic circaid leg garment. This first garment has been further developed to today’s circaid products (short for circulation aid).

\* Sathar F, Ludo Badlangana N, Manger PR. Variations in the thickness and composition of the skin of the giraffe. Anatomical record 2010;293(9):1615-1627.





## mediven<sup>®</sup> medical compression stockings

mediven embodies appropriate medical compression stockings for several indications in modern oedema therapy. They are tailored in detail to the needs of the patients.

mediven lymphoedema garments stand for high wearability and convenience while at the same time offering optimum medical effect.

mediven products guarantee high standards with regard to important major quality criteria such as "material selection", "basic compression and graduated compression", "flexural stiffness" or "compression allowances for changing vital statistics" with the result of satisfied loyal patients. Provide your patient with the numerous variants and styles of mediven!

# mediven<sup>®</sup> features



Clima Comfort

## Clima Comfort\*

- Wicks moisture away from the skin's surface quickly
- Guarantees a high level of breathability

\* Scientifically tested by French institute IFTH<sup>1</sup>. The test results show that Clima Comfort technology from medi assures climate control between the skin and the stocking. With 11/12 points, medi achieved an outstanding overall score

<sup>1</sup> Gardon-Mollard C., Gogu -Meunier G. Etude comparative du transfert de la vapeur d'eau dans diff rents articles de compression m dicale. Ph lebologie 2011,64(2):53-60.



Clima Fresh

## Integrated Clima Fresh function

- Odour-inhibiting effect



Perfect Fit

## Perfect Fit

- A very precise fit
- Medically effective pressure gradient
- Optimal care conditions in standard and made-to-measure version



Variety

## A wide range of variants and special models

## Anatomical oblique borders as standard (distal and proximal)

- Unimpeded lymphatic drainage without circular constriction
- Optimum fit and better hold

## Easy to care for


- Suitable for the delicates programme of washing machines and tumble driers



# medi Therapy Concept Lymphology – Lower Extremities

Optimum treatment with medi during the entire therapy.

---

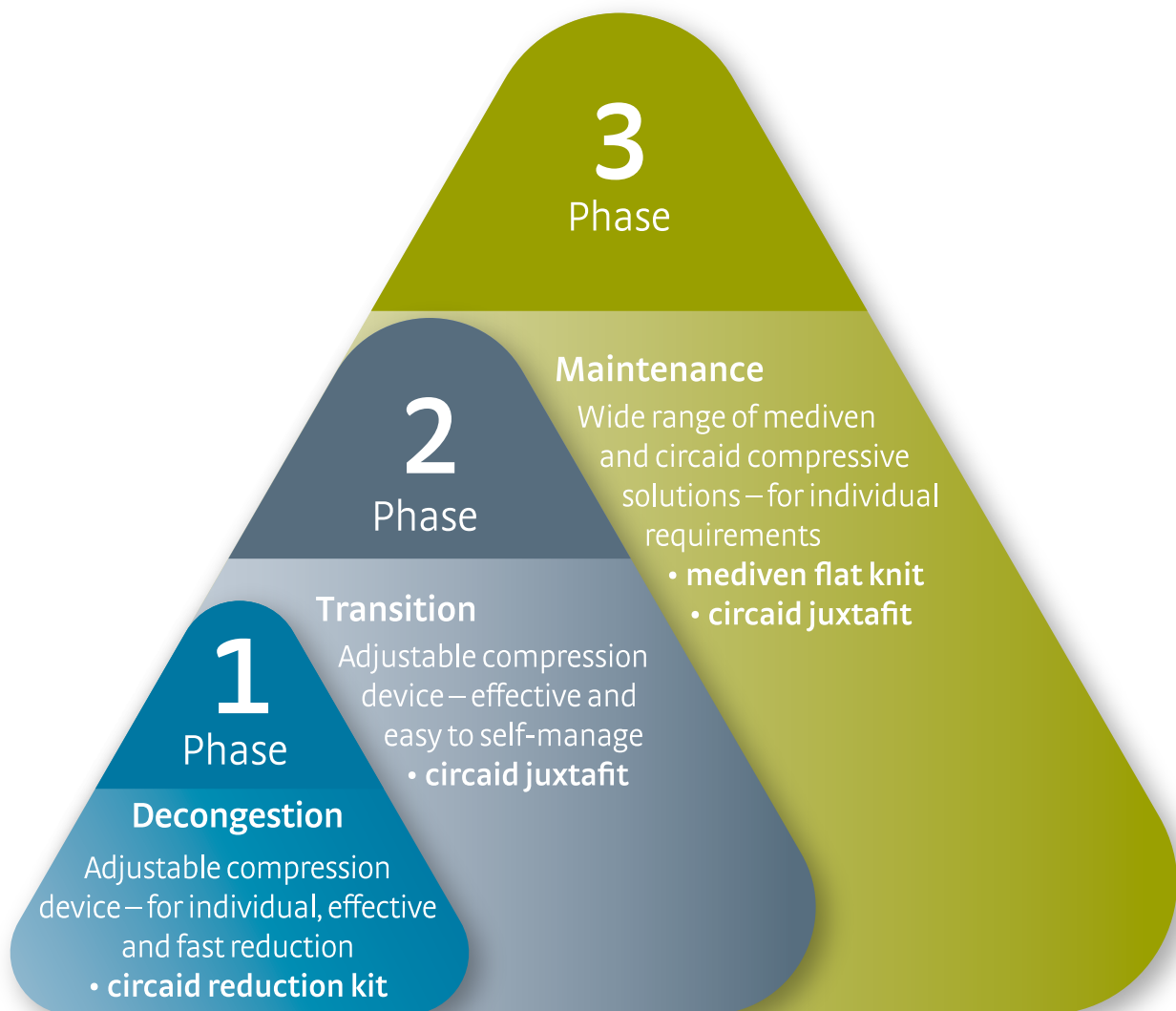



1

## Decongestion Phase



Adjustable and patient-individual compression for initial oedema reduction.

- **circaid® reduction kit**

2

## Transition Phase



Effective compression to optimise reduction results.

- **circaid® juxtafit®**

3

## Maintenance Phase



A huge variety of compression solutions to maintain therapy results in the long term.

- **mediven® flat knit**
- **circaid® juxtafit®**









# Decongestion Phase

Reducing the limb volume during the decongestion phase. A solution that drives decongestion and enables patient participation from day one.



Decongestion Phase

Transition Phase

Maintenance Phase



1

Phase



# circaid<sup>®</sup> reduction kit leg

Customisable, adjustable compression device for the decongestion phase of leg lymphoedema.

## medi tip\*:

Due to product features we recommend circaid reduction kit for:

- Men / women
- Initial decongestion of leg lymphoedema
- Up to 24 hour compression therapy

## Indications

- Lymphoedema
- Dependent oedema

## Product features

- Easy and time saving handling – simple donning and doffing
- Adjustable sizes – garment is initially fitted to the patient's individual circumferences and limb shape
- Ability to reduce size of garment as the limb volume decreases
- Built-In-Tension System (BTS) to set, control and re-adjust the correct therapeutic tension level – enables consistent compression and effective decongestion
- Re-adjusting the individual bands avoids pressure loss and thus sliding down of the garment – supporting oedema reduction effectively<sup>1</sup>
- Breathe-O-Prene: breathable and inelastic material with high wall-stability for high working pressure
- Odour-inhibiting material

## Benefits

- Easy handling – time-saving and less susceptible to error<sup>2</sup>
- Compression therapy – essential part of the complex lymphoedema therapy
- Faster reduction of lymphoedema<sup>3</sup>
- Increased patient-adherence: 94.6 % of the users experience the wearing comfort as pleasant<sup>2</sup>
- Promotes self-management thanks to easy handling

## Studies

<sup>1</sup> Mosti G et al. Adjustable compression wrap devices are cheaper and more effective than inelastic bandages for venous leg ulcer healing. A Multicentric Italian Randomized Clinical Experience. *Phlebology* 2020;35(2):124-133.

<sup>2</sup> Protz K et al. Comparison of handling, pressure and wearing comfort of different compression devices for decongestion therapy. *Veins and Lymphatics* 2018;7:7989:137-138.

<sup>3</sup> Damstra R et al. Prospective, randomized, controlled trial comparing the effectiveness of adjustable compression Velcro wraps versus inelastic multicomponent compression bandages in the initial treatment of leg lymphedema. *Journal of Vascular Surgery: Venous and Lymphatic Disorders* 2013;1(1):13-9.

## Adjustable compression

Tension 1 – 8

## Product benefits



Juxtaposed band




Variety



Easy handling



Inelastic

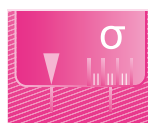

Built-In-Tension



Breathe-O-Prene



STANDARD  
100

16.0.87402  
Hohenstein HTTI

[www.oeko-tex.com](http://www.oeko-tex.com)

## Colours

Beige

## Size

Trimable (customisable)

## Note

A video for application and customisation of the circaid reduction kit can be found here: [medi.biz/rk](http://medi.biz/rk)

\* Please consider, that this recommendation does not replace the diagnosis and consultation of a medical healthcare professional.

Decongestion  
Phase

Transition  
Phase

Maintenance  
Phase




circaid reduction kit lower leg

## circaid® reduction kit lower leg

Art. No.	Description
JV23101000	circaid reduction kit lower leg short
JV23111000	circaid reduction kit lower leg short wide
JV23201000	circaid reduction kit lower leg standard
JV23211000	circaid reduction kit lower leg standard wide
JV23301000	circaid reduction kit lower leg long
JV23311000	circaid reduction kit lower leg long wide

Circumference in cm	Standard	Wide	
cC*	< 65	< 85	
cB	< 55	< 65	
Length measurement in cm	Short	Standard	Long
∠ malleolus – knee crease	< 32	32 – 36	> 36

\*cC should not be more than 30cm bigger than cB.

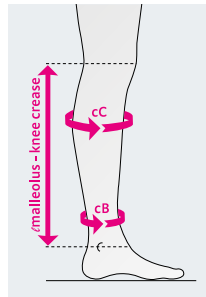

### Your order for circaid reduction kit lower leg includes:

- 1 adjustable compression device
- 1 Built-In-Tension System (BTS) guide card
- 1 pair of circaid undersleeves leg
- 1 circaid shelf strap with 6 hook tabs
- 6 small additional hook tabs

### Additional for the wide variants:

- circaid reduction kit extension




circaid reduction kit knee

## circaid® reduction kit knee

Art. No.	Description
JV21101000	circaid reduction kit knee standard
JV32001000	circaid reduction kit knee spine

Circumference in cm	Standard
cE2*	< 110
cE	< 102
cC	< 80
Length in cm	Standard
∠ C – E2	30

\*cE2 should not be more than 30 cm bigger than cC. cE2 is 10 cm above the knee

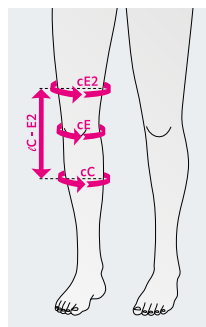

### Your order for circaid reduction kit knee includes:

- 1 adjustable compression device
- 2 circaid knee spine (connection piece)
- 1 circaid shelf strap with 6 hook tabs






circaid reduction kit upper leg

## circaid® reduction kit upper leg

Art. No.	Description
JV22101000	circaid reduction kit upper leg short
JV22111000	circaid reduction kit upper leg short wide
JV22201000	circaid reduction kit upper leg standard
JV22211000	circaid reduction kit upper leg standard wide

Circumference in cm	Standard	Wide
cG*	< 80	< 125
cE2**	< 65	< 110
Length measurement in cm	Short	Standard
knee crease – K1	< 37	> 41

\* cG should not be 30 cm bigger than cE2  
 \*\* cE2 is 10 cm above the knee

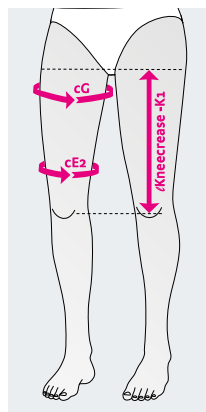

### Your order for circaid reduction kit upper leg includes:

- 1 adjustable compression device
- 1 Built-In-Tension System (BTS) guide card
- 1 circaid shelf strap with 6 hook tabs
- 6 small additional hook tabs

### Additional for the wide variants:

- circaid reduction kit extension

### Note

A foot option has to be ordered separately.



circaid reduction kit combination of upper leg, knee, lower leg and foot option



Built-In-Tension system card (BTS)



Fitting of the lower leg

# circaid<sup>®</sup> foot options

The following foot options are recommended to wear in combination with circaid reduction kit.

## Note

It is recommended to always wear a circaid foot option in combination with a circaid leg garment. The indications of the circaid foot options are identical to the indications of the corresponding circaid leg garment. Please order separately.




circaid pac band

## circaid<sup>®</sup> power added compression band (pac band)

**Additional compression for the arch of the foot and ankle.**

### Product details

- Additional compression to the foot with mild to severe tendency for oedema
- Targets the arch of the foot and around the malleoli
- Can be trimmed to fit almost every foot shape
- Easy handling
- Lightweight and breathable material
- Available in standard, large and extra large
- Bilateral use (left or right foot)

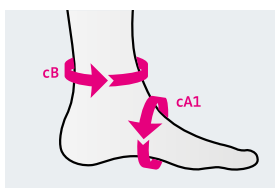

Circumferences in cm	Standard	Large	Extra large
cB	19 – 27	25 – 50	45 – 56
cA1	19 – 36	22 – 41	22 – 41
Art. No.	JU5P001000	JU5P002000	JU5P003000




circaid customizable interlocking ankle foot wrap

## circaid<sup>®</sup> customizable interlocking ankle foot wrap

**Customisable foot option for oedema decongestion.**

### Product details


- Customisable to patient-individual circumferences and shape
- Adjustable to reducing circumferences during decongestion
- Interlocking heel band for foot length adjustment
- Heel band offers targeted compression to the Bisgaard's region and ankle
- Easy handling
- One size fits all

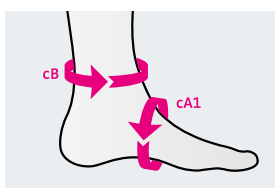

Circumferences in cm	
cB	< 56
cA1	< 44
Art. No.	JU5S001000



circaid toe cap

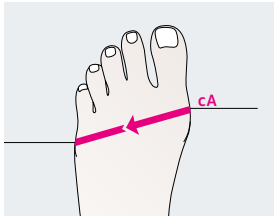

## circaid® toe cap

### Additional compression for the toes.

#### Product details

- Provides light to moderate compression
- Individually adjustable length of the toes
- Designed for left or right limb use
- It is recommended to always wear the circaid toe cap with a circaid foot option

Circumference in cm	2	3	4	3	6
cA	20 – 22	22 – 24	24 – 26	26 – 28	28 – 30
Art. No.	JU5T002000	JU5T003000	JU5T004000	JU5T005000	JU5T006000

#### Tip

Video for application and customisation of the circaid foot options can be found here: [medi.biz/rk](https://medi.biz/rk)





# circaid<sup>®</sup> reduction kit accessories

## Note

It is recommended to always use a circaid supporting band in combination with a circaid leg garment. The indications of the circaid supporting bands are identical to the indications of the corresponding circaid leg garment. Please order separately.



circaid reduction kit shelf strap

## circaid<sup>®</sup> reduction kit shelf strap

### Additional support.

- Offers additional support and coverage
- Covers potential gaps that may appear

Art. No.	Description
JV33001000	circaid reduction kit shelf strap customizable



circaid reduction kit lobe lifter

## circaid<sup>®</sup> reduction kit lobe lifter

### Additional compression and coverage for extreme lobules.

- Offers additional support and coverage
- Customisable for patients' individual needs

Art. No.	Description
JV33002000	circaid reduction kit lobe lifter








# Transition Phase

Following the intensive therapy of reduction, some patients may benefit from a period of transition management before progressing into long-term therapy.

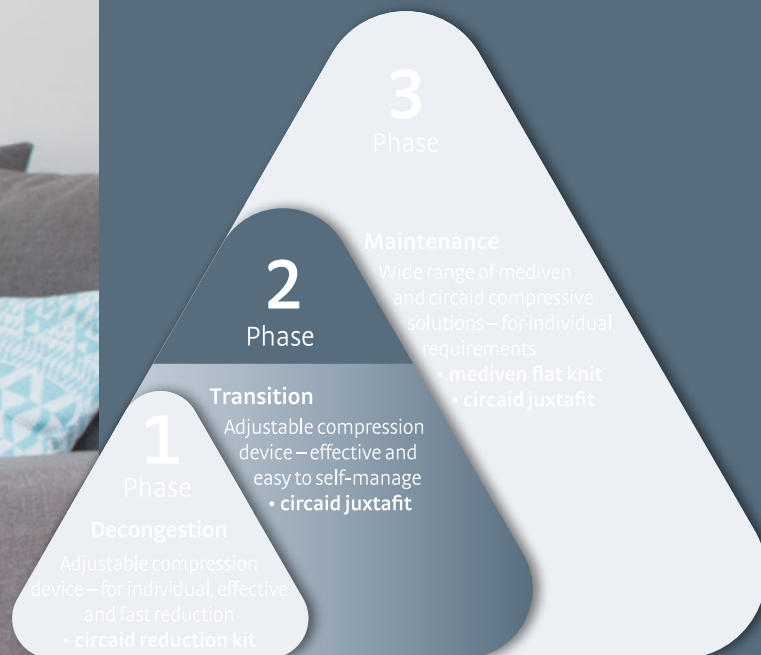

Decongestion  
Phase

Transition  
Phase

Maintenance  
Phase



2

Phase



# circaid® juxtafit® premium leg

Premium adjustable compression device for optimising and maintaining decongestion results in lymphoedema treatment.

## medi tip\*:

Due to product features we recommend circaid juxtafit for:

- Men / women
- Optimising and maintaining decongestion results
- Patients who have challenges with medical compression stockings
- Up to 24 hour compression therapy
- Mobile / immobile patients

## Indications

- Lymphoedema
- Lipoedema
- Dependent oedema

## Product features

- High quality band fasteners with reinforced landing straps
- Easy handling – simple donning and doffing
- circaid Built-In-Pressure System (BPS): adjustable, controllable and re-adjustable compression
- Re-adjusting the individual bands avoids pressure loss and thus sliding down of the garment
- Breathe-O-Prene: breathable and inelastic material with high wall-stability for high working pressure
- Odour-inhibiting material
- Features for upper leg: Smooth material in the back of the knee – for more wearing comfort
- Also available with two compressive undersocks moderate
  - Applies graduated compression to the foot and ankle
  - Graduated compression continues into the leg portion of the undersock
- Twelve month lifespan


## Benefits

- Offering an alternative therapy solution for patients who are not able to apply or tolerate compression stockings
- Compression therapy – essential part of the complex lymphoedema therapy<sup>1</sup>
- Increased patient-adherence: 94.6 % of the users experience the wearing comfort as pleasant<sup>2</sup>
- Increased mobility: possibility of wearing regular footwear<sup>2</sup>
- Easy handling allows for self-management of the patient

## Adjustable compression

20 – 30 mmHg  
30 – 40 mmHg  
40 – 50 mmHg  
50+ mmHg

## Product benefits

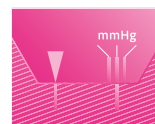

Built-In-Pressure®



Variety



Juxtaposed band



Easy handling



Inelastic



Breathe-O-Prene



STANDARD  
100

16.0.87402  
Hohenstein HTTI

www.oeko-tex.com

## Colours

S  
Beige

S = Standard

## Note

A video for application of the circaid juxtafit can be found here: [medi.biz/jf](https://medi.biz/jf)

\* Please consider, that this recommendation does not replace the diagnosis and consultation of a medical healthcare professional.

## Studies

<sup>1</sup> Lymphoedema Framework. Best Practice for the Management of Lymphoedema. International consensus. London: MEP Ltd, 2006.

<sup>2</sup> Protz K et al. Comparison of handling, pressure and wearing comfort of different compression devices for decongestion therapy. Veins and Lymphatics 2018;7:7989:137-138.

Decongestion  
Phase

Transition  
Phase

Maintenance  
Phase





circaid juxtafit premium lower leg

## circaid® juxtafit® premium lower leg standard styles


Styles – Standard	BD	BD
Size / Art. No.	Short	Long
S	JU35702000	JU35002000
M	JU35703000	JU35003000
L	JU35704000	JU35004000
XL	JU35705000	JU35005000
XXL	JU35706000	JU35006000
M extra wide	JU35713000	JU35013000
L extra wide	JU35714000	JU35014000
XL extra wide	JU35715000	JU35015000

Circumferences in cm	S	M	M – X	L
cC	30 – 38	36 – 43	43 – 50	41 – 48
cB1	20 – 28	25 – 33	33 – 40	30 – 38
cB	15 – 23	20 – 28	20 – 28	25 – 33
	L – X	XL	XL – X	XXL
cC	48 – 56	41 – 51	51 – 61	53 – 64
cB1	38 – 46	33 – 41	41 – 51	43 – 53
cB	25 – 33	30 – 38	30 – 38	33 – 41

Length measurement in cm	Short	Long
ℓ malleolus – knee crease	< 36	> 36



### Your order for circaid juxtafit premium lower leg includes:

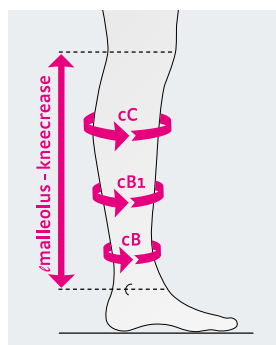- 1 adjustable compression device
- 1 circaid Built-In-Pressure System (BPS) guide card
- 1 pair of circaid compression anklets
- 1 circaid power added compression band (pac band)
- 1 pair of circaid undersleeves lower leg silver
- 1 circaid cover up



circaid juxtafit premium lower leg



circaid juxtafit premium lower leg (readjustment)





circaid juxtafit premium lower leg band fasteners with landing straps



circaid juxtafit premium lower leg with circaid compressive undersock

## circaid®juxtafit® premium lower leg with circaid® compressive undersock


Styles – Standard	BD	BD
		
Size / Art. No.	Short	Long
S	JU3M702000	JU3M002000
M	JU3M703000	JU3M003000
L	JU3M704000	JU3M004000
XL	JU3M705000	JU3M005000
XXL	JU3M706000	JU3M006000
M extra wide	JU3M713000	JU3M013000
L extra wide	JU3M714000	JU3M014000
XL extra wide	JU3M715000	JU3M015000

Circumferences in cm	S	M	M – X	L
cC	30 – 38	36 – 43	43 – 50	41 – 48
cB1	20 – 28	25 – 33	33 – 40	30 – 38
cB	15 – 23	20 – 28	20 – 28	25 – 33
	L – X	XL	XL – X	XXL
cC	48 – 56	41 – 51	51 – 61	53 – 64
cB1	38 – 46	33 – 41	41 – 51	43 – 53
cB	25 – 33	30 – 38	30 – 38	33 – 41

Length measurement in cm	Short	Long
ℓ malleolus – knee crease	< 36	> 36



### Your order for circaid juxtafit premium lower leg includes:

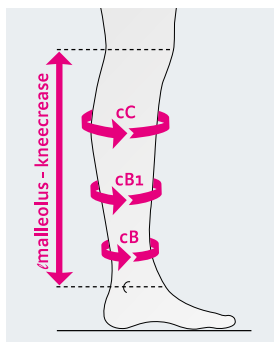- 1 adjustable compression device
- 1 circaid Built-In-Pressure System (BPS) guide card
- 1 pair of circaid compressive undersocks moderate
- 1 circaid power added compression band (pac band)
- 1 circaid cover up

Decongestion  
Phase

Transition  
Phase

Maintenance  
Phase

2

Phase





# circaid® juxtafit® essentials leg

Ready-to-wear adjustable compression device for optimising and maintaining decongestion results in lymphoedema treatment.

## medi tip\*:

**Due to product features we recommend circaid juxtafit for:**

- Men / women
- Optimising and maintaining decongestion results
- Patients who have challenges with medical compression stockings
- Up to 24 hour compression therapy
- Mobile / immobile patients

## Indications

- Lymphoedema
- Lipoedema
- Dependent oedema

## Product features

- Easy handling – simple donning and doffing
- circaid Built-In-Pressure System (BPS): adjustable, controllable and re-adjustable compression
- Re-adjusting the individual bands avoids pressure loss and thus sliding down of the garment
- Sustained compression with defined pressure for up to 24 hours if required
- Breathe-O-Prene: breathable and inelastic material with high wall-stability for high working pressure
- Also available with two compressive undersocks moderate
  - Applies graduated compression to the foot and ankle
  - Graduated compression continues into the leg portion of the undersock
- Odour-inhibiting effect
- Features for upper leg
  - Smooth material in the back of the knee – for more wearing comfort
  - Garment straps can be trimmed for better handling and wearing comfort
  - Garment can be trimmed in length for an optimum fit
- Six month lifespan


## Benefits

- Offering an alternative therapy solution for patients who are not able to apply or tolerate compression stockings
- Compression therapy – essential part of the complex lymphoedema therapy<sup>1</sup>
- Increased patient-adherence: 94.6 % of the users experience the wearing comfort as pleasant<sup>2</sup>
- Increased mobility: possibility of wearing regular footwear
- Easy handling allows for self-management of the patient

## Adjustable compression

20 – 30 mmHg  
30 – 40 mmHg  
40 – 50 mmHg  
50+ mmHg

## Product benefits

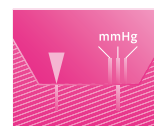

Built-In-Pressure®



Variety




Juxtaposed band



Easy handling



Inelastic

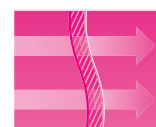

Breathe-O-Prene



## Colours

S  
Beige

S = Standard

## Note

A video for application of the circaid juxtafit can be found here: [medi.biz/jf](https://medi.biz/jf)

\* Please consider, that this recommendation does not replace the diagnosis and consultation of a medical healthcare professional.

## Studies

<sup>1</sup> Lymphoedema Framework. Best Practice for the Management of Lymphoedema. International consensus. London: MEP Ltd, 2006.

<sup>2</sup> Protz K et al. Comparison of handling, pressure and wearing comfort of different compression devices for decongestion therapy. Veins and Lymphatics 2018;7:7989:137-138.

Decongestion  
Phase

Transition  
Phase

Maintenance  
Phase



circaid juxtafit essentials lower leg

## circaid®juxtafit® essentials lower leg


Styles – Standard	BD	BD
Size / Art. No.	Short	Long
S	JU25702000	JU25002000
M	JU25703000	JU25003000
L	JU25704000	JU25004000
XL	JU25705000	JU25005000
XXL	JU25706000	JU25006000
XXXL	JU25707000	JU25007000
M extra wide	JU25713000	JU25013000
L extra wide	JU25714000	JU25014000
XL extra wide	JU25715000	JU25015000

Circumferences in cm	S	M	M – X	L	L – X
cC	30 – 38	36 – 43	43 – 50	41 – 48	48 – 56
cB1	20 – 28	25 – 33	33 – 40	30 – 38	38 – 46
cB	15 – 23	20 – 28	20 – 28	25 – 33	25 – 33
	XL	XL – X	XXL	XXXL	
cC	41 – 51	51 – 61	53 – 64	58 – 70	
cB1	33 – 41	41 – 51	43 – 53	50 – 60	
cB	30 – 38	30 – 38	33 – 41	42 – 52	

Length measurement in cm	Short	Long
ℓ malleolus – knee crease	< 36	> 36



### Your order for circaid juxtafit essentials lower leg includes:

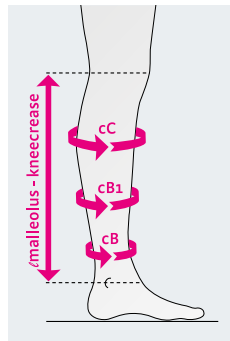- 1 adjustable compression device
- 1 circaid Built-In-Pressure System (BPS) guide card
- 1 pair of circaid compression anklets\*
- 1 circaid power added compression band (pac band)
- 1 pair of circaid undersleeves lower leg standard\*\*
- 1 circaid cover up

\* Not included with size XXXL, \*\* For size XXXL with foot



circaid juxtafit essentials combination, circaid juxtafit essentials upper leg with knee, lower leg and foot wrap (pac band)



circaid juxtafit essentials lower leg with pac band





circaid juxtafit essentials lower leg band fasteners



circaid juxtafit essentials lower leg with circaid compressive undersock

## circaid®juxtafit® essentials lower leg with circaid® compressive undersock


Styles – Standard	BD	BD
		
Size / Art. No.	Short	Long
S	JU2M702000	JU2M002000
M	JU2M703000	JU2M003000
L	JU2M704000	JU2M004000
XL	JU2M705000	JU2M005000
XXL	JU2M706000	JU2M006000
M extra wide	JU2M713000	JU2M013000
L extra wide	JU2M714000	JU2M014000
XL extra wide	JU2M715000	JU2M015000

Circumferences in cm	S	M	M – X	L
cC	30 – 38	36 – 43	43 – 50	41 – 48
cB1	20 – 28	25 – 33	33 – 40	30 – 38
cB	15 – 23	20 – 28	20 – 28	25 – 33
	L – X	XL	XL – X	XXL
cC	48 – 56	41 – 51	51 – 61	53 – 64
cB1	38 – 46	33 – 41	41 – 51	43 – 53
cB	25 – 33	30 – 38	30 – 38	33 – 41

Length measurement in cm	Short	Long
ℓ malleolus – knee crease	< 36	> 36



### Your order for circaid juxtafit essentials lower leg includes:

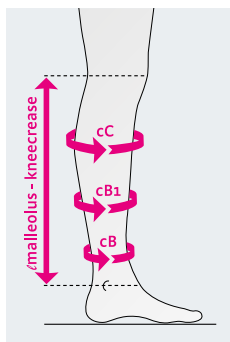- 1 adjustable compression device
- 1 circaid Built-In-Pressure System (BPS) guide card
- 1 pair of circaid compressive undersocks moderate
- 1 circaid power added compression band (pac band)
- 1 circaid cover up

Decongestion Phase

Transition Phase

Maintenance Phase





circaid juxtafit essentials  
upper leg with knee

## circaid® juxtafit® essentials upper leg with knee


Styles – Standard		DG	DG	DG
Size / Art. No.		Extra Short	Short	Long
<b>Left</b>	XXS	JU2878X000	JU2873X000	JU2803X000
	XS	JU28781000	JU28731000	JU28031000
	S	JU28782000	JU28732000	JU28032000
	M	JU28783000	JU28733000	JU28033000
	L	JU28784000	JU28734000	JU28034000
	XL	JU28785000	JU28735000	JU28035000
<b>Right</b>	XXS	JU2879X000	JU2874X000	JU2804X000
	XS	JU28791000	JU28741000	JU28041000
	S	JU28792000	JU28742000	JU28042000
	M	JU28793000	JU28743000	JU28043000
	L	JU28794000	JU28744000	JU28044000
	XL	JU28795000	JU28745000	JU28045000

Circumferences in cm	XXS	XS	S	M	L	XL
cG	40 – 52	46 – 62	54 – 70	60 – 80	69 – 90	77 – 101
cE1*	32 – 42	37 – 48	42 – 55	50 – 65	58 – 76	66 – 87
cD	23 – 30	27 – 35	31 – 41	35 – 46	40 – 52	44 – 58

Length measurement in cm	Extra Short	Short	Long
∅knee crease – K1	< 25	25 – 35	> 35

\*cE1 is 5 cm above the patella

\*\*product length can be shortened by 5 cm

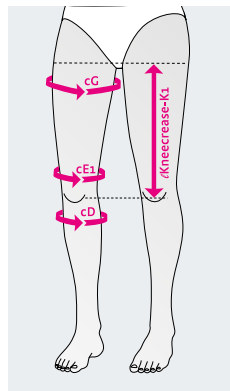

### Your order for circaid juxtafit essentials upper leg with knee includes:

- 1 adjustable compression garment upper leg
- 1 circaid Built-In-Pressure System (BPS) guide card
- 1 pair of circaid undersleeves whole leg
- 1 circaid cover up
- 8 additional hook and loop fasteners



Decongestion  
Phase

Transition  
Phase

Maintenance  
Phase

# circaid<sup>®</sup> foot options

The following foot options are recommended to wear in combination with circaid juxtafit.

## Note

It is recommended to always wear a circaid foot option in combination with a circaid leg garment. The indications of the circaid foot options are identical to the indications of the corresponding circaid leg garment. Please order separately.




circaid power added compression band (pac band)

## circaid<sup>®</sup> power added compression band (pac band)

**Additional compression for the arch of the foot and ankle.**

### Product details

- Additional compression to the foot with mild to severe tendency for oedema
- Targets the arch of the foot and around the malleoli
- Can be trimmed to fit almost every foot shape
- Easy handling
- Lightweight and breathable material
- Available in standard, large and extra large
- Bilateral use (left or right foot)

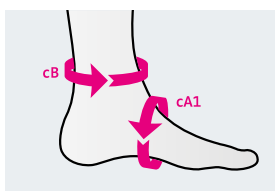

Circumferences in cm	Standard	Large	Extra large
cB	19 – 27	25 – 50	45 – 56
cA1	19 – 36	22 – 41	22 – 41
Art. No.	JU5P001000	JU5P002000	JU5P003000




circaid toe cap

## circaid<sup>®</sup> toe cap

**Additional compression for the toes.**

### Product details

- Provides light to moderate compression
- Individually adjustable length of the toes
- Designed for left or right limb use
- It is recommended to always wear the circaid toe cap with a circaid foot option

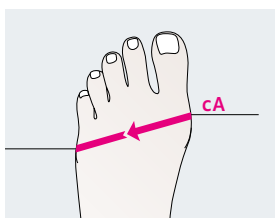


Circumference in cm	2	3	4	5	6
cA	20 – 22	22 – 24	24 – 26	26 – 28	28 – 30
Art. No.	JU5T002000	JU5T003000	JU5T004000	JU5T005000	JU5T006000





**circaid juxtafit premium ankle foot wrap**

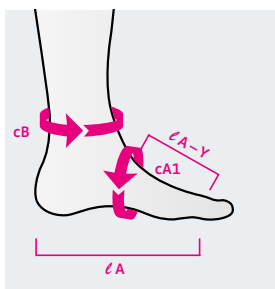

## circaid® juxtafit® premium ankle foot wrap

**Adjustable compression device for the foot with closed heel.**


### Product details

- Provides compression to the foot and ankle
- High quality band fasteners with reinforced landing straps
- Easy handling
- Can be trimmed in length to provide perfect fit
- Available in standard sizes

Measurements in cm	S	M	L
lA-Y	>9	>9	>9
cB	<36	<36	<41
cA1	19-24	24-29	29-34
lA	>16	>16	>16
Art. No.	JU3W002000	JU3W003000	JU3W004000



**circaid juxtafit premium interlocking ankle foot wrap**

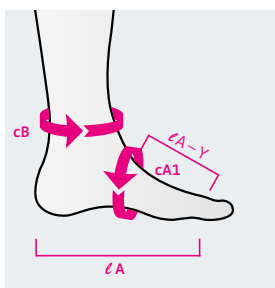

## circaid® juxtafit® premium interlocking ankle foot wrap

**Adjustable compression device for the foot with foot length adjustment.**

### Product details

- Provides compression to the foot and ankle
- Interlocking heel band allows foot length adjustment
- High quality band fasteners with reinforced landing straps
- Heel band offers targeted compression to the Bisgaard's region and ankle
- Easy handling
- Available in standard sizes only

Measurements in cm	S	M	L
lA-Y	>9	>9	>9
cB	<36	<36	<41
cA1	19-24	24-29	29-34
lA	13-25	13-25	13-25
Art. No.	JU3I002000	JU3I003000	JU3I004000

### Tip

Video for application and customisation of the circaid foot options can be found here: [medi.biz/jf](https://medi.biz/jf)



**circaid single band ankle foot wrap**

## circaid® single band ankle foot wrap

**Additional compression for the arch of the foot and ankle.**

### Product details

- Provides compression to foot and ankle
- Suitable for most foot sizes and shapes
- Ideal for individual bandaging of the foot
- One size fits all

### Measurements

<b>Width</b>	7.6 cm
<b>Length</b>	203 cm
<b>Art. No.</b>	JU5Q001000




**circaid compression anklet**

## circaid® compression anklet

**Standard compression option for foot and ankle.**

### Product details

- Provides compression to the foot and ankle
- Can be combined with other circaid foot options
- Available in standard and large

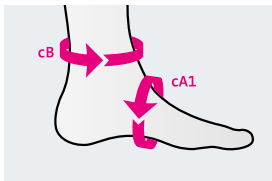

Circumferences in cm	Standard	Large
<b>cB</b>	19 – 27	25 – 50
<b>cA1</b>	19 – 36	22 – 41
<b>Art. No.</b>	JU51001000	JU51002000




**circaid compressive undersock**

## circaid® compressive undersock

**Easy-to-apply liner with targeted compression for foot and ankle.**

### Product details

- Applies graduated compression to the foot and ankle
- Graduated compression continues into the leg portion of the undersock
- Soft material for pleasant wearing comfort
- Available in 15 – 25 mmHg (moderate compression) and 25 – 35 mmHg (firm compression)
- Available in standard and large

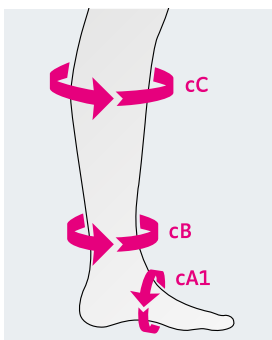

	Standard	Large
<b>cC</b>	26 – 54	41 – 64
<b>cB</b>	19 – 27	25 – 42
<b>cA1</b>	19 – 32	24 – 41

Size	Strength	Article No.
<b>Standard</b>	Moderate	JU5M001000
<b>Large</b>	Moderate	JU5M002000
<b>Standard</b>	Firm	JU5F001000
<b>Large</b>	Firm	JU5F002000

# circaid® juxtafit® accessories



circaid undersleeve lower leg, silver



circaid undersock lower leg with foot, silver

## circaid® undersleeves

For whole leg and lower leg.

- Can be worn directly against the skin underneath circaid products to provide added comfort
- Note: does not provide therapeutic compression

### Leg versions

circaid undersleeves (without foot)

- lower leg
- whole leg
- lower leg, silver

circaid undersock (with foot)

- lower leg
- lower leg, silver

Standard size	Art. No.	Type	Style
Undersleeve lower leg	JULS709000		BD
	JUSS709000	Silver	BD
Undersleeve whole leg	JULS009000		BG
Undersock lower leg	JULT709000		AD
	JUST709000	Silver	AD

Decongestion  
Phase

Transition  
Phase

Maintenance  
Phase






circaid cover up, black

## circaid® cover up

### Silky oversleeve for circaid garments.

- Can be worn over circaid arm and leg garments
- Prevents the garments from snagging on clothing or bedding
- Gives adjustable compression garments a refined look
- Available in black and beige
- Note: does not provide therapeutic compression

Standard size	Art. No.	Colour	Size
BD	JU5C7B2000	Beige	S
	JU5C7B4000	Beige	L
	JU5C7A2000	Black	S
	JU5C7A4000	Black	L
	JU5C700000		Made-to-measure

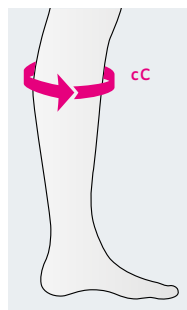

Circumferences in cm	Small	Large
cC	34 – 50	51 – 80

Measuring point cC for BD garment.



circaid juxtafit premium shelf strap

## circaid® juxtafit® premium shelf strap

### Additional support and coverage.

Offers additional support and coverage

Art. No.	Size	Size in cm
JU2Q001000	S	20.5 x 6.5 (without hook tabs)
JU2Q002000	L	25.5 x 9.0 (without hook tabs)
JU2Q000000	Made-to-measure	–

#### Note

It is recommended to always use a circaid supporting band in combination with a circaid leg garment. The indications of the circaid supporting band are identical to the indications of the corresponding circaid leg garment. Please order separately.



Decongestion  
Phase

Transition  
Phase

Maintenance  
Phase







# Maintenance Phase

Long-term treatment of lymphoedema focuses on the improvement of the lymphatic vessels' function, reducing further swelling and achieving long-term control of the lymphoedema. A successful maintenance phase depends on effective compression hosiery.



Decongestion  
Phase

Transition  
Phase

Maintenance  
Phase

## Care compass leg: perfectly supplied with mediven® flat knit

Each mediven fabric has its own unique features, therefore please refer to the information below to find the most suitable mediven fabric for your patient.\*

**Lipoedema**  
(Pain as the leading symptom –  
no staging applies)



**Lymphoedema**  
(Stage 1 – 3  
in the maintenance phase)



Connective tissue

normal to firm

soft to firm

very soft to hardened

Weight



Deepened skin folds /  
calibre jumps

none

moderate

strong

Recommended  
mediven compression  
fabric

mediven mondi®

mediven® cosy<sup>1</sup>

mediven® 550 leg

<sup>1</sup> medi recommendation: can be used for Lymphoedema up to stage 2

The softer / firmer the connective tissue – the higher the weight – the stronger the skin folds /  
caliber jumps...

... the stronger the stocking material should be.

→ The most prominent condition indicates fabric choice.




\* Please note that this recommendation does not replace the diagnosis of a doctor or the advice of a medical specialist. The choice of medically necessary and customised compression fabric depends on the indication and the patient's individual needs.



# LATERAL SEAM

FIGURES · IMPRESSIONS · FEEDBACK

medi surveyed 336 people about their experiences with the lateral seam for mediven cosy or mediven 550 – here are the results.\*

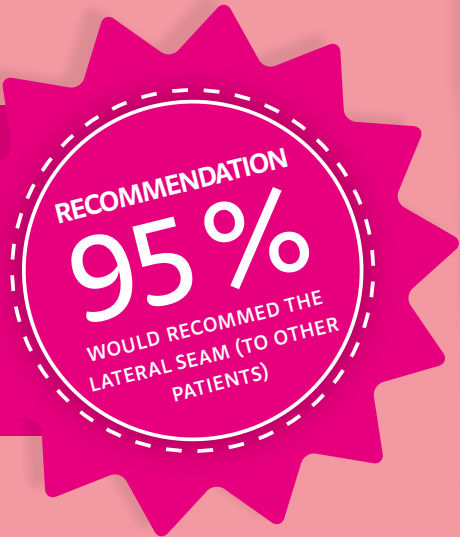

## FEATURES

Rated as **VERY GOOD/GOOD**

SPECIAL

- 98 %** SEAMLESS SURFACE WHEN SEATED
- 97 %** SEAMLESS FOOT SECTION
- 93 %** EXTRA COMFORT AT THE BACK OF THE KNEE

<input checked="" type="checkbox"/>	<input type="checkbox"/>
<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>

## CHARACTERISTICS

Rated as **VERY GOOD/GOOD**

GENERAL

- 93 %** WEARING COMFORT
- 92 %** FIT
- 91 %** EASE OF DONNING
- 87 %** LOOK / PLACEMENT OF THE SEAM

<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
--------------------------	--------------------------	--------------------------	--------------------------	--------------------------	--------------------------

## COMPARISON

to medical compression garments with back seam, the product received ratings of **VERY GOOD / GOOD**

IN

- 88 %** WEARING COMFORT
- 63 %** FIT
- 46 %** DONNING

<input checked="" type="checkbox"/>	<input type="checkbox"/>
<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>

<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
--------------------------	--------------------------	--------------------------	--------------------------	--------------------------	--------------------------

\* medi GmbH & Co. KG.  
Patient survey lateral seam  
05/2025 (n=336).



3

Phase





# mediven® 550 leg

**Strong. Very high wall stability. Reliable hold.**

## medi tip<sup>1</sup>:

**Because of the material, the fit and the product features, we recommend mediven 550 leg for:**

- Very soft to hardened connective tissue
- Strong deepened skin wrinkles / calibre jumps
- Associated weight disorders
- Pronounced lymphoedema

## Indications<sup>2</sup>


- Primary and secondary lymphoedema stages 1, 2 and 3, also obesity associated lymphoedema
- Lipoedema
- Post-operative oedema
- Post-traumatic oedema
- Mixed forms: e.g. phlebo-lymphoedema

## Product features

- Very high wall stability
- High working pressure
- Firm, strong fabric


## Special features

- UV protection factor 40
- Wide range of accessories such as soft toe, gluteal shaper, hallux or varus toe ease zone, flexure functional zones
- Numerous variants such as leg sections for double and multipart garments
- Clima Comfort guarantees a high level of breathability
- Fashionable variety: 8 standard colours, 4 trend colours, and 4 patterns for Design-Elements and Fashion-Elements

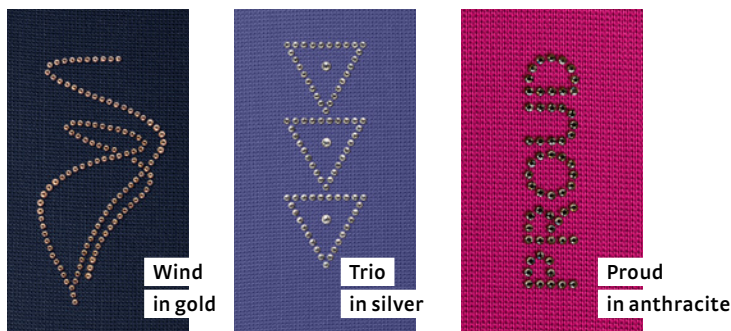

Available with sparkling crystals

## Patterns for Design-Elements<sup>3</sup> and Fashion-Elements<sup>4</sup>

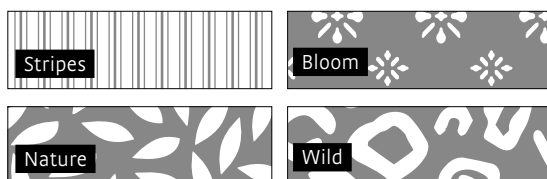

## Compression Class

**CCL 1, 2, 3, 4**

## Product benefits



**Clima Comfort**



**Clima Fresh**



**Perfect Fit**



**Variety**



**STANDARD 100**  
97.0.3231  
Hohenstein HTTI




## Colours

### Standard colours



### Trend colours

Delivery period up to 10 working days

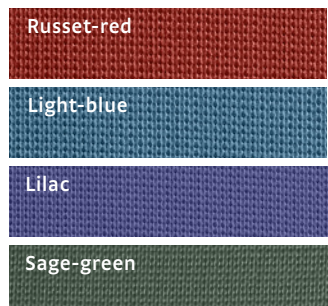

\* Standard colours with a delivery period up to 10 working days

<sup>1</sup> The choice of the optimum compression stocking depends on the indication and the individual needs of the patient.

<sup>2</sup> Please note that this recommendation does not replace the diagnosis or the advice from a medical professional.

<sup>3</sup> The Design-Elements can be freely combined with the standard and trend colours (no extra charge).

<sup>4</sup> Please note: Fashion-Elements are at extra costs



Combination of below knee stocking (AD), capri panty (CG) and cycling shorts (ET)





















Combination of below knee stocking (AD) and CT leg section



One-legged pantyhose (ATE)

## mediven® 550 leg made-to-measure styles

Styles – made-to-measure	AD	AF	AG	AT / ATH / ATU		
			 optional with belt			 optimised maternity pantytop
Styles – made-to-measure	ATE / ATEH	BT / BTH	B1T / B1TH CT / CTH	ET / ETH FT / FTH	AB	
	 left right					
Styles – made-to-measure	AB1	BD	BF	BG	CF	
						
Styles – made-to-measure	CG	DG				
						

### Standard models

Border	Flat oblique with all stocking variants
Styles	Flat oblique at the thigh and perforated topband with all pantyhose variants
Gusset	Men's gusset: ATH, ATEH, FTH, ETH, CTH, BTH non-compressive front panty top: ATU, ATEU, FTU, ETU, CTU, BTU
Foot	Oblique, open foot
Seam	back seam, lateral seam


### Material composition

	Polyamide (PA)	Elastane (EL)
CCL 1	77 %	23 %
CCL 2	75 %	25 %
CCL 3	72 %	28 %
CCL 4	68 %	32 %





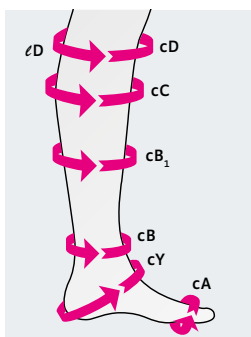

Below knee stocking (AD)  
with open toe



l = length measurements  
c = circumferences

## mediven® 550 leg standard styles (ready-to-wear)

Styles – Standard	AD	AD pte.
		
	calf	calf petite
<b>Article No.</b>	B2407*	B2477*
	*add size number 01 – 14	

<b>CCL</b>	2
<b>Length</b>	2 lengths for AD / calf and AD / calf petite
<b>Foot</b>	Open toe straight ending
<b>Topband</b>	Silicone dot topband
<b>Standard sizes</b>	IV – XIV
<b>Colour</b>	Caramel

Size / Art. No.		IV	V	VI	VII	VIII
AD, open toe		B240701000	B240702000	B240703000	B240704000	B240705000
AD, closed toe		B240711000	B240712000	B240713000	B240714000	B240715000
AD calf petite, open toe		B247701000	B247702000	B247703000	B240704000	B240705000
AD calf petite, closed toe		B247711000	B247712000	B247713000	B247714000	B247715000
IX		X	XI	XII	XIII	XIV
AD, open toe	B240706000	B240707000	B240708000	B240709000	B240723000	B240724000
AD, closed toe	B240716000	B240717000	B240718000	B240719000		
AD calf petite, open toe	B247706000	B247707000	B247708000	B247709000	B247723000	B247724000
AD calf petite, closed toe	B247716000	B247717000				

Circumference in cm (Tension dimension)	IV	V	VI	VII	VIII
cD	29 – 33	30 – 34	31 – 35	32 – 36	33 – 37
cC	30 – 34	31 – 35	32 – 36	33 – 37	34 – 38
cB <sub>1</sub>	24 – 26	25 – 27	26 – 28	27 – 29	28 – 30
cB	19 – 20	20 – 21	21 – 22	22 – 23	23 – 24
cY	27 – 29	28 – 30	29 – 32	29 – 33	30 – 34
cA	18 – 20	19 – 21	20 – 22	21 – 23	22 – 24

Circumference in cm (Tension dimension)	IX	X	XI	XII	XIII	XIV
cD	34 – 38	35 – 39	36 – 40	37 – 41	38 – 42	39 – 43
cC	35 – 39	36 – 40	37 – 41	38 – 42	39 – 43	40 – 44
cB <sub>1</sub>	29 – 32	29 – 33	30 – 34	31 – 35	32 – 36	33 – 37
cB	24 – 25	25 – 26	26 – 27	27 – 28	28 – 29	29 – 30
cY	31 – 35	32 – 36	33 – 37	34 – 38	35 – 39	36 – 40
cA	23 – 25	24 – 26	25 – 27	26 – 28	27 – 29	28 – 30

Length measurements in cm	
lD normal	39 – 41
lD short	37 – 39

Decongestion  
Phase

Transition  
Phase

Maintenance  
Phase



3

Phase





# mediven<sup>®</sup> 550 toe cap

**Strong. Very high wall stability. Reliable hold.**


## Indications<sup>1</sup>

- Patients with lymphoedema in the toes (Stemmer positive) and on the dorsum of the foot

## Product features

- Toe cap with Clima Fresh: Odour-inhibiting effect
- Top and bottom principle:
  - Seams moved to the upper and lower surfaces of the toes (no seams between the toes)
  - Each single toe made to measure
- medi tip: optional integration of medi lymphpads for additional pressure in cases of dorsum foot lymphedema

## Patterns for Design-Elements<sup>3</sup> and Fashion-Elements<sup>4</sup>

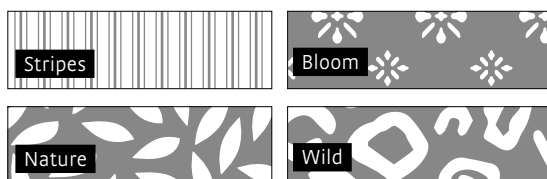

<sup>1</sup> Please note that this recommendation does not replace the diagnosis or the advice from a medical professional.

<sup>2</sup> The Design-Elements can be freely combined with the standard and trend colours (no extra charge).

<sup>3</sup> Please note: Fashion-Elements are at extra costs

## Compression Class

CCL 1, 2, 3

## Product benefits



Clima Comfort



Clima Fresh



Perfect Fit



Variety



STANDARD  
100  
97.0.3231  
Hohenstein HTTI



dermatest-  
guarantee.com


## Colours

### Standard colours



### Trend colours

Delivery period up to 10 working days

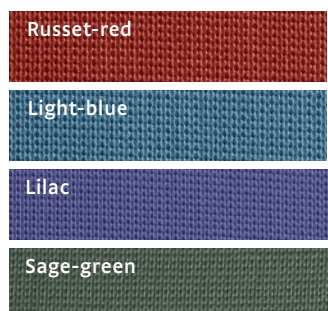

\* Standard colours with a delivery period up to 10 working days

Decongestion  
Phase

Transition  
Phase

Maintenance  
Phase





Fully-knitted toe cap

## Fully-knitted toe cap

- More reliable measurements (no longer necessary to calculate the overlap)
- Care to suit the indication for optimum lymphatic drainage (no circular seams)
- Very comfortable to wear, without circular seams
- Outstanding non-slip grip
- Available in variants AB, AB1, AD, AF, AG, AT, ATH, ATU, ATE/ATEH
- Different compression classes in toe cap and leg garment available
- UV protection factor 40



Individual toe cap

## Individual toe cap

- Increased wall stability due to double compression garments for pronounced forms of oedema over the dorsum of the foot
- Increased treatment compliance due to quick and easy donning
- Individual care options, e.g. nightwear or combination with other product variants
- Displacement of the seam on the bottom of the foot (medial seam at big toe as special request available)
- UV protection factor 40

### Styles

Closed toe cap



Closed toe cap without small toe



Open toe cap



Open toe cap without small toe





Decongestion  
Phase

Transition  
Phase

Maintenance  
Phase



3

Phase



# mediven® 550 SL toe cap

**Strong. High wall stability. Seamless.**

## Indications<sup>1</sup>

- Patients with lymphoedema in the toes (Stemmer positive) and on the back of the foot

## Features

- Seamless made-to-measure designed for maximum wearing comfort
- Very comfortable and easy to put on
- Each single toe made-to-measure for optimal fit
- The integrated Clima Fresh function has an Odour-inhibiting material
- Individual care options, e.g. nightwear or combination with other product variants or circaid products

## Styles

Closed toe cap



Closed toe cap without small toe



Open toe cap



Open toe cap without small toe



mediven 550 SL single toe cap



Multi-part garment: mediven 550 SL with circaid juxtafit

Compression Class

CCL 1, 2, 3

Product benefits



Clima Comfort



Clima Fresh



Perfect Fit



Variety



STANDARD 100  
97.0.3231  
Hohenstein HTTI



Colours

Standard colours

Sand

Cashmere

Caramel

Black

Anthracite

Navy

Grey\*

medi Magenta\*

Trend colours

Delivery period up to 10 working days

Russet-red

Light-blue

Lilac

Sage-green

\* Standard colours with a delivery period up to 10 working days

<sup>1</sup> Please note that this recommendation does not replace the diagnosis or the advice from a medical professional.



3

Phase





# mediven<sup>®</sup> cosy

Flexible. Supple. Freedom of movement.

## medi tip1:

Because of the material, the fit and the product features, we recommend mediven cosy for:

- Soft to firm connective tissue
- Moderate deepened skin wrinkles / calibre jumps
- Associated weight disorders
- Lipoedema patients
- Travelling / sports

## Indications<sup>2</sup>

- Primary and secondary lymphoedema stages 1 and 2, also obesity associated lymphoedema
- Lipoedema
- Scar treatment
- Post-operative oedema
- Post-traumatic oedema
- Mixed forms: e.g. phlebo-lymphoedema

## Product features


- Medium wall stability
- High longitudinal elongation
- Flexible, supple fabric

## Special features

- UV protection factor 80
- Particularly flexible, soft knitted fabric
- High longitudinal elongation for easy donning
- Special anatomical foot shape
- Special durable sole and heel



Wind  
in gold

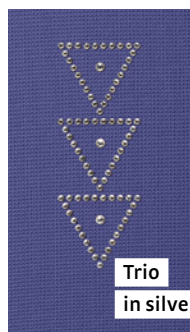

Trio  
in silver



Proud  
in anthracite

Available with sparkling crystals

## Patterns for Design-Elements<sup>3</sup> and Fashion-Elements<sup>4</sup>



Stripes



Bloom



Nature



Wild

<sup>1</sup> The choice of the optimum compression stocking depends on the indication and the individual needs of the patient.

<sup>2</sup> Please note that this recommendation does not replace the diagnosis or the advice from a medical professional.

<sup>3</sup> The Design-Elements can be freely combined with the standard and trend colours (no extra charge).

<sup>4</sup> Please note: Fashion-Elements are at extra costs

## Compression Class

CCL 1, 2, 3

## Product benefits



Clima Comfort



Clima Fresh



Perfect Fit



Variety



STANDARD  
100  
97.0.3231  
Hohenstein HTTI



## Colours

### Standard colours



Sand



Cashmere



Caramel



Black




Anthracite



Navy



Grey\*

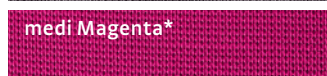


medi Magenta\*

### Trend colours

Delivery period up to 10 working days



Russet-red

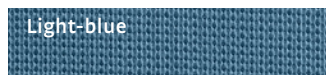

Light-blue



Lilac



Sage-green

\* Standard colours with a delivery period up to 10 working days

Decongestion  
Phase

Transition  
Phase

Maintenance  
Phase





Combination of below knee stocking (AD), capri panty (CG) and cycling shorts (ET)



Combination of below knee stocking (AD) and CT leg section



One-legged pantyhose (ATE)

## mediven® cosy made-to-measure styles

Styles – made-to-measure	AD	AF	AG	AT /	ATH /	ATU
Styles – made-to-measure	ATE / ATEH	BT / BTH	B1T / B1TH CT / CTH	ET / ETH FT / FTH	AB	
Styles – made-to-measure	AB1	BD	BF	BG	CF	
Styles – made-to-measure	CG	DG				

### Standard models

Border	Flat oblique with all stocking variants
Styles	Flat oblique at the thigh and perforated topband with all pantyhose variants
Gusset	Men's gusset: ATH, ATEH, FTH, ETH, CTH, BTH non-compressive front panty top: ATU, ATEU, FTU, ETU, CTU, BTU
Foot	Oblique, open foot
Seam	back seam, lateral seam

### Material composition

	Polyamide (PA)	Elastane (EL)
CCL 1	85 %	15 %
CCL 2	80 %	20 %
CCL 3	72 %	28 %



Decongestion  
Phase

Transition  
Phase

Maintenance  
Phase



3

Phase





# mediven<sup>®</sup> cosy toe cap

**Flexible. Supple. Freedom of movement.**


## Indications<sup>1</sup>

- Patients with lymphoedema in the toes (Stemmer positive) and on the dorsum of the foot

## Product features

- Toe cap with Clima Fresh: Odour-inhibiting material
- Top and bottom principle:
  - Seams moved to the upper and lower surfaces of the toes (no seams between the toes)
  - Each single toe made to measure
- medi tip: Optional integration of medi lymphpads for additional pressure in lymphoedema of the dorsum of the foot

## Patterns for Design-Elements<sup>3</sup> and Fashion-Elements<sup>4</sup>

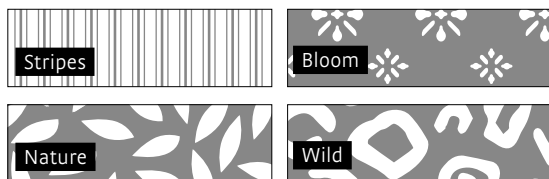

## Compression Class

**CCL 1, 2**

## Product benefits



**Clima Comfort**



**Clima Fresh**



**Perfect Fit**



**Variety**



**STANDARD 100**  
97.0.3231  
Hohenstein HTTI




## Colours

### Standard colours



### Trend colours

Delivery period up to 10 working days

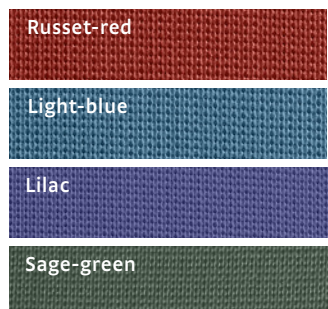

\* Standard colours with a delivery period up to 10 working days

<sup>1</sup> Please note that this recommendation does not replace the diagnosis or the advice from a medical professional.

<sup>2</sup> The Design-Elements can be freely combined with the standard and trend colours (no extra charge).

<sup>3</sup> Please note: Fashion-Elements are at extra costs



Fully-knitted toe cap

## Fully-knitted toe cap

- More reliable measurements (no longer necessary to calculate the overlap)
- Care to suit the indication for optimum lymphatic drainage (no circular seams)
- Very comfortable to wear, without circular seams
- Outstanding non-slip grip
- Available in variants AB, AB1, AD, AF, AG, AT, ATH, ATU, ATE/ATEH
- Different compression classes in toe cap and leg garment available



Individual toe cap

## Individual toe cap

- Increased wall stability due to double compression garments for pronounced forms of oedema over the dorsum of the foot
- Individual care options, e.g. nightwear or combination with other product variants
- Displacement of the seam on the bottom of the foot (medial seam at big toe as special request available)
- medi tip: The individual toe cap makes it easy and quick to put on and take off.

### Styles





Decongestion  
Phase

Transition  
Phase

Maintenance  
Phase



3

Phase





# mediven mondi®

Soft. Fine. Looks great.

## medi tip<sup>1</sup>:

**Because of the material, the fit and the product features, we recommend mediven mondi for:**

- Normal to firm connective tissue
- No deepened skin wrinkles / calibre jumps
- Normal weight (normal leg shape)
- Atopic dermatitis (mondi with silver segments)
- CVI with severe oedema tendency

## Indications<sup>2</sup>


- Primary and secondary lymphoedema stages 1 and 2, also obesity associated lymphoedema
- Lipoedema
- Post-operative oedema
- Post-traumatic oedema
- Scar treatment
- Severe chronic venous insufficiency
- Prevention of recurrent venous ulcer

## Product features


- Light wall stability
- High lateral elongation
- Soft, fine fabric

## Special features


- Available with silver segments:
  - Y-C, Y-D, Y-G, A-C, A-D, A-G
  - Odour-inhibiting effect for patients with e. g. atopic dermatitis
- UV protection factor 80
- Many combination options with special extras such as Y-knitting marks
- Fashionable variety: 8 standard colours and 4 trend colours

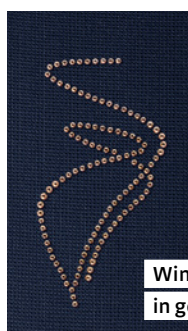

Wind  
in gold

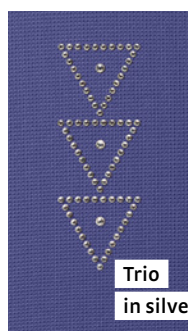

Trio  
in silver

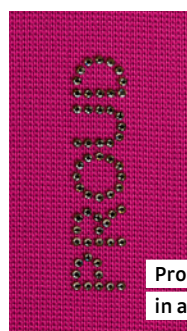

Proud  
in anthracite

Available with sparkling crystals

Compression Class

CCL 1, 2, 3

Product benefits



Clima Comfort



Clima Fresh



Perfect Fit



Variety




STANDARD  
100  
97.0.3231  
Hohenstein HTTI



Colours


Standard colours

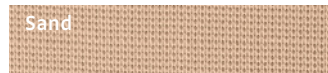

Sand



Cashmere

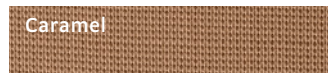

Caramel



Black



Anthracite



Navy



Grey\*



medi Magenta\*

Trend colours

Delivery period up to 10 working days



Russet-red



Light-blue



Lilac



Sage-green

\* Standard colours with a delivery period up to 10 working days

<sup>1</sup> The choice of the optimum compression stocking depends on the indication and the individual needs of the patient.

<sup>2</sup> Please note that this recommendation does not replace the diagnosis or the advice from a medical professional.



Below knee stocking (AD)  
with open toe



Thigh-length stocking (AG)  
with open toe



Maternity panty (ATU)

## mediven mondi® made-to-measure styles

Styles – made-to- measure	AD	AF	AG	AT /	ATH /	ATU
Styles – made-to- measure	ATE / ATEH	BT / BTH	B1T / B1TH CT / CTH	ET / ETH FT / FTH	AB	
Styles – made-to- measure	AB1	BD	BF	BG	CF	
Styles – made-to- measure	CG	DG				

### Standard models

Border	Flat oblique with all stocking variants
Style	Flat oblique at the thigh and perforated topband with all pantyhose variants
Gusset	Men's gusset: ATH, ATEH, FTH, ETH, CTH, BTH non-compressive front panty top: ATU, ATEU, FTU, ETU, CTU, BTU
Foot part	Oblique, open foot

### Material composition





	Polyamide (PA)	Elastane (EL)
CCL 1	78 %	22 %
CCL 2	76 %	24 %
CCL 3	72 %	28 %





Below knee stocking (AD)  
with open toe

## mediven mondi® standard styles (ready-to-wear)

Styles – Standard	AD	AD pte.	AG / PrB	AG pte. / PrB
				
	calf	calf petite	thigh with topband	thigh petite with topband
Article No.	C2407*	C2477*	C26P7*	C25P7*
	*add size number 01 – 07			

### Standard models

CCL	2
Length	2 lengths for AD / calf and AG / thigh
Foot	Open toe straight ending, closed toe
Topband	Tricot border, silicone dot topband, narrow silicone dot topband
Standard sizes	I – VII
Colour	Caramel, Black (AD)

Circumference in cm (Tension dimension)	I	II	III	IV	V	VI	VII
cG	44 – 47	46 – 50	50 – 54	53 – 58	56 – 61	60 – 65	64 – 70
cF	40 – 47	42 – 51	45 – 55	48 – 58	51 – 62	54 – 67	58 – 72
cE	30 – 37	33 – 40	36 – 42	38 – 44	40 – 46	42 – 48	44 – 50
cD	28 – 33	30 – 35	32 – 37	34 – 39	36 – 41	38 – 43	40 – 45
cC	29 – 34	31 – 36	33 – 38	35 – 40	37 – 42	39 – 44	41 – 46
cB1	24 – 26	26 – 28	28 – 31	30 – 33	32 – 35	34 – 37	36 – 39
cB	18 – 20	20 – 22	22 – 24	24 – 26	26 – 28	28 – 30	30 – 32
cY	28 – 30	29 – 32	31 – 34	33 – 36	34 – 38	36 – 40	37 – 41
cA	18 – 21	20 – 23	22 – 25	24 – 27	26 – 29	28 – 31	30 – 33


### Length measurements in cm

AD standard 38 – 43

AG standard 72 – 83

AD short 33 – 37

AG short 62 – 71

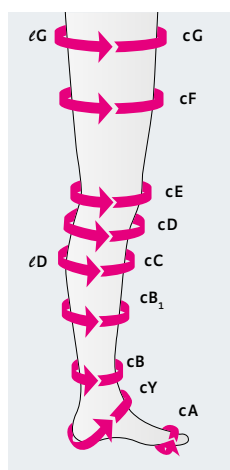

l = length measurements  
c = circumferences

Decongestion  
Phase

Transition  
Phase

Maintenance  
Phase

3

Phase





# mediven mondi® toe cap

Soft. Fine. Looks great.

## Indications<sup>1</sup>

- Patients with lymphoedema in the toes (Stemmer positive) and on the dorsum of the foot

## Product features

- Toe cap with Clima Fresh: reliably prevents odour and colonisation by bacteria
- Top and bottom principle:
  - lower surfaces of the toes
  - (no seams between the toes)
  - Each single toe made to measure



Fully-knitted individual toe cap

## Fully-knitted individual toe cap

- More reliable measurements (no longer necessary to calculate the overlap)
- Care to suit the indication for optimum lymphatic drainage (no circular seams)
- Very comfortable to wear, without circular seams
- Outstanding non-slip grip
- Available in variants AB, AB1, AD, AF, AG, AT, ATH, ATU, ETE/ATEH
- Different compression classes in toe cap and leg garment available



Individual toe cap

## Individual toe cap

- Increased wall stability due to double compression garments for pronounced forms of oedema over the dorsum of the foot
- Increased treatment compliance due to quick and easy donning
- Individual care options, e.g. nightwear or combination with other product variants
- Displacement of the seam on the bottom of the foot (lateral seam as special request available)
- medi tip: The individual toe cap makes it easy and quick to put on and take off.

## Styles



## Compression Class

CCL 1, 2

## Product benefits



Clima Comfort



Clima Fresh



Perfect Fit



Variety



STANDARD  
100  
97.0.3231  
Hohenstein HTTI



## Colours

### Standard colours

Sand

Cashmere

Caramel

Black

Anthracite

Navy

Grey\*

medi Magenta\*

### Trend colours

Delivery period up to 10 working days

Russet-red

Light-blue

Lilac

Sage-green

\* Standard colours with a delivery period up to 10 working days

<sup>1</sup>Please note that this recommendation does not replace the diagnosis or the advice from a medical professional.



3

Phase



# circaid® juxtafit® premium and essentials leg

Adjustable compression products for maintaining long term therapy results. For patients who have challenges with the established medical compression stockings.

## medi tip\*:

**Due to product features we recommend circaid juxtafit for:**

- Men / women
- Optimising and maintaining decongestion results
- Patients who have challenges with medical compression stockings
- Up to 24 hour compression therapy
- Mobile / immobile patients

## Indications

- Lymphoedema
- Other forms of oedema
- Lipoedema

## Product features

- Easy handling – simple donning and doffing
- circaid Built-In-Pressure System (BPS): adjustable, controllable and re-adjustable compression
- Re-adjusting the individual bands avoids pressure loss and thus sliding down of the garment
- Breathe-O-Prene: breathable and inelastic material with high wall-stability for high working pressure
- Odour-inhibiting material

## Benefits


- Offering an alternative therapy solution for patients who are not able to apply or tolerate compression stockings
- Compression therapy – essential part of the complex lymphoedema therapy<sup>1</sup>
- Increased patient-adherence: 94.6 % of the users experience the wearing comfort as pleasant<sup>2</sup>
- Increased mobility: possibility of wearing regular footwear<sup>2</sup>
- Easy handling allows for self-management of the patient

For further product information and details please refer to pages 38 – 47 (see Phase 2, Transition Phase)

## Adjustable compression

20 – 30 mmHg  
30 – 40 mmHg  
40 – 50 mmHg  
50+ mmHg

## Product benefits

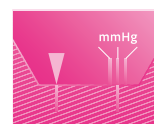

Built-In-Pressure®



Variety



Juxtaposed band



Easy handling



Inelastic



Breathe-O-Prene



## Colours

S  
Beige

S = Standard

## Note

A video for application of the circaid juxtafit can be found here: [medi.biz/jf](https://medi.biz/jf)

\* Please consider, that this recommendation does not replace the diagnosis and consultation of a medical healthcare professional.

## Studies

<sup>1</sup> Lymphoedema Framework. Best Practice for the Management of Lymphoedema. International consensus. London: MEP Ltd, 2006.

<sup>2</sup> Protz K et al. Comparison of handling, pressure and wearing comfort of different compression devices for decongestion therapy. Veins and Lymphatics 2018;7:7989:137-138.

Decongestion  
Phase


Transition  
Phase

Maintenance  
Phase

# medi Therapy Concept Lymphology – Upper Extremities

Optimum treatment with medi during the entire therapy.

---

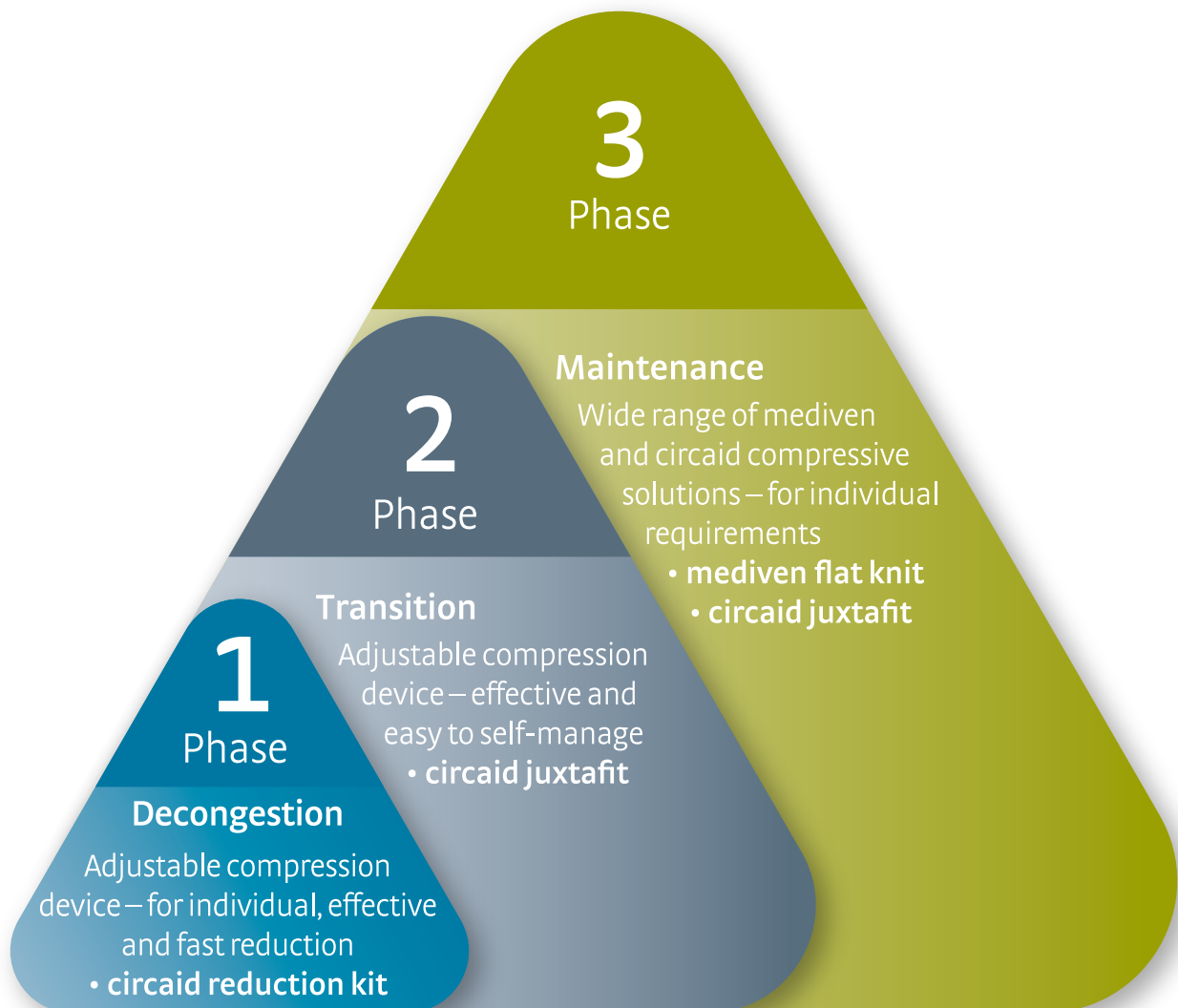



## 1 Decongestion Phase



Adjustable and patient-individual compression for initial oedema reduction.

- **circaid® reduction kit**

## 2 Transition Phase



Effective compression to optimise reduction results.

- **circaid® juxtafit®**

## 3 Maintenance Phase



A huge variety of compression solutions to maintain therapy results in the long term.


- **mediven® flat knit**
- **circaid® juxtafit®**





# Decongestion Phase

Reducing the limb volume during the decongestion phase.  
A solution that drives decongestion and enables patient participation from day one.

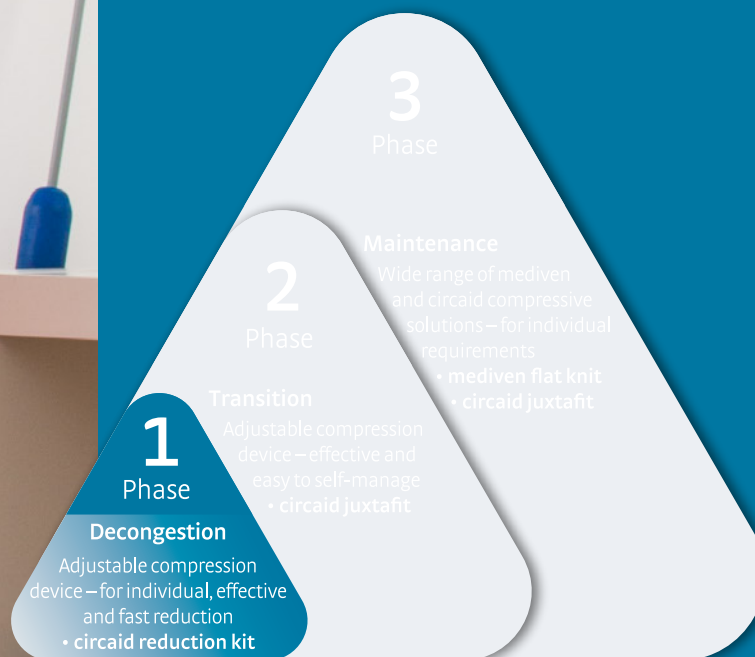

Decongestion Phase

Transition Phase

Maintenance Phase



**1**  
Phase



# circaid<sup>®</sup> reduction kit arm

Customisable, adjustable compression device for the decongestion phase of arm lymphoedema.

## medi tip\*:

Due to product features we recommend circaid reduction kit for:

- Men / women
- Initial decongestion of arm lymphoedema
- Up to 24 hour compression therapy

## Indications

- Lymphoedema
- Dependent oedema

## Product features

- Easy and time saving handling – simple donning and doffing
- Adjustable sizes – garment is initially fitted to the patient's individual circumferences and limb shape
- Ability to reduce size of garment as the limb volume decreases
- Built-In-Tension System (BTS) to set, control and re-adjust the correct therapeutic tension level – enables consistent compression and effective decongestion
- Re-adjusting the individual bands avoids pressure loss and thus sliding down of the garment – supporting oedema reduction effectively
- Breathe-O-Prene: breathable and inelastic material with high wall-stability for high working pressure
- Odour-inhibiting material
- Can also be worn at night if required

## Benefits

- Easy handling – time-saving and less susceptible to error
- Easy handling – can be applied using one hand
- Compression therapy – essential part of the complex lymphoedema therapy<sup>1</sup>
- Faster reduction of lymphoedema<sup>2</sup>
- Promotes selfmanagement thanks to easy handling



circaid reduction kit arm including hand wrap customizable



Built-In-Tension system card (BTS)



Fitting of the arm component

## Adjustable compression

## Tension 1 – 8

### Product benefits



Juxtaposed band




Variety



Easy handling



Inelastic

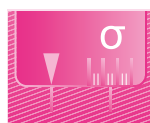

Built-In-Tension



Breathe-O-Prene



### Colours

Beige

### Size

Trimable (customisable)

### Note

A video for application and customisation of the circaid reduction kit arm can be found here:

[medi.biz/rk](https://www.medi.biz/rk)

\* Please consider, that this recommendation does not replace the diagnosis and consultation of a medical healthcare professional.

### Studies

<sup>1</sup> Lymphoedema Framework. Best Practice for the Management of Lymphoedema. International consensus. London: MEP Ltd, 2006.

<sup>2</sup> Damstra R et al. Prospective randomized, controlled trial comparing the effectiveness of adjustable compression Velcro wraps versus inelastic multicomponent compression bandages in the initial treatment of leg lymphedema. Journal of Vascular Surgery: Venous and Lymphatic Disorders 2013;1(1): 13-9.

Decongestion Phase

Transition Phase

Maintenance Phase





circaid reduction kit arm

## circaid® reduction kit arm

Art. No.	Description
JV11001000	circaid reduction kit arm standard
JV11011000	circaid reduction kit arm standard wide
JV12001000	circaid reduction kit arm long
JV12011000	circaid reduction kit arm long wide

Circumference in cm	Standard	Wide
cF	< 50	50 – 65
cB*	< 30	< 30
Length measurement in cm	Standard	Long
∠C – G	< 45	> 45

\* If cB is > 30 cm hand wrap will not fit, though circaid reduction kit arm component may still be used.



### Your order for circaid reduction kit arm includes:

- 1 adjustable compression device
- 1 Built-In-Tension System (BTS) guide card
- 1 circaid customizable hand wrap
- 1 pair of circaid undersleeve arm



# circaid® hand options & accessories

The following hand options are recommended to wear in combination with circaid reduction kit.

## Note

It is recommended to always wear a circaid hand option in combination with a circaid arm garment. The indications of the circaid foot options are identical to the indications of the corresponding circaid leg garment, ordered separately.

## Tip

Video for application and customisation of the circaid hand options can be found here: [medi.biz/rk](https://medi.biz/rk)



circaid customizable interlocking hand wrap

## circaid® customizable hand wrap

Customisable hand option for oedema decongestion.

### Product details

- Provides additional compression to the hand from the wrist to the base of the fingers
- Customisable to patient- individual circumferences and shape
- Adjustable to decreasing circumferences during decongestion
- Easy handling
- Bilateral use (left and/or right hand)

Art. No.	Description	Circumference in cm
JULV409000	circaid customizable hand wrap	cC < 30
		cB < 30



circaid glove

## circaid® glove

Additional compression for the fingers.

### Product details

- Provides light to moderate compression
- Individually adjustable length of the fingers
- Intuitive and simple donning
- Designed for left or right hand use
- It is recommended to always wear the circaid glove with a circaid arm garment



Circumference in cm	2	3	4	5	6
cB	18 – 21	20 – 23	22 – 25	24 – 27	26 – 29
Art. No.	JU5G002000	JU5G003000	JU5G004000	JU5G005000	JU5G006000



circaid reduction kit undersleeve arm

## circaid® reduction kit undersleeve arm

Additional wearing comfort.

- Can be worn directly against the skin underneath circaid products
- Extra comfort
- Note: does not provide therapeutic compression

Art. No.	Description
JULV909000	circaid reduction kit undersleeve arm wide

Decongestion Phase

Transition Phase

Maintenance Phase








# Transition Phase

Following the intensive therapy of reduction, some patients may benefit from a period of transition management before progressing into long-term therapy.

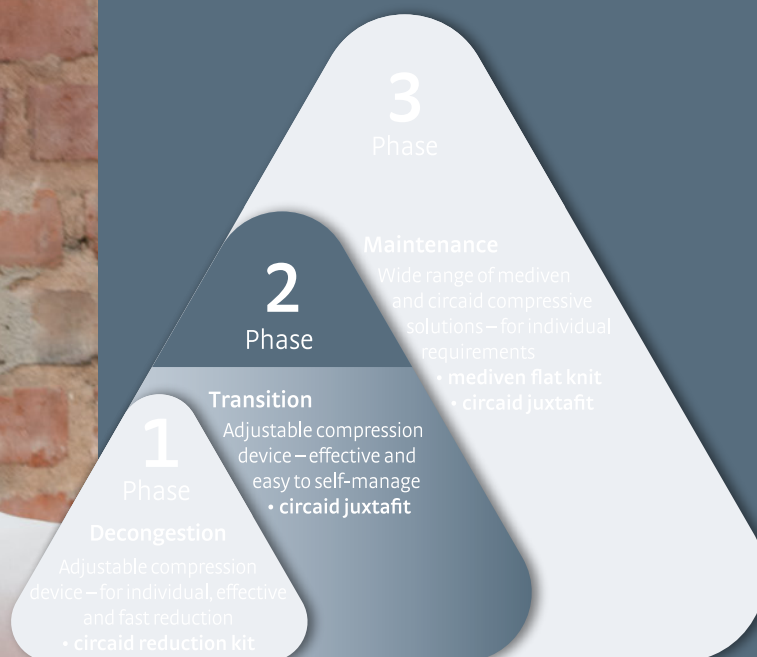

Decongestion  
Phase

Transition  
Phase

Maintenance  
Phase



2

Phase



# circaid® juxtafit® essentials arm

Adjustable compression device for optimising and maintaining decongestion results in lymphoedema treatment.

## medi tip\*:

**Due to product features we recommend circaid juxtafit for:**

- Men / women
- Optimising and maintaining decongestion results
- Patients who have challenges with medical compression armsleeves
- Up to 24 hour compression therapy

## Indications

- Lymphoedema
- Lipoedema
- Dependent oedema

## Product features

- Easy handling – can be applied using one hand
- Easy handling – simple donning and doffing
- circaid Built-In-Pressure System (BPS): adjustable, controllable and re-adjustable compression
- Re-adjusting the individual bands avoids pressure loss and thus sliding down of the garment
- Inelastic material with high wall-stability for high working pressure
- Odour-inhibiting material

## Benefits

- Offering an alternative therapy solution for patients who are not able to apply or tolerate compression armsleeves
- Compression therapy – essential part of the complex lymphoedema therapy<sup>1</sup>
- Easy handling allows for self-management of the patient
- Optimum addition to medical compression stockings – attractive armsleeves during the day and night compression with circaid garments
- Compression solution for optimising decongestion results and stabilising oedema fluctuations


## Adjustable compression

10 – 20 mmHg

20 – 30 mmHg

30 – 40 mmHg

## Product benefits

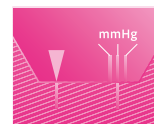

Built-In-Pressure®



Variety



Juxtaposed band



Easy handling



Inelastic



Breathe-O-Prene



STANDARD  
100  
16.0.87402  
Hohenstein MTI  
www.oeko-tex.com

## Colours

S  
Beige

S = Standard

## Note

A video for application of the circaid juxtafit essentials arm can be found here: [medi.biz/jf](https://medi.biz/jf)

\* Please consider, that this recommendation does not replace the diagnosis and consultation of a medical healthcare professional.

## Studies


<sup>1</sup> Lymphoedema Framework. Best Practice for the Management of Lymphoedema. International consensus. London: MEP Ltd, 2006.

Decongestion  
Phase

Transition  
Phase

Maintenance  
Phase



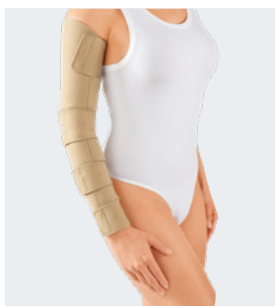


circaid juxtafit essentials arm

## circaid® juxtafit® essentials arm standard styles

Styles – Standard		CG	CG	CG
Size / Art. No.		Extra Short	Short	Long
<b>Left</b>	1	JU2A781000	JU2A731000	JU2A031000
	2	JU2A782000	JU2A732000	JU2A032000
	3	JU2A783000	JU2A733000	JU2A033000
	4	JU2A784000	JU2A734000	JU2A034000
	5	JU2A785000	JU2A735000	JU2A035000
	6	JU2A786000	JU2A736000	JU2A036000
<b>Right</b>	1	JU2A791000	JU2A741000	JU2A041000
	2	JU2A792000	JU2A742000	JU2A042000
	3	JU2A793000	JU2A743000	JU2A043000
	4	JU2A794000	JU2A744000	JU2A044000
	5	JU2A795000	JU2A745000	JU2A045000
	6	JU2A796000	JU2A746000	JU2A046000

Circumferences in cm	1	2	3	4	5	6
cG	28 – 32	30 – 34	32 – 37	35 – 40	38 – 44	42 – 48
cF	26 – 30	28 – 32	30 – 35	33 – 38	36 – 41	39 – 45
cE	25 – 28	27 – 30	29 – 32	31 – 34	33 – 36	36 – 40
cD	23 – 26	25 – 28	27 – 30	29 – 32	31 – 34	33 – 37
cC	15 – 17	17 – 19	19 – 21	21 – 23	23 – 25	25 – 28

Length measurement in cm	Extra Short	Short	Long
ℓ C – G	< 38	38 – 48	> 48

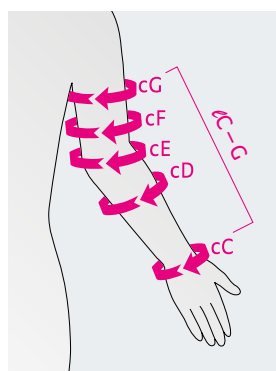

### Your order for circaid juxtafit essentials arm includes:

- 1 adjustable compression garment
- 1 circaid Built-In-Pressure system (BPS) guide card
- 1 elbow band
- 1 circaid cover up arm

# circaid<sup>®</sup> hand options

The following hand options are recommended to wear in combination with circaid juxtafit.

**Note**

It is recommended to always wear a circaid hand option in combination with a circaid arm garment. The indications of the circaid hand options are identical to the indications of the corresponding circaid arm garment.




circaid juxtafit essentials hand wrap

## circaid<sup>®</sup> juxtafit<sup>®</sup> essentials hand wrap

**Adjustable compression device for the hand.**

**Product details**

- Compresses the hand from the wrist to the base of the fingers
- Adjustable for additional pressure on the dorsum of the hand
- Simple handling – ideal for self-care
- Available in standard sizes only – for the left or right hand

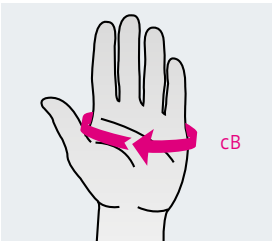

Size	Circumferences in cm / cB Palm	Art. No. / Left	Art. No. / Right
XS	14 – 17	JU2B031000	JU2B041000
S	17 – 20	JU2B032000	JU2B042000
M	20 – 23	JU2B033000	JU2B043000
L	23 – 26	JU2B034000	JU2B044000
XL	26 – 29	JU2B035000	JU2B045000




## circaid<sup>®</sup> glove

**Additional compression for the fingers.**

**Product details**


- Provides light to moderate compression
- Individually adjustable length of the fingers
- Intuitive and simple donning
- Designed for left or right hand use
- It is recommended to always wear the circaid glove with a circaid arm garment

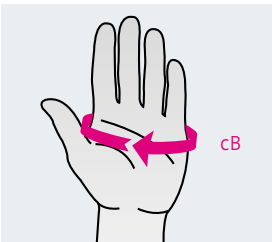

Circumference in cm	2	3	4	5	6
cB	18 – 21	20 – 23	22 – 25	24 – 27	26 – 29
Art. No.	JU5G002000	JU5G003000	JU5G004000	JU5G005000	JU5G006000



# circaid® juxtafit® accessories

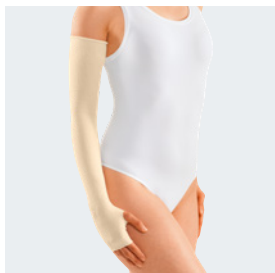

circaid undersleeve arm  
with thumb opening

## circaid® undersleeves

### Additional wearing comfort.

- Can be worn directly against the skin underneath circaid products to provide added comfort
- Note: does not provide therapeutic compression

Standard size	Art. No.	Style
Undersleeve arm	JULV009000	AG



circaid juxtafit premium  
shelf strap

## circaid® juxtafit® premium shelf strap

### Additional support and coverage.

Offers additional support and coverage

Art. No.	Size	Size in cm
JU2Q001000	S	20.5 x 6.5 (without hook tabs)
JU2Q002000	L	25.5 x 9.0 (without hook tabs)
JU2Q000000	Made-to-measure	—

#### Note

It is recommended to always use a circaid supporting band in combination with a circaid arm garment. The indications of the circaid supporting band are identical to the indications of the corresponding circaid arm garment. Please order separately.



Decongestion  
Phase

Transition  
Phase

Maintenance  
Phase










# Maintenance Phase

The long-term management of lymphoedema focuses on enhancing the function of the lymphatics, limiting further deterioration of swelling and gaining long-term control of the condition. Success relies on compression garments.

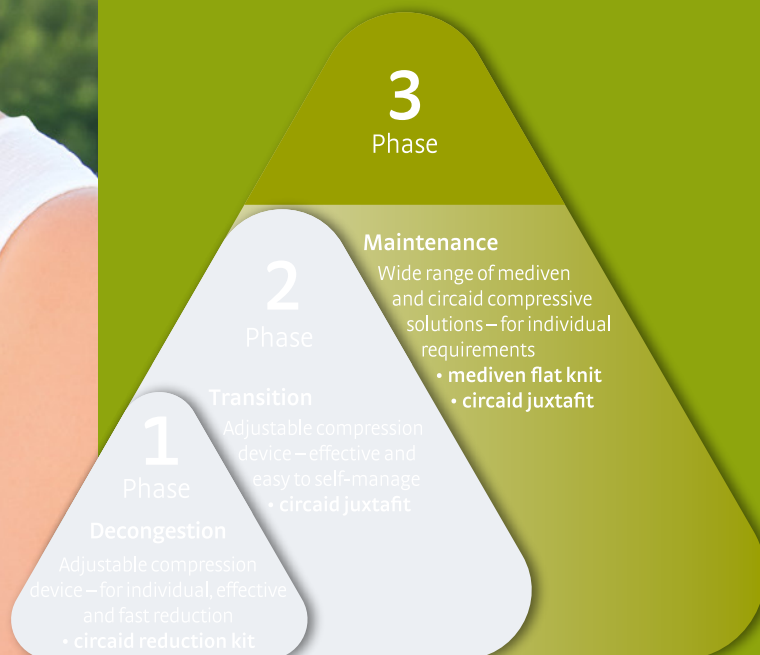

Decongestion Phase

Transition Phase



Maintenance Phase

## Care compass arm: perfectly supplied with mediven® flat knit

Each mediven fabric has its own unique features, therefore please refer to the information below to find the most suitable mediven fabric for your patient.\*

### Lipoedema

(Pain as the leading symptom –  
no staging applies)

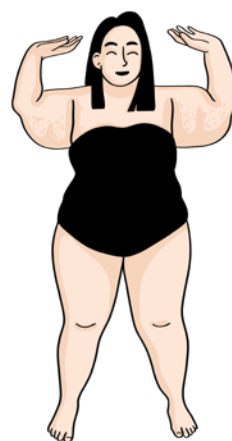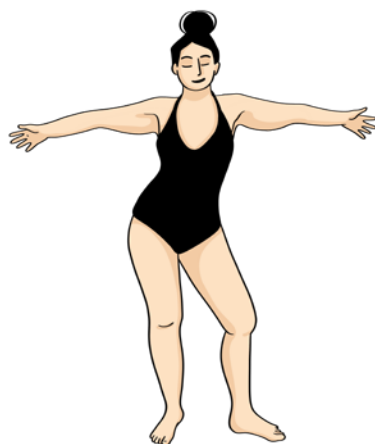

### Lymphoedema

(Stage 1 – 3  
in the maintenance phase)



Connective tissue

normal to firm

very soft to hardened

Weight



Deepened skin folds /  
calibre jumps

none / moderate

moderate / strong

Recommended  
mediven compression  
fabric

mediven esprit®<sup>1</sup>

mediven® 550 arm

<sup>1</sup> medi recommendation: can be used for Lymphoedema up to stage 2

The softer / firmer the connective tissue – the higher the weight – the stronger the skin folds /  
caliber jumps...

... the stronger the stocking material should be.

→ The most prominent condition indicates fabric choice.



\* Please note that this recommendation does not replace the diagnosis of a doctor or the advice of a medical specialist. The choice of medically necessary and customised compression fabric depends on the indication and the patient's individual needs.





Decongestion  
Phase

Transition  
Phase

Maintenance  
Phase



3

Phase





# mediven<sup>®</sup> 550 arm

**Strong. Very high wall stability. Reliable hold.**

## medi tip<sup>1</sup>:

**Because of the material, the fit and the product features, we recommend mediven 550 arm for:**

- Very soft to hardened connective tissue
- Strong deepened skin wrinkles /calibre jumps
- Associated weight disorders
- Pronounced lymphoedema

## Suitable for

soft to firm connective tissue

## Indications<sup>2</sup>


- Primary and secondary lymphoedema stages 1, 2 and 3
- Lipoedema
- Post-operative oedema
- Post-traumatic oedema
- Scar treatment

## Product features

- Very high wall stability
- High working pressure
- Firm, strong fabric
- High durability


## Special features

- UV protection factor 20
- Many combination options with special extras
- Numerous variants, such as arm cuffs for double and multipart garments
- Fashionable variety: 8 standard colours, 4 trend colours and 4 patterns for Design-Elements and Fashion-Elements

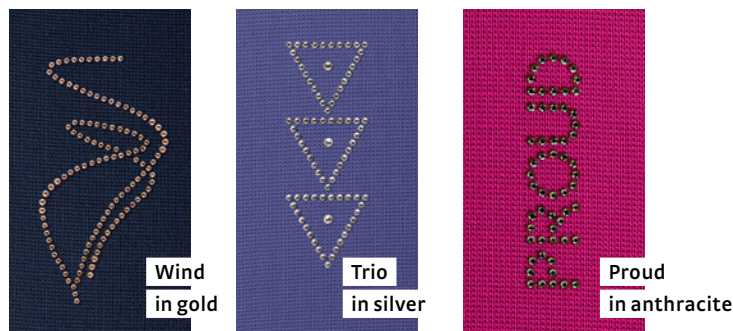

Available with sparkling crystals

## Patterns for Design-Elements<sup>3</sup> and Fashion-Elements<sup>4</sup>

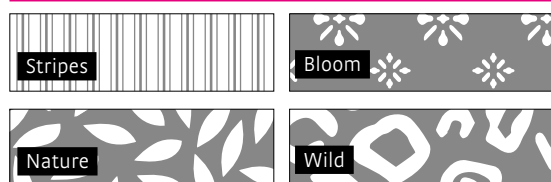

<sup>1</sup> The choice of the optimum compression stocking depends on the indication and the individual needs of the patient.

<sup>2</sup> Please note that this recommendation does not replace the diagnosis or the advice from a medical professional.

<sup>3</sup> The Design-Elements can be freely combined with the standard and trend colours (no extra charge).

<sup>4</sup> Please note: Fashion-Elements are at extra costs


## Compression Class

CCL 1, 2, 3

## Product benefits



Perfect Fit

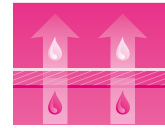

Clima Comfort



Variety



Clima Fresh





STANDARD  
100  
97.0.3231  
Hohenstein HTTI



## Colours

### Standard colours

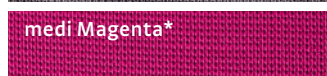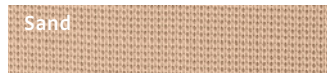

### Trend colours

Delivery period up to 10 working days




\* Standard colours with a delivery period up to 10 working days

Decongestion  
Phase

Transition  
Phase


Maintenance  
Phase

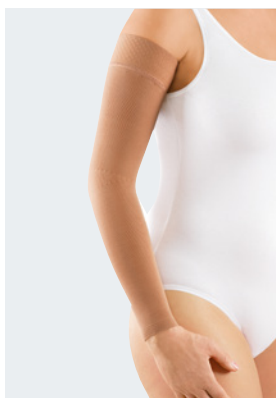


Armsleeve CG



With non-slip topband at G and hand piece with long fingers and thumb base

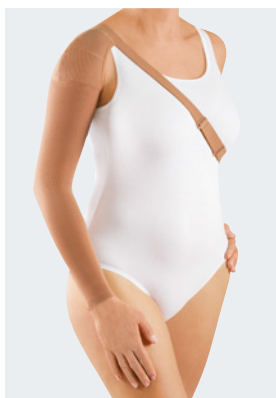

With shoulder strap

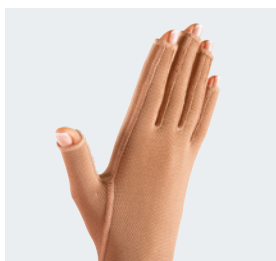

Hand piece (seam outside)

## mediven® 550 arm made-to-measure styles

Styles made-to-measure	AD	AE	AF	AG
	<p>long glove to D</p>	<p>long glove to E</p>	<p>Armsleeve with hand to F</p>	<p>Armsleeve with hand to G</p>
Styles made-to-measure	CD	CE	CF	CG
	<p>Armsleeve without hand to D</p>	<p>Armsleeve without hand to E</p>	<p>Armsleeve without hand to F</p>	<p>Armsleeve without hand to G</p>
Styles made-to-measure	DG	EG	Bolero	
	<p>Arm cuff</p>	<p>Arm cuff</p>		
Attachment options	Topband		Shoulder strap	Bra attachment
<b>One-piece (AG / AF)</b> Arm with hand knitted on	<p>At C or F</p>			
<b>Two-piece (CG + AC1)</b> Arm with separate hand piece				
Hand section AC1				
<b>Finger piece</b> 	<b>Hand piece without fingers, with thumb opening</b> 	<b>Hand piece without fingers, with base of thumb</b> 	<b>Hand piece with open fingers, with base of thumb</b> 	
<b>Hand piece with open fingers, with base of thumb</b> 	<b>Hand piece with closed fingers</b> 			
Standard models				
<b>Border proximal</b>	Flat oblique with all variants apart from AC1. For AC1 porous, straight border.			
<b>Border distal</b>	Porous straight border: CD, CE, CF, CG			
<b>Finger</b>	Without fingers with base of thumb, open / closed fingers with base of thumb: AE, AF, AG, AC1, AD			
<b>Fingerpiece</b>	Open or closed			

### Material composition

	Polyamide (PA)	Elastane (EL)
CCL 1	88 %	12 %
CCL 2	84 %	16 %
CCL 3	74 %	26 %





Decongestion  
Phase

Transition  
Phase

Maintenance  
Phase



3

Phase





# mediven esprit®

Soft. Fine. Looks great.

## medi tip<sup>1</sup>:

Because of the material, the fit and the product features, we recommend mediven esprit for:

- Normal to firm connective tissue
- No deepened skin wrinkles /calibre jumps
- Normal weight

## Indications<sup>2</sup>


- Primary and secondary lymphoedema stages 1 and 2
- Lipoedema
- Post-operative oedema
- Post-traumatic oedema
- Scar treatment

## Product features

- Light to medium wall stability
- Supple fabric
- High durability


## Special features

- UV protection factor 40
- Many combination options with special extras
- Numerous variants for double and multipart garments
- Odour-inhibiting effect by Clima Fresh
- Breathable fabric by Clima Comfort
- Fashionable variety: 8 standard colours, 4 trend colours and 4 patterns for Design-Elements and Fashion-Elements

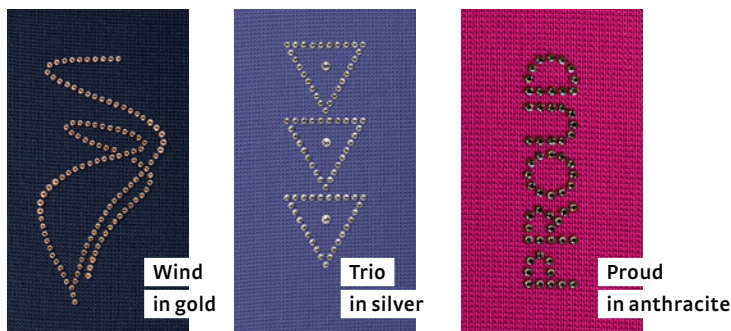

Available with sparkling crystals

## Patterns for Design-Elements<sup>3</sup> and Fashion-Elements<sup>4</sup>

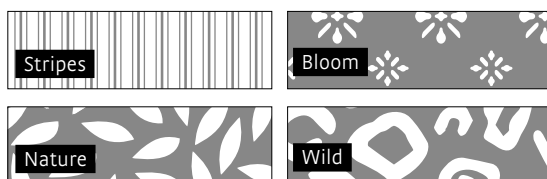

## Compression Class

CCL 1, 2, 3

## Product benefits



STANDARD  
100  
97.0.3231  
Hohenstein HTTI




## Colours

### Standard colours



### Trend colours

Delivery period up to 10 working days

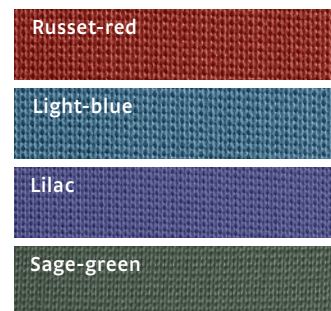


\* Standard colours with a delivery period up to 10 working days

<sup>1</sup> The choice of the optimum compression stocking depends on the indication and the individual needs of the patient.

<sup>2</sup> Please note that this recommendation does not replace the diagnosis or the advice from a medical professional.

<sup>3</sup> The Design-Elements can be freely combined with the standard and trend colours (no extra charge).

<sup>4</sup> Please note: Fashion-Elements are at extra costs

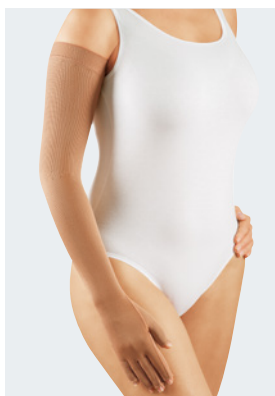

Armsleeve with hand to G (AG)



With non-slip topband at G and hand part with long fingers and thumb base

## mediven esprit® made-to-measure styles

Styles made-to-measure	AD	AE	AF	AG
	long glove to D	long glove to E	Armsleeve with hand to F	Armsleeve with hand to G
Styles made-to-measure	CD	CE	CF	CG
	Armsleeve without hand to D	Armsleeve without hand to E	Armsleeve without hand to F	Armsleeve without hand to G
Styles made-to-measure	DG	EG	Bolero	
	Arm cuff	Arm cuff		
Attachment options	Topband		Shoulder strap	Bra attachment
One-piece (AG / AF) Arm with hand knitted on	At C or F			
Two-piece (CG + AC1) Arm with separate hand piece				

Hand section AC1			
Finger piece		Hand piece without fingers, with thumb opening	
Hand piece with open fingers, with base of thumb		Hand piece without fingers, with base of thumb	
Hand piece with closed fingers		Hand piece with open fingers, with base of thumb	

Material composition		
	Polyamide (PA)	Elastane (EL)
CCL 1	88 %	12 %
CCL 2	84 %	16 %
CCL 3	74 %	26 %

Standard models	
Border proximal	Flat oblique with all variants apart from AC1. For AC1 porous, straight border.
Border distal	Porous straight border: CD, CE, CF, CG
Finger	Without fingers with base of thumb, open / closed fingers with base of thumb: AE, AF, AG, AC1, AD





Armsleeve (CG) with non-slip toband at G


## mediven esprit® standard styles (ready-to-wear)

Styles – Standard	CG / SN	CG / SN Long	AG / SN	AG / SN Long	AC1
	armsleeve with topband	armsleeve long with topband	combined with topband	combined with topband long	glove with compr. fingers
CCL1	J1H1701-5	J1G1701-5	J151701-5	J161701-5	J110711-5
CCL2	J2H1701-5	J2G1701-5	J251701-5	J261701-5	J210711-5

Circumferences in cm (Tension dimension) / Armsleeve	I	II	III	IV	V
cA	17.0 – 19.0	18.0 – 20.0	19.0 – 22.0	21.0 – 24.0	23.0 – 27.0
cC	15.0 – 17.0	17.0 – 19.0	19.0 – 21.0	21.0 – 23.0	23.0 – 25.0
cE	25.0 – 27.5	27.0 – 29.5	29.0 – 32.0	31.0 – 34.0	33.0 – 37.0
cG	28.0 – 32.0	30.0 – 34.0	32.0 – 37.0	35.0 – 40.0	38.0 – 44.0
Length CG: standard 40 – 44 cm, long 45 – 49 cm					
Colour: caramel					

Circumferences in cm (Tension dimension) / Hand part	I	II	III	IV	V
a	17.0 – 20.0	19.0 – 22.0	21.0 – 24.0	23.0 – 26.0	25.0 – 28.0
b	18.0 – 21.0	20.0 – 23.0	22.0 – 25.0	24.0 – 27.0	26.0 – 29.0
c	15.0 – 17.0	17.0 – 19.0	19.0 – 21.0	21.0 – 23.0	23.0 – 25.0
c <sub>1</sub>	15.5 – 18.0	17.5 – 20.0	19.5 – 22.0	21.5 – 24.0	23.5 – 26.0

Circumferences in cm (Tension dimension) / Fingers		I	II	III	IV	V
Thumb	top	4.5	5.5	6.5	7.5	8.5
	bottom	6.5	7.5	8.5	9.5	10.5
Index finger	top	4.5	5.5	6.5	7.5	8.5
	bottom	5.5	6.5	7.5	8.5	9.5
Middle finger	top	4.5	5.5	6.5	7.5	8.5
	bottom	5.5	6.5	7.5	8.5	9.5
Ring finger	top	4.0	5.0	6.0	7.0	8.0
	bottom	5.0	6.0	7.0	8.0	9.0
Little finger	top	3.5	4.5	5.5	6.5	7.5
	bottom	4.5	5.5	6.5	7.5	8.5

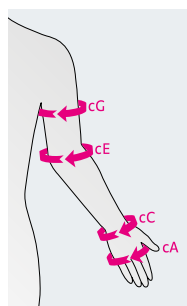

Length / Hand part (cm), all sizes	
AB	3.5 – 4.5
AC	10.0 – 12.0
AC <sub>1</sub>	17.0 – 18.0

Length / Fingers (cm)	
Thumb	3.5 – 4.5
Index finger	4.5 – 5.5
Middle finger	5.5 – 6.5
Ring finger	4.5 – 5.5
Little finger	3.5 – 4.5

Decongestion Phase

Transition Phase

Maintenance Phase





# mediven esprit® SL glove

**Soft. Fine. Seamless.**

## Indications<sup>1</sup>

- Primary and secondary lymphoedema stages 1 and 2
- Post-operative oedema
- Post-traumatic oedema

## Product features

- Light to medium wall stability
- Supple fabric
- Many special additions, including a Smart-Touch option for closed fingers
- 5 different styles for the hand
- UV protection factor 80
- Breathable fabric by Clima Comfort

### Hand section AC1

Hand piece without fingers, with thumb opening



Hand piece without fingers, with base of thumb




Hand piece with open fingers, with base of thumb




Hand piece with open fingers, with base of thumb



Hand piece with closed fingers

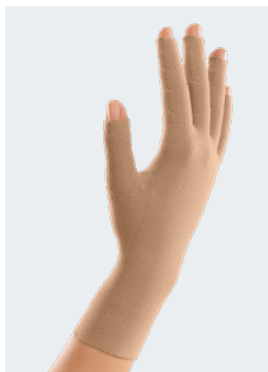

mediven esprit SL glove

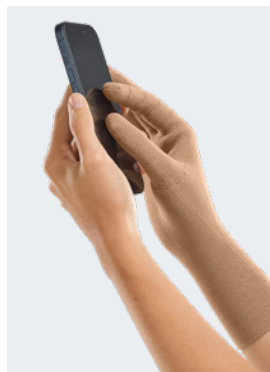

Smart-Touch for hand piece with closed fingers

## Compression Class

CCL 1, 2, 3

## Product benefits



Perfect Fit



Clima Comfort



Variety



Clima Fresh



STANDARD 100  
97.0.3231  
Hohenstein HTTI



## Colours

### Standard colours

Sand

Cashmere

Caramel

Black

Anthracite

Navy

Grey\*

medi Magenta\*

### Trend colours

Delivery period up to 10 working days

Russet-red

Light-blue

Lilac

Sage-green

\* Standard colours with a delivery period up to 10 working days

<sup>1</sup> Please note that this recommendation does not replace the diagnosis or the advice from a medical professional.



3

Phase





# mediven harmony<sup>®</sup>

Comfortable round-knitted armsleeve.

## Indications<sup>1</sup>

- Arm vein thrombosis
- Post-operative and post-traumatic oedema (medi note: not suitable for pronounced oedema!)

## Product features

- The double ellipses on the point and in the crease of the elbow offer freedom of movement. To reduce wrinkles to a minimum to prevent soreness.
- The anatomical shape of the armsleeve ensures easy and correct donning and doffing
- The silicone topband is knitted to the armsleeve with a flat and inconspicuous seam which secures hold whilst retaining air permeability.
- The skin feels comfortable all year round with Clima Comfort
- Added lanolin<sup>2</sup>

Compression Class

CCL 1, 2

## Product benefits



Perfect Fit



Clima Comfort



Variety



Double ellipses




STANDARD  
100  
96.0.1369/1  
Hohenstein HTTI



STANDARD  
100  
97.0.3231  
Hohenstein HTTI

## Colours

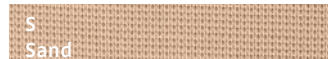

S = Standard

\* Black: minimum order quantities of 20 pieces per size and variant; approx. 4 weeks delivery period



Narrow napped silicone topband




Ellipse on the point of the elbow



Ellipse in the elbow crease

<sup>1</sup> The garment model and the strength of pressure required, i.e. the compression class, depend on the diagnosis, the treatment site, the clinical findings and the patient's susceptibility to oedema. Please also consider the patient's individual requirements and the circumstances (comorbidities, mobility, BMI, state of their skin and tissues, etc.).

<sup>2</sup> Natural substance for the treatment of dry and rough skin, used in many ways, for example in child and baby care. Nourishing protective effect remains even after washing.

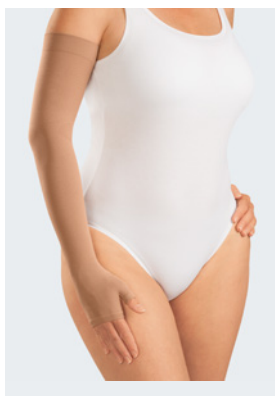

Combined armsleeve with hand part AG, sand

## mediven harmony® standard styles (ready-to-wear)

Styles – Standard*	AG combined armsleeve	AG combined armsleeve with topband	AG combined armsleeve with shouldercap
Styles – Standard*	CG	CG with topband	CG with shouldercap

Size / Art. No. Standard length	Regular	Extra Wide
<b>AG combined armsleeve</b>	CCL 1 Art. No.: 2Y10x01 – 8 CCL 2 Art. No.: 2Z10x01 – 8	CCL 1 Art. No.: 2Y10x11 – 8 CCL 2 Art. No.: 2Z10x11 – 8
<b>AG combined armsleeve with topband</b>	CCL 1 Art. No.: 2Y11x01 – 8 CCL 2 Art. No.: 2Z11x01 – 8	CCL 1 Art. No.: 2Y11x11 – 8 CCL 2 Art. No.: 2Z11x11 – 8
<b>AG combined armsleeve with shouldercap</b>	CCL 1 Art. No.: 2Y15x01 – 8 CCL 2 Art. No.: 2Z15x01 – 8	CCL 1 Art. No.: 2Y15x11 – 8 CCL 2 Art. No.: 2Z15x11 – 8
<b>CG</b>	CCL 1 Art. No.: 2Y00x01 – 8 CCL 2 Art. No.: 2Z00x01 – 8	CCL 1 Art. No.: 2Y00x11 – 8 CCL 2 Art. No.: 2Z00x11 – 8
<b>CG with topband</b>	CCL 1 Art. No.: 2Y01x01 – 8 CCL 2 Art. No.: 2Z01x01 – 8	CCL 1 Art. No.: 2Y01x11 – 8 CCL 2 Art. No.: 2Z01x11 – 8
<b>CG with shouldercap</b>	CCL 1 Art. No.: 2Y05x01 – 8 CCL 2 Art. No.: 2Z05x01 – 8	CCL 1 Art. No.: 2Y05x11 – 8 CCL 2 Art. No.: 2Z05x11 – 8

x = colour; replace with 5 for black, 7 for caramel or 8 for sand

Size / Art. No. Short length**	Regular	Extra Wide
<b>AG combined armsleeve with topband</b>	CCL 1 Art. No.: 2Y41701 – 8 CCL 2 Art. No.: 2Z41701 – 8	CCL 1 Art. No.: 2Y41711 – 8 CCL 2 Art. No.: 2Z41711 – 8
<b>CG with topband</b>	CCL 1 Art. No.: 2Y31701 – 8 CCL 2 Art. No.: 2Z31701 – 8	CCL 1 Art. No.: 2Y31711 – 8 CCL 2 Art. No.: 2Z31711 – 8

Material composition armsleeve		
	Polyamide (PA)	Elastane (EL)
<b>CCL 1</b>	66 %	34 %
<b>CCL 2</b>	57 %	43 %


Material composition hand part		
	Polyamide (PA)	Elastane (EL)
<b>CCL 1</b>	82 %	18 %
<b>CCL 2</b>	78 %	22 %

\*all variants available as regular or extra wide top  
\*\*short length only available in caramel



Circumference in cm	Size I		Size II		Size III		Size IV	
	Regular	Extra wide	Regular	Extra wide	Regular	Extra wide	Regular	Extra wide
cG	25.5 – 28.0	28.5 – 31.0	28.0 – 30.5	30.5 – 33.5	30.0 – 33.0	33.0 – 36.0	32.0 – 35.0	35.0 – 38.5
cF	23.5 – 25.5	25.5 – 28.0	25.0 – 27.5	27.5 – 30.0	27.5 – 30.0	30.0 – 33.0	29.5 – 32.0	32.0 – 35.0
cE	23.0 – 25.0	24.0 – 26.0	24.5 – 26.5	26.5 – 29.0	26.0 – 28.5	28.5 – 31.0	27.5 – 30.0	30.0 – 33.0
cD	22.0 – 24.0	24.0 – 26.0	23.0 – 25.0	25.0 – 27.5	24.0 – 26.0	26.0 – 28.5	25.0 – 27.5	28.0 – 30.5
cC1*	15.0 – 16.5		16.5 – 18.0		18.0 – 19.5		19.5 – 21.0	
cC	13.5 – 14.5		14.5 – 15.5		16.0 – 17.5		17.5 – 19.0	
cA	15.0 – 18.0		16.0 – 19.0		17.0 – 20.0		19.0 – 22.0	
Circumference in cm	Size V		Size VI		Size VII		Size VIII**	
	Regular	Extra wide	Regular	Extra wide	Regular	Extra wide	Regular	Extra wide
cG	33.5 – 36.5	37.0 – 40.5	36.0 – 39.5	39.5 – 43.0	38.0 – 41.5	41.5 – 45.5	40.5 – 44.5	45.0 – 49.5
cF	31.0 – 34.0	34.0 – 37.0	33.0 – 36.0	36.0 – 39.5	35.0 – 38.5	38.5 – 42.0	37.5 – 41.0	41.5 – 45.5
cE	29.0 – 31.5	31.5 – 34.5	30.5 – 33.5	33.5 – 36.5	32.5 – 35.5	35.5 – 39.0	35.0 – 38.5	38.5 – 42.0
cD	27.0 – 29.5	29.5 – 32.0	28.5 – 31.0	31.0 – 34.0	30.5 – 33.5	33.5 – 36.5	33.0 – 36.0	36.0 – 39.5
cC1*	21.0 – 23.0		23.0 – 25.0		25.0 – 27.5		27.5 – 30.0	
cC	19.0 – 20.5		20.5 – 22.5		22.5 – 24.5		25.0 – 27.5	
cA	20.0 – 23.0		22.0 – 25.0		24.0 – 27.0		26.0 – 29.0	

Length (cm)	Standard length	Short length
	41 – 47	36 – 40

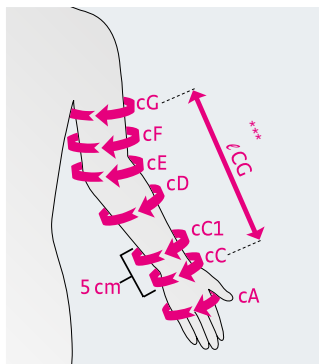

\* Measurement cC1 is 5 cm above cC

\*\* Size VIII: Minimum order quantity of 20 pieces per colour and variant; delivery period approx. 4 weeks

\*\*\* Length cG is measured on the inside of the arm from the wrist to the anterior axillary fold.



Seamless AC1 hand piece



Seamless AC1 glove



Seamless AC1 glove with armsleeve CG

## mediven harmony® SL standard styles (ready-to-wear)

### Styles – Standard

AC1 seamless hand piece  
without fingers  
with thumb base

AC1 seamless glove  
with compressive fingers



seamless

CCL 1: Art.No.: 2Y90x02-7  
CCL 2: Art.No.: 2Z90x02-7

CCL 1: Art.No.: 2Y91x02-7  
CCL 2: Art.No.: 2Z91x02-7

x = colour; replace with 5 for black, 7 for caramel or 8 for sand

### Circumference / Hand part (cm)

Sizes	II	III	IV	V	VI	VII
cC <sub>1</sub> <sup>*</sup>	15.0 – 18.0	17.0 – 20.0	19.0 – 21.5	21.0 – 23.5	23.0 – 26.0	25.5 – 30.0
cC	13.5 – 15.5	15.0 – 18.0	17.0 – 20.0	19.0 – 21.0	20.0 – 23.5	22.5 – 27.5
cB	15.5 – 20.0	17.5 – 21.0	19.5 – 23.0	20.5 – 24.0	22.5 – 26.0	24.5 – 30.0
cA	15.0 – 19.0	17.0 – 20.0	19.0 – 22.0	20.0 – 23.0	22.0 – 25.0	24.0 – 29.0

### Circumference / Fingers (cm) / Sizes


Sizes		II	III	IV	V	VI	VII
Thumb	Bottom cX	7.0	7.5	8.0	8.5	9.0	10.0
	Top cZ	6.0	6.5	6.5	7.0	7.5	8.0
Index finger	Bottom cX	6.5	7.0	7.5	8.0	8.5	9.5
	Top cZ	5.0	5.5	6.0	6.0	6.5	7.0
Middle finger	Bottom cX	6.5	7.0	7.5	8.0	8.5	9.5
	Top cZ	5.0	5.5	6.0	6.0	6.5	7.0
Ring finger	Bottom cX	6.0	6.5	7.0	7.0	7.5	9.0
	Top cZ	5.0	5.0	5.5	5.5	6.0	6.5
Little finger	Bottom cX	5.5	6.0	6.0	6.5	7.0	8.0
	Top cZ	4.5	4.5	5.0	5.0	5.5	6.0

### Length / Hand part (cm), all sizes


AB	3.5 – 4.5
AC	10.0 – 12.0
AC <sub>1</sub>	15.0 – 16.0

### Length / Fingers (cm)

Thumb	3.5 – 4.5
Index finger	4.5 – 5.5
Middle finger	5.5 – 6.5
Ring finger	4.5 – 5.5
Little finger	3.5 – 4.5

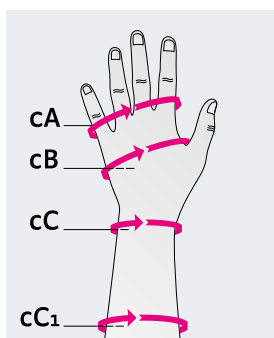

\* Measurement cC1 is 5 cm above cC

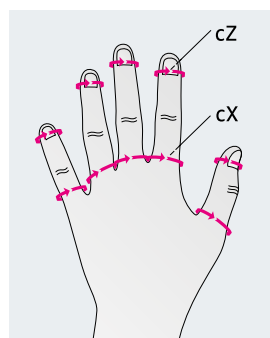

Circumference cZ is measured at the nailbed.  
Circumference cX is measured  
at the base of the finger





Decongestion  
Phase

Transition  
Phase

Maintenance  
Phase



3

Phase





# circaid<sup>®</sup> juxtafit<sup>®</sup> essentials arm

Adjustable compression device for optimising and maintaining decongestion results in lymphoedema treatment.

## medi tip<sup>1</sup>:

**Due to product features we recommend circaid juxtafit for:**

- Men / women
- Optimising and maintaining decongestion results
- Patients who have challenges with medical compression armsleeves
- Up to 24 hour compression therapy

## Indications

- Lymphoedema
- Lipoedema
- Dependent oedema

## Product features

- Easy handling – can be applied using one hand
- Easy handling – simple donning and doffing
- circaid Built-In-Pressure System (BPS): adjustable, controllable and re-adjustable compression
- Re-adjusting the individual bands avoids pressure loss and thus sliding down of the garment
- Sustained compression with defined pressure for up to 24 hours if required
- Breathe-O-Prene: breathable and inelastic material with high wall-stability for high working pressure
- Odour-inhibiting material

## Benefits


- Offering an alternative therapy solution for patients who are not able to apply or tolerate compression armsleeves
- Compression therapy – essential part of the complex lymphoedema therapy
- Easy handling allows for self-management of the patient
- Compression solution for optimising decongestion results and stabilising oedema fluctuations

For further product information and details please refer to pages 95 – 96 (see Phase 2, Transition Phase)

## Adjustable compression

10 – 20 mmHg  
20 – 30 mmHg  
30 – 40 mmHg

## Product benefits

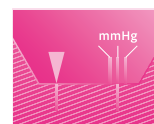

Built-In-Pressure<sup>®</sup>



Variety



Juxtaposed band



Easy handling



Inelastic



Breathe-O-Prene



## Colours

S  
Beige

S = Standard

## Note

A video for application of the circaid juxtafit essentials arm can be found here: [medi.biz/jf](https://medi.biz/jf)

\* Please consider, that this recommendation does not replace the diagnosis and consultation of a medical healthcare professional.

Decongestion  
Phase

Transition  
Phase

Maintenance  
Phase







## Commercial products

Sometimes it is difficult for patients to put on their compression stockings. medi offers a large selection of practical donning and doffing aids. For patients with sensitive skin there is also a special skin care series by medi. The rich ingredients pamper the skin and help it regenerate.


# medi Donning and doffing aids – medi Butler

## Your benefits:

- Different Butlers: the right solution for every customer requirement
- Increase add-on sales
- Time saving with uncomplicated donning and doffing

## Benefits for your customers:

- Quick and simple donning of compression stockings
- Less effort required
- Wide choice for the most individual demands

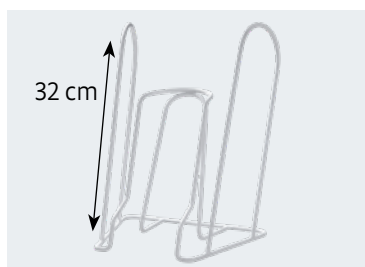

### medi Export Butler with long handles

The ideal donning aid for all compression stockings wearers who can comfortably don their stockings without assistance. The long handle ensures maximum stability when putting on compression stockings and pantyhose. The handle length is 32 cm. The stirrup width is 11 cm.


**Article No.** 6900041000



### medi Hospitalbutler

Adjustable diameter with hard paint finish coating.  
The stirrup width is 11cm.


**Article No.** 6900062000

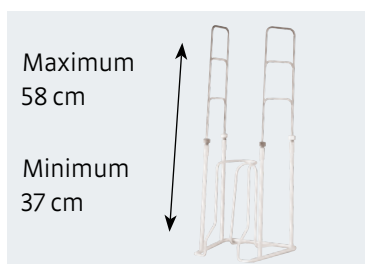

### medi Butler with variable handles

The donning aid for stocking wearers with considerably restricted freedom of movement.  
The handle length can be adjusted between 37 and 58 cm.  
The stirrup width is 11 cm.

**Article No.** 6900051000

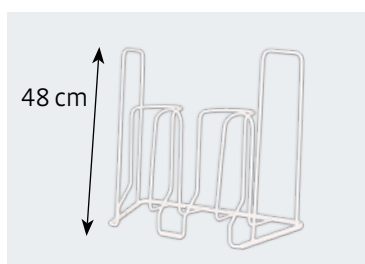


### medi Butler for pantyhose

The medi pantyhose Butler eases donning of compression pantyhose.  
The handle length is 48 cm. The stirrup width is 11 cm.

**Article No.** 6900055000



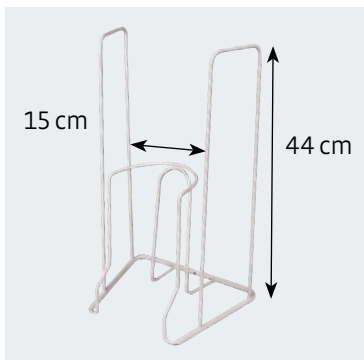

### medi Big (XXL) Butler

The stirrup of the medi Big Butler has a larger diameter. This makes allowance for larger circumferences which are often associated with less freedom of movement. The handle length is 44 cm. The stirrup width is 15 cm.


**Article No.** 6900102000



### medi Arm Butler

The medi Arm Butler eases donning of compression armsleeves.

**Article No.** 6900057000 (shaft diameter: 11 cm)

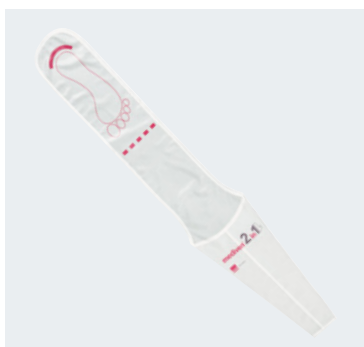

### medi 2in1

Your compression stocking glides up over your heel effortlessly

- For all garment lengths
- For open and closed toes
- Hard-wearing and tear-resistant
- Light and folds up – very compact and portable

**Article No.** 6900013000



Dressing aid

## Dressing aid for compression stockings with open toe

Easy-on-slipper, one size fits all.

Article No.	Description
6900022000	Dressing aids (unit with 40 pieces)




Rubber gloves

## Rubber gloves

- Sizes: small, medium and large
- Material: Natural rubber

Article No.	Description	Size	Quantity
6901022000	rubber gloves	S 6.5 – 7	1 pair
6901023000	rubber gloves	M 7.5 – 8	1 pair
6901024000	rubber gloves	L 8.5 – 9	1 pair
6901122000	rubber gloves	S 6.5 – 7	12 pairs / carton
6901123000	rubber gloves	M 7.5 – 8	12 pairs / carton
6901124000	rubber gloves	L 8.5 – 9	12 pairs / carton

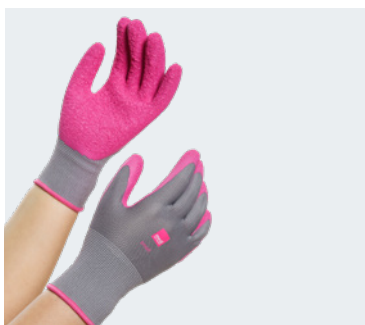

medi textile gloves

## medi textile gloves with latex


- Provide a better grip
- Breathable and skin friendly material
- Easy sizing system

Article No. single pack	Article No. multi pack	Description
6911004000	6911014000	Size large
6911005000	6911015000	Size x-large



Sizing for medi rubber and textile gloves

Place your right hand on the marked line for the inner edge of the hand and the thumb. Extend the fingers without overstretching them. Use the markings to read your glove size.

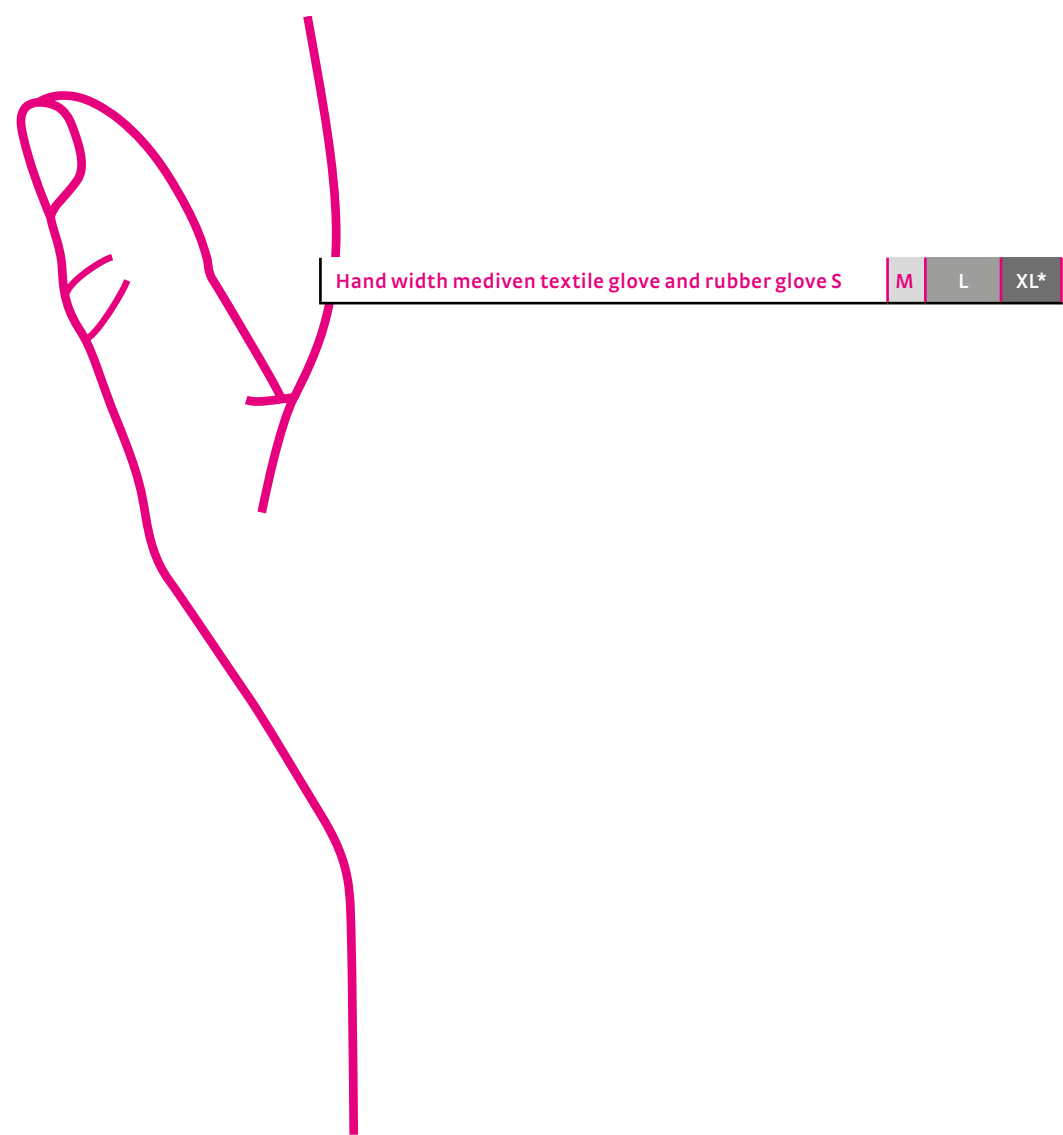

\* The size XL is only available for medi textile gloves.







# medi skin care

**Compression garments are the central component of lymph therapy. Skin care is an important complement to compression garments.**

## Your benefits

- Increase add-on sales / present as a full-line provider
- Confidence that the ingredients are perfectly matched to the compression garments
- Increase customer satisfaction by supplying ideal care for stressed skin
- Decreases wear and tear on the compression garments by encouraging smooth soft skin

## An important tip for patients with allergies

All products meet the highest demands and are produced free of parabens (preservatives) and wool fat alcohols, and are latex-free.

The medi care products do not have a use-by or expiry date. They are marked with an “opened tub” and a month. This means that as soon as the tube has been opened and the contents are being used, the product should be used within 6, 9, or 12 months, as with shower gels, shampoos, etc.

### medi Note

medi care products are cosmetic products and may only be used on intact skin.

medi day, medi fresh and medi soft are not available to all countries.

# medi skin care

The development of a lymph disease increases the burden on the skin. The stretched wall of the vein leaks fluids into the surrounding tissue. The increased loss of water from the surface of the skin makes it more prone to injury. In order to enhance the success of compression therapy, products have been developed to meet the specific requirements of an affected leg.



medi day



\*

## medi day – cooling gel with horse chestnut

New Formula  
(without alcohol)

### Benefits

- Refreshing and cooling effect
- Well-nourished, smooth skin
- Reduces the feeling of tightness
- Supports user comfort of medical compression stockings

Use medi day directly before putting on the compression stockings and let the day begin.

Article No.	Description
P300025000	medi day with 50 ml (unit with 6 pieces)
P300026000	medi day with 150 ml (unit with 6 pieces)



medi night



\*\*

## medi night – skin care cream with regenerative witch Hamamelis

### Benefits

- To soothe and rejuvenate during the night.
- Soothes the skin and also supports regeneration

Apply medi night after putting on the compression stockings.

Article No.	Description
P500025000	medi night with 50 ml (unit with 6 pieces)
P500026000	medi night with 150 ml (unit with 6 pieces)

\* You can find more information about the test carried out at [www.dermatest-guarantee.com](http://www.dermatest-guarantee.com)

\*\* You can find more information about the test carried out at [www.dermatest-guarantee.de](http://www.dermatest-guarantee.de)





medi fresh



\*

## medi fresh – refreshing spray with menthol

New Formula  
(without alcohol)

### Benefits

- For a hint of freshness throughout the day
- Feel refreshed straight away
- Gentle on the stocking fabric

Easy to use: spray directly onto the garment at any time and enjoy an immediate soothing effect.

Article No.	Description
P700026000	medi fresh with 100 ml (unit with 6 pieces)



medi soft



\*\*

## medi soft – skin care with horse chestnut and moisturising urea

### Benefits

- Gentle foam lotion to care for the skin around the clock
- Well-nourished, smooth skin
- Reduces the feeling of tightness
- Supports user comfort of medical compression stockings
- Provides additional support for fabrics and topbands
- Can: 100 % recycled, except spray head, cap and label

Treat your skin in the mornings and evenings to some extra moisture.

Article No.	Description
P800026000	medi soft, 125 ml can (unit with 6 pieces)

### Note

Suitable for diabetics.

\* You can find more information about the test carried out at [www.dermatest-guarantee.com](http://www.dermatest-guarantee.com)

\*\* You can find more information about the test carried out at [www.dermatest-guarantee.de](http://www.dermatest-guarantee.de)

# medi textile care

Gentle care for hand washing your garments.



medi clean

## medi clean – hand detergent for medical compression stockings

### Easy care of compression stockings – anytime, anywhere.

The special washing liquid is formulated to clean your medical compression stockings.

Article No.	Description
P920026000	medi clean with 200 ml (unit with 6 pieces)



medi spot ex

## medi spot ex – removal stick

### The clever solution to remove stains.

Used before washing, the spot removal stick medi spot ex removes shoe marks from the toes or the heel of the fabric.

Article No.	Description
P930025000	medi spot ex with 15 ml (unit with 6 pieces)



medi fix

## medi fix

Adhesive lotion for securing medical compression stockings.

Article No.	Description
1023025000	medi fix with 50 ml (unit with 6 pieces)



\*



medi laundry net

## medi laundry net

### Gentle washing

Gentle care helps compression stockings maintain their comfort and their medical efficacy.

The medi laundry net is the ideal aid for caring for compression stockings. The net structure protects the fabric against damage during the daily wash, but does not hinder the removal of dirt particles or soap residues.

Article No.	Description
1020585000	medi laundry net (unit with 15 pieces)

\* You can find more information about the test carried out at [www.dermatest-guarantee.de](http://www.dermatest-guarantee.de)











## Service

On the following pages we have compiled all the relevant information you will need for ordering our products. "medi. I feel better".

# Overview – what's included in your order?

## Standard sizes

	1 inelastic adjustable compression garment	1 pair of circaid® compression anklets	1 pair of circaid® undersleeves*	1 pair of circaid® compressive undersocks (moderate)	1 circaid® cover up	1 circaid® power added compression band (pac band)
circaid® reduction kit						
circaid® reduction kit lower leg	•		•			
circaid® reduction kit knee	•					
circaid® reduction kit upper leg	•					
circaid® reduction kit arm	•		•			
circaid® juxtafit® premium						
circaid® juxtafit® premium lower leg	•	•	•		•	•
circaid® juxtafit® premium lower leg with circaid® compressive undersock	•			•	•	•
circaid® juxtafit® essentials						
circaid® juxtafit® essentials lower leg	•	•**	•***		•	•
circaid® juxtafit® essentials lower leg with circaid® compressive undersock	•			•	•	•
circaid® juxtafit® essentials upper leg with knee	•		•		•	
circaid® juxtafit® essentials arm	•				•	

\* Silver for circaid® juxtafit® premium

\*\* Not included with size XXXL

\*\*\* For size XXXL with foot



1 circaid® Built-In- Pressure System (BPS) guide card	1 circaid® Built-In- Tension System (BTS) guide card	2 circaid® knee spine (connection piece)	6 additional hook tabs small	8 additional hook and loop fasteners	1 circaid® shelf strap	1 circaid® customizable hand wrap	1 elbow band
	•		•		•		
		•			•		
	•		•		•		
	•					•	
•							
•							
•							
•							
•				•			
•							•

# Order form for circaid® standard garments

Telefax: +49 921 912-781, Email: orders@medi.de

**medi**


Customer name \_\_\_\_\_ Customer no. \_\_\_\_\_

Delivery address \_\_\_\_\_ Patient name \_\_\_\_\_

Delivery date (client) \_\_\_\_\_ Order date \_\_\_\_\_ Measured by \_\_\_\_\_

Tel no. \_\_\_\_\_ Email address \_\_\_\_\_

Stamp / Signature \_\_\_\_\_

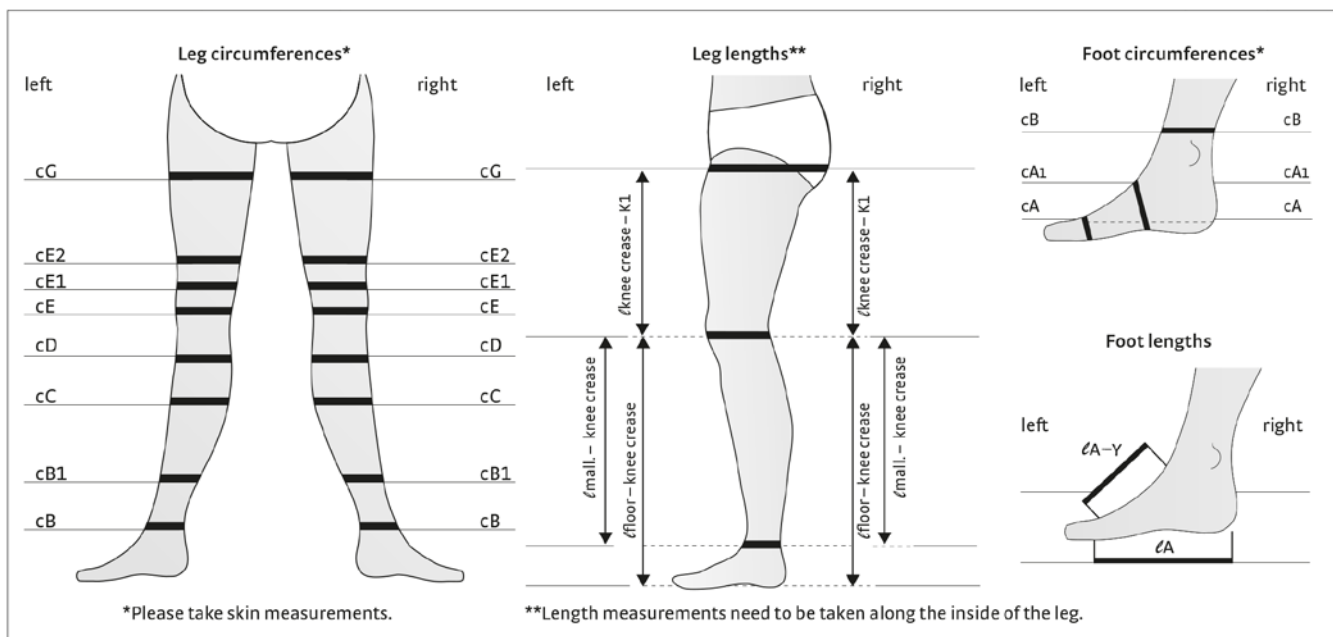

Garments	Quantity	Sizes								Length			Variant***	
circaid juxtacures***										short	standard	long	anklet	undersock
circaid juxtalite***		S	M	M-X	L	L-X	XL	XL-X	XXL	short		long	anklet	undersock
circaid juxtalite hd		S	M	M-X	L	L-X	XL	XL-X	XXL	short		long		
circaid reduction kit lower leg		standard		wide						short	standard	long		
circaid reduction kit knee														
circaid reduction kit upper leg		standard		wide						short	standard	long		
circaid juxtafit premium lower leg***		S	M	M-X	L	L-X	XL	XL-X	XXL	short		long	anklet	undersock
circaid juxtafit essentials lower leg***		S	M	M-X	L	L-X	XL	XL-X	XXL	short		long	anklet	undersock
circaid juxtafit essentials	left	XXS	XS	S	M	L	XL			x-short	short	long		
upper leg with knee	right	XXS	XS	S	M	L	XL			x-short	short	long		
circaid compression anklet		S	L											
circaid compressive undersock		standard		medium									moderate	firm
circaid power added compression band (pac band)		S	L	XL										
circaid juxtalite ankle foot wrap		S	M											
circaid customizable interlocking ankle foot wrap														
circaid juxtafit premium interlocking afw		S	M	L										
circaid juxtafit premium afw		S	M	L										
circaid single band afw														
circaid toe cap		2	3	4	5	6								

## Note

For arm garments and accessories please see next page.

\*\*\* These products are available with circaid compression anklet or circaid compressive undersock moderate. Please choose accordingly.



# Order form for circaid® standard garments

Telefax: +49 921 912-781, Email: orders@medi.de




Customer name \_\_\_\_\_ Customer no. \_\_\_\_\_

Delivery address \_\_\_\_\_ Patient name \_\_\_\_\_

Delivery date (client) \_\_\_\_\_ Order date \_\_\_\_\_ Measured by \_\_\_\_\_

Tel no. \_\_\_\_\_ Email address \_\_\_\_\_

Stamp/Signature \_\_\_\_\_

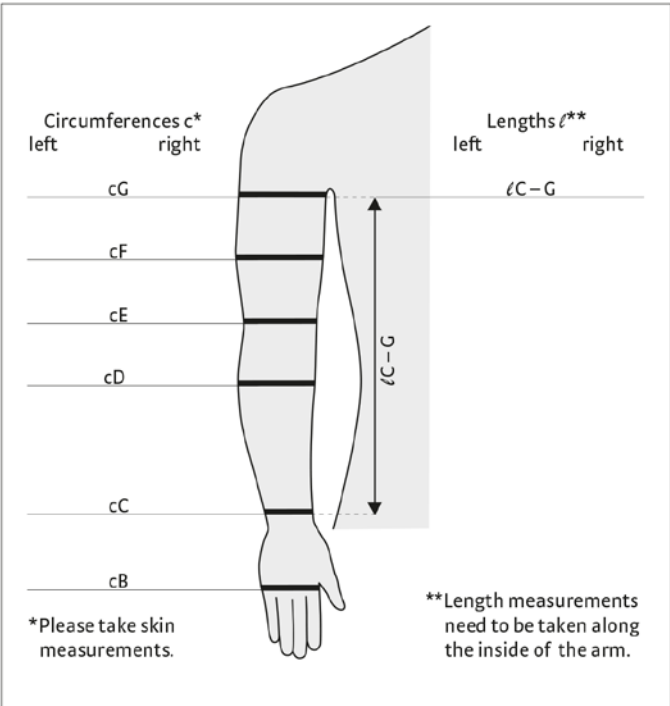

## Notes

\_\_\_\_\_

\_\_\_\_\_

\_\_\_\_\_

\_\_\_\_\_

\_\_\_\_\_

\_\_\_\_\_

\_\_\_\_\_

\_\_\_\_\_

Garments	Quantity	Sizes						Length			Side	
circaid reduction kit arm		standard		wide		extra wide		standard	long			
circaid customizable hand wrap												
circaid juxtafit essentials arm		1	2	3	4	5	6	x-short	short	long	left	right
circaid juxtafit essentials hand wrap		XS S M L XL									left	right
circaid glove		2 3 4 5 6										

Accessories	Quantity	Sizes		Style		Colour	
circaid undersleeve leg				lower leg	whole leg		
circaid undersleeve leg silver				lower leg			
circaid undersock				lower leg			
circaid undersock silver				lower leg			
circaid undersleeve arm							
circaid undersleeve arm silver							
circaid cover up leg		S	L	lower leg		beige	black
circaid reduction kit shelf strap							
circaid reduciton kit lobe lifter							
circaid juxtafit premium shelf strap		S	L				


0097C66 / 07.2024

Commercial products

Service


# Correct fitting of mediven<sup>®</sup> flat knit compression stockings

Before taking measurements for medical compression stockings with seams, any swelling should be reduced as much as possible. Circumference and length measurements are taken in a distal to proximal direction. The measuring points on the leg must be marked so that the circumference and length measurements are taken at the same location. For lymphatic compression stockings, some measurements will be performed with tension applied.

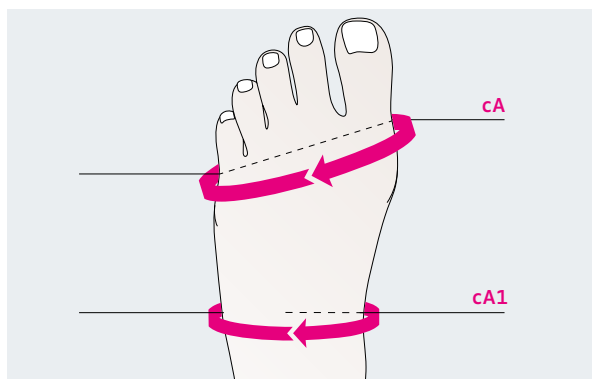


## 1. Foot circumference

- Leg is resting relaxed on a horizontal support
- During this measurement, the foot is relaxed, not held in a 90° position

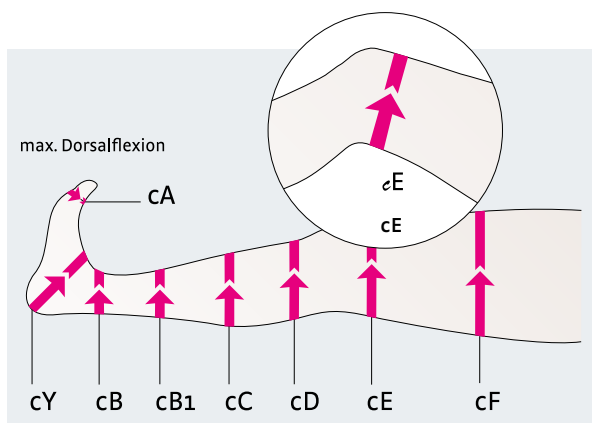

## 2. Leg circumference

- Leg is lying in a relaxed position on a support.
- For circumference measurement **cY**, flex foot in dorsal direction to maximum extent
- Measure the remaining circumferences up to **cF** with the leg in a relaxed position
- For circumference measurement **cE**, bend knee slightly
- Mark measuring points
- Tension measurements: cB, cB1, cC, cF,

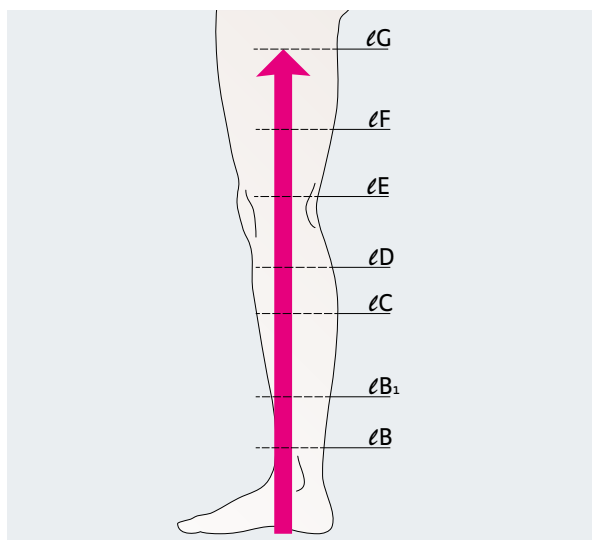


## 3. Leg length

Proximal length measurements need to be taken along the inside of the leg while the patient is standing.


medi uses the measurement method according to the guidelines of the Federal Academy for Orthopaedic Technology in Dortmund.



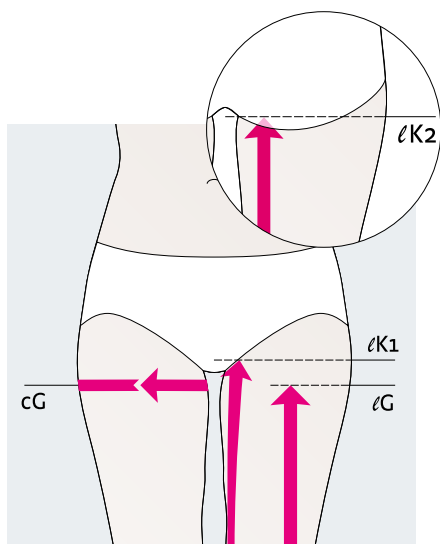


#### 4. Border on thigh

- Measure circumference cG at the widest part on the thigh / below crotch
- Measure length lG along the leg with patient standing up
- Measure lK2 from ground to gluteal fold along body with patient standing up  $\hat{=}$  posterior leg length
- Measure lK1 from ground to symphysis along body with patient standing up  $\hat{=}$  anterior leg length  
(Difference between lK1 and lK2  $\hat{=}$  high front / low back)

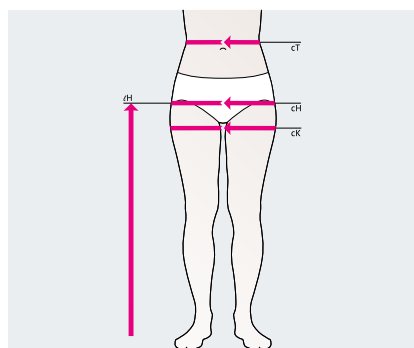


#### 5. Waist / hip circumference / length


- Measure circumference cK with patient standing up at the widest section around both thighs below the gluteal fold
- Measure circumference cH with patient standing up at the widest section of the hips
- Measure circumference cT with patient standing up at the waist or the desired border
- Measure length lH along body up to desired height = Exterior leg length
- Tension measurements: cK, CH

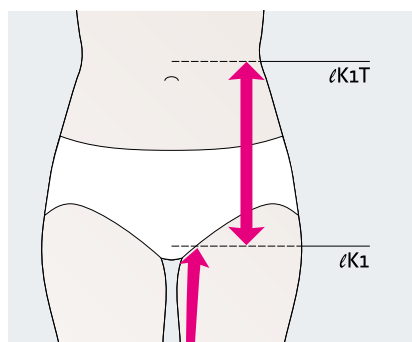

#### 6. Waist / hip lengths

- Measure anterior pelvic height lK1T along body from crotch (symphysis) to waist or desired height
- Measure posterior pelvic height lK2T along body from gluteal fold to waist or desired height

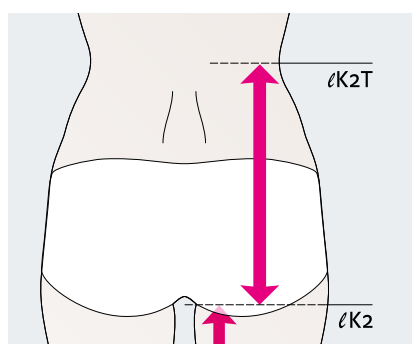


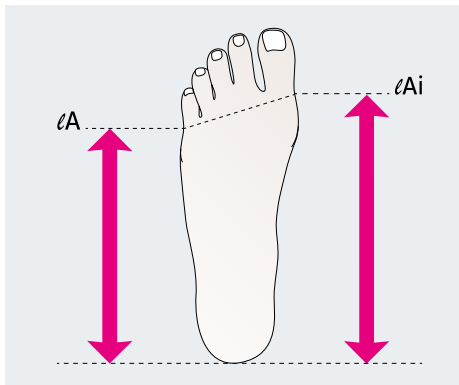

## 7. Foot length

### Open toe / oblique border


- Measure lengths on the inside and outside of weight-bearing foot.

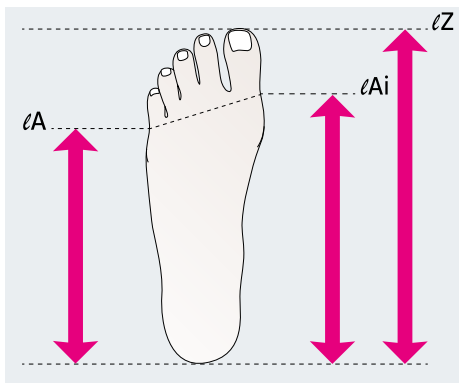

## 8. Foot length

### Closed toe / oblique border


- Measure lengths on inside, outside and entire weight-bearing foot.

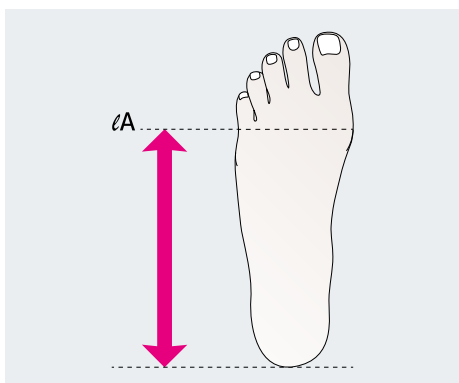

## 9. Foot length

### Open toe / straight border

- Measure length on outside of weight-bearing foot.

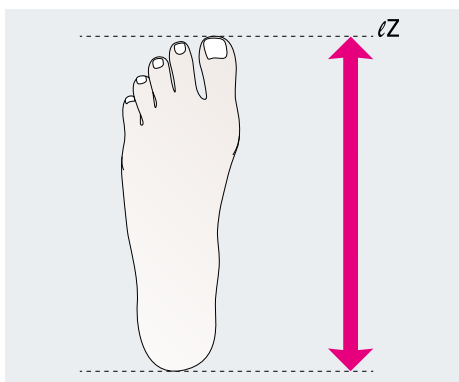

## 10. Foot length

### Closed toe / straight border

- Generally, this design is made to allow a 6 cm long toe area.
- Measure length of entire, weight-bearing foot.



☐ Estimate ☐ Estimate by last MTM-number: \_\_\_\_\_  
☐ Order ☐ Order by last MTM-number: \_\_\_\_\_

E-mail order to medi • mtm-export@medi.de

Retailer: \_\_\_\_\_ Stamp/Signature: \_\_\_\_\_


Customer No.: \_\_\_\_\_ Delivery address: \_\_\_\_\_  
(please always specify)

Commission: \_\_\_\_\_


Delivery date: (client) \_\_\_\_\_ Order date: \_\_\_\_\_

medi

**Circumferences c – left** **Circumferences c – right**  
normal Tight tape measure Tight tape measure normal

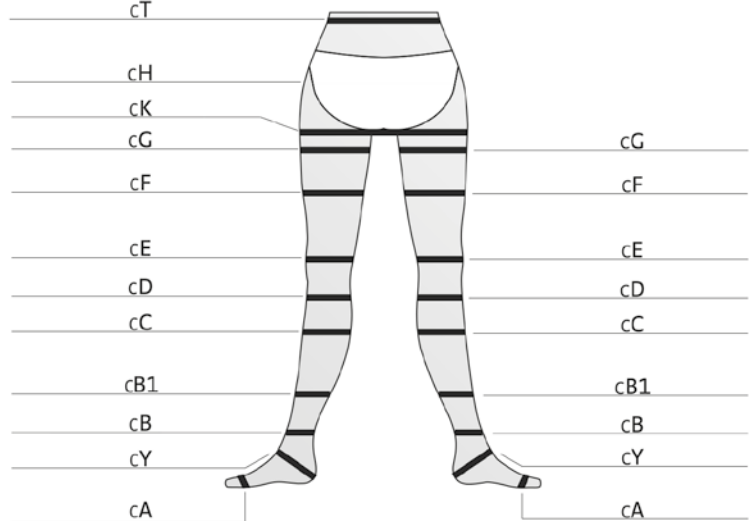

**Lengths ℓ**

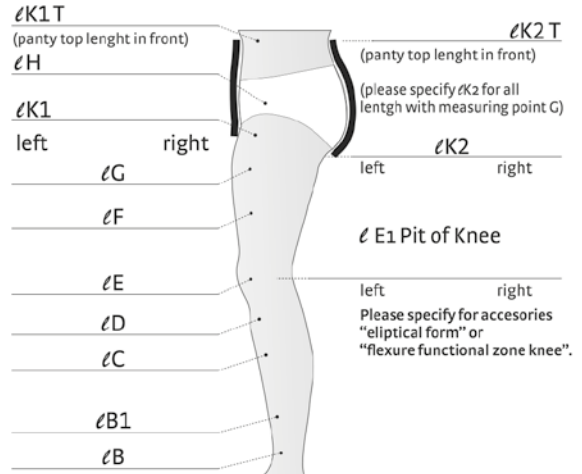

☐ lateral seam (except mediven mondi)

Model	Compression CCL	Standard colours	Trend colours**	Quantity	Foot	Distal border
<input type="checkbox"/> mediven mondi (CCL 1, 2, 3)	1 2 3 4	<input type="checkbox"/> Sand	<input type="checkbox"/> Lilac	<input type="checkbox"/> pair	<input type="checkbox"/> closed toe	oblique border forefoot
<input type="checkbox"/> mediven cosy (CCL 1, 2, 3)	<input type="checkbox"/> Panty section	<input type="checkbox"/> Caramel	<input type="checkbox"/> Sage-green	<input type="checkbox"/> pcs.	<input type="checkbox"/> open toe	Please specify for integration of oblique foot:
<input type="checkbox"/> mediven 550 (CCL 1, 2, 3, 4)	<input type="checkbox"/> Left leg	<input type="checkbox"/> Black	<input type="checkbox"/> Light-blue		<input type="checkbox"/> hallux ease zone (except mediven mondi)	ℓZ (total in cm)
	<input type="checkbox"/> Right leg	<input type="checkbox"/> Cashmere	<input type="checkbox"/> Russet-red		<input type="checkbox"/> left <input type="checkbox"/> right	ℓAi (inner in cm)
		<input type="checkbox"/> Navy			<input type="checkbox"/> varus toe ease zone (except mediven mondi)	ℓA (outer in cm)
		<input type="checkbox"/> Anthracite			<input type="checkbox"/> left <input type="checkbox"/> right	straight border
		<input type="checkbox"/> Grey**			<input type="checkbox"/> toe cap	<input type="checkbox"/> <input type="checkbox"/>
		<input type="checkbox"/> medi Magenta**				

Variations	Proximal border	Accessories	Waist	Gusset
<input type="checkbox"/> below knee (AD)	flat oblique (standard)	<input type="checkbox"/> Extension to sole of foot	perforated topband (standard)	tricot (standard)
<input type="checkbox"/> thigh-length (AG)	<input type="checkbox"/> steep oblique	<input type="checkbox"/> Y knitting mark (90° heel)	<input type="checkbox"/> waistband	<input type="checkbox"/> netting
<input type="checkbox"/> pantyhose (AT)	<input type="checkbox"/> straight	<input type="checkbox"/> elliptical form	<input type="checkbox"/> knitted border	<input type="checkbox"/> compressive
<input type="checkbox"/> men's leotard (ATH)		<input type="checkbox"/> flexure functional zone knee (except mediven mondi)	<input type="checkbox"/> silicone dot topband	<input type="checkbox"/> horizontal fly (for men's panty)
<input type="checkbox"/> maternity panty (ATU) (except mediven mondi)		<input type="checkbox"/> orientation mark at „D“ (only for AG and AT)	<input type="checkbox"/> silicone micro dot topband sensitive	<b>Panty top</b>
<input type="checkbox"/> one-legged panty (ATE)		<input type="checkbox"/> extra leg length (ℓK1 needed)	<input type="checkbox"/> silicone dot topband motive	<input type="checkbox"/> Gluteal shaper (except mediven mondi)
<input type="checkbox"/> BT / B1T / CT / ET / FT		<input type="checkbox"/> _____ cm porous	<input type="checkbox"/> plain silicone topband rose	
<input type="checkbox"/> cuff BD / BF / BG / CF / CG / DG				

Other accessories	Topband piece	Anti-slip-dots
Position	Standard	Individual
along the oblique border	<input type="checkbox"/> 15 x 5 cm	<input type="checkbox"/> _____ cm
lengthways over „E“	<input type="checkbox"/> 8 x 5 cm	<input type="checkbox"/> _____ cm
rear over seam	<input type="checkbox"/> 8 x 5 cm	<input type="checkbox"/> _____ cm
on the sole	<input type="checkbox"/> 5 x 5 cm	<input type="checkbox"/> 6 x 4,5 cm

<b>Silver</b> (only mediven mondi)	<input type="checkbox"/> „Y“ to C	<input type="checkbox"/> „Y“ to D	<input type="checkbox"/> „Y“ to G
<input type="checkbox"/> left <input type="checkbox"/> right	<input type="checkbox"/> „A“ to C	<input type="checkbox"/> „A“ to D	<input type="checkbox"/> „A“ to G
<input type="checkbox"/> Lymphpad*	<input type="checkbox"/> Lining*	<input type="checkbox"/> Levamed*	
<b>left</b> <input type="checkbox"/> inside <input type="checkbox"/> outside	<b>right</b> <input type="checkbox"/> inside <input type="checkbox"/> outside		

<b>Attachment</b>	
Silicone dot topband <input type="checkbox"/> narrow 2,5 cm <input type="checkbox"/> wide 5 cm	Circumference:
<input type="checkbox"/> Silicone dot topband motive (5 cm)	left _____ cm
<input type="checkbox"/> Silicone micro dot topband sensitive (5 cm)	right _____ cm
<input type="checkbox"/> Plain silicone topband rose (5 cm)	
<input type="checkbox"/> waist attachment <input type="checkbox"/> left <input type="checkbox"/> right <input type="checkbox"/> ℓT _____ cm (total length)	

<b>Patterns***</b>	<input type="checkbox"/> Design-Elements	<input type="checkbox"/> Fashion-Elements
(except mediven mondi)	<input type="checkbox"/> Nature	<input type="checkbox"/> Stripes
	<input type="checkbox"/> Bloom	<input type="checkbox"/> Wild

<b>Special requests</b>

\* Please specify / draw exact area.  
\*\* Delivery period up to 10 working days.  
\*\*\* For all standard and trend colours!




0099109 / 012025

Commercial products


Service

# Correct fitting of mediven<sup>®</sup> flat knit toe caps

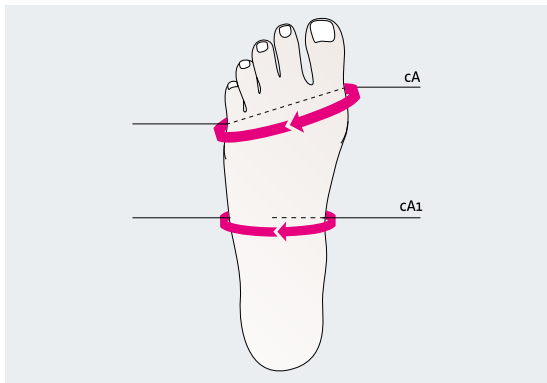

## 1. Foot circumference

- Leg is placed flat on a support
- Measure circumference
- Mark the measuring points

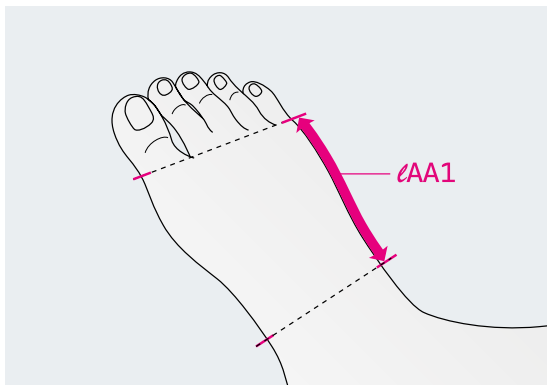


## 2. Foot length

- Proceed as in (1)
- Measure the length on the inside and outside of the foot
- Measure the distance from mark to mark (see measured circumference)



## 3. Toe circumference

- Proceed as in (1)
- Measure circumference cX at web area
- Measure the circumference cZ at the nail bed

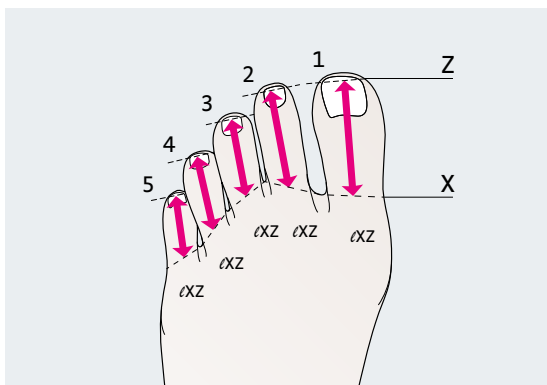

## 4. Toe lengths

- Proceed as in (1)
- Measure length from web to the end of the toe

medi uses the measurement method according to the guidelines of the Federal Academy for Orthopaedic Technology in Dortmund.

☐ Estimate      ☐ Estimate by last MTM-number: \_\_\_\_\_  
☐ Order      ☐ Order by last MTM-number: \_\_\_\_\_

E-mail order to medi • [mtm-export@medi.de](mailto:mtm-export@medi.de)

**medi**


Retailer: \_\_\_\_\_ Stamp/Signature: \_\_\_\_\_

Customer No.: \_\_\_\_\_ Delivery address: \_\_\_\_\_  
(please always specify)


Commission: \_\_\_\_\_

Delivery date: (client) \_\_\_\_\_ Order date: \_\_\_\_\_

### Circumferences c and lengths $\ell$ – left

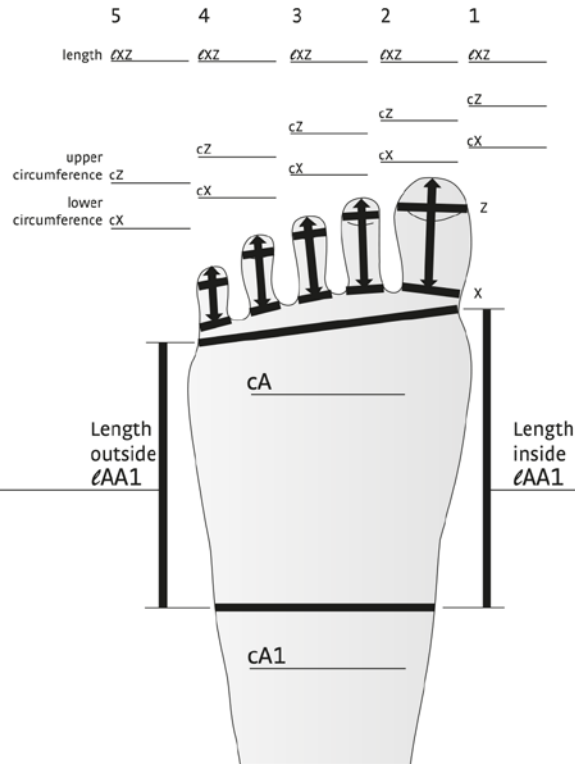

### Circumferences c and lengths $\ell$ – right

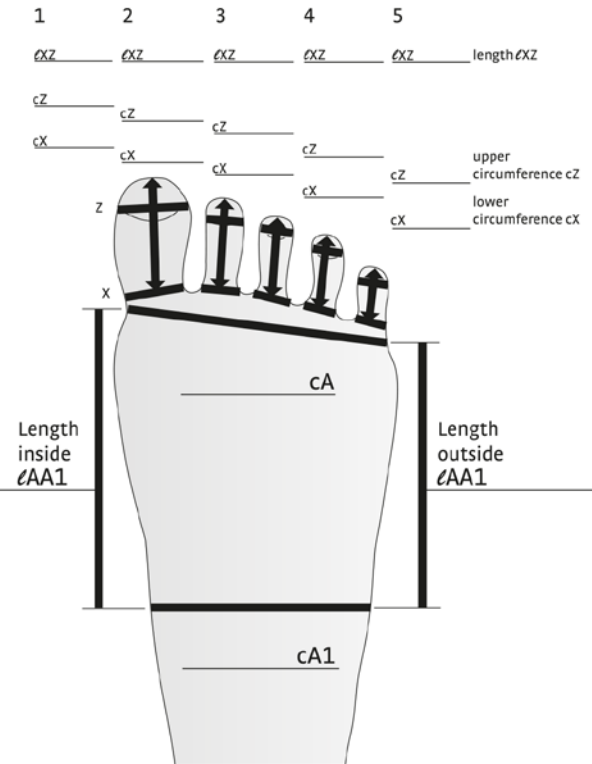

Model	Compression CCL	Standard colours	Trend colours*	Quantity left	Quantity right
<input type="checkbox"/> mediven mondi <input type="checkbox"/> mediven cosy <input type="checkbox"/> mediven 550 <input type="checkbox"/> mediven 550 SL	<b>1 2 3</b> <input type="checkbox"/> <input type="checkbox"/> <input type="checkbox"/> <input type="checkbox"/> <input type="checkbox"/> <input type="checkbox"/> <input type="checkbox"/> <input type="checkbox"/> <input type="checkbox"/> <input type="checkbox"/>	<input type="checkbox"/> Sand <input type="checkbox"/> Cashmere <input type="checkbox"/> Caramel <input type="checkbox"/> Navy <input type="checkbox"/> Black <input type="checkbox"/> Anthracite <input type="checkbox"/> Grey* <input type="checkbox"/> medi Magenta*	<input type="checkbox"/> Lilac <input type="checkbox"/> Sage-green <input type="checkbox"/> Light-blue <input type="checkbox"/> Russet-red	pcs. _____	pcs. _____

Other accessories / left variant	Other accessories / right variant
<input type="checkbox"/> without small toe <input type="checkbox"/> with small toe <input type="checkbox"/> closed toes <input type="checkbox"/> open toes <input type="checkbox"/> separate toe cap <input type="checkbox"/> lateral seam for single toe <input type="checkbox"/> toe cap seamless knitted to the stocking***      cap on big toe***	<input type="checkbox"/> without small toe <input type="checkbox"/> with small toe <input type="checkbox"/> closed toes <input type="checkbox"/> open toes <input type="checkbox"/> separate toe cap <input type="checkbox"/> lateral seam for single toe <input type="checkbox"/> toe cap seamless knitted to the stocking***      cap on big toe***
<input type="checkbox"/> Lymphpad*** <input type="checkbox"/> Lining*** <input type="checkbox"/> Pocket*** Please specify position exactly! (length / width)	<input type="checkbox"/> Lymphpad*** <input type="checkbox"/> Lining*** <input type="checkbox"/> Pocket*** Please specify position exactly! (length / width)

\* Delivery period up to 10 working days,

\*\* For all standard and trend colours!

\*\*\* not available for mediven 550 SL

**Patterns\*\***  
(mediven cosy and  
mediven 550)

☐ Design-Elements    ☐ Fashion-Elements

☐ Nature    ☐ Stripes    ☐ Bloom    ☐ Wild

### Special requests



4 046114 349228

0098111 / 07.2024


Commercial products

Service




# Correct fitting of mediven<sup>®</sup> flat knit arm compression sleeves

Before taking measurements for the armsleeve and glove, any swelling should be reduced as much as possible. At the circumferences of cC, cC1, and at the elbow, take measurements without applying any tension. For all other circumference measurements, it depends on how much pressure the patient can tolerate. Please always use a soft marker (e. g. a kohl pencil) to prevent damage to the skin.

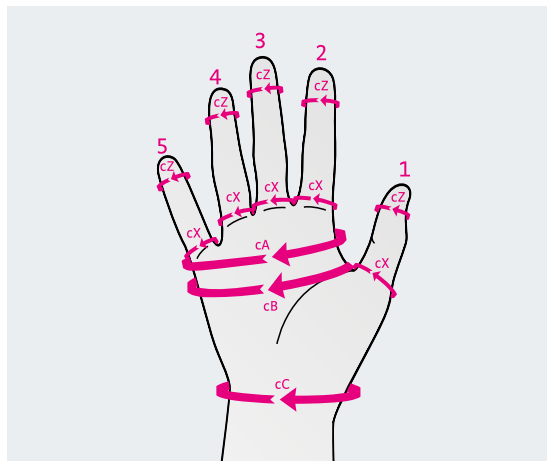


## 1. Finger circumference

- Hand is resting lightly on a surface with the back of the hand facing up
- Measure circumference cX at the web between fingers
- cZ is measured at the nail bed
- Mark the measuring points for length measurement

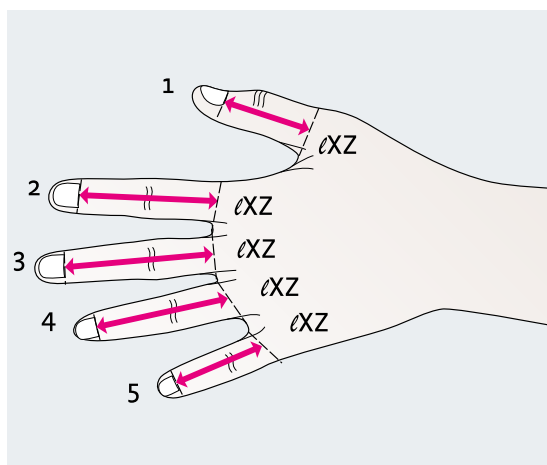


## 2. Finger length

- Hand is resting lightly on a surface with the palm of the hand facing up
- Length is measured from mark to mark (see "Finger circumference", Fig. 2)

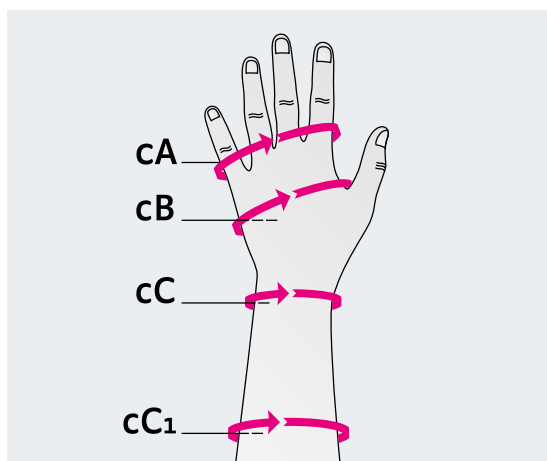


## 3. Hand circumference

- Hand should be pressed lightly on surface with the back of the hand facing up
- Measure circumference
- Mark the measuring points on the inside of the hand

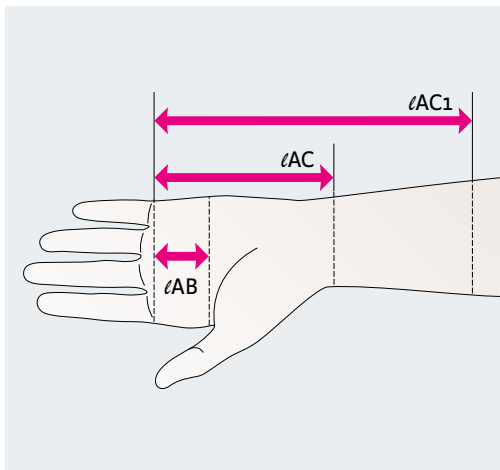


#### 4. Hand length

- Hand is resting lightly on a surface with the palm of the hand facing up
- Length is measured from mark to mark (see "Hand circumference", Fig. 1)

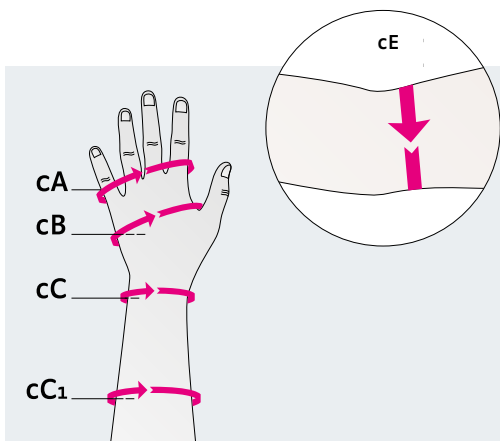


#### 5. Arm circumference

- Lightly rest hand and forearm on a surface
- Measure circumference as shown in drawing
- Mark measuring points for length measurement

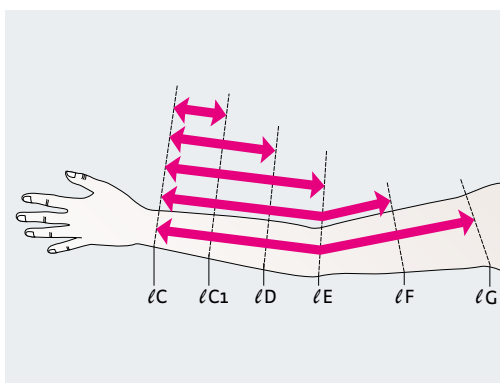


#### 6. Arm length

- Lightly rest hand and forearm on a surface.  
Arm is extended
- Measure lengths from lC to lC<sub>1</sub>, lC to lD and so on to lG in a line extending back from the groove between the thumb and index finger to the anterior axillary fold, from mark to mark (see circumference dimensions)
- tension measurements: cF, cD

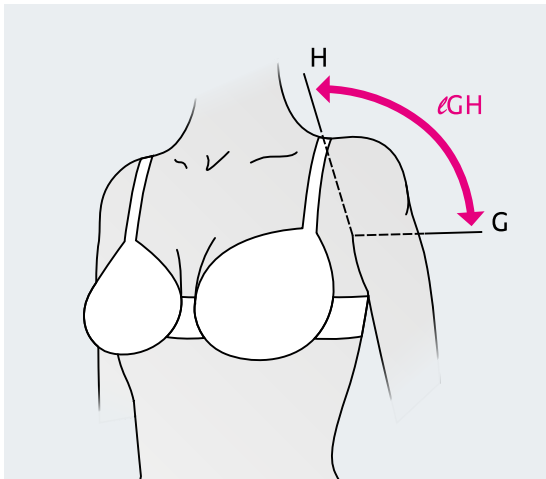

## 7. Upper arm circumference

For the shoulder cap, measure from point G to H, where H is where the bra strap sits. Do not measure past point H.

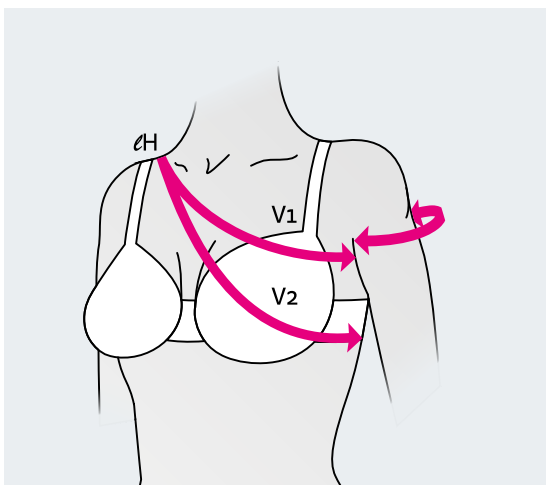

## 8. Attachments

- a) Shoulder strap  
The length of the shoulder is measured according to method 1 or 2 (full circumference)
- b) Anatomical shoulder caps are measured from the anterior to the posterior axillary fold

medi uses the measurement method according to the guidelines of the Federal Academy for Orthopaedic Technology in Dortmund.



- ☐ Estimate      ☐ Estimate by last MTM-number: \_\_\_\_\_
- ☐ Order        ☐ Order by last MTM-number: \_\_\_\_\_

E-mail order to [medi-mtm-export@medi.de](mailto:medi-mtm-export@medi.de)

Retailer: \_\_\_\_\_ Stamp/Signature: \_\_\_\_\_

Customer No.: \_\_\_\_\_ Delivery address: \_\_\_\_\_  
(please always specify)

Commission: \_\_\_\_\_

Delivery date: (client) \_\_\_\_\_ Order date: \_\_\_\_\_



**HAND SECTION – Circumference C and lengths  $\ell$**

Diagram illustrating hand measurements. The hand is divided into five fingers (1-5) and the thumb (Z). Measurements include upper circumference (cZ), lower circumference (cX), and lengths ( $\ell$ ) for various segments (AB, AC, AC1, AD, AE). A 'Tight tape measure' is shown for the upper circumference.

**ARMSLEEVE – Circumference C and lengths  $\ell$**

Diagram illustrating arm sleeve measurements. The arm is divided into segments (G, F, E, D, C1, C). Measurements include upper circumference (cG), lower circumference (cC), and lengths ( $\ell$ ) for various segments (GH, GF, FE, ED, DC1, CC1). A 'Tight tape measure' is shown for the upper circumference. A 'Shoulder strap' is also indicated.

<b>Model</b>	<b>Compression CCL</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>Standard colours</b>	<b>Trend colours*</b>	<b>Quantity</b>	<b>Options</b>
<input type="checkbox"/> mediven esprit SL (AC1) <input type="checkbox"/> mediven esprit <input type="checkbox"/> mediven 550 arm	Hand section <input type="checkbox"/> <input type="checkbox"/> <input type="checkbox"/> Armsleeve <input type="checkbox"/> <input type="checkbox"/> <input type="checkbox"/>				<input type="checkbox"/> Sand <input type="checkbox"/> Cashmere <input type="checkbox"/> Caramel <input type="checkbox"/> Navy <input type="checkbox"/> Black <input type="checkbox"/> Anthracite <input type="checkbox"/> Grey* <input type="checkbox"/> medi Magenta*	<input type="checkbox"/> Lilac <input type="checkbox"/> Sage-green <input type="checkbox"/> Light-blue <input type="checkbox"/> Russet-red	pcs. _____	<input type="checkbox"/> left <input type="checkbox"/> right <input type="checkbox"/> Fingers open <input type="checkbox"/> Fingers closed
<b>Variants</b>	<b>Standard border</b> (proximal)	<b>Variable Border</b>			<b>Other accessories</b>			
<b>Hand section</b> <input type="checkbox"/> AC1	straight (porous – 2 cm)	<input type="checkbox"/> flat oblique	<input type="checkbox"/> steep oblique			<b>Position</b>	<b>Topband piece</b>	<b>Anti-slip-dots</b>
<input type="checkbox"/> AD/AE	flat oblique	<input type="checkbox"/> steep oblique	<input type="checkbox"/> straight			along the oblique border	<input type="checkbox"/> 5 x 2,5 cm <input type="checkbox"/> 5 x 5 cm <input type="checkbox"/> 10 x 5 cm	<input type="checkbox"/> 6 x 4,5 cm
<b>Armsleeve</b> <input type="checkbox"/> CD/CE/CF/CG	flat oblique	<input type="checkbox"/> steep oblique	<input type="checkbox"/> straight			on the palm	<input type="checkbox"/> 5 x 5 cm	<input type="checkbox"/> 6 x 4,5 cm
<input type="checkbox"/> AF/AG (1-part)	flat oblique	<input type="checkbox"/> steep oblique	<input type="checkbox"/> straight			<input type="checkbox"/> Lymphpad <input type="checkbox"/> Lining <input type="checkbox"/> Pocket (except mv esprit SL) (Please specify / draw position exactly!)		
					_____ cm porous in hand section (proximal – standard 2 cm)			
					_____ cm porous in armsleeve (distal – standard 2 cm)			
					<b>Patterns**</b> <input type="checkbox"/> Design-Elements <input type="checkbox"/> Fashion-Elements <input type="checkbox"/> Nature <input type="checkbox"/> Stripes <input type="checkbox"/> Bloom <input type="checkbox"/> Wild			
<b>Attachment</b>					<b>Special requests</b>			
Silicone dot topband <input type="checkbox"/> narrow 2,5 cm <input type="checkbox"/> wide 5 cm					Circumference _____cm			
<input type="checkbox"/> Silicone dot topband motive (5 cm) <input type="checkbox"/> Silicone micro dot topband sensitive (5 cm) <input type="checkbox"/> Plain silicone topband rose (5 cm)								
<input type="checkbox"/> shoulder cap standard <input type="checkbox"/> shoulder cap anatomical _____ cm <input type="checkbox"/> with shoulder strap K      strap width: <input type="checkbox"/> 2,5 cm <input type="checkbox"/> 5 cm <input type="checkbox"/> with bra attachment      width of bra strap: _____ cm								
<b>Accessories</b>								
Knitting marks elbow <input type="checkbox"/> 160° (Standard) <input type="checkbox"/> 150° <input type="checkbox"/> 135° <input type="checkbox"/> Flexure functional zone elbow <input type="checkbox"/> Smart-Touch***					* Delivery period up to 10 working days, ** For all standard and trend colours! *** For mediven esprit SL fingers closed			

\*\* For all standard and trend colours!

\*\*\* For mediven esprit SL, fingers closed


\*\*\* For mediven esprit SL, fingers closed



0098L10 / 11.2025


Service

# Measuring Bolero

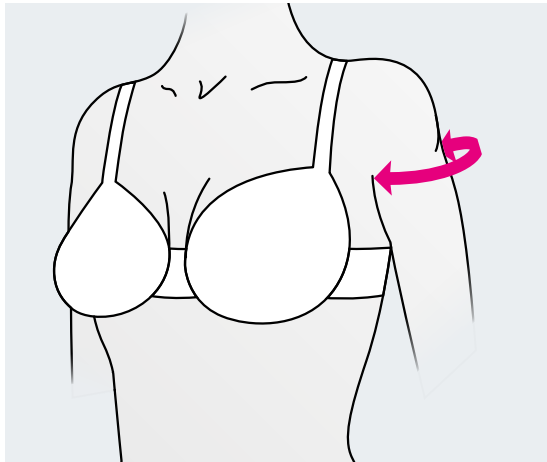

## Anatomical shoulder cap


Determine the anatomical shoulder cap on the outside of the arm. Place the tape measure at the anterior axillary fold and measure the distance to the posterior axillary fold.

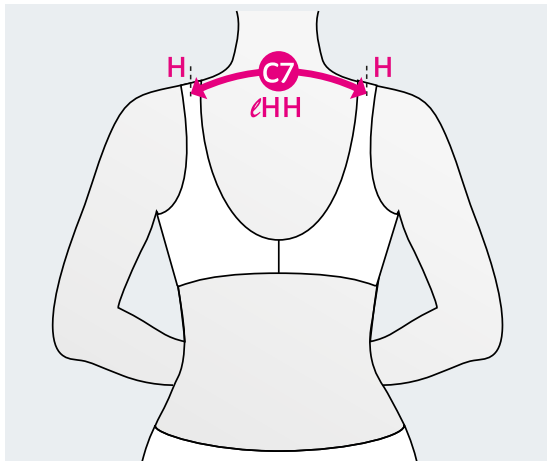

## Length HH

Place the tape measure at measuring point H (offset to the shoulder back) and measure along the body contour over the 7th cervical vertebra (C7) to measuring point H on the opposite side.

The measuring point H is located between the acromion and the clavicle, at the level of the bra strap. Please note: The torso is erect, and the shoulders are stretched through.

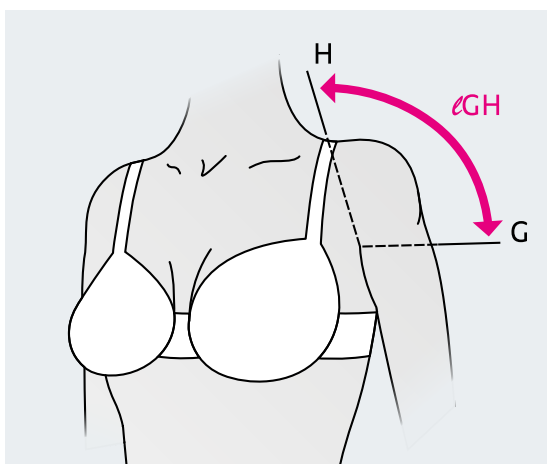

## Length GH

Determine the length IGH on the outside of the arm. Place the tape measure between the front and back creases and measure straight up to the measuring point H. The measuring point H is at shoulder height (acromion).

☐ Estimate      ☐ Estimate by last MTM-number: \_\_\_\_\_  
☐ Order      ☐ Order by last MTM-number: \_\_\_\_\_



E-mail order to medi • [mtm-export@medi.de](mailto:mtm-export@medi.de)

Retailer: \_\_\_\_\_ Stamp/Signature: \_\_\_\_\_

Customer No.: \_\_\_\_\_ Delivery address: \_\_\_\_\_  
(please always specify)

Commission: \_\_\_\_\_

Delivery date: (client) \_\_\_\_\_ Order date: \_\_\_\_\_

ℓGH (Length shoulder cap)			ℓGH (Length shoulder cap)			
Shoulder cap anatomical circumference (cG) semicircular (distance anterior till posterior axillary)			Shoulder cap anatomical circumference (cG) semicircular (distance anterior till posterior axillary)			
lengths	Normal	Tight tape measure			Tight tape measure	lengths
ℓCG		cG			cG	ℓCG
ℓCF		cF			cF	ℓCF
ℓCE		cE			cE	ℓCE
ℓCD		cD			cD	ℓCD
ℓCC1		cC1			cC1	ℓCC1
		cC			cC	
<b>BOLERO left –</b> Circumference C and lengths ℓ			<b>BOLERO right –</b> Circumference C and lengths ℓ			

Model	Compression CCL 1 2 3	Standard colours	Trend colours*	Quantity
<input type="checkbox"/> mediven esprit <input type="checkbox"/> mediven 550 Arm	Armsleeve <input type="checkbox"/> <input type="checkbox"/> <input type="checkbox"/>	<input type="checkbox"/> Sand <input type="checkbox"/> Cashmere <input type="checkbox"/> Caramel <input type="checkbox"/> Navy <input type="checkbox"/> Black <input type="checkbox"/> Anthracite <input type="checkbox"/> Grey* <input type="checkbox"/> medi Magenta*	<input type="checkbox"/> Lilac <input type="checkbox"/> Sage-green <input type="checkbox"/> Light-blue <input type="checkbox"/> Russet-red	pcs. _____

Variants	Border		
CG	flat oblique	<input type="checkbox"/> steep oblique	<input type="checkbox"/> straight
<b>Accessoires</b> Knitting marks elbow 160° (Standard) <input type="checkbox"/> 150° <input type="checkbox"/> 135° <input type="checkbox"/> Flexure functional zone elbow (only mediven 550)			

\* Delivery period up to 10 working days,

\*\* For all standard and trend colours!

Other accessories		Anti-slip-dots
<b>Position</b> along the oblique border	<b>Topband pieces</b> <input type="checkbox"/> 5 x 2,5 cm <input type="checkbox"/> 5 x 5 cm <input type="checkbox"/> 10 x 5 cm	<input type="checkbox"/> 6 x 4,5 cm
_____ cm porous in armsleeve (distal – standard 2 cm)		
<input type="checkbox"/> Lymphpad <input type="checkbox"/> Lining <input type="checkbox"/> Pocket (Please specify / draw position exactly!)		
<b>Patterns**</b> <input type="checkbox"/> Design-Elements <input type="checkbox"/> Fashion-Elements <input type="checkbox"/> Nature <input type="checkbox"/> Stripes <input type="checkbox"/> Bloom <input type="checkbox"/> Wild		

Special requests



0098L76/072024

Commercial products

Service



# Tools for retailers and medical professionals



## Fan overview flat knit arm / leg

Art. No. 0025303

Fan overview for mediven flat knit arm and leg with all available colours and patterns



## Broschure (fan) flat knit News


only available as PDF file

Information about the mediven flat knit product variants



## medi StyleBook

Art. No. 0025068

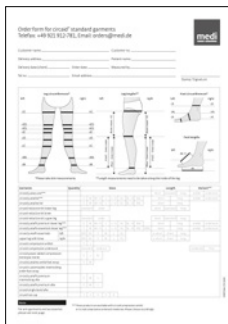


## Ordering form circular standard garments leg

only available as PDF file




## Ordering form circular standard garments arm

only available as PDF file

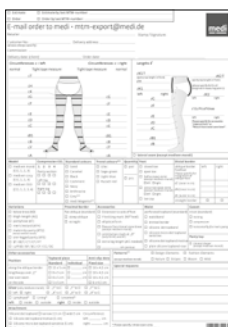


## Ordering form mediven flat knit leg – made-to-measure Flat knit

only available as PDF file  
Art. No. 0099L09

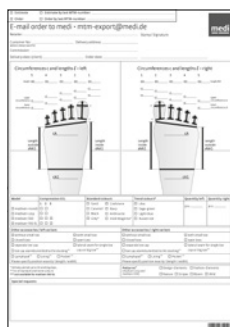


## Ordering form mediven flat knit toe – made-to-measure Flat knit

only available as PDF file  
Art. No. 0098L11

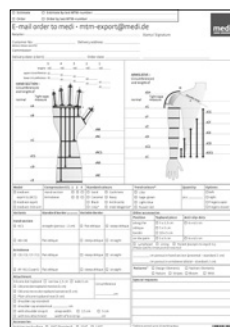

## Ordering form mediven flat knit arm – made-to-measure Flat knit

only available as PDF file  
Art. No. 0098L10

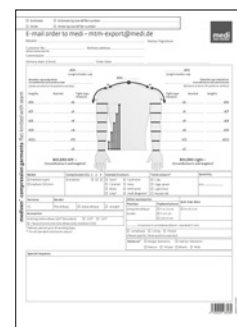

## Ordering form mediven flat knit bolero – made-to-measure Flat knit

only available as PDF file  
Art.No. 0097L76

# Education for patients

Detailed information all about lymphoedema and compression.



## **My guide to lipoedema or lymphoedema**

Art. No. 0099E90

Information about lymphoedema and lipoedema, disease pattern, therapy, prevention etc.

Ideal for first customer consultations and events.



## **Brochure "Guide Oedema"**

Art. No. 00998E6

Detailed information about lip- and lymphoedema. Ideal for first customer consultations and events.

# Intended purposes

## **circaid® leg garments**

---

The compression system is designed to provide compression to the leg for patients with venous and lymphatic disorders.

## **circaid® foot options**

---

The compression system is designed to provide compression to the foot and ankle for patients with venous and lymphatic disorders.

## **circaid® toe cap**

---

The compression system is designed to provide compression to the toes for patients with venous and lymphatic disorders.

## **circaid® supporting bands**

---

Supporting band for application in combination with another circaid compression garment.

## **circaid® glove**

---

The compression system is designed to provide compression to the fingers for patients with venous or lymphatic disorders.

## **circaid® arm garments**

---

The compression system is designed to provide compression to the arm for patients with venous and lymphatic disorders.

## **circaid® hand options**

---

The compression system is designed to provide compression to the hand for patients with venous and lymphatic disorders.

## **mediven® 550 arm, mediven esprit®, mediven esprit® SL**

---

Flat-knitted medical compression garment used for compression of the upper extremities, mainly for the treatment of disorders of the lymphatic system.

## **mediven® 550 leg, mediven® 550 toe cap, mediven® 550 SL toe cap, mediven® cosy, mediven® cosy toe cap, mediven mondi®, mediven mondi® toe cap**

---

Flat-knitted medical compression garment used for compression of the lower extremities, mainly for the treatment of disorders of the lymphatic system.

## **mediven® harmony**

---

Round-knitted medical compression garment used for compression of the upper extremities, mainly for the treatment of disorders of the venous or lymphatic systems.

## **mediven® harmony SL**

---

Flat-knitted medical compression garment used for compression of the upper extremities, mainly for the treatment of disorders of the venous or lymphatic systems.

## **medi Butler**

---

Donning aids for medical/clinical compression socks and stockings. Donning aid for medical/clinical compression sleeves.

## **medi 2in1**


---

Donning and doffing aids for medical/clinical compression socks and stockings.



# medi Therapy Concept Lymphology

Effective lymphoedema treatment.

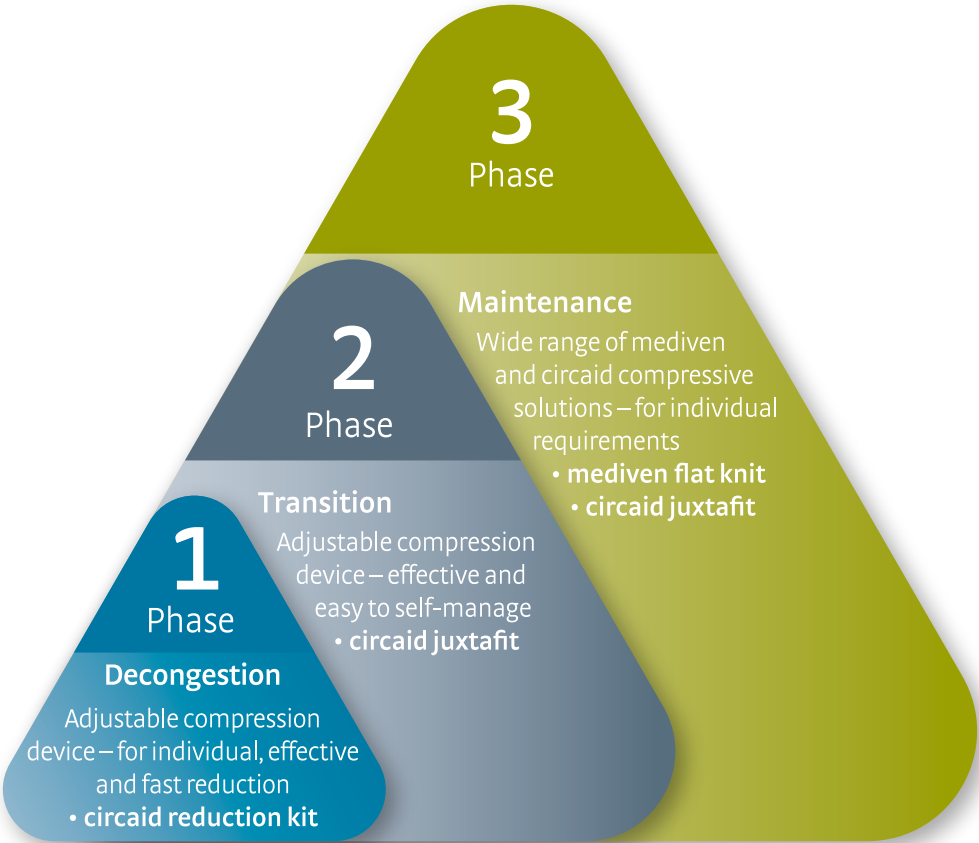

Adjustable and patient-individual compression for initial oedema reduction.

- circaid® reduction kit



Effective compression to optimise reduction results.


- circaid® juxtafit®



A huge variety of compression solutions to maintain therapy results in the long term.

- mediven® flat knit
- circaid® juxtafit®

# Guideline-compliant<sup>1</sup> and patient-individual\* treatment of lymphoedema

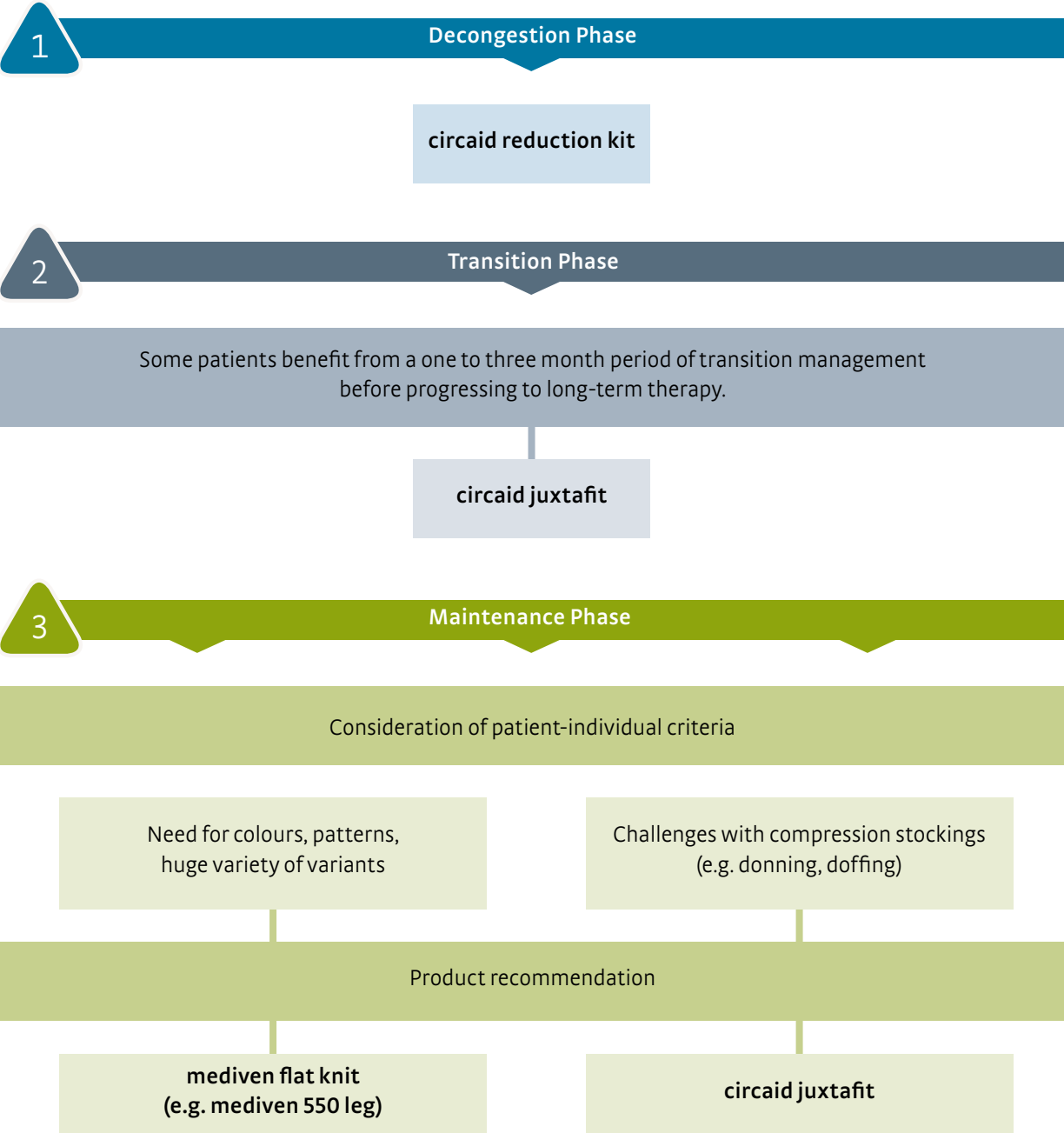

<sup>1</sup> Lymphoedema Framework. Best Practice for the Management of Lymphoedema. International consensus. London: MEP Ltd, 2006.

\* Please consider all further contraindications of compression therapy and further patient-individual criteria.

medi GmbH & Co. KG  
Medicusstraße 1  
95448 Bayreuth  
Germany  
T +49 921 912-0  
F +49 921 912-781  
export@medi.de  
www.medi.de/en

medi Australia Pty Ltd  
Unit 4/13-21 Hallmark Street  
Pendle Hill NSW 2145  
Australia  
T +61 2 9890 8696  
F +61 2 9890 8439  
sales@medi.au  
www.mediaustralia.com.au

medi Austria GmbH  
Adamgasse 16 / 7  
6020 Innsbruck  
Austria  
T +43 512 57 95 15  
F +43 512 57 95 15 45  
vertrieb@medi-austria.at  
www.medi-austria.at

medi Belgium NV  
Staatsbaan 77/0099  
3945 Ham  
Belgium  
T +32 11 24 25 60  
F +32 11 24 25 64  
info@medibelgium.be  
www.medibelgium.be

medi Canada Inc / médi Canada Inc  
104-1375 Lionel-Boulet,  
Varenes, Québec,  
QC Canada J3X 1P7  
T +1 450-583-3317 / +1 800 361 3153  
F +1 888-583-6827  
service@medicanada.ca  
www.medicana.ca

medi Danmark ApS  
Vejleåvej 66  
2635 Ishøj  
Denmark  
T +45 70 25 56 10  
kundeservice@sw.dk  
www.medit danmark.dk

medi Bayreuth Espana SL  
Carrer Primer de Maig 21  
Hospitalet de Llobregat  
08908 Barcelona  
Spain  
T +34-932 60 04 00  
F +34-932 60 23 14  
medi@mediespana.com  
www.mediespana.com

medi France  
Z. I. Charles de Gaulle  
25, rue Henri Farman  
93297 Tremblay en France Cedex  
France  
T +33 1 48 61 76 10  
F +33 1 49 63 33 05  
infos@medi-france.com  
www.medi-france.com

medi Hungary Kft.  
Bokor u. 21.  
1037 Budapest  
Hungary  
T +36 1 371-0090  
F +36 1 371-0091  
info@medi.hu  
www.medi.hu

medi Italia s. r. l.  
Via Giuseppe Ghedini, 2  
40069 Zola Predosa (BO)  
Italy  
T +39 051 6 13 24 84  
F +39 051 6 13 29 56  
info@medi-italia.it  
www.medi-italia.it

medi Japan K.K.  
5-1 Nihonbashi Kabutocho Chuoku  
103-0026 Tokyo  
Japan  
T +81 3 6778 2590  
F +81 3 5847 7901  
info@medi-japan.jp  
www.medi-japan.co.jp

medi Nederland BV  
Heusing 5  
4817 ZB Breda  
The Netherlands  
T +31 76 57 22 555  
F +31 76 57 22 565  
info@medi.nl  
www.medi.nl

medi Norway AS  
Vestheimvegen 35  
4250 Kopervik  
Norway  
T +47 52 84 45 00  
F +47 52 84 45 09  
post@medinorway.no  
www.medinorway.no

medi Polska Sp. z o. o.  
ul. Łabędzka 22  
44-121 Gliwice  
Poland  
T +48 32 230 60 21  
F +48 32 202 87 56  
info@medi-polska.pl  
www.medi-polska.pl

medi Bayreuth Unipessoal, Lda  
Rua do Centro Cultural, no. 43  
1700-106 Lisbon  
Portugal  
T +351 21 843 71 60  
F +351 21 847 08 33  
medi.portugal@medibayreuth.pt  
www.medi.pt

medi RUS LLC  
Business Center NEO GEO  
Butlerova Street 17  
117342 Moscow  
Russia  
T +7 495 374 04 56  
F +7 495 374 04 56  
info@medirus.ru  
www.medirus.ru

medi Sweden AB  
Box 6034  
19206 Sollentuna  
Sweden  
T +46 8 969798  
F +46 8 6266870  
info@medi.se  
www.medi.se

medi UK Ltd.  
Plough Lane  
Hereford HR4 OEL  
Great Britain  
T +44 1432 37 35 00  
F +44 1432 37 35 10  
enquiries@mediuk.co.uk  
www.mediuk.co.uk

medi Ukraine LLC  
Evghena Sverstiuka str, 11  
Kiev 02002  
Ukraine  
T +380 44 591 11 63  
F +380 44 392 73 73  
info@medi.ua  
http://medi.ua

medi USA L.P.  
6481 Franz Warner Parkway  
Whitsett, N. C. 27377-3000  
USA  
T +1 336 4 49 44 40  
F +1 888 5 70 45 54  
info@mediusa.com  
www.mediusa.com

medi Trading (Shanghai) Co. Ltd.  
Room 613, HnR Group Building  
No.1178, Tian Yao Qiao Road  
Xuhui District, Shanghai, 200232  
People's Republic of China  
T +86-21 50582318  
infocn@medi.cn  
www.medi.cn

Medi Turk Ortopedi Medikal İc ve Dis Tic. AS  
Mustafa Kemal Mah. 2156 Sokak No: 16 / 4 – 5  
06510 Çankaya Ankara  
Turkey  
T +90 312 435 20 26  
F +90 312 434 17 67  
info@medi-turk.com  
www.medi-turk.com

MAXIS a. s.  
medi group company  
Slezská 2127 / 13  
120 00 Prague 2  
Czech Republic  
T +420 571 633 510  
F +420 571 616 271  
info@maxis-medica.com  
www.maxis-medica.com

Tachezy Sanit Srl.  
Via Druso 48 / a  
39100 Bolzano  
Italy  
T +39 0471 288 068  
F +39 0471 282 773  
info@tachezysanit.com  
www.tachezysanit.com



Headquarter  
medi GmbH & Co. KG  
Medicusstraße 1  
95448 Bayreuth  
Germany  
T + 49 921 912-0  
F + 49 921 912-781  
export@medi.de  
www.medi.de/en



0098050 / 01.2026