medi GmbH & Co. KG  
Medicusstraße 1  
95448 Bayreuth  
Germany  
T +49 921 912-0  
F +49 921 912-57  
medi@medi.de  
www.medi.de

medi Australia Pty Ltd  
83 Fennell Street  
North Parramatta NSW 2151  
Australia  
T +61 2 9890 8696  
F +61 2 9890 8439  
sales@mediaustralia.com.au  
www.mediaustralia.com.au

medi Austria GmbH  
Adamgasse 16/7  
6020 Innsbruck  
Austria  
T +43-512 57 95 15  
F +43-512 57 95 15 45  
vertrieb@medi-austria.at  
www.medi-austria.at

medi Belgium NV  
Staatsbaan 77/0099  
3945 Ham  
Belgium  
T: +32-11 24 25 60  
F: +32-11 24 25 64  
info@medibelgium.be  
www.medibelgium.be

medi Canada Inc / médi Canada Inc  
104-1375 Lionel-Boulet,  
Varennes, Québec,  
QC Canada J3X 1P7  
T: +1 450-583-3317 / +1 800 361 3153  
F: +1 888-583-6827  
service@medicanada.ca  
www.medicanada.ca

MAXIS a.s.,  
medi group company  
Slezská 2127/13  
120 00 Prague 2  
Czech Republic  
T: +420 571 633 510  
F: +420 571 616 271  
info@maxis-medica.com  
www.maxis-medica.com

UK Responsible Person  
medi UK Ltd.  
Plough Lane  
Hereford HR4 OEL  
Great Britain  
T +44 1432 37 35 00  
F +44 1432 37 35 10  
enquiries@mediuk.co.uk  
www.mediuk.co.uk

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# **Instructions for use**

### **Intended purpose**

Circular-knit compression stocking system used for compression  of the lower extremities, primarily in the treatment of disorders of the venous system.

### **Indications**

- Therapy for venous ulcers (CEAP classification C6)
- Therapy for mixed ulcers (due to arterial and venous issues), taking the contraindications into account
- Prevention of recurrence of venous ulcer (CEAP classification C5)

### **Contraindications**

- Advanced peripheral arterial occlusive disease (if one of these parameters is present: ABPI < 0.5, ankle arterial pressure < 60 mmHg, toe pressure < 30 mmHg or TcPO<sub>2</sub> < 20 mmHg on dorsum of foot)
- Decompensated heart failure (NYHA III + IV)
- Septic phlebitis
- Phlegmasia cerulea dolens

### **Particular attention to**

- Severe weeping dermatosis
- Intolerance to compressive material
- Severe paraesthesia in the limbs
- Advanced peripheral neuropathy (e.g. as seen in diabetes mellitus)
- Primary chronic polyarthritis

### **Patient target group**

Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device, provided that they have received briefing on this from healthcare professionals. Healthcare professionals

should provide care to adults and children, applying the available information on the measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their own responsibility.

### Appropriate usage

Taking measurements correctly is a prerequisite for enabling the system to do its job optimally. Here, the size of the ankle (cB) dictates the size to be selected. Measure below the knee (cD) to confirm the selected size.

Contact with chemical substances (e.g. benzenes, methanols, salicylic acids in high concentrations) may impair the compressive function of the product. Oils and ointments should not be applied to the limb prior to application of the garment.

When a wound is present, always cover with appropriate dressing prior to application of compression. The skin should be inspected on a daily basis.

Only one stocking should be worn if you are immobile for long periods of time.

In order to ensure safe use, the under-stocking mediven ulcer should be washed at least every 2-3 days.

### Content

2 mediven ulcer  
1 mediven ulcer plus  
1 donning aid for mediven ulcer plus open toe

### Storage

Keep this product out of direct sunlight  
Store in a dry place

### Service life / life span

A precisely graduated compression pattern is required to ensure that these stockings work optimally. Daily wearing and washing of the stockings can cause the compression and elasticity of your stockings, which are required to make them medically effective, to deteriorate over time. The stockings should therefore only be worn for a maximum of six months before being replaced. Observe the instructions for putting on the stockings and instructions for care to ensure that the compression pressure is maintained as well as possible.

### Disposal

Please dispose of your medical compression stockings via your general waste or household waste. There are no specific criteria for disposing of the stockings.

### Care instructions

This compressive product can be washed on a delicate cycle and dried in a tumble dryer set to a low heat.

- Delicate wash cycle (mediven ulcer plus)
- Hot cycle up to 95°C (mediven ulcer)
- Do not bleach
- Dry in the tumble dryer at a low heat
- Do not iron
- Do not dry-clean

### Wearing recommendation

Unless otherwise specified by the doctor, wear your mediven compression stocking from morning to night for optimal therapeutic results. The under-stocking mediven ulcer can also be worn at night or when you are at rest.

### Donning without Butler

#### Putting on mediven ulcer without

- Gather the stocking or turn the stocking inside out up to the heel.
- First, pull the foot section over the foot until the size marking is positioned over the heel and the dorsum of the foot.
- Now pull the stocking over your ankle and calf until it is under the knee, at which point you can pull the stocking to the desired length. Do not twist the stocking while doing this and ensure that there are no wrinkles in it.
- Then check that it fits correctly and make sure that it is well-positioned and wrinkle-free. Ensure that the colour mark is sitting over the middle of the heel.

#### Putting on the mediven ulcer plus without the medi Butler

- Gather the stocking or turn the stocking inside out up to the heel
- Pull the foot section over the foot until the size marking is positioned over the heel and the dorsum of the foot.
- Now pull the stocking over your ankle and calf until it is under the knee, at which point you can pull the stocking to the desired length. Do not twist the stocking while doing this and ensure that there are no wrinkles in it.

**Ensure that the bands of both stockings are sitting on top of each other and that there are no wrinkles in the toes.**

### Donning with Butler

#### Putting on the mediven ulcer with the medi Butler with long handles

- Roll the mediven ulcer over the step-in frame of the medi Butler until the colour mark appears over the edge of the frame.
- Step into the stocking, placing your foot through the stocking and onto the floor until you are standing on the floor.
- Now pull the stocking up to the middle of your calf using the handles. Tilt the medi Butler backwards to remove it.

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### ارتداء mediven ulcer دون medi Butler

- Pull the stocking up evenly until it sits below the knee. You can then pull it up further if you wish.
- Then check that it fits correctly and make sure that it is well-positioned and wrinkle-free. Ensure that the colour mark is sitting over the middle of the heel.

### Putting on the mediven ulcer plus with the medi Butler

- Roll the mediven ulcer over the step-in frame of the medi Butler until the colour mark appears over the edge of the frame.
- Step into the stocking, placing your foot through the stocking and onto the floor until you are standing on the floor.
- Now pull the stocking up to the middle of your calf using the handles. Tilt the medi Butler backwards to remove it.
- Pull the stocking up evenly until it sits below the knee. You can then pull it up further if you wish.
- Now pull the stocking up until it sits under the knee. You can then pull it up further if you wish.

Ensure that the bands of both stockings are sitting on top of each other and that there are no wrinkles in the toes.

## Complaints

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).

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## Important notes

This medical device **MD** is made only for multiple uses on a single patient. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If you experience excessive pain or any discomfort whilst wearing, remove the product and immediately contact your doctor or medical supply store. Only wear the product on unbroken skin or over injured skin that has been medically treated, not directly on injured or damaged skin, and only after previous consultation with a medical professional.

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