**medi**

# lipomed

Kompressionsbekleidung  
compression garments

Gebrauchsanweisung. Instructions for use. Istruzioni per l'uso. 使用说明.  
Bruksanvisning. Инструкция по использованию. Mode d'emploi.  
Gebruiksaanwijzing. Instrucciones de uso. Navodila za uporabo.  
دليل الاستخدام. Manual de instruções. Brugsvejledning. Інструкція з  
використання. Instrucțiuni de utilizare. Instrukcja zakładania.  
Naudojimo instrukcija.

**medi. I feel better.**

## Inhalt / Content

Deutsch / German	3–4
Englisch / English	4–5
Italiano / Italian	5–6
中文 / Chinese	6–7
Norsk / Norwegian	7–8
Русский / Russian	9–10
Français / French	10–11
Nederlands / Dutch	11–12
Español / Spanish	12–13
Slovenščina / Slovenian	14–15
Svenska / Swedish	15–16
عربي / Arabic	16–17
Português / Portuguese	18–19
Lietuvių / Lithuanian	19–20
Dansk / Danish	20–21
Українська / Ukrainian	21–22
Română / Romanian	22–23
Polski / Polish	24–25

# lipomed

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## Instructions for Use

### Intended purpose

Post-surgical garment for compression of tissue of abdomen and leg.

### Product features

The lipomed clothing exerts even pressure on the tissue over the entire area. This compressive clothing supports the retraction of stretched skin and simultaneously reduces build-up of oedema and haematoma.

### Indications

All indications where compression in the area of abdomen and leg is necessary, such as:

- After liposuction
- After liposculpture
- After abdominoplasty

### Contraindications

- Arterial circulatory disorders
- Patient is at high risk of thrombosis
- Breathing difficulties

- Intolerance to material

## Risks and side effects

If swelling in the genital area or paraesthesia are observed while wearing the compressive clothing, please contact your doctor immediately. You should only wear this product for long periods of time if recommended by your physician/Health Care Professional.

## Intended users and patient target groups

Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device, provided that the persons supporting carer has received a briefing on this from healthcare professionals. Healthcare professionals should provide care to adults and children, applying the available information on measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their own responsibility.

## Instructions for fitting and wear

The doctor or trained specialist must select the correct size of the lipomed item of clothing based on the patient's measurements. Lipomed products are designed for unique use by one patient (lasting 4 - 6 weeks) following an operation (single patient use). If the product is used in multiple instances or if it is modified, then its medical safety is no longer guaranteed. Changes to the design, cut and standard model can result in impairments in the therapeutic medical efficacy of the product and may put the patient at risk. Changes of any kind executed by third parties shall render the EU declaration of conformity (CE marking) required by law for medical devices which is issued by the manufacturer null and void. Contact with chemical substances (e.g. benzenes, methanols, salicylic acids in high concentrations) may impair the compressive function of the products. Oils and ointments should not be applied to when compression garments are used.

## Length of time that compression clothing should be worn and used for

The doctor will stipulate the period of time that the device needs to be worn for depending on the type and extent of the surgery. Generally, the device must be worn for around four weeks. To elicit optimal results, the lipomed clothing should be worn day and night.

## In combination with other products

In order to minimise the potential risk of thrombosis, the mediven struva 23 knee bandage stocking or compression stockings from mediven should be implemented as prophylactic care.

---

## Instructions for care

The lipomed compression wear can be washed in a normal washing machine, using a mild, non-bleaching detergent. The wash temperature should not exceed 40°C. Do not use fabric conditioner. Wash your lipomed compression clothing separately or with clothing that is similar in colour.

**Note:** Before every wash, close any hook-and-loop fasteners in order to preserve their functionality and to prevent damage to other items of clothing. Air dry, but not in direct sunlight or on heating elements. Excessively high temperatures may damage the elasticity of the material and thus the medical efficacy of the lipomed clothing. In order to prevent longer interruptions in compression therapy, it is recommended that the patient procures two identical items of the lipomed clothing in the correct size so that they can change between the two.

## Storage

The products must be stored in a dry place and out of direct sunlight.

## Material

The hypoallergenic material has excellent active breathability and moisture wicking properties.

## Composition:

55% polyamide  
45% elastane

## Disposal

Please dispose of your compression wear via your general waste or household waste. There are no specific criteria for disposing of the compression wear.

## Complaints

In the event of any complaints regarding the product such as damage to the knitted fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).

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## Important notes

This medical device **[MD]** is made only for multiple uses on a single patient . If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If you experience excessive pain or any discomfort whilst wearing, remove the product and immediately contact your doctor or medical supply store. Only wear the product on unbroken skin or over injured skin that has been medically treated, not directly on injured or damaged skin, and only after previous consultation with a medical professional.

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Tachezy Sanit Srl.