medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-57
medi@medi.de
www.medi.de

medi Australia Pty Ltd
83 Fennell Street
North Parramatta NSW 2151
Australia
T +61 2 9890 8696
F +61 2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43-512 57 95 15
F +43-512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Belgium bvba
Posthoornstraat 13/1
3582 Koersel
Belgium
T: +32-11 24 25 60
F: +32-11 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Brasil
Rua Itapeva 538 cj. 42
Bela Vista, São Paulo - SP
CEP: 01332-000
Brasil
T +55 (11) 3500 8000
sac@medibrasil.com
www.medibrasil.com

medi Danmark ApS
Vejlegardsvej 59
2665 Vallensbaek Strand
Denmark
T +45 70 25 56 10
F +45 70 25 56 20
kundeservice@sw.dk
www.medit danmark.dk

medi Bayreuth España SL
C/Canigó 2 – 6 bajos
Hospitalet de Llobregat
08901 Barcelona
Spain
T +34 93 260 04 00
F +34 93 260 23 14
medi@mediespana.com
www.mediespana.com

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33 1 48 61 76 10
F +33 1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Hungary Kft.
Bokor u. 21.
1037 Budapest
Hungary
T +36 1371 0090
F +36 1371 0091
info@medi.hu
www.medi.hu

medi Italia s.r.l.
Via Giuseppe Ghedini, 2
40069 Zola Predosa (BO)
Italy
T: +39-051 6 13 24 84
F: +39-051 6 13 29 56
info@medi-italia.it
www.medi-italia.it

medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31 76 57 22 555
F +31 76 57 22 565
info@medi.nl
www.medi.nl

medi Norway AS
P.B. 238
4291 Kopervik
Norway
T: +47 52 84 45 00
F: 47 52 84 45 09
post@medinorway.no
www.medinorway.no

medi

medi Butler Off

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medi Butler Off

General Instructions

Please read all of this leaflet carefully before you start.

It contains important information about the purpose, handling, care and disposal of this product.

Keep this leaflet as you might want to read it again later.

If you have further questions, please ask the doctor treating you, your specialist retailer or the medi Consumer Service.

Purpose

Doffing aid for medical compression stockings

Description

Aids for compression therapy include medical compression stockings that are manufactured in different lengths, compression classes and knits and which are used in venous disorders and to treat oedema in lymphoedema and lipoedema, for example*.

The medi Butler Off is an aid for the removal of medical compression hosiery and is classed as an accessory to treatment with medical compression stockings and tights.

The doffing aid medi Butler Off makes it simple to take the compression hosiery off.

The compressive knitted fabric is moved downwards and around the heel by the removal aid.

medi Butler Off enables the wearer of medical compression stockings and tights to remove them easily without great physical effort.

*See mediven Instructions for Use about the indications for medical compression stockings.

Important general information

Take whatever time you need. With this removal aid you don't have to rush, in order to reach your goal

If you find it difficult to bend down, the medi Butler Off will be a big help, no matter what your age. At last you will be able to take your stockings off without assistance from others!

The medi Butler Off is suitable for both left- and right-handed users.

When used properly, it will not damage your compression or ordinary stockings. Please read the section „How to use your medi Butler Off“.

Other uses: the medi Butler Off can also be used as a removal aid for ordinary stockings or as a shoehorn.

Please don't continue to use the product if it shows signs of wear and tear or other damage.

Important information for use

Not recommended for mediven forte or any mediven flat-knit variants.

Storage instructions

Store dry.

Protect against direct sunlight

How to use your medi Butler Off

1. Slide the tip of the horn under the topband of the stocking.
2. Now push the handle gently downwards; the front end of the horn will slide down along the calf towards your heel. The inner surface of the horn should thereby always remain in contact with the skin. The stocking is simultaneously pushed down with the help of the notch on the horn. If the horn loses contact with the skin, you can just start again with the horn a little higher up and then continue the downward movement.
3. Once you reach ankle level, lift your heel up and rest the ball of your foot on a firm surface. Now tilt the handle downwards a little and guide the horn of the doffing aid around your heel and along the underside of your foot.
Caution: Make sure that you don't press the stocking with the doffing aid against the floor as this could damage the stocking or break the horn.
Now gently push the handle forwards; the stocking will slide off your foot, but remains on the horn.
4. Now you can simply take the stocking off the aid, without having to bend down.

Care and washing instructions

The medi Butler Off can be cleaned using normal washing agents or disinfectants if necessary.

Material composition

Disposal

Please dispose of medi Butler Off correctly at the appropriate collection point for plastics and old metal.

There are no particular criteria for disposal as hazardous waste.

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