**medi GmbH & Co. KG**  
Medicusstraße 1  
D-95448 Bayreuth  
Germany  
T +49 921 912-0  
F +49 921 912-57  
info@medi.de  
www.medi.de

medi Australia Pty Ltd  
Unit 4/13-21 Hallmark Street  
Pendle Hill NSW 2145  
Australia  
T +61 2 9890 8696  
F +61 2 9890 8439  
sales@medi.au  
www.mediaustralia.com.au

medi Austria GmbH  
Adamgasse 16/7  
6020 Innsbruck  
Austria  
T +43 512 57 95 15  
F +43 512 57 95 15 45  
vertrieb@medi-austria.at  
www.medi-austria.at

medi Belgium NV  
Staatsbaan 77/0099  
3945 Ham  
Belgium  
T: +32-11 24 25 60  
F: +32-11 24 25 64  
info@medibelgium.be  
www.medibelgium.be

medi Canada Inc / médi Canada Inc  
104-1375 Lionel-Boulet,  
Varenes, Québec,  
QC Canada J3X 1P7  
T +1 450-583-3317 / +1 800 361 3153  
F +1 888-583-6827  
service@medicanada.ca  
www.medicanada.ca

MAXIS a.s.,  
medi group company  
Slezská 2127/13  
120 00 Prague 2  
Czech Republic  
T: +420 571 633 510  
F: +420 571 616 271  
info@maxis-medica.com  
www.maxis-medica.com

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# medi 2in1

## General Instructions

Please read through the entire instructions for use carefully. They contain important information concerning the product itself, as well as its handling, maintenance and disposal.

Keep the instructions for use. You may wish to read them again at a later point in time.

If you have any questions, please contact the medical supply store or the medi Consumer Service.

The medi 2in1 donning and doffing aid can be used with any stocking length (knee, ankle or full-length stocking). The same method is applied for all stocking lengths.

---

Compression therapy aids include, among other items, medical compression stockings which are manufactured in different lengths, compression classes and with different knitting methods. Compression stockings are used for a variety of purposes, including treating venous disorders and conditions\* and for treating oedema caused by lymphoedema and lipoedema.

<sup>1</sup> Donning and doffing aids protect and preserve the stocking material and make it easier to put on and take off medical compression stockings. In particular, they also enable patients with limited mobility to use the products by reducing the amount of motion and force needed to put them on and take them off.

\* For indications for medical compression stockings see medi and/or mediven instructions for use.

## Purpose and use

Accessories : Donning and doffing aids for medical/clinical compression socks and stockings

## Indications for donning and doffing aids include

- Hemiplegia
- Loss of power due to age
- Osteoarthritis/rheumatism
- Morbid obesity
- Extensive spinal/hip/knee stiffness
- Degenerative condition of the hands/related to the hands
- Sequellae from injuries/amputations

<sup>1</sup> S2k guideline: Medical compression therapy of the extremities using medical compression stockings (MCS), venous compression bandage (VCB) and medical adaptive compression systems (MACS). Published online at: <https://www.awmf.org/leitlinien/detail/II/037-005.html>  
Chapter: 2.1.3 Donning and Doffing Aids

## Contraindications

No contraindications are known.

---

### Important general instructions

To prevent injuries and damage to the delicate knitted fabric, please make sure your finger and toenails are well-kempt and the skin on your heels is smooth.

Please dispose of this product as soon as it shows signs of wear or damage.

---

### Storage instructions

Please store the medi 2in1 in a dry place and keep it out of direct sunlight.

---

### Explanation

You have selected a top-class product by choosing the medi 2in1 donning and doffing aid.

The medi 2in1 is specially adapted to meet the needs of all wearers of compression stockings and simplifies two crucial steps in putting on and taking off compression stockings.

- The medi 2in1 makes it easy and simple to bring the compression stocking over the instep and heel when putting it on (Figs. 1 - 7).
- This is also true for taking off the compression stocking (Figs. A-E). The force and mobility needed to perform these tasks are reduced to minimum levels, making the process more comfortable for the user.

### Putting on compression stockings with the medi 2in1

1. Lie the medi 2in1 donning aid flat on the floor. Fold the long end of the donning aid along the dashed line.
2. Place your bare foot on the medi 2in1 donning aid. Your toes should be pointing towards the folded over edge. Please ensure that there is enough fabric around the front of the toes and at the back of the heel (at least 3 cm).
3. Push the material lightly between your toes so that you can hold it with your toes.
4. Pull the stocking on over the instep and heel, working towards your knee.
5. Here, the medi 2in1 donning aid remains in the stocking and pokes out of the back of the calf.
6. Pull your compression clothing up until it is around one handspan under your knee (at the highest) so that you can pull out the medi donning aid easily.

- To do this, grip the section of the medi 2in1 donning aid that is still poking out of your stocking and lift the heel slightly. Now you can easily pull the medi 2in1 donning aid out of your compression sock / compression stocking without exerting much force.

### **Taking off compression stockings with the medi 2in1**

- Lie the medi 2in1 doffing aid flat on the floor.
- Slide your foot into the pocket section, as far as it will go.
- Pull down your compression stocking until it is at the edge of the doffing aid (calf area). Then bring the cuff or the top band of your compression stocking over the medi 2in1.
- Now you can easily pull the stocking off towards the toes, over the heel and instep.
- When you have taken the compression stocking off, the medi 2in1 doffing aid will still be inside the stocking.

### **How do I clean the medi 2in1?**

You can easily wash the medi 2in1 in the washing machine on a delicate cycle at temperatures of up to 40°C.

Please do not clean it chemically and do not use stain remover on it.

Simply leave the medi 2in1 donning and doffing aid to air dry. Please keep it away from sources of heat such as ovens, radiators and tumble dryers.

### **Material composition**

Non-dyed, naturally white polyamide that has been smoothed under pressure

### **Disposal**

Please dispose of your medi 2in1 aid via your general waste or household waste. There are no specific criteria for disposing of the stockings.

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