**medi GmbH & Co. KG**  
**Medicusstraße 1**  
**95448 Bayreuth / Germany**  
**T +49 921 912-0**  
**F +49 921 912-780**  
**ortho@medi.de**  
**www.medi.de/en**

medi Australia Pty Ltd  
 83 Fennell Street  
 North Parramatta NSW 2151  
 Australia  
 T +61-2 9890 8696  
 F +61-2 9890 8439  
 sales@mediaustralia.com.au  
 www.mediaustralia.com.au

medi Austria GmbH  
 Adamgasse 16/7  
 6020 Innsbruck  
 Austria  
 T +43 512 57 95 15  
 F +43 512 57 95 15 45  
 vertrieb@medi-austria.at  
 www.medi-austria.at

medi Belgium NV  
 Staatsbaan 77/0099  
 3945 Ham  
 Belgium  
 T: + 32-11 24 25 60  
 F: +32-11 24 25 64  
 info@medibelgium.be  
 www.medibelgium.be

medi Canada Inc / médi Canada Inc  
 104-1375 Lionel-Boulet,  
 Varennes, Québec,  
 QC Canada J3X 1P7  
 T +1 450-583-3317 / +1 800 361 3153  
 F +1 888-583-6827  
 service@medicanada.ca  
 www.medicanada.ca

MAXIS a.s.,  
 medi group company  
 Slezská 2127/13  
 120 00 Prague 2  
 Czech Republic  
 T: +420 571 633 510  
 F: +420 571 616 271  
 info@maxis-medica.com  
 www.maxis-medica.com

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### Important notes

This medical device **MD** is made only for multiple uses on a single patient. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Only wear the product on unbroken skin or over injured skin that has been medically treated, not directly on injured or damaged skin, and only after previous consultation with a medical professional.

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# protect. CAT Walker

## Intended purpose

The protect.CAT Walker is a lower-leg foot brace for immobilising this region in a specific position.

## Indications

For all indications which require immobilisation of the lower leg and foot in a specified position, such as:

- Following injuries of the achilles tendon (postoperative / nonsurgical)
- For stable foot and joint fractures, fracture postoperative care (nonsurgical)
- Following luxation of the upper ankle joint
- Following ligament-, soft tissue-, or tendon injuries (postoperative / nonsurgical)

## Contraindications

Unstable fractures or fractures of the proximal tibia or fibia

## Risks / Side Effects

If the aids are too tight, this can cause local pressure sores or constriction of blood vessels or nerves. Therefore you should consult the doctor treating you before you use it if the following circumstances apply to you:

- Condition or injury of the skin in the area of application, particularly with infectious indications (excessive heat, swelling or reddening)
- Paraesthesia and circulatory disorders (e.g. as seen in diabetes, varicose veins)
- Lymph drainage disorders – equally, swelling of soft tissue adjacent to the area of application which is not related to a condition

When wearing snug-fitting aids, this can

result in localised skin inflammation or irritation that is due to mechanical irritation of the skin (particularly in conjunction with sweating) or due to the composition of the material.

## Intended users and patient target groups

Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device, provided that they have received briefing on this from healthcare professionals.

Patient target group: Healthcare professionals should provide care to adults and children, applying the available information on the measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their own responsibility.

## Advice

The protect.CAT Walker is not designed for patients who weigh more than 120 kg.

## Fitting instructions

- Open the belts, loosen the front shin from the inner shoe and finally take the inner shoe out of the Walker.
- (Fig.1) Open the leg and foot part of the inner shoe by loosening the hook and loop fasteners. Ask the patient to place his foot and lower leg into the inner shoe. Make sure that the air chambers are empty!
- (Fig.2) Close the inner shoe and make sure that it is firmly fixed to the patient's leg. Align the inner shoe in such a way that the integrated pump is sitting in the middle and so that later it fits into the space provided in the shin.
- (Fig.3) Now ask the patient to place their foot in the half shell of the Walker. The lower leg and foot should

positioned so that they fit exactly into the shell

- (Fig. 4) Check to see that the foot is properly set and positioned in the Walker.  
Make sure that the patient's toes are not pushing against the toe protector. If necessary, the toe protector's position can be adjusted or it can be removed completely.
- (Fig. 5) Now place the loose front shin onto the patient's shinbone and foot. Make sure that the pump and ventilator fit into the spaces provided.
- (Fig. 6) Close the belts to stabilise the system. Start by closing the belt at the front and then close all the other belts one after another. The belts should fit tightly.  
Extra material can be cut away.
- Ask the patient to walk around in the orthosis to make sure that the Walker is sitting properly.
- (Fig. 7) To tighten the Walker's fit, the air cushions can be filled using the pump system. Adding or letting out air can be operated using the valve. The padded pieces can be used as additional padding.

### Care instructions

Hook and loop fasteners should be closed for washing. Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the product by hand, preferably using medi clean washing agent.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

### Storage instructions

Keep the product in a dry place and do not expose to direct sunlight.

### Material Composition

PP, PU, EVA, Polyester

### Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

### Disposal

The product can be disposed of in the domestic waste.

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).

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