**medi GmbH & Co. KG**  
**Medicusstraße 1**  
**95448 Bayreuth**  
**Germany**  
**T +49 921 912-0**  
**F +49 921 912-783**  
**ortho@medi.de**  
**www.medi.de**

medi Australia Pty Ltd  
 83 Fennell Street  
 North Parramatta NSW 2151  
 Australia  
 T +61-2 9890 8696  
 F +61-2 9890 8439  
 sales@mediaustralia.com.au  
 www.mediaustralia.com.au

medi Austria GmbH  
 Adamgasse 16/7  
 6020 Innsbruck  
 Austria  
 T +43 512 57 95 15  
 F +43 512 57 95 15 45  
 vertrieb@medi-austria.at  
 www.medi-austria.at

medi Belgium NV  
 Staatsbaan 77/0099  
 3945 Ham  
 Belgium  
 T: + 32-11 24 25 60  
 F: +32-11 24 25 64  
 info@medibelgium.be  
 www.medibelgium.be

medi Canada Inc / médi Canada Inc  
 104-1375 Lionel-Boulet,  
 Varennes, Québec,  
 QC Canada J3X 1P7  
 T +1 450-583-3317 / +1 800 361 3153  
 F +1 888-583-6827  
 service@medicanada.ca  
 www.medicanada.ca

MAXIS a.s.,  
 medi group company  
 Slezská 2127/13  
 120 00 Prague 2  
 Czech Republic  
 T: +420 571 633 510  
 F: +420 571 616 271  
 info@maxis-medica.com  
 www.maxis-medica.com

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medi Trading (Shanghai) Co. Ltd.  
 Room 1703, T1, CES WEST BUND CENTER,  
 No. 277, Long Lan Road  
 Xuhui District  
 200232 Shanghai  
 People's Republic of China  
 T: +86-21 50582319  
 F: +86-21 50582319

UK Responsible Person  
 medi UK Ltd.  
 Plough Lane  
 Hereford HR4 0EL  
 Great Britain  
 T +44-1432 37 35 00  
 F +44-1432 37 35 10  
 enquiries@mediuk.co.uk  
 www.mediuk.co.uk

## Lumbamed® facet

### LWS-Orthese zur Entlastung der Facettengelenke · Lumbar support for targeted relief of the facet joints

Gebrauchsanweisung. Instructions for use. Mode d'emploi. Instrucciones de uso.  
 Instruções para aplicação. Istruzioni per l'uso. Gebruiksaanwijzing. Brugsvejledning.  
 Bruksanvisning. Návod k použití. Uputa za uporabu. Инструкция по использованию.  
 Kullanma kılavuzu. Instrukcja załadania. Οδηγία εφαρμογής. Használati útmutató.  
 Uputstvo za upotrebu. Інструкція з використання. Navodila za uporabo.  
 دليل الاستخدام. Návod na použitie. 使用说明. הוראות שימוש.  
 Инструкции за употреба. 使用説明書。

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#### Important notes

This medical device **MD** is made only for multiple uses on a single patient. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If you experience excessive pain or any discomfort whilst wearing, remove the product and immediately contact your doctor or medical supply store. Only wear the product on unbroken skin or over injured skin that has been medically treated, not directly on injured or damaged skin, and only after previous consultation with a medical professional.

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# Lumbamed<sup>®</sup> facet

## Intended purpose

Lumbamed facet is a support for partial stress relief  of the lumbar vertebrae via correction of lordosis.

## Indications

All indications requiring partial relief by reduction of lordosis of the lumbar spine such as:

- Facet joint arthrosis
- Protrusion/prolapse of an intervertebral disk
- Lower back pain (lumbago)
- Lower back pain with sciatica
- Lumbar vertebral canal stenosis
- Postural muscle imbalance in the lumbar spine
- Spondylolisthesis
- Spondylosis
- Painful conditions of the lumbosacral junction

## Contraindications

- Open wounds in the area of the orthosis
- Respiratory diseases such as COPD

## Risks / Side Effects

If the aids are too tight, this can cause local pressure sores or constriction of blood vessels or nerves. Therefore you should consult the doctor treating you before you use it if the following circumstances apply to you:

- Condition or injury of the skin in the area of application, particularly with infectious indications (excessive heat, swelling or reddening)
- Paraesthesia and circulatory disorders (e.g. as seen in diabetes, varicose veins)
- Lymph drainage disorders – equally, swelling of soft tissue adjacent to the

area of application which is not related to a condition

When wearing snug-fitting aids, this can result in localised skin inflammation or irritation that is due to mechanical irritation of the skin (particularly in conjunction with sweating) or due to the composition of the material.

## Intended users and patient target groups

Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device, provided that they have received briefing on this from healthcare professionals.

Patient target group: Healthcare professionals should provide care to adults and children, applying the available information on the measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their own responsibility.

## Note on wearing

Basically, the orthotic can be worn all day long. The special cut in the groin area (Comfort Zone) ensures that the fit of the support is comfortable and just right. However, the length of time you wear it should depend on your needs.

## Fitting instructions

- Before putting on the orthotic, please loosen the straps, so that it can be put on without pulling. To do so, unfasten the hook and loop fastener ends of the straps and loosen these by placing them further back on the velour on the belt (Fig. 1).
- The label on the right-hand inner fastener has information on what way to put on the orthotic (Fig. 2).
- Now, slide your left hand into the inner hand strap on the left-hand fastener

side (Fig. 3).

- Wrap the support around your body and check once again on the label that you have the orthotic the right way around (Fig. 4).
- Now, slide your right hand into the outer hand strap on the right-hand fastener side (Fig. 5).
- Pull both fastener sides together in the middle, so that they can be fixed over each other as far at the same distance. When fastening, ensure that the back brace is in the middle (Fig. 6 + 7). The orthotic is on properly when the lower fastener edge is just above the pubic bone (see Fig. 8).
- Now unfasten the strap ends of both lower pull straps and pull these forward at the same time. You can adjust the tightness to suit your own requirements and manage your pain levels (Fig. 8 + 9).
- Do the same for the upper pull straps (Fig. 10 + 11).
- Information for technicians: The strap length may be adjusted by cutting it at the edge of the Y fastener (Fig. 12).

### Care instructions

Hook and loop fasteners should be closed for washing. We recommend using a laundry bag. Fabric softeners, fats, oils, lotions, ointments and soap residues can affect the material, hinder the Clima Comfort effect and cause skin irritations.

- You can wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

### Storage instructions

Keep the product in a dry place and do not expose to direct sunlight.

### Material composition

Polyamide, Polyester, Elastane

### Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

### Disposal

The product can be disposed of in the domestic waste.

### Your medi team

wishes you a speedy recovery!

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).

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