medi GmbH & Co. KG
Medicusstraße 1
95448 Bayreuth
Germany
T +49 921 912-0
www.medi.de

medi Australia Pty Ltd
83 Fennell Street
North Parramatta NSW 2151
Australia
T +61-2 9890 8696
F +61-2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Belgium NV
Staatsbaan 77/0099
3945 Ham
Belgium
T: +32-11 24 25 60
F: +32-11 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Canada Inc / médi Canada Inc
104-1375 Lionel-Boulet,
Varenes, Québec,
QC Canada J3X 1P7
T +1 450-583-3317 / +1 800 361 3153
F +1 888-583-6827
service@medicanada.ca
www.medicanada.ca

MAXIS a.s.,
medi group company
Slezská 2127/13
120 00 Prague 2
Czech Republic
T: +420 571 633 510
F: +420 571 616 271
info@maxis-medica.com
www.maxis-medica.com

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Important notes

This medical device **MD** is made only for multiple uses on a single patient. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If you experience excessive pain or any discomfort whilst wearing, remove the product and immediately contact your doctor or medical supply store. Only wear the product on unbroken skin or over injured skin that has been medically treated, not directly on injured or damaged skin, and only after previous consultation with a medical professional.

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Polyurethan, Nylon, Elastan, Polyethylen

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circaid® juxtafit® premium foot options

Intended purpose

The compression system is designed to provide compression to the foot for patients with venous and lymphatic disorders.

Indications

- Lymphedema
- Other forms of edema
- Lipedema
- CVI (C3-C6)
- Varicose veins (C3)
- DVT/Thrombosis Prevention
- Acute deep vein thrombosis
- Post Thrombotic Syndrome
- Venous Stasis Ulcer (C6)
- Post Sclerotherapy
- Lipodermatosclerosis (C4b)

Contraindication

- Severe Peripheral Arterial Disease
- Uncontrolled Congestive Heart Failure
- Septic Phlebitis
- Phlegmasia Cerulea Dolens
- Untreated infection
- Any circumstance where increased venous and lymphatic return is undesirable

Risks / Side Effects

- Mild to Moderate Peripheral Arterial Disease
- Impaired Sensation - must be able to sense applied pressure
- Intolerance to material(s)
- Pediatric & assisted use - must be able to sense & communicate applied pressure

Intended users and patient target groups

Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device, provided that they have received briefing on this from healthcare professionals.

Patient target group: Healthcare professionals should provide care to adults and children, applying the available information on the measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their own responsibility.

Wearing recommendations

Slightly loosen the bands of the garment for night-time wear. If

experiencing any pain, or discomfort, reduce the compression (mmHg) applied or remove device. Inform your clinician of your action. When bands are placed correctly they should slightly overlap, leaving no space between bands. Do not tuck bands. The Velcro tabs should never touch the skin!

Donning the circaid juxtafit premium interlocking ankle foot wrap

- Apply a circaid sock first. Ensure that there are no wrinkles in the sock.
- Slide the foot into the circaid juxtafit interlocking ankle foot wrap. Unroll the two bands closest to the toes and secure them to the opposing side with firm and comfortable compression. While attaching the second band, pull the ankle-band into the opposing direction.
- Ensure that the foot is properly aligned within the foot wrap. The front edge of the foot wrap should align with the base of the toes. Adjust the interlocking heel-bands to fit snugly around the back of the heel and attach them to the landing pads with firm and comfortable compression.
- Detach the ankle-band and pull it around the ankle with firm and comfortable compression. Attach it to the landing pad. Ensure that there are no gaps. Adjust the ankle-band if any gaps or pressure points occur. The ankle-band should overlap with the circaid leg garment.

Note: *The ankle-band may pull from a different side depending on which foot, left or right, the garment is applied to.*

- Readjust the bands as necessary throughout the day to maintain a firm and comfortable compression level.

Donning the circaid juxtafit premium ankle foot wrap

- Apply a circaid sock first. Ensure that there are no wrinkles in the sock.
- Unroll the band closest to the toes and secure it to the opposing side to create a kind of slipper. Slide the foot into the slipper shape and firmly press the heel into the heel cup.
- The front edge of the foot options should align with the base of the toes.
- Tighten the band closest to the toes to firm and comfortable compression.
- While pulling the ankle band in the opposing direction, attach the second band on the opposing landing

pad with firm and comfortable compression. If necessary, adjust the band closest to the toes to firm and comfortable compression. Adjust the ankle band if any gaps or pressure points occur. The ankle-band should overlap with the circaid leg garment.

Note: *The ankle band may pull from a different side depending on which foot, left or right, the garment is applied to.*

- Readjust the bands as necessary throughout the day to maintain a firm and comfortable compression level.

Individualisation

This product is not designed to be cut or modified except as outlined in the directions for use. The circaid juxtafit premium ankle foot wrap can be tailored to fit the patient's foot size and contour. The ankle foot wrap should be fitted by a medical professional. Doing other than instructed will void the three month limited warranty. The following steps will permanently reduce the foot-length of the circaid juxtafit premium ankle foot wrap. The afw can be trimmed by up to 1.25 in. (3 cm) (see notches on both sides).

Foot-length adjustment of the circaid juxtafit premium ankle foot wrap

- With the beige side facing up, mark the desired length.
- Draw a straight line connecting the notches.
- Carefully cut along the drawn line, assuring that the reinforced seam section of the Velcro material is not cut.

Care instructions

The garment can be machine washed using a gentle cycle and dried on a low heat setting. We recommend using a laundry bag. To extend the life of the garment, hand wash and drip dry.

-
- | | |
|--|----------------------|
|  | Delicates wash cycle |
|  | Do not bleach |
|  | Tumble dry, low heat |
|  | Do not iron |
|  | Do not dry clean |
|  | Do not wring |
-

Storage instructions

Keep the product in a dry place and do not expose to direct sunlight.

Lifespan

3 months - due to material wear-and-tear the medical efficacy can only be expected for a defined lifespan. This assumes correct handling of the garment (e.g. proper care, application, and removal).

Material composition

Polyurethane, Nylon, Elastane, Polyethylene

Disposal

The product can be disposed of in the domestic waste.

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).

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