**medi GmbH & Co. KG**  
**Medicusstraße 1**  
**D-95448 Bayreuth**  
**Germany**  
**T +49 921 912-0**  
**www.medi.de**

medi Australia Pty Ltd  
 83 Fennell Street  
 North Parramatta NSW 2151  
 Australia  
 T +61 2 9890 8696  
 F +61 2 9890 8439  
 sales@mediaustralia.com.au  
 www.mediaustralia.com.au

medi Austria GmbH  
 Adamgasse 16/7  
 6020 Innsbruck  
 Austria  
 T +43-512 57 95 15  
 F +43-512 57 95 15 45  
 vertrieb@medi-austria.at  
 www.medi-austria.at

medi Belgium NV  
 Posthoornstraat 13/1  
 3582 Koersel  
 Belgium  
 T: +32-11 24 25 60  
 F: +32-11 24 25 64  
 info@medibelgium.be  
 www.medibelgium.be

medi Brasil  
 Rua Cristovan de Vita, n.º 260, Un. 30,  
 Centro Logístico Raposo Tavares,  
 Vila Camargo  
 Vargem Grande Paulista - SP  
 CEP 06730-000  
 Brasil  
 T: +55-11-3500 8005  
 sac@medibrasil.com  
 www.medibrasil.com

medi Canada Inc / médi Canada Inc  
 597, Rue Duvernay, Verchères  
 QC Canada J0L 2R0  
 T: +1 450-583-3317 / +1 800 361 3153  
 F: +1 888-583-6827  
 service@medicanada.ca  
 www.medicanada.ca

MAXIS a.s.,  
 medi group company  
 Slezská 2127/13  
 120 00 Prague 2  
 Czech Republic  
 T: +420 571 633 510  
 F: +420 571 616 271  
 info@maxis-medica.com  
 www.maxis-medica.com

medi Danmark ApS  
 Vejlegardsvej 59  
 2665 Vallensbaek Strand  
 Denmark  
 T +45 70 25 56 10  
 F +45 70 25 56 20  
 kundeservice@sw.dk  
 www.meditandmark.dk

**medi**

## circaid® single band ankle foot wrap

Instructions for use. Gebrauchsanweisung.  
 Instrucciones de uso. Instruções para aplicação.  
 Инструкция по использованию.

medi Bayreuth España SL  
 C/Canigó 2 – 6 bajos  
 Hospitalet de Llobregat  
 08901 Barcelona  
 Spain  
 T +34 93 260 04 00  
 F +34 93 260 23 14  
 medi@mediespana.com  
 www.mediespana.com

medi France  
 Z.I. Charles de Gaulle  
 25, rue Henri Farman  
 93297 Tremblay en France Cedex  
 France  
 T +33 1 48 61 76 10  
 F +33 1 49 63 33 05  
 infos@medi-france.com  
 www.medi-france.com

medi Hungary Kft.  
 Bokor u. 21.  
 1037 Budapest  
 Hungary  
 T +36 1371 0090  
 F +36 1371 0091  
 info@medi.hu  
 www.medi.hu

medi Italia s.r.l.  
 Via Giuseppe Ghedini, 2  
 40069 Zola Predosa (BO)  
 Italy  
 T: +39-051 6 13 24 84  
 F: +39-051 6 13 29 56  
 info@medi-italia.it  
 www.medi-italia.it

medi Japan K.K.  
 5-1 Nihonbashi Kabutocho Chuoku  
 103-0026 Tokyo  
 Japan  
 T: +81 3 6778 2590  
 F: +81 3 5847 7901  
 info@medi-japan.jp  
 www.medi-japan.co.jp

Tachezy Sanit Srl.  
 Via Druso 48/a  
 39100 Bolzano  
 Italy  
 T: +39 0471 288 068  
 F: +39 0471 282 773  
 info@tachezysanit.com  
 www.tachezysanit.com

medi Nederland BV  
 Heusing 5  
 4817 ZB Breda  
 The Netherlands  
 T +31 76 57 22 555  
 F +31 76 57 22 565  
 info@medi.nl  
 www.medi.nl

medi Norway AS  
 Vestheimvegen 35  
 4250 Kopervik  
 Norway  
 T: +47 52 84 45 00  
 F: +47 52 84 45 09  
 post@medinorway.no  
 www.medinorway.no

E012037/05.2020

4 049772 365647

medi. I feel better.

English

## circaid® single band ankle foot wrap

### Purpose

The compression system is designed to provide compression to the foot and ankle region for patients with venous and lymphatic disorders.

### Indications

- Lymphedema
- other forms of edema
- Lipedema
- Chronic venous insufficiency
- Varicose veins
- DVT/Thrombosis Prevention
- Post Thrombotic Syndrome
- Venous Stasis Ulcer
- Post Sclerotherapy
- Lipodermatosclerosis

### Contraindications

- Severe Peripheral Arterial Disease

- Uncontrolled Congestive Heart Failure
- Septic Phlebitis
- Phlegmasia Cerulea Dolens
- Untreated infection
- Any circumstance where increased venous and lymphatic return is undesirable
- Suspected or known untreated acute DVT

### Cautions

- Mild to Moderate Peripheral Arterial Disease
- Impaired Sensation - must be able to sense applied pressure
- Intolerance to material(s)
- Pediatric & assisted use - must be able to sense & communicate applied pressure

### This Package contains

- 1 circaid single band afw
- 1 booklet instructions for use

medi Polska Sp. z o.o.  
ul. Łąbędzka 22  
44-121 Gliwice  
Poland  
T: +48-32 230 60 21  
F: +48-32 202 87 56  
info@medi-polska.pl  
www.medi-polska.pl

medi Bayreuth Unipessoal Lda.  
Rua do Centro Cultural, no. 43  
1700-106 Lisbon  
Portugal  
T +351 21 843 71 60  
F +351 21 847 08 33  
medi.portugal@medibayreuth.pt  
www.medi.pt

medi RUS LLC  
Business Center NEO GEO  
Butlerova Street 17  
117342 Moscow  
Russia  
T +7-495 374 04 56  
F +7-495 374 04 56  
info@medirus.ru  
www.medirus.ru

medi Trading (Shanghai) Co. Ltd.  
Room 1703, T1, CES WEST BUND  
CENTER,  
No. 277, Long Lan Road  
Xuhui District  
200232 Shanghai  
People's Republic of China  
T: +86-21 50582319  
F: +86-21 50582319

medi Sweden AB  
Box 6034  
192 06 Sollentuna  
Sweden  
T +46 8 96 97 98  
F +46 8 626 68 70  
info@medi.se  
www.medi.se

Medi Turk Ortopedi Medikal Iç ve  
Dis Tic. AS  
Mustafa Kemal Mah. 2156 Sokak  
No: 16 /4-5  
06510 Çankaya Ankara  
Turkey  
T: +90 312 435 20 26  
F: +90 312 434 17 67  
info@medi-turk.com  
www.medi-turk.com

medi UK Ltd.  
Plough Lane  
Hereford HR4 OEL  
Great Britain  
T +44 1432 37 35 00  
F +44 1432 37 35 10  
enquiries@mediuk.co.uk  
www.mediuk.co.uk

medi Ukraine LLC  
Evhena Sverstiuka str, 11.  
Kiev 02002  
Ukraine  
T: +380 44 591 11 63  
F: +380 44 392 73 73  
info@medi.ua  
http://medi.ua

## Storage instructions

Store in a dry place

Keep out of sunlight

## Lifespan

circaid single band afw has a lifespan of 3 months. Due to material wear-and-tear the medical efficacy can only be expected for a defined lifespan. This assumes correct handling of the garment (e.g. proper care, application, and removal).

## Disposal

Please dispose of the garment in household waste. No special criteria for disposal apply for medical compression garments.

## Material composition

circaid compression garment  
Polyester 72%  
Nylon 24%  
Elastane 4%

This garment does not contain latex.

## Care instructions

The garment can be machine washed using a gentle cycle and dried on a low heat setting. To extend the life of the garment, hand wash and drip dry.

Machine wash warm

Do not bleach

Tumble dry, low heat

Do not iron

Do not dry clean

Do not wring

### **Wearing recommendations**

The garment can be worn during the day or as directed by your clinician.

The circaid compression garment should never hurt. It should feel firm but comfortable.

If experiencing any pain, or discomfort, reduce the compression (mmHg) applied or remove device. Inform your clinician of your action.

If pressure increases or decreases noticeably during wear, readjust the garment as needed.

### **Donning instructions**

Put on an undersock before donning. The circaid single band afw should not be applied directly against the skin

#### **Step 1:**

Place the hook end of the circaid single band afw on top of the foot, facing away from the skin.

Holding the hook tab in place, wrap the band around the foot once, and secure it to the hook end.

#### **Step 2:**

Continue wrapping the circaid single band ankle foot wrap around the foot with firm and comfortable compression. Cover the foot from the base of the toes to the heel making sure there are no gaps between the layers of elastic. Apply double layers as desired to target problem areas.

#### **Step 3:**

If applying for the first time, the band can be shortened as needed. Secure the loose end of the circaid single band ankle foot wrap using one of the provided hook tabs.

### **Doffing instructions**

Disengage the hook tab and remove it from the circaid single band ankle foot wrap. Unwrap the foot and roll the band into a cylinder to prepare it for the next use. Make sure the sewn-in hook tab is at the tail-end of the roll.

# circaid® single band ankle foot wrap

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