medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
www.medi.de

medi Australia Pty Ltd
 83 Fennell Street
 North Parramatta NSW 2151
 Australia
 T +61 2 9890 8696
 F +61 2 9890 8439
 sales@mediaustralia.com.au
 www.mediaustralia.com.au

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
 T +43-512 57 95 15
 F +43-512 57 95 15 45
 vertrieb@medi-austria.at
 www.medi-austria.at

medi Belgium NV
 Posthoornstraat 13/1
 3582 Koersel
 Belgium
 T: +32-11 24 25 60
 F: +32-11 24 25 64
 info@medibelgium.be
 www.medibelgium.be

medi Brasil
 Rua Cristovan de Vita, n.º 260, Un. 30,
 Centro Logístico Raposo Tavares,
 Vila Camargo
 Vargem Grande Paulista - SP
 CEP 06730-000
 Brasil
 T: +55-11-3500 8005
 sac@medibrasil.com
 www.medibrasil.com

medi Canada Inc / médi Canada Inc
 597, Rue Duvernay, Verchères
 QC Canada J0L 2R0
 T: +1 450-583-3317 / +1 800 361 3153
 F: +1 888-583-6827
 service@medicanada.ca
 www.medicanada.ca

MAXIS a.s.,
 medi group company
 Slezská 2127/13
 120 00 Prague 2
 Czech Republic
 T: +420 571 633 510
 F: +420 571 616 271
 info@maxis-medica.com
 www.maxis-medica.com

medi Danmark ApS
 Vejlegardsvej 59
 2665 Vallensbaek Strand
 Denmark
 T +45 70 25 56 10
 F +45 70 25 56 20
 kundeservice@sw.dk
 www.meditandmark.dk

medi

circaid® single band ankle foot wrap

Instructions for use. Gebrauchsanweisung.
 Instrucciones de uso. Instruções para aplicação.
 Инструкция по использованию.

medi Bayreuth España SL
 C/Canigó 2 – 6 bajos
 Hospitalet de Llobregat
 08901 Barcelona
 Spain
 T +34 93 260 04 00
 F +34 93 260 23 14
 medi@mediespana.com
 www.mediespana.com

medi France
 Z.I. Charles de Gaulle
 25, rue Henri Farman
 93297 Tremblay en France Cedex
 France
 T +33 1 48 61 76 10
 F +33 1 49 63 33 05
 infos@medi-france.com
 www.medi-france.com

medi Hungary Kft.
 Bokor u. 21.
 1037 Budapest
 Hungary
 T +36 1371 0090
 F +36 1371 0091
 info@medi.hu
 www.medi.hu

medi Italia s.r.l.
 Via Giuseppe Ghedini, 2
 40069 Zola Predosa (BO)
 Italy
 T: +39-051 6 13 24 84
 F: +39-051 6 13 29 56
 info@medi-italia.it
 www.medi-italia.it

medi Japan K.K.
 5-1 Nihonbashi Kabutocho Chuoku
 103-0026 Tokyo
 Japan
 T: +81 3 6778 2590
 F: +81 3 5847 7901
 info@medi-japan.jp
 www.medi-japan.co.jp

Tachezy Sanit Srl.
 Via Druso 48/a
 39100 Bolzano
 Italy
 T: +39 0471 288 068
 F: +39 0471 282 773
 info@tachezysanit.com
 www.tachezysanit.com

medi Nederland BV
 Heusing 5
 4817 ZB Breda
 The Netherlands
 T +31 76 57 22 555
 F +31 76 57 22 565
 info@medi.nl
 www.medi.nl

medi Norway AS
 Vestheimvegen 35
 4250 Kopervik
 Norway
 T: +47 52 84 45 00
 F: +47 52 84 45 09
 post@medinorway.no
 www.medinorway.no

E012037/05.2020

4 049772 365647

medi. I feel better.

English

circaid® single band ankle foot wrap

Purpose

The compression system is designed to provide compression to the foot and ankle region for patients with venous and lymphatic disorders.

Indications

- Lymphedema
- other forms of edema
- Lipedema
- Chronic venous insufficiency
- Varicose veins
- DVT/Thrombosis Prevention
- Post Thrombotic Syndrome
- Venous Stasis Ulcer
- Post Sclerotherapy
- Lipodermatosclerosis

Contraindications

- Severe Peripheral Arterial Disease

- Uncontrolled Congestive Heart Failure
- Septic Phlebitis
- Phlegmasia Cerulea Dolens
- Untreated infection
- Any circumstance where increased venous and lymphatic return is undesirable
- Suspected or known untreated acute DVT

Cautions

- Mild to Moderate Peripheral Arterial Disease
- Impaired Sensation - must be able to sense applied pressure
- Intolerance to material(s)
- Pediatric & assisted use - must be able to sense & communicate applied pressure

This Package contains

- 1 circaid single band afw
- 1 booklet instructions for use

medi Polska Sp. z o.o.
ul. Łąbędzka 22
44-121 Gliwice
Poland
T: +48-32 230 60 21
F: +48-32 202 87 56
info@medi-polska.pl
www.medi-polska.pl

medi Bayreuth Unipessoal Lda.
Rua do Centro Cultural, no. 43
1700-106 Lisbon
Portugal
T +351 21 843 71 60
F +351 21 847 08 33
medi.portugal@medibayreuth.pt
www.medi.pt

medi RUS LLC
Business Center NEO GEO
Butlerova Street 17
117342 Moscow
Russia
T +7-495 374 04 56
F +7-495 374 04 56
info@medirus.ru
www.medirus.ru

medi Trading (Shanghai) Co. Ltd.
Room 1703, T1, CES WEST BUND
CENTER,
No. 277, Long Lan Road
Xuhui District
200232 Shanghai
People's Republic of China
T: +86-21 50582319
F: +86-21 50582319

medi Sweden AB
Box 6034
192 06 Sollentuna
Sweden
T +46 8 96 97 98
F +46 8 626 68 70
info@medi.se
www.medi.se

Medi Turk Ortopedi Medikal Iç ve
Dis Tic. AS
Mustafa Kemal Mah. 2156 Sokak
No: 16 /4-5
06510 Çankaya Ankara
Turkey
T: +90 312 435 20 26
F: +90 312 434 17 67
info@medi-turk.com
www.medi-turk.com

medi UK Ltd.
Plough Lane
Hereford HR4 OEL
Great Britain
T +44 1432 37 35 00
F +44 1432 37 35 10
enquiries@mediuk.co.uk
www.mediuk.co.uk

medi Ukraine LLC
Evhena Sverstiuka str, 11.
Kiev 02002
Ukraine
T: +380 44 591 11 63
F: +380 44 392 73 73
info@medi.ua
http://medi.ua

Storage instructions

Store in a dry place

Keep out of sunlight

Lifespan

circaid single band afw has a lifespan of 3 months. Due to material wear-and-tear the medical efficacy can only be expected for a defined lifespan. This assumes correct handling of the garment (e.g. proper care, application, and removal).

Disposal

Please dispose of the garment in household waste. No special criteria for disposal apply for medical compression garments.

Material composition

circaid compression garment
Polyester 72%
Nylon 24%
Elastane 4%

This garment does not contain latex.

Care instructions

The garment can be machine washed using a gentle cycle and dried on a low heat setting. To extend the life of the garment, hand wash and drip dry.

Machine wash warm

Do not bleach

Tumble dry, low heat

Do not iron

Do not dry clean

Do not wring

Wearing recommendations

The garment can be worn during the day or as directed by your clinician.

The circaid compression garment should never hurt. It should feel firm but comfortable.

If experiencing any pain, or discomfort, reduce the compression (mmHg) applied or remove device. Inform your clinician of your action.

If pressure increases or decreases noticeably during wear, readjust the garment as needed.

Donning instructions

Put on an undersock before donning. The circaid single band afw should not be applied directly against the skin

Step 1:

Place the hook end of the circaid single band afw on top of the foot, facing away from the skin.

Holding the hook tab in place, wrap the band around the foot once, and secure it to the hook end.

Step 2:

Continue wrapping the circaid single band ankle foot wrap around the foot with firm and comfortable compression. Cover the foot from the base of the toes to the heel making sure there are no gaps between the layers of elastic. Apply double layers as desired to target problem areas.

Step 3:

If applying for the first time, the band can be shortened as needed. Secure the loose end of the circaid single band ankle foot wrap using one of the provided hook tabs.

Doffing instructions

Disengage the hook tab and remove it from the circaid single band ankle foot wrap. Unwrap the foot and roll the band into a cylinder to prepare it for the next use. Make sure the sewn-in hook tab is at the tail-end of the roll.

circaid® single band ankle foot wrap

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