medi GmbH & Co. KG
 Medicusstraße 1
 D-95448 Bayreuth
 Germany
 T +49 921 912-0
 www.medi.de

medi Australia Pty Ltd
 83 Fennell Street
 North Parramatta NSW 2151
 Australia
 T +61 2 9890 8696
 F +61 2 9890 8439
 sales@mediaustralia.com.au
 www.mediaustralia.com.au

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
 T +43-512 57 95 15
 F +43-512 57 95 15 45
 vertrieb@medi-austria.at
 www.medi-austria.at

medi Belgium NV
 Posthoornstraat 13/1
 3582 Koersel
 Belgium
 T: + 32-11 24 25 60
 F: +32-11 24 25 64
 info@medibelgium.be
 www.medibelgium.be

medi Brasil
 Rua Cristovan de Vita, n.º 260, Un. 30,
 Centro Logístico Raposo Tavares,
 Vila Camargo
 Vargem Grande Paulista - SP
 CEP 06730-000
 Brasil
 T: +55-11-3500 8005
 sac@medibrasil.com
 www.medibrasil.com

medi Canada Inc / médi Canada Inc
 597, Rue Duvernay, Verchères
 QC Canada J0L 2R0
 T: +1 450-583-3317 / +1 800 361 3153
 F: +1 888-583-6827
 service@medicanada.ca
 www.medicanada.ca

MAXIS a.s.,
 medi group company
 Slezská 2127/13
 120 00 Prague 2
 Czech Republic
 T: +420 571 633 510
 F: +420 571 616 271
 info@maxis-medica.com
 www.maxis-medica.com

medi Danmark ApS
 Vejlegårdsvej 59
 2665 Vallensbaek Strand
 Denmark
 T +45 70 25 56 10
 F +45 70 25 56 20
 kundeservice@sw.dk
 www.medit danmark.dk

E012047/01.2020


medi

circaid® cover up arm / cover up leg

Instructions for use. Gebrauchsanweisung.
 Инструкция по использованию. Mode d'emploi.
 Instrucciones de uso. Instruções de uso.
 Gebruiksaanwijzing. Istruzioni per l'uso. Bruksanvisning.
 Brugsvejledning.

circaid cover up arm
 beige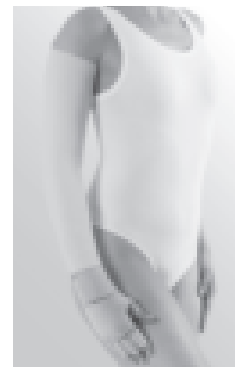

circaid cover up arm
 pink

circaid cover up arm
 black


medi. I feel better.

4 049772 365746


circaid cover up whole leg beige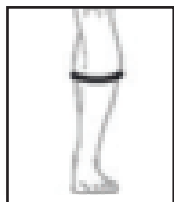

circaid cover up lower leg
small: 34cm -50cm
large: 51cm - 80cm

circaid cover up lower leg black

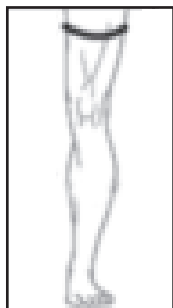circaid cover up whole leg
small: 50cm - 71cm
large: 72cm - 108cm

English

circaid® cover up arm

The circaid cover up can be worn over any compression garment. The sleek design prevents your compression garments from irritating other parts of your body or snagging on surrounding fabrics. The circaid cover up also fits well under clothing.

Disposal

Please dispose of the garment in household waste. No special criteria for disposal apply for medical compression garments.

Material composition

circaid cover up arm:
80% Nylon
18% Elastane
2% Polyethylene

This garment does not contain latex.

Washing instructions

The garment can be machine washed using a gentle cycle and dried on a low heat setting. If doing so, it is recommended that you place the garment in a pillow case to aid in keeping the Velcro tab material lint-free and to protect other clothing from getting damaged.

-  Machine wash warm
-  Do not bleach
-  Tumble dry, low heat
-  Do not iron
-  Do not dry clean

Donning

Ensure that all bands of your compression garment are securely fastened to the garment. No Velcro tabs should be sticking off the garment. Remove the piece of felt between the Velcro tabs of the cover up. Fold over the top edge so that the two Velcro tabs are visible. This will prevent the tabs from catching on the

medi Bayreuth España SL
C/Canigó 2 – 6 bajos
Hospitalet de Llobregat
08901 Barcelona
Spain
T +34 93 260 04 00
F +34 93 260 23 14
medi@mediespana.com
www.mediespana.com

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33 1 48 61 76 10
F +33 1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Hungary Kft.
Bokor u. 21.
1037 Budapest
Hungary
T +36 1371 0090
F +36 1371 0091
info@medi.hu
www.medi.hu

medi Italia s.r.l.
Via Giuseppe Ghedini, 2
40069 Zola Predosa (BO)
Italy
T: +39-051 6 13 24 84
F: +39-051 6 13 29 56
info@medi-italia.it
www.medi-italia.it

medi Japan K.K.
5-1 Nihonbashi Kabutocho Chuoku
103-0026 Tokyo
Japan
T: +81 3 6778 2590
F: +81 3 5847 7901
info@medi-japan.jp
www.medi-japan.co.jp

Tachezy Sanit Srl.
Via Druso 48/a
39100 Bolzano
Italy
T: +39 0471 288 068
F: +39 0471 282 773
info@tachezysanit.com
www.tachezysanit.com

medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31 76 57 22 555
F +31 76 57 22 565
info@medi.nl
www.medi.nl

medi Norway AS
Vestheimvegen 35
4250 Kopervik
Norway
T: +47 52 84 45 00
F: +47 52 84 45 09
post@medinorway.no
www.medinorway.no

garment while sliding it on. Then, slide the cover up over the garment. Finally, attach the Velcro tabs of the cover up to the garment once it is in the desired position.

Doffing: To remove the circaid cover up, slide it off the arm and reattach the piece of felt to the Velcro tabs to keep them lint-free.

Sizing chart

The cover up arm is available in size S and L

Note

This product does not provide therapeutic compression.

circaid® cover up leg

The circaid cover up leg can be worn over any compression garment. The sleek design prevents your compression garments from irritating other parts of your body or snagging on surrounding fabrics. The circaid cover up also fits well under clothing.

Disposal

Please dispose of the garment in household waste. No special criteria for disposal apply for medical compression garments.

Material composition

circaid cover up lower leg / circaid cover up whole leg
80% Nylon
20% Elastane

This garment does not contain latex.

Washing instructions

The garment can be machine washed using a gentle cycle and dried on a low heat setting. If doing so, it is recommended that you place the garment in a pillow case to aid in keeping the Velcro tab material lint-free and to protect other clothing from getting damaged.

Machine wash warm

Do not bleach

Tumble dry, low heat

Do not iron

Do not dry clean

Donning:

Ensure that all bands of your compression garment are securely fastened to the garment. No Velcro tabs should be sticking off the garment. Remove the

piece of felt between the Velcro tabs of the cover up. Fold over the top edge so that the two Velcro tabs are visible. This will prevent the tabs from catching on the garment. while sliding it on. Then, slide the cover up over the garment. Finally, attach the Velcro tabs of the cover up to the garment once it is in the desired position.

Doffing:

To remove the circaid cover up leg, slide it off the leg and reattach the piece of felt to the Velcro tabs to keep them lint-free.

Notes:

This product does not provide therapeutic compression.

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medi UK Ltd.
medi Ukraine LLC