medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
www.medi.de

medi Australia Pty Ltd
Unit 4/13-21 Hallmark Street
Pendle Hill NSW 2145
Australia
T +61 2 9890 8696
F +61 2 9890 8439
sales@medi.au
www.mediaustralia.com.au

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Belgium NV
Staatsbaan 77/0099
3945 Ham
Belgium
T: +32-11 24 25 60
F: +32-11 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Canada Inc / médi Canada Inc
104-1375 Lionel-Boulet,
Varennnes, Québec,
QC Canada J3X 1P7
T +1 450-583-3317 / +1 800 361 3153
F +1 888-583-6827
service@medicanada.ca
www.medicanada.ca

MAXIS a.s.,
medi group company
Slezská 2127/13
120 00 Prague 2
Czech Republic
T: +420 571 633 510
F: +420 571 616 271
info@maxis-medica.com
www.maxis-medica.com

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circaid® cover up leg

The circaid cover up leg can be worn over any compression garment. The sleek design prevents your compression garments from irritating other parts of your body or snagging on surrounding fabrics. The circaid cover up also fits well under clothing.

Donning

Ensure that all bands of your compression garment are securely fastened to the garment. No Velcro tabs should be sticking off the garment. Remove the piece of felt between the Velcro tabs of the cover up. Fold over the top edge so that the two Velcro tabs are visible. This will prevent the tabs from catching on the garment while sliding it on. Then, slide the cover up over the garment. Finally, attach the Velcro tabs of the cover up to the

garment once it is in the desired position.

Doffing

To remove the circaid cover up leg, slide it off the leg and reattach the piece of felt to the Velcro tabs to keep them lint-free.

Notes:

This product does not provide therapeutic compression.

Care instructions

This product can be washed on a delicate cycle and dried in a tumble dryer set to a low heat. We recommend using a mesh laundry bag. To extend the life of the product, hand wash and drip dry.

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medi Trading (Shanghai) Co. Ltd.
medi Ukraine LLC