**medi GmbH & Co. KG**  
**Medicusstraße 1**  
**95448 Bayreuth / Germany**  
**T +49 921 912-0**  
**F +49 921 912-780**  
**ortho@medi.de**  
**www.medi.de**

medi Australia Pty Ltd  
 83 Fennell Street  
 North Parramatta NSW 2151  
 Australia  
 T +61-2 9890 8696  
 F +61-2 9890 8439  
 sales@mediaustralia.com.au  
 www.mediaustralia.com.au

medi Austria GmbH  
 Adamgasse 16/7  
 6020 Innsbruck  
 Austria  
 T +43 512 57 95 15  
 F +43 512 57 95 15 45  
 vertrieb@medi-austria.at  
 www.medi-austria.at

medi Belgium NV  
 Staatsbaan 77/0099  
 3945 Ham  
 Belgium  
 T: + 32-11 24 25 60  
 F: +32-11 24 25 64  
 info@medibelgium.be  
 www.medibelgium.be

medi Canada Inc / médi Canada Inc  
 104-1375 Lionel-Boulet,  
 Varennes, Québec,  
 QC Canada J3X 1P7  
 T +1 450-583-3317 / +1 800 361 3153  
 F +1 888-583-6827  
 service@medicanada.ca  
 www.medicanada.ca

MAXIS a.s.,  
 medi group company  
 Slezská 2127/13  
 120 00 Prague 2  
 Czech Republic  
 T: +420 571 633 510  
 F: +420 571 616 271  
 info@maxis-medica.com  
 www.maxis-medica.com

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### Important notes

This medical device **MD** is made only for multiple uses on a single patient. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If you experience excessive pain or any discomfort whilst wearing, remove the product and immediately contact your doctor or medical supply store. Only wear the product on unbroken skin or over injured skin that has been medically treated, not directly on injured or damaged skin, and only after previous consultation with a medical professional.

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medi Trading (Shanghai) Co. Ltd.  
Room 1703, T1, CES WEST BUND CENTER,  
No. 277, Long Lan Road  
Xuhui District  
200232 Shanghai  
People's Republic of China  
T: +86-21 50582319  
F: +86-21 50582319

Medi Turk Ortopedi Medikal İc ve Dis Tic. AS  
Mustafa Kemal Mah. 2156 Sokak No: 16 /4-5  
06510 Çankaya Ankara  
Turkey  
T: +90 312 435 20 26  
F: +90 312 434 17 67  
info@medi-turk.com  
www.medi-turk.com

medi Ukraine LLC  
Evhena Sverstiuika str, 11.  
Kiev 02002  
Ukraine  
T: +380 44 591 11 63  
F: +380 44 392 73 73  
info@medi.ua  
http://medi.ua

medi USA L.P.  
6481 Franz Warner Parkway  
Whitsett, N.C. 27377-3000  
USA  
T +1-336 4 49 44 40  
F +1-888 5 70 45 54  
info@mediusa.com  
www.mediusa.com

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# protect. CAT Walker short

## Intended purpose

The protect.CAT Walker short is a lower-leg foot brace for immobilising this region in a specified position.

## Indications

For all indications which require immobilisation of the lower leg and foot in a specified position, such as:

- Following ligament-, soft tissue-, or tendon injuries (postoperative / Nonsurgical)
- For Stable foot and joint fractures, fracture postoperative care (Nonsurgical)
- Following midfoot and forefoot surgery
- Following serious distorsion

## Contraindications

Unstable fractures or fractures of the proximal tibia or fibula

## Risks / Side Effects

If the aids are too tight, this can cause local pressure sores or constriction of blood vessels or nerves. Therefore you should consult the doctor treating you before you use it if the following circumstances apply to you:

- Condition or injury of the skin in the area of application, particularly with infectious indications (excessive heat, swelling or reddening)
- Paraesthesia and circulatory disorders (e.g. as seen in diabetes, varicose veins)
- Lymph drainage disorders – equally, swelling of soft tissue adjacent to the area of application which is not related to a condition

When wearing snug-fitting aids, this can result in localised skin inflammation or irritation that is due to mechanical

irritation of the skin (particularly in conjunction with sweating) or due to the composition of the material.

## Intended users and patient target groups

Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device, provided that they have received briefing on this from healthcare professionals.

Patient target group: Healthcare professionals should provide care to adults and children, applying the available information on the measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their own responsibility.

## Advice

The protect.CAT Walker short is not designed for patients who weigh more than 120 kg.

## Fitting instructions

- Open the two lower belts and the upper belt on the left-hand side. Fold the shin and the belts away to the right. Remove the inner shoe from the Walker.
- (Fig.1) Open the leg and foot part of the inner shoe by loosening the hook and loop fasteners. Ask the patient to place his foot and lower leg into the inner shoe. Make sure that the air chambers are empty and that the valve is loosened anti-clockwise (deflate).
- (Fig.2) Close the inner shoe and make sure that it is firmly fixed to the patient's leg. (Fig. 3a/b) Align the pump in such a way that it is sitting in the middle and so that later it fits into the space provided for the shin.
- (Fig.4) Now ask the patient to place

their foot in the half shell of the Walker. The lower leg and foot should be positioned so that they fit exactly into the shell.

- (Fig. 5) Check to see that the foot is properly set and positioned in the Walker. Make sure that the patient's toes are not pushing against the toe protector. If necessary, the toe protector's position can be adjusted or it can be removed completely.
- (Fig. 6) Now place the loose shin onto the patient's shinbone and foot from the front. Make sure that the pump and ventilator fit into the spaces provided.
- (Fig.7) Close the upper belt.
- (Fig.8) Retighten the right-hand belt again as well in order to stabilise the system.
- (Fig.9,10) Then close all the other belts one after the other. The belts should fit tightly. Extra material can be cut away.
- Ask the patient to walk around in the orthosis to make sure that the Walker is sitting properly.
- (Fig.11) To tighten the Walker's fit, the air cushion can be filled using the pump system. To this end, close the valve by turning it clockwise (inflate) and then pump air into the chamber. The padded pieces can be used as additional padding.

---

### Care instructions

Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the product by hand, preferably using medi clean washing agent.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

### Storage instructions

Keep the product in a dry place and do not expose to direct sunlight.

---

### Material Composition

PP, PU, EVA, Polyester

### Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

### Disposal

The product can be disposed of in the domestic waste.

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).

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