**medi GmbH & Co. KG**  
**Medicusstraße 1**  
**95448 Bayreuth**  
**Germany**  
**T +49 921 9120**  
**F +49 921 912780**  
**ortho@medi.de**  
**www.medi.de**

medi Australia Pty Ltd  
 83 Fennell Street  
 North Parramatta NSW 2151  
 Australia  
 T +61-2 9890 8696  
 F +61-2 9890 8439  
 sales@mediaustralia.com.au  
 www.mediaustralia.com.au

medi Austria GmbH  
 Adamgasse 16/7  
 6020 Innsbruck  
 Austria  
 T +43 512 57 95 15  
 F +43 512 57 95 15 45  
 vertrieb@medi-austria.at  
 www.medi-austria.at

medi Belgium NV  
 Staatsbaan 77/0099  
 3945 Ham  
 Belgium  
 T: +32-11 24 25 60  
 F: +32-11 24 25 64  
 info@medibelgium.be  
 www.medibelgium.be

medi Canada Inc / médi Canada Inc  
 104-1375 Lionel-Boulet,  
 Varennes, Québec,  
 QC Canada J3X 1P7  
 T +1 450-583-3317 / +1 800 361 3153  
 F +1 888-583-6827  
 service@medicanada.ca  
 www.medicanada.ca

MAXIS a.s.,  
 medi group company  
 Slezská 2127/13  
 120 00 Prague 2  
 Czech Republic  
 T: +420 571 633 510  
 F: +420 571 616 271  
 info@maxis-medica.com  
 www.maxis-medica.com

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medi Trading (Shanghai) Co. Ltd.  
 Room 1703, T1, CES WEST BUND CENTER,  
 No. 277, Long Lan Road  
 Xuhui District  
 200232 Shanghai  
 People's Republic of China  
 T: +86-21 50582319  
 F: +86-21 50582319

Medi Turk Ortopedi Medikal İc ve Dis Tic. AS  
 Mustafa Kemal Mah. 2156 Sokak No: 16 /4-5  
 06510 Çankaya Ankara  
 Turkey  
 T: +90 312 435 20 26  
 F: +90 312 434 17 67  
 info@medi-turk.com  
 www.medi-turk.com

UK Responsible Person  
 medi UK Ltd.  
 Plough Lane  
 Hereford HR4 OEL  
 Great Britain  
 T +44-1432 37 35 00  
 F +44-1432 37 35 10  
 enquiries@mediuk.co.uk  
 www.mediuk.co.uk

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#### Important notes

This medical device **MD** is made only for multiple uses on a single patient. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If you experience excessive pain or any discomfort whilst wearing, remove the product and immediately contact your doctor or medical supply store. Only wear the product on unbroken skin or over injured skin that has been medically treated, not directly on injured or damaged skin, and only after previous consultation with a medical professional.

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# Lumbamed<sup>®</sup> disc

## Intended purpose

Lumbamed disc is a support for partial stress relief (👉) via correction of lordosis and functional mobilisation (👉) of the lumbar spine.

## Indications

All indications requiring partial relief by reduction of lordosis of the lumbar spine with subsequent functional mobilisation of the lumbar spine such as:

- Intervertebral disc protrusion or disc prolaps (Nonsurgical, postoperative)
- Pseudo-radicular pain syndrome
- Postural muscle imbalance in the lumbar spine
- Moderate lower back pain (lumbago)
- Moderate lower back pain with sciatica

## Contraindications

None known at present.

## Risks / Side Effects

If the aids are too tight, this can cause local pressure sores or constriction of blood vessels or nerves. Therefore you should consult the doctor treating you before you use it if the following circumstances apply to you:

- Condition or injury of the skin in the area of application, particularly with infectious indications (excessive heat, swelling or reddening)
- Paraesthesia and circulatory disorders (e.g. as seen in diabetes, varicose veins)
- Lymph drainage disorders – equally, swelling of soft tissue adjacent to the area of application which is not related to a condition

When wearing snug-fitting aids, this can result in localised skin inflammation or irritation that is due to mechanical

irritation of the skin (particularly in conjunction with sweating) or due to the composition of the material.

## Intended users and patient target groups

Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device, provided that they have received briefing on this from healthcare professionals.

Patient target group: Healthcare professionals should provide care to adults and children, applying the available information on the measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their own responsibility.

## Note on wearing

Supports are particularly beneficial during physical exercise. The supports is designed to fit comfortably and may be worn all day. It should be removed during prolonged periods of inactivity (e.g. sitting for a long time or a long car drive, sleeping).

The support is suitable for use in water. After use, please rinse thoroughly.

## Fitting instructions

- Before putting on the orthotic, please loosen the straps, so that it can be put on without pulling. To do so, unfasten the velcro ends of the straps and loosen these by placing them further back on the velour on the belt (Fig. 1).
- The orientation of the label on the righthand inner fastener has information on how to put on the brace (Fig. 2).
- Now, slide your left hand into the inner hand strap on the lefthand fastener side (Fig. 3).
- Wrap the brace around your body and

check once again on the label that you are holding the brace the right way (Fig. 4).

- Now, slide your right hand into the outer hand strap on the righthand fastener side (Fig. 5).
- Pull both fastener sides together in the middle, so that they can be velcroed over each other as far at the same distance. When fastening, ensure that the back brace is in the middle. The orthotic is on properly when the lower fastener edge is just above the pubic bone (see Fig. 9) (Fig. 6 + 7).
- Now unfasten the hook and loop ends of the pull straps and pull these forward at the same time. You can adjust the tightness to suit your own requirements and manage your pain levels (Fig. 8) .
- Information for technicians: The strap length may be adjusted by cutting it at the edge of the Y fastener (Fig. 10).

### Removing / Changing the stabilising rods

Hold the lower end of the rod and pull it slightly downwards with the knitted fabric. At the same time, slightly pull apart the upper opening in the knitted fabric. You can now remove the rod easily from the pocket. For inserting the rods, slide them into the individual pockets again and reposition them in the same way.

### Care instructions

Before washing, please remove the internal rods (cf. Fig. 12) as well as the strap supporting system by detaching it from the Y connecting piece (cf. Fig. 11). During replacement make sure that the medi logo on the back stays is at the top. Hook and loop tabs must be closed for washing. We recommend using a laundry bag. Fabric softeners, fats, oils, lotions, ointments and soap residues can affect the material, hinder the Clima Comfort

effect and cause skin irritations.

- Wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

### Storage instructions

Keep the product in a dry place and do not expose to direct sunlight.

### Composition

Polyamide, Polyester, elastane, viscose, ABS plastic

### Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

### Disposal

The product can be disposed of in the domestic waste.

### Your medi Team

Wishes you a rapid recovery!

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).

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