medi GmbH & Co. KG  
 Medicusstraße 1  
 95448 Bayreuth  
 Germany  
 T +49 921 912-0  
 F +49 921 912-780  
 ortho@medi.de  
 www.medi.de

medi Australia Pty Ltd  
 Unit 4/13-21 Hallmark Street  
 Pendle Hill NSW 2145  
 Australia  
 T +61 2 9890 8696  
 F +61 2 9890 8439  
 sales@medi.au  
 www.medi.australia.com.au

medi Austria GmbH  
 Adamgasse 16/7  
 6020 Innsbruck  
 Austria  
 T +43 512 57 95 15  
 F +43 512 57 95 15 45  
 vertrieb@medi-austria.at  
 www.medi-austria.at

medi Belgium NV  
 Staatsbaan 77/0099  
 3945 Ham  
 Belgium  
 T: + 32-11 24 25 60  
 F: +32-11 24 25 64  
 info@medibelgium.be  
 www.medibelgium.be

medi Canada Inc / médi Canada Inc  
 104-1375 Lionel-Boulet,  
 Varennes, Québec,  
 QC Canada J3X 1P7  
 T +1 450-583-3317 / +1 800 361 3153  
 F +1 888-583-6827  
 service@medicanada.ca  
 www.medicanada.ca

4 046114 562009

MAXIS a.s.,  
 medi group company  
 Slezská 2127/13  
 120 00 Prague 2  
 Czech Republic  
 T: +420 571 633 510  
 F: +420 571 616 271  
 info@maxis-medica.com  
 www.maxis-medica.com

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### Important notes

This medical device **MD** is made only for multiple uses on a single patient . If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If you experience excessive pain or any discomfort whilst wearing, remove the product and immediately contact your doctor or medical supply store. Only wear the product on unbroken skin or over injured skin that has been medically treated, not directly on injured or damaged skin, and only after previous consultation with a medical professional.

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medi Trading (Shanghai) Co. Ltd.  
Room 1703, T1, CES WEST BUND CENTER,  
No. 277, Long Lan Road  
Xuhui District  
200232 Shanghai  
People's Republic of China  
T: +86-21 50582319  
F: +86-21 50582319

Medi Turk Ortopedi Medikal İc ve Dis Tic. AS  
Mustafa Kemal Mah. 2156 Sokak No: 16/4-5  
06510 Çankaya Ankara  
Turkey  
T: +90 312 435 20 26  
F: +90 312 434 17 67  
info@medi-turk.com  
www.medi-turk.com

medi Ukraine LLC  
Evhena Sverstiuka str, 11.  
Kiev 02002  
Ukraine  
T: +380 44 591 11 63  
F: +380 44 392 73 73  
info@medi.ua  
http://medi.ua

medi USA L.P.  
6481 Franz Warner Parkway  
Whitsett, N.C. 27377-3000  
USA  
T +1-336 4 49 44 40  
F +1-888 5 70 45 54  
info@mediusa.com  
www.mediusa.com

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# protect. Lumbostyle

## Intended purpose

protect.Lumbostyle is a support for partial stress relief  of the lumbar vertebrae via correction of lordosis.

## Indications

All indications which require partial stress relief of the lumbar vertebrae via correction of lordosis, such as:

- Lower back pain (lumbago)
- Lower back pain with sciatica
- Lumbar vertebral canal stenosis
- Postural muscle imbalance in the lumbar spine
- Spondylolisthesis
- Spondylosis
- Painful conditions of the lumbosacral junction
- Intervertebral disc protrusion, prolapse

## Contraindications

- Open wounds in the area of the orthosis
- Respiratory diseases such as COPD

## Risks / Side Effects

If the aids are too tight, this can cause local pressure sores or constriction of blood vessels or nerves. Therefore you should consult the doctor treating you before you use it if the following circumstances apply to you:

- Condition or injury of the skin in the area of application, particularly with infectious indications (excessive heat, swelling or reddening)
- Paraesthesia and circulatory disorders

(e.g. as seen in diabetes, varicose veins)

- Lymph drainage disorders – equally, swelling of soft tissue adjacent to the area of application which is not related to a condition

When wearing snug-fitting aids, this can result in localised skin inflammation or irritation that is due to mechanical irritation of the skin (particularly in conjunction with sweating) or due to the composition of the material.

## Intended users and patient target groups

Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device, provided that they have received briefing on this from healthcare professionals.

Patient target group: Healthcare professionals should provide care to adults and children, applying the available information on the measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their own responsibility.

## Note on wearing

We recommend you do not wear the product directly on your skin. Wear a thin t-shirt or undershirt underneath it. Unless otherwise directed by your doctor, do not wear the support for more than 8 hours at a time. After a break of approx. 2 hours it may be refitted.

## Fitting instructions

- Before putting on the orthosis, please loosen the straps so that it can be put on without pulling. Wrap the orthosis

around your abdomen and make sure it is positioned on the centre of your back (fig. 1).

- Afterwards, place the left lock side on your stomach and then pull the right lock side far enough over the left side that it is possible to fasten (fig. 2).
- Now, pull both tension bands forward at the same time. You can adjust the tightness to suit your own requirements and manage your pain levels (fig. 3).
- First, fasten the left tension band and then the right (fig. 4).
- The orthosis is on properly when the lower fastener edge is just above the groin (fig. 5).

---

#### Care instructions

Hook and loop fasteners should be closed for washing. Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the product by hand, preferably using medi clean washing agent.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

---

#### Storage instructions

Keep the product in a dry place and do not expose to direct sunlight.

---

#### Material composition

Polyester, poliamide, elastane

#### Liability

The manufacturer's liability will become

void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

#### Disposal

Dispose of properly after use.

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).

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