medi GmbH & Co. KG
 Medicusstraße 1
 D-95448 Bayreuth
 Germany
 T +49 921 912-0
 www.medi.de

medi Australia Pty Ltd
 83 Fennell Street
 North Parramatta NSW 2151
 Australia
 T +61 2 9890 8696
 F +61 2 9890 8439
 sales@mediaustralia.com.au
 www.mediaustralia.com.au

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
 T +43-512 57 95 15
 F +43-512 57 95 15 45
 vertrieb@medi-austria.at
 www.medi-austria.at

medi Belgium NV
 Posthoornstraat 13/1
 3582 Koersel
 Belgium
 T: +32-11 24 25 60
 F: +32-11 24 25 64
 info@medibelgium.be
 www.medibelgium.be

medi Brasil
 Rua Cristovan de Vita, n.º 260, Un. 30,
 Centro Logístico Raposo Tavares,
 Vila Camargo
 Vargem Grande Paulista - SP
 CEP 06730-000
 Brasil
 T: +55-11-3500 8005
 sac@medibrasil.com
 www.medibrasil.com

medi Canada Inc / médi Canada Inc
 597, Rue Duvernay, Verchères
 QC Canada J0L 2R0
 T: +1 450-583-3317 / +1 800 361 3153
 F: +1 888-583-6827
 service@medicanada.ca
 www.medicanada.ca

MAXIS a.s.,
 medi group company
 Slezská 2127/13
 120 00 Prague 2
 Czech Republic
 T: +420 571 633 510
 F: +420 571 616 271
 info@maxis-medica.com
 www.maxis-medica.com

medi Danmark ApS
 Vejlegårdsvej 59
 2665 Vallensbaek Strand
 Denmark
 T +45 70 25 56 10
 F +45 70 25 56 20
 kundeservice@sw.dk
 www.meditandmark.dk

medi Bayreuth España SL
 C/Canigó 2 – 6 bajos
 Hospitalet de Llobregat
 08901 Barcelona
 Spain
 T +34 93 260 04 00
 F +34 93 260 23 14
 medi@mediespana.com
 www.mediespana.com

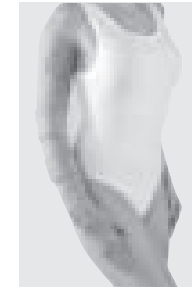
medi France
 Z.I. Charles de Gaulle
 25, rue Henri Farman
 93297 Tremblay en France Cedex
 France
 T +33 1 48 61 76 10
 F +33 1 49 63 33 05
 infos@medi-france.com
 www.medi-france.com

medi Hungary Kft.
 Bokor u. 21.
 1037 Budapest
 Hungary
 T +36 1371 0090
 F +36 1371 0091
 info@medi.hu
 www.medi.hu

medi

**circaid® juxtafit®
 essentials arm**

**circaid® juxtafit®
 essentials arm
 with
 circaid® juxtafit®
 essentials hand
 wrap**

circaid® juxtafit® essentials arm

Instructions for use. Gebrauchsanweisung. Инструкция по использованию. Instruções para aplicação. Instrucciones de uso. Mode d'emploi. Gebruiksaanwijzing. Istruzioni per l'uso. Bruksanvisning. Brugsvejledning. دليل الاستخدام. Інструкція з використання, Instrukcja użytkowania. הוראות שימוש.

4 049772 764662

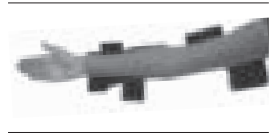
EO1261/04.2020

medi. I feel better.

circaid® juxtafit® essentials arm

Step 1

Step 2

Step 3

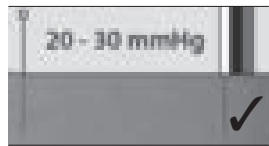
circaid® juxtafit® essentials elbow strap

Step 1


A

B

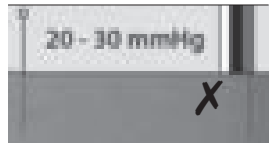C

English

circaid® juxtafit® essentials arm

Purpose

The compression system is designed to provide compression to the arm for patients with venous and lymphatic disorders. The device consists of a series of juxtapositioned, inelastic bands extending from the central portion of the garment. The wrap contours the arm and is designed to cover from the wrist to the axilla.

Indications

- Lymphedema
- other forms of edema
- Lipedema

Contraindications

- Severe Peripheral Arterial Disease
- Uncontrolled Congestive Heart Failure
- Septic Phlebitis
- Phlegmasia Cerulea Dolens
- Untreated infection
- Any circumstance where increased venous and lymphatic return is undesirable
- Suspected or known untreated acute DVT

Cautions

- Mild to Moderate Peripheral Arterial Disease
- Impaired Sensation - must be able to sense applied pressure
- Intolerance to material(s)
- Pediatric & assisted use - must be able to sense & communicate applied pressure

This Package contains

- 1 circaid juxtafit essentials arm
- 1 circaid cover up arm
- 1 Built-In-Pressure System™ (BPS™) card
- 1 booklet instructions for use

Storage instructions

Store in a dry place

Keep out of sunlight

Lifespan

circaid juxtafit essentials has a lifespan of 6 months. Due to material wear-and-tear the medical efficacy can only be expected for a defined lifespan. This assumes correct handling of the garment (e.g. proper care, application, and removal).

Disposal

Please dispose of the garment in household waste. No special criteria for disposal apply for medical compression garments.

Material composition

circaid compression garment
Nylon 53%
Polyurethane 40%
Elastane 7%
For all other components, please refer to the inserts included with each item.

This garment does not contain latex.

medi Italia s.r.l.
Via Giuseppe Ghedini, 2
40069 Zola Predosa (BO)
Italy
T: +39-051 6 13 24 84
F: +39-051 6 13 29 56
info@medi-italia.it
www.medi-italia.it

medi Japan K.K.
5-1 Nihonbashi Kabutocho Chuoku
103-0026 Tokyo
Japan
T: +81 3 6778 2590
F: +81 3 5847 7901
info@medi-japan.jp
www.medi-japan.co.jp

Tachezy Sanit Srl.
Via Druso 48/a
39100 Bolzano
Italy
T: +39 0471 288 068
F: +39 0471 282 773
info@tachezysanit.com
www.tachezysanit.com

medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31 76 57 22 555
F +31 76 57 22 565
info@medi.nl
www.medi.nl

medi Norway AS
Vestheimvegen 35
4250 Kopervik
Norway
T: +47 52 84 45 00
F: +47 52 84 45 09
post@medinorway.no
www.medinorway.no

medi Polska Sp. z o.o.
ul. Łąbedzka 22
44-121 Gliwice
Poland
T: +48-32 230 60 21
F: +48-32 202 87 56
info@medi-polska.pl
www.medi-polska.pl

medi Bayreuth Unipessoal Lda.
Rua do Centro Cultural, no. 43
1700-106 Lisbon
Portugal
T +351 21 843 71 60
F +351 21 847 08 33
medi.portugal@medibayreuth.pt
www.medi.pt

medi RUS LLC
Business Center NEO GEO
Butlerova Street 17
117342 Moscow
Russia
T +7-495 374 04 56
F +7-495 374 04 56
info@medirus.ru
www.medirus.ru

medi Trading (Shanghai) Co. Ltd.
Room 1703, T1, CES WEST BUND CENTER,
No. 277, Long Lan Road
Xuhui District
200232 Shanghai
People's Republic of China
T: +86-21 50582319
F: +86-21 50582319

medi Sweden AB
Box 6034
192 06 Sollentuna
Sweden
T +46 8 96 97 98
F +46 8 626 68 70
info@medi.se
www.medi.se

Medi Turk Ortopedi Medikal Ic ve Dis Tic. AS
Mustafa Kemal Mah. 2156 Sokak No: 16 /4-5
06510 Çankaya Ankara
Turkey
T: +90 312 435 20 26
F: +90 312 434 17 67
info@medi-turk.com
www.medi-turk.com

medi UK Ltd.
Plough Lane
Hereford HR4 OEL
Great Britain
T +44 1432 37 35 00
F +44 1432 37 35 10
enquiries@mediuk.co.uk
www.mediuk.co.uk

medi Ukraine LLC
Evhen Sverstiuka str, 11.
Kiev 02002
Ukraine
T: +380 44 591 11 63
F: +380 44 392 73 73
info@medi.ua
http://medi.ua

medi USA L.P.
6481 Franz Warner Parkway
Whitsett, N.C. 27377-3000
USA
T +1-336 4 49 44 40
F +1-888 5 70 45 54
info@mediusa.com
www.mediusa.com

Care instructions

The garment can be machine washed using a gentle cycle and dried on a low heat setting. If doing so, it is recommended that you place the garment in a pillow case to aid in keeping the hook and loop tap lint-free and to protect other clothing from getting damaged. To extend the life of the garment, hand wash and drip dry.

-
-  Machine wash warm
 -  Do not bleach
 -  Tumble dry, low heat
 -  Do not iron
 -  Do not dry clean
 -  Do not wring
-

Wearing recommendation

Always ensure that the correct prescribed pressure range is being applied. If desired, temporarily detach the elbow band in order to increase mobility. Slightly loosen the bands of the garment for night-time wear. Each vertical BPS line on the bands may be at a different location on the arm depending on the size and shape of the arm. They do not need to be aligned down the front center of the arm or in any other specific location on the arm. If experiencing any pain, or discomfort, reduce the compression (mmHg) applied or remove device. Inform your clinician of your action. When bands are placed correctly they should slightly overlap, leaving no space between bands. Do not tuck bands. The Velcro tabs should never touch the skin!

Donning instructions

Step 1: Don optional circaid

undersleeve. The circaid juxtafit essentials arm can also be worn directly against the skin.

Step 2: Slide the circaid juxtafit essentials arm onto the arm. The top of the armsleeve should stop comfortably below the axilla. The elastic material should be on the inner side of the arm with the elbow section of the sleeve centered with the elbow crease.

Step 3: Unroll the band closest to the wrist, pull it across the elastic section of the sleeve and attach it onto the opposite side with firm and comfortable compression. Repeat this process with every band continuing up the arm. The garment should lay flat and wrinkle-free against the arm. Adjust each band to a firm and comfortable compression level. Ensure that the bands are placed in an alternating order.

Note:

Once all bands are secured inspect the garment for any gaps or creases. Adjust the bands as necessary.

Note

The circaid juxtafit essentials elbow strap will be pre-applied in its proper position for ease of use however some adjusting to maximize comfort and coverage may be needed when wearing the arm garment.

Step 1: Locate the elbow crease.

Step 2: Attach the two hook tabs of one side of the elbow strap to the arm garment next to the elbow crease.

Step 3: Pull the elbow strap across the elbow crease and

secure the remaining two hook tabs to the other side of the garment.

Ensure that the hook tabs never make contact with the thin, elastic material.

The circaid juxtafit essentials elbow strap is symmetrical, so either end may be used for alignment.

Doffing instructions

To remove the garment, detach all bands starting with the top band. Fold each band onto itself to keep the hook tabs lint-free.

Note

If you have decided to wear a circaid undersleeve beneath the garment, remove the undersleeve.

Using the Built-In-Pressure System (BPS)

Step 1: Locate the Built-In-Pressure card in your packaging.

Step 2: Identify your prescribed pressure scale using the color-coded system.

Step 3: Starting with the bottom band, line up the black triangle on the card with one of the BPS lines on the bottom band.

Step 4: Note where the second BPS line lines up with the card's compression ranges.

Step 5: If the BPS line on the garment either falls short (Fig. B) or goes beyond the correct compression range (Fig. C), readjust the band as necessary so that the second BPS line on the garment is aligned with the

prescribed compression range (Fig. A). Completely loosen the band before reapplying it if too much tension has been applied.

Step 6: Repeat the steps 3 - 5 for each band going up the arm. Be sure to adjust all bands to the same compression level. Bands may need adjusting throughout the day to maintain the prescribed compression range.

Note

If using multiple circaid juxtafit made-to-measure garments, only lightly apply bands overlapping the legging below to ensure gradient compression. Set the remaining bands to firm and comfortable compression or prescribed compression level.

Important information for practitioners and patients:

Please note that the limb is always treated in its entirety with medical compression.

This means that compression products for the arm begin at the hand or the wrist, and some extend up to just below the elbow or beyond to below the axilla.

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