medi GmbH & Co. KG
Medicusstraße 1
95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-780
ortho@medi.de
www.medi.de

medi Australia Pty Ltd
83 Fennell Street
North Parramatta NSW 2151
Australia
T +61-2 9890 8696
F +61-2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Belgium NV
Staatsbaan 77/0099
3945 Ham
Belgium
T: +32-11 24 25 60
F: +32-11 24 25 64
info@medibelgium.be
www.medi Belgium.be

medi Canada Inc / médi Canada Inc
104-1375 Lionel-Boulet,
Varennes, Québec,
QC Canada J3X 1P7
T +1 450-583-3317 / +1 800 361 3153
F +1 888-583-6827
service@medicanada.ca
www.medicana.ca

MAXIS a.s.,
medi group company
Slezská 2127/13
120 00 Prague 2
Czech Republic
T: +420 571 633 510
F: +420 571 616 271
info@maxis-medica.com
www.maxis-medica.com

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medi Trading (Shanghai) Co. Ltd.
Room 1703, T1, CES WEST BUND CENTER,
No. 277, Long Lan Road
Xuhui District
200232 Shanghai
People's Republic of China
T: +86-21 50582319
F: +86-21 50582319

UK Responsible Person
medi UK Ltd.
Plough Lane
Hereford HR4 0EL
Great Britain
T +44-1432 37 35 00
F +44-1432 37 35 10
enquiries@mediuk.co.uk
www.mediuk.co.uk

medi

Lumbamed® maternity

Lumbar support for stabilisation during pregnancy

Gebrauchsanweisung. Instructions for use. Mode d'emploi. Instrucciones de uso. Instruções para aplicação. Istruzioni per l'uso. Gebruiksaanwijzing. Návod k použití. Uputa za uporabu. Инструкция по использованию. Instrukcja zakładowania. Használati útmutató. Інструкція з використання. Navodila za uporabo. Návod na použitie. دليل الاستخدام. Instrucțiuni de utilizare. 使用说明. הוראות שימוש. Kasutusjuhend. Uzvilksšanas instrukcija. Užsimovimo instrukcija.

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Important notes

This medical device **[MD]** is made only for multiple uses on a single patient. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If you experience excessive pain or any discomfort whilst wearing, remove the product and immediately contact your doctor or medical supply store. Only wear the product on unbroken skin or over injured skin that has been medically treated, not directly on injured or damaged skin, and only after previous consultation with a medical professional.

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Lumbamed[®] maternity

Intended purpose

Lumbamed maternity is a brace for stabilising the lumbar spine using pressure that can be applied at different levels.

Indications

All indications which require stabilisation of the lumbar vertebrae with a range of pressure levels, such as:

- Pain in the lumbar region during pregnancy
- Lower back pain

Contraindications

Inguinal hernia

Risks / Side Effects

If the aids are too tight, this can cause local pressure sores or constriction of blood vessels or nerves. Therefore you should consult the doctor treating you before you use it if the following circumstances apply to you:

- Condition or injury of the skin in the area of application, particularly with infectious indications (excessive heat, swelling or reddening)
- Paraesthesia and circulatory disorders (e.g. as seen in diabetes, varicose veins)
- Lymph drainage disorders – equally, swelling of soft tissue adjacent to the area of application which is not related to a condition

When wearing snug-fitting aids, this can result in localised skin inflammation or irritation that is due to mechanical irritation of the skin (particularly in conjunction with sweating) or due to the composition of the material.

Intended users and patient target groups

Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device, provided that they have received briefing on this from healthcare professionals.

Patient target group: Healthcare professionals should provide care to adults and children, applying the available information on the measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their own responsibility.

Note on wearing

Supports cause the greatest effect during physical exercise. In principal, the support can be worn all day. However, this depends on how comfortable the support is to wear. The support should be taken off for longer breaks (e.g. sitting for a long time or a long car drive, sleeping). If you suffer excessive pain or discomfort during wear, please take off the product immediately and seek medical advice. The support is also suitable for use in water. After use, please rinse out the product thoroughly.

Fitting instructions

Please wash the brace before its first use. Please note that initial fitting should be supervised by a healthcare professional.

- Before you put the brace on, loosen the fastening straps so that the brace can be put on easily. Release the belts' fasteners and fix them loosely at the back of the belt.
- First, slip your hand into the hand strap on the right outer fastening side.
- Wrap the brace around your body and place the left fastening side on your

stomach, pulling it so that it is snug (Fig. 2).

- Draw the right fastening side over the left one, and fasten it to the left side (Fig. 3).
- Then release the fastening straps' fasteners and pull these to the front simultaneously (Fig. 4). You can adjust the tightness of the brace according to your needs and level of pain.
- The brace is fitted correctly if the fastening is closed under the stomach and supports the stomach (Fig. 5).

Care instructions

Hook and loop fasteners should be closed for washing. We recommend using a laundry bag. Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the product by hand, preferably using medi clean washing agent.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

Storage instructions

Keep the product in a dry place and do not expose to direct sunlight.

Material composition

Polyamide, polyester, polyurethane

Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

Disposal

The product can be disposed of in the domestic waste.

Your medi Team

wishes you a healthy and happy pregnancy!

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).

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