**medi GmbH & Co. KG**  
**Medicusstraße 1**  
**D-95448 Bayreuth**  
**Germany**  
**T +49 921 912-0**  
**www.medi.de**

medi Australia Pty Ltd  
 83 Fennell Street  
 North Parramatta NSW 2151  
 Australia  
 T +61-2 9890 8696  
 F +61-2 9890 8439  
 sales@mediaustralia.com.au  
 www.mediaustralia.com.au

medi Austria GmbH  
 Adamgasse 16/7  
 6020 Innsbruck  
 Austria  
 T +43 512 57 95 15  
 F +43 512 57 95 15 45  
 vertrieb@medi-austria.at  
 www.medi-austria.at

medi Belgium bvba  
 Posthoornstraat 13/1  
 3582 Koersel  
 Belgium  
 T: + 32-11 24 25 60  
 F: +32-11 24 25 64  
 info@medibelgium.be  
 www.medibelgium.be

medi Brasil  
 Rua Cristovan de Vita, n.º 260, Un. 30,  
 Centro Logístico Raposo Tavares,  
 Vila Camargo  
 Vargem Grande Paulista - SP  
 CEP 06730-000  
 Brasil  
 T: +55-11-3500 8005  
 sac@medibrasil.com  
 www.medibrasil.com

medi Canada Inc / médi Canada Inc  
 597, Rue Duvernay, Verchères  
 QC Canada J0L 2R0  
 T: +1 450-583-3317 / +1 800 361 3153  
 F: +1 888-583-6827  
 service@medicanada.ca  
 www.medicana.ca

MAXIS a.s.,  
 medi group company  
 Slezská 2127/13  
 120 00 Prague 2  
 Czech Republic  
 T: +420 571 633 510  
 F: +420 571 616 271  
 info@maxis-medica.com  
 www.maxis-medica.com

medi Danmark ApS  
 Vejlegårdsvej 59  
 2665 Vallensbæk Strand  
 Denmark  
 T +45-70 25 56 10  
 F +45-70 25 56 20  
 kundeservice@sw.dk  
 www.meditandmark.dk

medi Bayreuth Espana SL  
 C/Canigo 2-6 bajos  
 Hospitalet de Llobregat  
 08901 Barcelona  
 Spain  
 T +34-932 60 04 00  
 F +34-932 60 23 14  
 medi@mediespana.com  
 www.mediespana.com

medi France  
 Z.I. Charles de Gaulle  
 25, rue Henri Farman  
 93297 Tremblay en France Cedex  
 France  
 T +33-1 48 61 76 10  
 F +33-1 49 63 33 05  
 info@medi-france.com  
 www.medi-france.com

medi Hungary Kft.  
 Bokor u. 21.  
 1037 Budapest  
 Hungary  
 T +36 1 371-0090  
 F +36 1 371-0091  
 info@medi.hu  
 www.medi.hu

medi Nederland BV  
 Heusing 5  
 4817 ZB Breda  
 The Netherlands  
 T +31-76 57 22 555  
 F +31-76 57 22 565  
 info@medi.nl  
 www.medi.nl

medi Polska Sp. z o.o.  
 ul. Łąbiedzka 22  
 44-121 Gliwice  
 Poland  
 T: +48-32 230 60 21  
 F: +48-32 202 87 56  
 info@medi-polska.pl  
 www.medi-polska.pl

medi Bayreuth Unipessoal, Lda  
 Rua do Centro Cultural, no. 43  
 1700-106 Lisbon  
 Portugal  
 T +351-21 843 71 60  
 F +351-21 847 08 33  
 medi.portugal@medibayreuth.pt  
 www.medi.pt

medi RUS LLC  
 Business Center NEO GEO  
 Butlerova Street 17  
 117342 Moscow  
 Russia  
 T +7-495 374 04 56  
 F +7-495 374 04 56  
 info@medirus.ru  
 www.medirus.ru

medi Trading (Shanghai) Co. Ltd.  
 Room 105, Building No. 4,  
 Lane 500 ShengXia Road  
 Pudong New District  
 201210 Shanghai  
 People's Republic of China  
 T: +86-21 61761988  
 F: +86-21 61769607  
 info@medi.cn

Medi Türk Ortopedi Medikal İç ve Dis Tic. AS  
 Mustafa Kemal Mah. 2156 Sokak No: 16 /4-5  
 06510 Çankaya Ankara  
 Turkey  
 T: +90 312 435 20 26  
 F: +90 312 434 17 67  
 info@medi-turk.com  
 www.medi-turk.com

medi UK Ltd.  
 Plough Lane  
 Hereford HR4 0EL  
 Great Britain  
 T +44-1432 37 35 00  
 F +44-1432 37 35 10  
 enquiries@mediuk.co.uk  
 www.mediuk.co.uk

**medi**

## circaid® juxtafit® essentials upper leg with knee inelastic compression system

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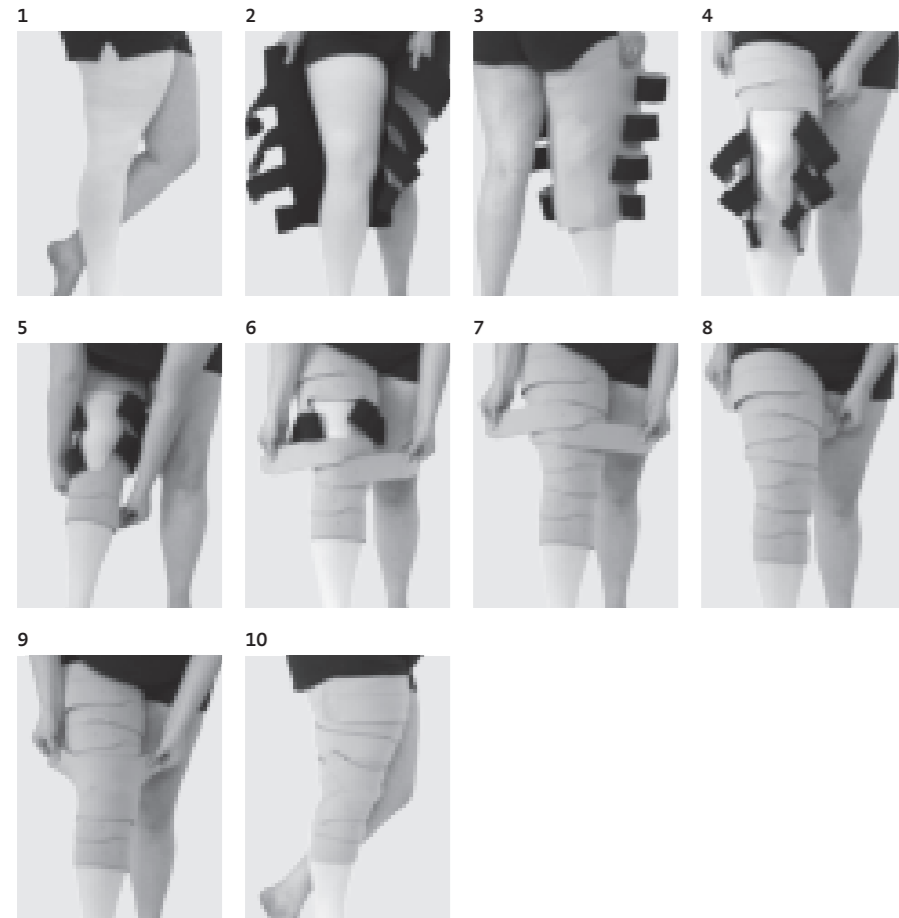
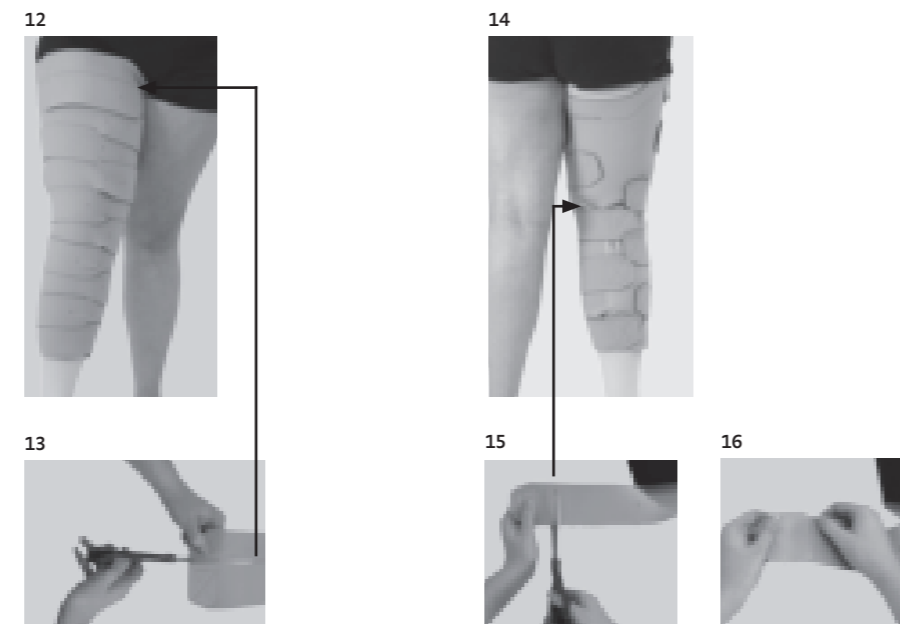
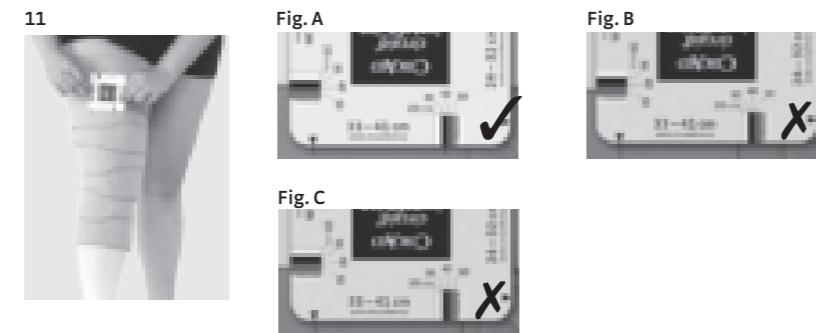
medi. I feel better.

### Built-In-Pressure System

## circaid® juxtafit® essentials upper leg with knee

### Purpose

The compression system is designed to provide compression to the leg for patients with venous and lymphatic disorders.

The device consists of a series of juxtapositioned inelastic bands extending from the central portion of the garment. The garment contours to the leg and is designed to cover from just below the knee to the groin.

### Indications

- Lymphedema
- other forms of edema
- Lipedema
- Chronic venous insufficiency
- Varicose veins
- DVT/Thrombosis Prevention
- Post Thrombotic Syndrome
- Venous Stasis Ulcer
- Post Sclerotherapy
- Lipodermatosclerosis

### Contraindications

- Severe Peripheral Arterial Disease
- Uncontrolled Congestive Heart Failure
- Septic Phlebitis
- Phlegmasia Cerulea Dolens
- Untreated infection
- Any circumstance where increased venous and lymphatic return is undesirable
- Suspected or known untreated acute DVT

### Cautions

- Mild to Moderate Peripheral Arterial Disease
- Impaired Sensation - must be able to sense applied pressure
- Intolerance to material(s)

- Pediatric & assisted use - must be able to sense & communicate applied pressure

### Storage instructions

-  Store in a dry place
-  Keep out of sunlight

### Lifespan

circaid juxtafit essentials has a lifespan of 6 months. Due to material wear-and-tear the medical efficacy can only be expected for a defined lifespan. This assumes correct handling of the garment (e.g. proper care, application, and removal).

### Disposal

 Please dispose of the garment in household waste. There are no specific criteria for disposal of medical compression garments.

### Material composition

circaid compression garment  
Nylon 52%  
Polyurethane 40,9%  
Elastane 7%  
Polyethylene 0,1%  
For all other components, please refer to the inserts included with each item.

This garment does not contain latex.

### Care instructions

The garment can be machine washed using a gentle cycle and dried on a low heat setting. If doing so, it is recommended that you place the garment in a pillow case to aid in

medi Ukraine LLC  
Evhena Sverstiuka str, 11.  
Kiev 02002  
Ukraine  
T: +380 44 591 11 63  
F: +380 44 392 73 73  
info@medi.ua  
http://medi.ua

medi USA L.P.  
6481 Franz Warner Parkway  
Whitsett, N.C. 27377-3000  
USA  
T +1-336 4 49 44 40  
F +1-888 5 70 45 54  
info@mediusa.com  
www.mediusa.com

keeping the Velcro tab material lint-free and to protect other clothing from getting damaged.  
To extend the life of the garment, hand wash and drip dry.

- 
-  Machine wash warm
  -  Do not bleach
  -  Tumble dry, low heat
  -  Do not iron
  -  Do not dry clean
  -  Do not wring
- 

### **Wearing recommendation**

Always ensure that the correct prescribed pressure range is being applied.

Slightly loosen the bands of the garment for night-time wear.

Each vertical BPS line on the bands may be at a different location on the leg depending on the size and shape of the leg. They do not need to be aligned down the front center of the leg or in any other specific location on the leg.

If experiencing any pain, or discomfort, reduce the compression (mmHg) applied or remove device. Inform your clinician of your action.

When the bands are positioned correctly they should overlap slightly, leaving no space between bands.

Do not tuck the bands in. The Velcro tabs should never touch the skin!

### **Donning instructions**

**Step 1:** Slide the circaid undersleeve onto the leg so that it covers the entire leg up to the groin. Ensure that there are no wrinkles in the sock.

### **Note:**

If using a juxtafit lower leg (not included), apply the lower leg, before donning the upper leg. Please refer to the instructions included within your juxtafit lower leg for instructions.

---

**Step 2:** Align the upper leg component with the black side facing the skin and the top of the component far enough below the groin so as not to cause iscomfort. (2) The curved edge of the top of the component should go along he lateral (outer) side of the leg. (3)

**Step 3:** Unroll the top two bands and loosely secure them to the outside of the component to anchor the component in place. (4) This enables you to raise or lower the component easily for correct positioning.

**Step 4:** Unroll the bottom two bands and secure the bottom most band to the outside of the legging to a firm and comfortable compression level (5).

**Step 5:** While holding the second band, detach the next band. Secure the second band and continue this process with the other bands, going up the leg (6-8) Ensure that the bands are positioned in an alternating order.

**Step 6:** The garment should lay flat and wrinkle free against the leg. Adjust each band to a firm and comfortable compression level. See section "Using the Built-in-Pressure System" for instructions on using the BPS card to set the compression to a prescribed level.

**Step 7:** Once all bands are secured, inspect the garment for any gaps or creases. Adjust the bands if necessary.

**Step 8:** Applying the knee cover: Place the knee cover over the knee cap and

pull the tabs at each corner to firm and comfortable compression. Secure the tabs to the legging once the knee cover is snugly against the knee. (9) The circaid juxtafit essentials upper leg with knee garment has now been applied. (10) Adjust the bands throughout the day to provide firm and comfortable compression. Slightly loosen the bands if wearing at night-time.

---

**Note:**

After application is complete, you may fold the edges of the circaid undersleeve over the top and bottom edges of the leg component if desired.

---

Recommendation for combination of lower and upper leg with knee garments:

If combining a circaid juxtafit essentials lower leg with a circaid juxtafit essentials upper leg with knee, ensure that the upper leg with knee overlaps with the lower leg.

**Doffing instructions**

To remove the component, first remove the cover up, if using. Then unfold the circaid undersleeve. Detach all bands of the component, starting with the top band. Fold each band back onto itself to keep the hook tabs free of lint. Remove the component and then remove the circaid undersleeve.

**Using the Built-In-Pressure card:**

Step 1: Locate the BPS card in your packaging.

Step 2: Identify the appropriate side of the BPS card based on your ankle circumference.

Step 3: Identify your prescribed pressure scale using the color-coded system.

Step 4: Starting with the bottom band, line up the arrow on the BPS card with one of the BPS lines on the bottom band of the component. (Fig. A)

Step 5: Note where the second BPS line lines up with the card's compression ranges.

Step 6: If the BPS line on the component either falls short (Fig. B) or goes beyond the correct compression range (Fig. C), readjust the band as necessary so that the second BPS line on the component is aligned with the prescribed compression range. (Fig. A shows the application of 30-40 mmHg of pressure for illustration purposes, although patients' prescribed ranges of pressure may vary.) Completely loosen the band before reapplying if too much tension has been applied.

Step 7: Repeat steps 4-6 for each band going up the leg (11). Be sure to adjust all bands to the same compression range.

---

**Note:**

If also using a circaid lower legging, only lightly apply the juxtafit upper legging bands that overlap with the lower legging to ensure gradient compression. The remaining bands can be set to a firm and comfortable compression, using the BPS system if desired.

---

**Adjusting the component length:**

If the length of the juxtafit essentials upper leg component is too long, it can be trimmed by 5 cm (12). The upper leg component is too long if it overlaps with the lower leg component by more than 2 bands.

---

**Note:**

If the component needs to be trimmed more than 5 cm, a different length of juxtafit component is required.

---

With the component flat on a table or other surface, trim the component length in a straight line all the way across the top edge of the component along the black trim line. (13)

**Adjusting the band length:**

If the length of any juxtafit band is too long, it can be trimmed as much as necessary, as long as you do not trim past the BPS lines. It is recommended to trim in small increments so as to avoid over-trimming. A band is too long and should be trimmed if bands are touching in the back of the leg or if it is causing excessive creasing in the back of the leg. (14) To adjust the length of a band, first identify how much the band needs to be trimmed. Mark the desired trim point on the component using a pen or pencil. Next, with the component flat on a table or other surface, trim the band length at the desired point in a straight line. (15) Then, using the included extra hook tabs, apply one of the extra hook tabs to the new band end. (16)

---

**Note:**

Do not trim the component while it is on the wearer's leg.

---

**Important information for practitioners and patients:**

Please note it is recommended that the limb is always treated in its entirety with medical compression.

This means that compression products for the leg begin at the foot or the ankle and extend up to below the hollow of the knee or beyond to include the thigh.

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