medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-783
ortho@medi.de
www.medi.de

medi Australia Pty Ltd
 83 Fennell Street
 North Parramatta NSW 2151
 Australia
 T +61-2 9890 8696
 F +61-2 9890 8439
 sales@mediaustralia.com.au
 www.mediaustralia.com.au

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
 T +43 512 57 95 15
 F +43 512 57 95 15 45
 vertrieb@medi-austria.at
 www.medi-austria.at

medi Belgium NV
 Staatsbaan 77/0099
 3945 Ham
 Belgium
 T: +32-11 24 25 60
 F: +32-11 24 25 64
 info@medibelgium.be
 www.medibelgium.be

medi Canada Inc / médi Canada Inc
 597, Rue Duvernay, Verchères
 QC Canada J0L 2R0
 T: +1 450-583-3317 / +1 800 361 3153
 F: +1 888-583-6827
 service@medicanada.ca
 www.medicanada.ca

MAXIS a.s.,
 medi group company
 Slezská 2127/13
 120 00 Prague 2
 Czech Republic
 T: +420 571 633 510
 F: +420 571 616 271
 info@maxis-medica.com
 www.maxis-medica.com

medi Danmark ApS
 Vejlegårdsvej 59
 2665 Vallensbaek Strand
 Denmark
 T +45-70 25 56 10
 F +45-70 25 56 20
 kundeservice@sw.dk
 www.meditandmark.dk

medi Bayreuth Espana SL
 C/Canigo 2-6 bajos
 Hospitalet de Llobregat
 08901 Barcelona
 Spain
 T +34-932 60 04 00
 F +34-932 60 23 14
 medi@mediespana.com
 www.mediespana.com

medi France
 Z.I. Charles de Gaulle
 25, rue Henri Farman
 93297 Tremblay en France Cedex
 France
 T +33-1 48 61 76 10
 F +33-1 49 63 33 05
 infos@medi-france.com
 www.medi-france.com

medi Hungary Kft.
 Bokor u. 21.
 1037 Budapest
 Hungary
 T +36 1 371-0090
 F +36 1 371-0091
 info@medi.hu
 www.medi.hu

medi Japan K.K.
 5-1 Nihonbashi Kabutocho Chuoku
 103-0026 Tokyo
 Japan
 T: +81 3 6778 2590
 F: +81 3 5847 7901
 info@medi-japan.jp
 www.medi-japan.co.jp

medi Nederland BV
 Heusing 5
 4817 ZB Breda
 The Netherlands
 T +31-76 57 22 555
 F +31-76 57 22 565
 info@medi.nl
 www.medi.nl

medi Polska Sp. z o.o.
 ul.Łabędzka 22
 44-121 Gliwice
 Poland
 T: +48-32 230 60 21
 F: +48-32 202 87 56
 info@medi-polska.pl
 www.medi-polska.pl

medi Bayreuth Unipessoal, Lda
 Rua do Centro Cultural, no. 43
 1700-106 Lisbon
 Portugal
 T +351-21 843 71 60
 F +351-21 847 08 33
 medi.portugal@medibayreuth.pt
 www.medi.pt

medi RUS LLC
 Business Center NEO GEO
 Butlerova Street 17
 117342 Moscow
 Russia
 T +7-495 374 04 56
 F +7-495 374 04 56
 info@medirus.ru
 www.medirus.ru

medi Sweden AB
 Box 6034
 192 06 Sollentuna
 Sweden
 T +46 8 96 97 98
 F +46 8 626 68 70
 info@medi.se
 www.medi.se

medi Trading (Shanghai) Co. Ltd.
 Room 1703, T1, CES WEST BUND CENTER,
 No. 277, Long Lan Road
 Xuhui District
 200232 Shanghai
 People's Republic of China
 T: +86-21 50582319
 F: +86-21 50582319

Medi Turk Ortopedi Medikal Ic ve Dis Tic. AS
 Mustafa Kemal Mah. 2156 Sokak No: 16 /4-5
 06510 Çankaya Ankara
 Turkey
 T: +90 312 435 20 26
 F: +90 312 434 17 67
 info@medi-turk.com
 www.medi-turk.com

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Important notes

This medical device **[MD]** is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Do not wear the product over open wounds, and use it only as instructed by your doctor or orthotist.

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Spinomed®

Instructions for the orthopedic technician

Intended purpose

Spinomed is a brace designed to actively relieve load and correct the lumbar spine/thoracic spine in the sagittal plane.

Indications

All indications requiring a relief and/or correction of the lumbar/thoracic spine as well as movement restriction in sagittal and frontal plane:

- Osteoporotic collapse of thoracic and/or lumbar vertebrae
- Juvenile Scheuermann's disease
- Kyphosis with chronic back pain

Contraindications

None known to date

Risks / Side Effects

If the aids are too tight, this can cause local pressure sores or constriction of blood vessels or nerves. Therefore you should consult the doctor treating you before you use it if the following circumstances apply to you:

- Condition or injury of the skin in the area of application, particularly with infectious indications (excessive heat, swelling or reddening)
- Paraesthesia and circulatory disorders (e.g. as seen in diabetes, varicose veins)
- Lymph drainage disorders – equally, swelling of soft tissue adjacent to the area of application which is not related to a condition

When wearing snug-fitting aids, this can result in localised skin inflammation or irritation that is due to mechanical irritation of the skin (particularly in conjunction with sweating) or due to the composition of the material.

Intended patient groups

Healthcare professionals should provide care to the adults and children, applying the available information on the measurements/sizes and necessary functions/indications and in line with the information provided by the manufacturer, and acting under their own responsibility.

Adaption of the brace (may only be executed by an orthopaedic technician!)

The brace must be worn over clothing

Adjusting the back support

When you take your brace out of the packaging, the back support has already been pre-moulded to fit human anatomy. It is, however, necessary to adjust the support to the shape of the spine of the individual patient. The pad is fixed on the support upon delivery but can be removed if needed, using the hook-and-loop strap, to mould the support or to clean the pad (Fig. 1). Hold the support on the patient's back so that the lower edge of the support is positioned level with the top of the buttock cleft. The upper edge should be 1 - 5cm under cervical vertebra C7. The back support can be reshaped when it is cold and can be set without tools. Start by setting the brace in the lumbar lordosis region (the natural inward curve of the spine) and then work your way up the spine.

Attention: The back support must be adjusted so that it fits the patient exactly. There should be no gaps nor pressure points. Tip: Additional hook-and-loop strips are supplied with the brace. These can be attached to the support if required. They increase the hold for the back padding.

Adjusting the band straps

Have the patient put on the brace and

close the stomach pad. Now set the band straps, starting with the pelvic belt and moving on to the shoulder strap and then the elastic middle belt (Fig. 2). You must follow this sequence. To achieve this, open the Y hook-and-loop fastening, pull the band strap so that it is snug and stick the fastening down again. Once all of the straps have been set correctly, they can be cut shorter using scissors if needed. The band strap eyes can be moulded thermoplastically. To mould the band strap eyes, set the hot air gun to a maximum of 300°C and focus it on the band strap eye at a distance of approx. 3 cm for a maximum of 30 seconds. Attention: The band strap must be removed from the brace beforehand. The band strap eyes on the support can be moved and can be removed without a tool and then refastened. To achieve this, turn the band strap eyes clockwise or anticlockwise until they unlock automatically (Fig. 3). To attach the band strap eyes, place them back on the strap and turn them until they lock. The middle eyes can be set individually at different heights.

Pelvic strap

Ensure that the pelvic strap runs below the pelvic bone. This strap should be tight.

Shoulder straps

The shoulder strap should straighten the patient's posture slightly but not cut into the patient. Here, it is important that the complete strap is tightened, from top to bottom. The shoulder strap must be tightened so that the tension ensures that it lies snugly on the body. However, it should still be possible to fit an entire flat hand between the strap padding and shoulder. Check that the shoulder pads are seated correctly. They should be positioned so that the pad lies across the front of the shoulder area (Fig. 4). The

band strap should roll in underneath the underarm in order to prevent friction. Attention: To prevent slipping of the back support, the pelvic belt must be done up tighter than the shoulder straps. If the support slips upwards sometimes, this has no impact on the function of the brace. However, the patient should pull the back support down again.

Elastic middle strap

Adjust the middle strap to the optimum length so that the reverse buckle is located in the extension of the armpit and the patient's shoulders are adequately held backwards.

Checking the fit of the brace

- Back pad adjusted to fit the patient's spinal curvature
- The pelvic strap runs firmly below the pelvic bone
- The shoulder strap is flat against the body and the flat hand can still be inserted between the strap and the shoulder
- The shoulder pad is in the optimum position
- The elastic connecting strap is adjusted to the correct length

Spinomed®

Instructions on putting on the brace for the patient

Dear patient,
Your doctor has recommended that you wear the Spinomed brace. Please read these instructions through carefully in order to make it easier to put the brace on.

Putting the brace on

When putting the brace on, it's useful to stand in front of a wardrobe mirror. This

makes it easier to put on the brace. Put the Spinomed on the same way as you would a rucksack:

- Open the large hook-and-loop fastening in the front.
- Put the brace on as you would a rucksack (Fig. 5). The back support must lie upon the back, supported by the padding (Fig. 8).
- Slip your right hand into the loop on the right side of the fastening. Hold the left half of the fastening in place with your left hand and close the broad stomach fastening (Fig. 6). The brace is fitted correctly when the lower edge of the fastening is directly above the groin (Fig. 7).

Now check the following in front of the mirror

- The position of the back support: this should fit snugly to your spine. The lower edge of the support is positioned level with the top of the buttock cleft (Fig. 8).
- The layout of the belt straps: ensure that the straps are not twisted, though they may be rotated when they are fed through the reversing clasp (Fig. 2).
- The seat of the shoulder pads: ensure that the strap does not cut into the armpit (Fig. 4).
- The reversing clasps: they should not press into the patient's side.

Caution

To avoid the possibility of the back brace slipping, the pelvic strap must be tightened more than the shoulder strap. Slight upward slippage of the brace will not have any effect on the function of the orthosis. For the sake of comfort, however, you should pull the brace down again.

Usage Instructions

The Spinomed should only be worn for 1 to 2 hours per day in the habituation

phase and in order to gradually restore the muscle activity. The length of time that the brace is worn can be increased in stages over the course of the therapy. The brace should be taken off during longer periods of sitting or lying down. If required, the brace must be realigned by your orthopaedic technician. Your acceptance and cooperation play an important role in this form of therapy. Therefore, should any issues occur, please discuss them with your doctor or therapeutic technician. They may be able to make any potentially required changes to your Spinomed brace. To ensure that this therapy is successful, you must follow your doctor's instructions at all times. Do not wear the Spinomed directly on the skin, instead put a t-shirt or undershirt on underneath the brace.

Care instructions

The straps, shoulder pad and the large hook-and-loop fastening can be washed by hand in water with a mild detergent. The back support and pads can be cleaned using a moistened cloth. If required, the back pad can be removed (Fig. 1) and washed in water by hand too. Soap residues, lotions and ointments can cause skin irritation and material wear.

- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

Storage instructions

Keep the product in a dry place and do not expose to direct sunlight.

Material composition

Aluminium Polyester, Elastan, Elastomer

Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

Disposal

The product can be disposed of in the domestic waste.

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR). The traceability of this product is assured via a UDI code UDI.

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