**medi GmbH & Co. KG**  
**Medicusstraße 1**  
**95448 Bayreuth / Germany**  
**T +49 921 912-0**  
**F +49 921 912-780**  
**ortho@medi.de**  
**www.medi.de/en**

**medi Australia Pty Ltd**  
**83 Fennell Street**  
**North Parramatta NSW 2151**  
**Australia**  
**T +61-2 9890 8696**  
**F +61-2 9890 8439**  
**sales@mediaustralia.com.au**  
**www.mediaustralia.com.au**

**medi Austria GmbH**  
**Adamgasse 16/7**  
**6020 Innsbruck**  
**Austria**  
**T +43 512 57 95 15**  
**F +43 512 57 95 15 45**  
**vertrieb@medi-austria.at**  
**www.medi-austria.at**

**medi Belgium NV**  
**Staatsbaan 77/0099**  
**3945 Ham**  
**Belgium**  
**T: +32-11 24 25 60**  
**F: +32-11 24 25 64**  
**info@medibelgium.be**  
**www.medibelgium.be**

**medi Canada Inc / médi Canada Inc**  
**104-1375 Lionel-Boulet,**  
**Varenes, Québec,**  
**QC Canada J3X 1P7**  
**T +1 450-583-3317 / +1 800 361 3153**  
**F +1 888-583-6827**  
**service@medicanada.ca**  
**www.medicanada.ca**

**MAXIS a.s.,**  
**medi group company**  
**Slezská 2127/13**  
**120 00 Prague 2**  
**Czech Republic**  
**T: +420 571 633 510**  
**F: +420 571 616 271**  
**info@maxis-medica.com**  
**www.maxis-medica.com**

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### Important notes

This medical device **[MD]** is made only for multiple uses on a single patient. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Only wear the product on unbroken skin or over injured skin that has been medically treated, not directly on injured or damaged skin, and only after previous consultation with a medical professional.

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medi Trading (Shanghai) Co. Ltd.  
Room 1703, T1, CES WEST BUND CENTER,  
No. 277, Long Lan Road  
Xuhui District  
200232 Shanghai  
People's Republic of China  
T: +86-21 50582319  
F: +86-21 50582319

Medi Turk Ortopedi Medikal Ic ve  
Dis Tic. AS  
Mustafa Kemal Mah. 2156 Sokak  
No: 16 /4-5  
06510 Çankaya Ankara  
Turkey  
T: +90 312 435 20 26  
F: +90 312 434 17 67  
info@medi-turk.com  
www.medi-turk.com

medi Ukraine LLC  
Evhena Sverstiuka str, 11.  
Kiev 02002  
Ukraine  
T: +380 44 591 11 63  
F: +380 44 392 73 73  
info@medi.ua  
http://medi.ua

medi USA L.P.  
6481 Franz Warner Parkway  
Whitsett, N.C. 27377-3000  
USA  
T +1-336 4 49 44 40  
F +1-888 5 70 45 54  
info@mediusa.com  
www.mediusa.com

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# medi Clavicle support Kidz

## **Intended purpose**

medi Clavicle support Kidz is a clavicle support for positioning and immobilising the clavicle.

---

## **Indications**

All indications which require positioning and immobilization of the collarbone for example following:

- Dislocations of the clavicle
- Sprains and fractures of the clavicle

## **Contraindications**

- Constriction of the airways
- Compression of the vascular nerve bundle

## **Risks / Side Effects**

If the aids are too tight, this can cause pressure sores or constriction of blood vessels or nerves. Consult your doctor before using this device if you have the following conditions:

- A condition affecting the application area
- Paraesthesia, circulatory disorders
- Lymph drainage disorders

Tightly fitting aids can cause skin irritation (pressure, sweat, material).

## **Intended users and patient target groups**

Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device, provided that they have received briefing on this from healthcare professionals. Patient target group: Healthcare professionals should provide care to adults and children, applying the available information on the measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their own responsibility.

## **Instructions for use**

- Place the cushion on the thoracic vertebrae (upper back) so that the straps lie on the shoulders (Fig. 1).
- Pull both of the straps under the armpits and thread them into the loops intended for them. Tighten the straps to an appropriate level and fasten them (Fig. 2).

---

## **Care instructions**

Hook and loop fasteners should be closed for washing. Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric

conditioners.

- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

### Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

### Material composition

Polyester, cotton, polyamide

### Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

### Disposal

The product can be disposed of in the domestic waste.

### Your medi team

wishes you a speedy recovery!

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).

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