**medi GmbH & Co. KG**  
Medicusstraße 1  
95448 Bayreuth  
Germany  
T +49 921 912-0  
F +49 921 912-780  
ortho@medi.de  
www.medi.de

medi Australia Pty Ltd  
Unit 4/13-21 Hallmark Street  
Pendle Hill NSW 2145  
Australia  
T +61 2 9890 8696  
F +61 2 9890 8439  
sales@medi.au  
www.mediaustralia.com.au

medi Austria GmbH  
Adamgasse 16/7  
6020 Innsbruck  
Austria  
T +43 512 57 95 15  
F +43 512 57 95 15 45  
vertrieb@medi-austria.at  
www.medi-austria.at

medi Belgium NV  
Staatsbaan 77/0099  
3945 Ham  
Belgium  
T: + 32-11 24 e25 60  
F: +32-11 24 25 64  
info@medibelgium.be  
www.medibelgium.be

medi Canada Inc / médi Canada Inc  
104-1375 Lionel-Boulet,  
Varenes, Québec,  
QC Canada J3X 1P7  
T +1 450-583-3317 / +1 800 361 3153  
F +1 888-583-6827  
service@medicanada.ca  
www.medicanada.ca

MAXIS a.s.,  
medi group company  
Slezská 212/13  
120 00 Prague 2  
Czech Republic  
T: +420 571 633 510  
F: +420 571 616 271  
info@maxis-medica.com  
www.maxis-medica.com

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medi Trading (Shanghai) Co. Ltd.  
Room 1703, T1, CES WEST BUND CENTER,  
No. 277, Long Lan Road  
Xuhui District  
200232 Shanghai  
People's Republic of China  
T: +86-21 50582319  
F: +86-21 50582319

UK Responsible Person  
medi UK Ltd.  
Plough Lane  
Hereford HR4 0EL  
Great Britain  
T +44-1432 37 35 00  
F +44-1432 37 35 10  
enquiries@mediuk.co.uk  
www.mediuk.co.uk

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#### Important notes

This medical device **MD** is made only for multiple uses on a single patient. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If you experience excessive pain or any discomfort whilst wearing, remove the product and immediately contact your doctor or medical supply store. Only wear the product on unbroken skin or over injured skin that has been medically treated, not directly on injured or damaged skin, and only after previous consultation with a medical professional.

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# medi Epico ROMs Kidz

## Intended purpose

medi Epico ROMs Kidz is an elbow brace for mobilisation  with immobilisation  of the proximal radioulnar articulation.

## Indications

All indications that require early functional mobilisation of the elbow joint with limitation of the range of motion and immobilisation of the proximal radioulnar articulation, such as:

- The surgical fixation of the medial and / or lateral collateral ligaments
- The conservative treatment of elbow dislocations
- The surgical treatment of elbow fractures
- Reconstruction of the ulnar or radial collateral ligaments
- The surgical treatment of biceps or triceps ruptures
- Treatment of radial head fracture (nonsurgical, postoperative)
- Treatment following insertion of a prothesis
- Distal humeral fractures (nonsurgical, postoperative)

## Contraindications

- Wound healing issues and wound infections
- Skin issues when open wounds are present
- Concomitant neurological damage, e.g. ulnar nerve
- Compartment syndrome

## Risks / Side Effects

If the aids are too tight, this can cause local pressure sores or constriction of

blood vessels or nerves. Therefore you should consult the doctor treating you before you use it if the following circumstances apply to you:

- Condition or injury of the skin in the area of application, particularly with infectious indications (excessive heat, swelling or reddening)
- Paraesthesia and circulatory disorders (e.g. as seen in diabetes, varicose veins)
- Lymph drainage disorders – equally, swelling of soft tissue adjacent to the area of application which is not related to a condition

When wearing snug-fitting aids, this can result in localised skin inflammation or irritation that is due to mechanical irritation of the skin (particularly in conjunction with sweating) or due to the composition of the material. The product is not suitable for children under 6 years of age. Danger of suffocation from small parts that can be swallowed.

## Intended users and patient target groups

Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device, provided that they have received briefing on this from healthcare professionals.

Patient target group: Healthcare professionals should provide care to adults and children, applying the available information on the measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their own responsibility.

## Setting the flexion and extension limits

The medi Epico ROMs Kidz brace comes with an extension and flexion angle of 0° pre-installed. The ChronoLock device ensures that the degree setting on the brace articulation cannot be changed by

accident (e.g. if the brace experiences an impact).

In order to limit flexion and/or extension, continuously turn the ChronoLock wheel until the control window is under the arrow of the grey fixation button (Fig. 1). Now, you can set first the limitation of flexion and then the limitation of extension by pressing the grey button (Figs. 2+3). To lock everything is securely in place, turn the ChronoLock wheel to a neutral position outside the setting range for flexion and extension (Fig. 4).  
Limitation of flexion: 0°-120°  
Limitation of extension: 0°-120°

### Fitting instructions

Open all of the hook-and-loop straps and put the brace on the child/patient so that the brace articulation is positioned exactly over the lateral epicondyle (Fig. 5). The length of the hand support can be set using both of the screws on the lower arm splint. Fasten both of the underarm straps first, and then the upper arm straps (Fig. 6+7). Bring the end of the neck strap through the eye on the upper arm sleeve and close the hook-and-loop fastening. Bring the other end around the shoulder, feed it through the eye on the hand support, and stick the end down (Fig. 8). The neck pad can be moved to suit the individual but should be positioned centrally over the shoulder. The pads can be trimmed to size using scissors, if necessary.

### Important notes

Adjust the extension and flexion limits on the instructions of your doctor only. Particularly during the first few days after traumatic events, the belts should be slackened often so as to avoid severe swelling.

### Care instructions

Hook and loop fasteners should be closed for washing. Soap residues, lotions and ointments can cause skin

irritation and material wear. The cushion for the hand support can be removed for washing. To remove the cushion, loosen the screw of the shoulder strap with the attached Allen key.

- Wash the paddings and straps by hand, preferably using medi clean washing agent.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

With normal use, the hinge mechanism should not require any maintenance, but it can be lubricated with PTFE spray.

### Storage instructions

Keep the product in a dry place and do not expose to direct sunlight.

### Material composition

Aluminum, PU foam, cotton

### Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

### Disposal

The product can be disposed of in the domestic waste.

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).

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