medi

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# Product information

## Lymph pad – Replacement pad and display model

Only for use in mediven flat-knit compression stockings.

### To support oedema therapy

Using underlying padding, the compressive pressure exercised by the prominent areas (e.g. the sides of the hands and feet or bones) is evenly redistributed towards the surfaces of the hands, external depressions around the bone and the lateral areas of the Achilles tendon in order to avoid high levels of pressure.

Lymph pads are attached to the flat-knit compression stocking in order to ensure that the compressive pressure of the stocking

- Is spread evenly over a greater surface area or
- Is increased in some areas, where necessary.

### The following designs are available:

- Ribbed lymph pad: Lymphoedema in legs (standard) and arms
- Fine nap lymph pad: Lymphoedema in arms(standard) and legs
- Coarse nap lymph pad: Lipoedema (for thighs)

Please observe the instructions for use for medical compression stockings, which are found in the packaging for every item.

Here, you will receive further information on the intended purpose, usage and care of your medical product, should you require it.

### Material composition:

SBR latex, foam created via air bubbles = styrene butadiene rubber

### Storage instructions:

- Store in a dry place.
- Keep out of direct sunlight

### Product life:

Due to material wear, the lymph pads should be used for a maximum of six months. This is the same lifetime stipulated for wearing a medical compression stocking.

### Disposal:

Please dispose of medi lymph pads as you would normal household waste. There are no specific criteria for disposal.

### Instructions for care:

Please wash and clean your medical compression sock and/or stocking in accordance with the instructions given by the specialist medical retailer. Please also ensure that you observe the detailed description for medical compression stockings in the mediven instructions for use and the information on the fabric label that is sewed into them.

The attached lymph pad is washed with the compression product during hand or machine washing and during drying.

Please do not use any disinfectants. This can have an undesirable influence on the foam material and affect the structure of the device.

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medi Sweden AB
medi Trading (Shanghai) Co. Ltd.