medi

medi  
footsupport  
insoles

medi. I feel better.

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# Instructions

## medi footsupport

medi footsupport insoles are aids that have been developed for feet with special demands.

---

### Intended purpose of foot orthotics

Foot orthotics are medical devices for specific malpositions or poor alignment of the patient's feet.

### Indications

Insoles can be implemented for functional therapy for the following diagnoses:

- **Osteoarthritis of the lower limbs**  
Comfort (pro)
- **Diabetic neuropathy**  
Comfort (pro)
- **Heel spurs / Plantar fasciitis**  
Heel Pain (pro)
- **Hallux rigidus**  
Control (slim)
- **Hallux valgus**  
Business (pro), Business slim (pro), Gentlemen pro, Ballerinas pro, High Heels (pro)
- **Metatarsalgia**  
Control (slim)
- **Pes cavus**  
Comfort (pro)
- **Pes valgus**  
Junior, Active, Business (pro), Business slim (pro), Gentlemen pro, Ballerinas pro
- **Fallen arches**  
Junior, Active, Business (pro), Business slim (pro), Gentlemen pro, Ballerinas pro
- **Flat foot**  
Junior, Active, Business (pro), Business slim (pro), Gentlemen pro, Ballerinas pro
- **Splay foot**  
Business (pro), Business slim (pro), Gentlemen pro, Ballerinas pro

### Contra-indications

None

### Features of the product

- **Cushion**  
The insoles embed the foot so that it is cushioned, protecting it from pressure peaks and relieving load on individual parts of the foot in a targeted manner.
- **Support**  
The supportive function of the insole can relieve load on specific parts of the foot, improve rolling of the foot and helps to optimise the functional stability of the foot.

- **Pain relief**

The structure of the insole and the combination of functional materials can help to reduce the pain.

- **Correction**

The corrective function of the insoles is achieved via various supportive elements positioned at specific points on the foot, e.g. a corrective support for the arch.

---

### **Important Notes**

#### **Instruction for use:**

- Insoles should always be worn as a pair and are only intended for use with intact skin - they cannot be worn on open wounds. Furthermore, the insoles are intended for multiple use by one individual patient. The products must be issued by experts or trained personnel. Furthermore, the product must be used in accordance with the manufacturer's instructions for use (in accordance with the indication, instructions for putting on, storage and care instructions) and the respective orders from the specialist personnel.
- The stress that the insoles are placed under due to general wearing, salves, skin moisture and environmental influences can reduce their service life and impair the medical effectiveness of the insoles. Therefore, we recommend that your doctor and/or specialist retailer checks the insoles regularly.

#### **Intended users:**

- Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device, provided that they have received briefing on this from healthcare professionals.

#### **Patient target group:**

- Healthcare professionals should provide care to adults and children, applying the available information on the measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their own responsibility.

#### **Disposal:**

- You can dispose of this product with your household waste.

#### **Side effects:**

- Sometimes, pressure points can be created or the insole can slip in the shoe. Therefore wear well-fitting shoes, otherwise using the insole will incur risks.
- 

### **Instructions for putting the insoles on**

- The insoles can be individually adjusted to the foot and shoe by an expert.
- If required, remove any insoles that are already in the shoe in order to achieve an optimal fit.
- Some individuals may need to allow their feet to get used to the insoles. This can last several days - up to two weeks.
- If pain and/or discomfort occur whilst wearing the insoles, remove the insoles immediately and contact your doctor or your specialist retailer.
- If you have experienced ongoing problems due to wearing your insoles, please consult your doctor.

---

### **Instructions for care**

- If the insoles need cleaning, clean them with a damp cloth.
  - Do not use any solvents containing acetone to clean them.
  - Do not dry your insoles on the radiator, do not expose them to other direct sources of heat.
- 

### **Information on the warranty, guarantee and liability**

- The product is intended to be used multiple times by one patient and is therefore not intended to be handed over to third parties. If the insole is used to treat more than one person or is altered in an incorrect manner, the product liability will be rendered null and void.
- No claims on the guarantee may be asserted for any damages which are caused by improper usage. Discolouration of the insoles does not constitute the grounds for a complaint.
- The product identification mark on the insole is an important component of the product. If you remove this identification from a medi product, the claim to a guarantee, repair or exchange is rendered null and void.
- In the event of any complaints regarding the product such as damage to the insole or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).

### **Transportation and storage conditions**

- The products must be stored in a dry place and out of direct sunlight.
- Insoles have a service life of 42 months. This includes the storage life and the length of time that they may be used for. The length of time that they may be used for is between 6 and 12 months, depending on how much and how heavily they are worn.

The date of the last revision of the instructions is stated on the last page of the instructions. Information on the manufacturer, the conformity symbol and, if required, the authorised representatives, is stated on the packaging.

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Heel Pain (pro)

- **Čekićasti prst**

Control (slim), Control multizone (slim)

- **Čukljevi**

Business (pro), Business slim (pro), Business multizone pro, Business multizone slim pro, Gentlemen pro, Ballerinas pro, High Heels (pro), High Heels + (pro)

- **Metatarzalgija**

Control (slim), Control multizone (slim)

- **Šuplje stopalo**

Comfort (pro)

- **Valgus stopala**

Junior, Active, Business (pro), Business slim (pro), Business multizone (pro), Business multizone slim (pro), Gentlemen pro, Ballerinas pro

- **Spušteni svodovi stopala**

Junior, Active, Business (pro), Business slim (pro), Business multizone (pro), Business multizone slim (pro), Gentlemen pro, Ballerinas pro, High Heels (pro), High Heels + (pro)

- **Ravno stopalo (ravni tabani)**

Junior, Active, Business (pro), Business slim (pro), Business multizone (pro), Business multizone slim (pro), Gentlemen pro, Ballerinas pro, High Heels (pro), High Heels + (pro)

- **Prošireno stopalo**

Business (pro), Business slim (pro), Business multizone pro, Business multizone slim pro, Gentlemen pro, Ballerinas pro

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medi Norway AS  
Vestheimvegen 35  
4250 Kopervik  
Norway  
T: +47 52 84 45 00  
F: +47 52 84 45 09  
post@medinorway.no  
www.medinorway.no

medi Polska Sp. z.o.o.  
ul.Łabędzka 22  
44-121 Gliwice  
Poland  
T: +48-32 230 60 21  
F: +48-32 202 87 56  
info@medi-polska.pl  
www.medi-polska.pl

medi Bayreuth Unipessoal Lda.  
Rua do Centro Cultural, no. 43  
1700-106 Lisbon  
Portugal  
T +351 21 843 71 60  
F +351 21 847 08 33  
medi.portugal@medibayreuth.pt  
www.medi.pt

medi RUS LLC  
Business Center NEO GEO  
Butlerova Street 17  
117342 Moscow  
Russia  
T +7-495 374 04 56  
F +7-495 374 04 56  
info@medirus.ru  
www.medirus.ru

medi Trading (Shanghai) Co. Ltd.  
Room 1703, T1, CES WEST BUND CENTER,  
No. 277, Long Lan Road  
Xuhui District  
200232 Shanghai  
People's Republic of China  
T: +86-21 50582319  
F: +86-21 50582319

medi Sweden AB  
Box 6034  
192 06 Sollentuna  
Sweden  
T +46 8 96 97 98  
F +46 8 626 68 70  
info@medi.se  
www.medi.se

Medi Turk Ortopedi Medikal İc ve Dis Tic. AS  
Mustafa Kemal Mah. 2156 Sokak No: 16 /4-5  
06510 Çankaya Ankara  
Turkey  
T: +90 312 435 20 26  
F: +90 312 434 17 67  
info@medi-turk.com  
www.medi-turk.com

medi UK Ltd.  
Plough Lane  
Hereford HR4 OEL  
Great Britain  
T +44 1432 37 35 00  
F +44 1432 37 35 10  
enquiries@mediuk.co.uk  
www.mediuk.co.uk

medi Ukraine LLC  
Evhena Sverstiuka str, 11.  
Kiev 02002  
Ukraine  
T: +380 44 591 11 63  
F: +380 44 392 73 73  
info@medi.ua  
http://medi.ua

medi USA L.P.  
6481 Franz Warner Parkway  
Whitsett, N.C. 27377-3000  
USA  
T +1-336 4 49 44 40  
F +1-888 5 70 45 54  
info@mediusa.com  
www.mediusa.com

**medi GmbH & Co. KG**  
Medicusstraße 1  
95448 Bayreuth  
Germany  
T +49 921 912-0  
F +49 921 912-510  
footcare@medi.de  
www.medi.de

medi Australia Pty Ltd  
83 Fennell Street  
North Parramatta NSW 2151  
Australia  
T +61 2 9890 8696  
F +61 2 9890 8439  
sales@mediaustralia.com.au  
www.mediaustralia.com.au

medi Austria GmbH  
Adamgasse 16/7  
6020 Innsbruck  
Austria  
T +43-512 57 95 15  
F +43-512 57 95 15 45  
vertrieb@medi-austria.at  
www.medi-austria.at

medi Belgium NV  
Staatsbaan 77/0099  
3945 Ham  
Belgium  
T: +32-11 24 25 60  
F: +32-11 24 25 64  
info@medibelgium.be  
www.medibelgium.be

medi Canada Inc / médi Canada Inc  
597, Rue Duvernay, Verchères  
QC Canada J0L 2R0  
T: +1 450-583-3317 / +1 800 361 3153  
F: +1 888-583-6827  
service@medicanada.ca  
www.medicanada.ca

MAXIS a.s.,  
medi group company  
Slezská 2127/13  
120 00 Prague 2  
Czech Republic  
T: +420 571 633 510  
F: +420 571 616 271  
info@maxis-medica.com  
www.maxis-medica.com

medi Danmark ApS  
Vejlegardsvej 59  
2665 Vallensbaek Strand  
Denmark  
T +45 70 25 56 10  
F +45 70 25 56 20  
kundeservice@sw.dk  
www.medit danmark.dk


medi Bayreuth España SL  
C/Canigó 2 – 6 bajos  
Hospitalet de Llobregat  
08901 Barcelona  
Spain  
T +34 93 260 04 00  
F +34 93 260 23 14  
medi@mediespana.com  
www.mediespana.com

medi France  
Z.I. Charles de Gaulle  
25, rue Henri Farman  
93297 Tremblay en France Cedex  
France  
T +33 1 48 61 76 10  
F +33 1 49 63 33 05  
infos@medi-france.com  
www.medi-france.com

medi Hungary Kft.  
Bokor u. 21.  
1037 Budapest  
Hungary  
T +36 1371 0090  
F +36 1371 0091  
info@medi.hu  
www.medi.hu

medi Italia s.r.l.  
Via Giuseppe Ghedini, 2  
40069 Zola Predosa (BO)  
Italy  
T: +39-051 6 13 24 84  
F: +39-051 6 13 29 56  
info@medi-italia.it  
www.medi-italia.it

medi Japan K.K.  
5-1 Nihonbashi Kabutocho Chuoku  
103-0026 Tokyo  
Japan  
T: +81 3 6778 2590  
F: +81 3 5847 7901  
info@medi-japan.jp  
www.medi-japan.co.jp

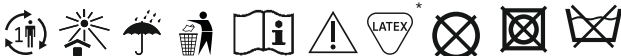\* only for the products: medi Ballerinas pro, High Heels (pro).  
They also contain material of animal origin.

E016017 / 03.2022