medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
www.medi.de

medi Australia Pty Ltd
83 Fennell Street
North Parramatta NSW 2151
Australia
T +61-2 9890 8696
F +61-2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Belgium NV
Staatsbaan 77/0099
3945 Ham
Belgium
T: + 32-11 24 25 60
F: +32-11 24 25 64
info@medibelgium.be
www.medi Belgium.be

medi Canada Inc / médi Canada Inc
597, Rue Duvernay, Verchères
QC Canada J0L 2R0
T: +1 450-583-3317 / +1 800 361 3153
F: +1 888-583-6827
service@medicanada.ca
www.medicana.ca

medi Danmark ApS
Vejlegardsvej 59
2665 Vallensbaek Strand
Denmark
T +45-70 25 56 10
F +45-70 25 56 20
kundeservice@sw.dk
www.medit danmark.dk

medi Bayreuth Espana SL
C/Canigo 2-6 bajos
Hospitalet de Llobregat
08901 Barcelona
Spain
T +34-932 60 04 00
F +34-932 60 23 14
medi@mediespana.com
www.mediespana.com

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33-1 48 61 76 10
F +33-1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Hungary Kft.
Bokor u. 21.
1037 Budapest
Hungary
T +36 1 371-0090
F +36 1 371-0091
info@medi.hu
www.medi.hu

medi Italia s.r.l.
Via Giuseppe Ghedini, 2
40069 Zola Predosa (BO)
Italy
T: +39-051 6 13 24 84
F: +39-051 6 13 29 56
info@medi-italia.it
www.medi-italia.it

medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31-76 57 22 555
F +31-76 57 22 565
info@medi.nl
www.medi.nl

medi Norway AS
Vestheimvegen 35
4250 Koprøvik
Norway
T: +47 52 84 45 00
F: 47 52 84 45 09
post@medinorway.no
www.medinorway.no

medi

circaid® reduction kit lobe lifter

Inelastic compression system

Instructions for use. Gebrauchsanweisung. Instructions
d'utilisation. Instrucciones de uso.


4 064984 707238


EO16213/11.2020

medi. I feel better.


1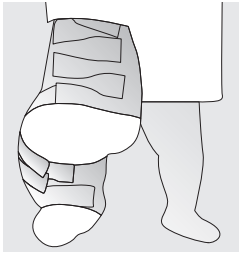


2

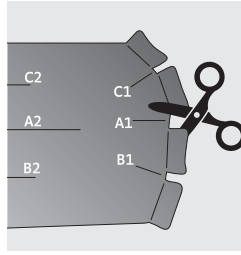
3

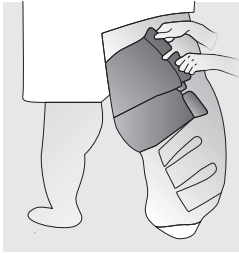
4

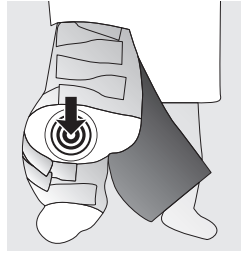
5

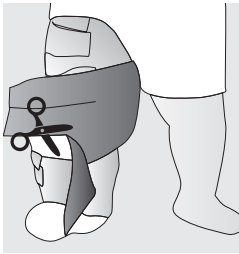
6

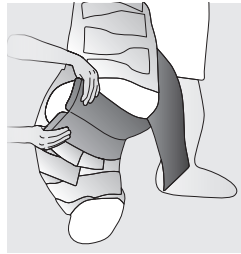
7

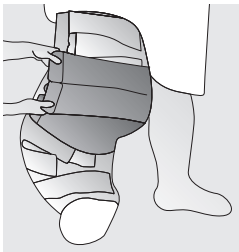
8a

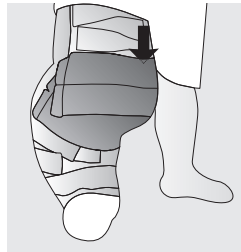8b

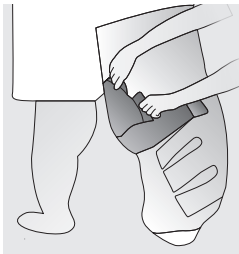9

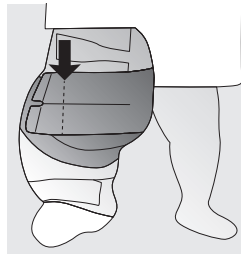Product video medi.biz/rk

medi Polska Sp. z o.o.
ul. Łąbiedzka 22
44-121 Gliwice
Poland
T: +48-32 230 60 21
F: +48-32 202 87 56
info@medi-polska.pl
www.medi-polska.pl

medi Bayreuth Unipessoal, Lda
Rua do Centro Cultural, no. 43
1700-106 Lisbon
Portugal
T +351-21 843 71 60
F +351-21 847 08 33
medi.portugal@medibayreuth.pt
www.medi.pt

medi RUS LLC
Business Center NEO GEO
Butlerova Street 17
117342 Moscow
Russia
T +7-495 374 04 56
F +7-495 374 04 56
info@medirus.ru
www.medirus.ru

medi Sweden AB
Box 6034
19206 Sollentuna
Sweden
T: +46-8-969798
F: +46-8-6266870
info@medi.se
www.medi.se

medi UK Ltd.
Plough Lane
Hereford HR4 OEL
Great Britain
T: +44-1432 37 35 00
F: +44-1432 37 35 10
enquiries@mediuk.co.uk
www.mediuk.co.uk

medi Ukraine LLC
Evhena Sverstiuka str, 11.
Kiev 02002
Ukraine
T: +380 44 591 11 63
F: +380 44 392 73 73
info@medi.ua
<http://medi.ua>

medi USA L.P.
6481 Franz Warner Parkway
Whitsett, N.C. 27377-3000
USA
T +1-336 4 49 44 40
F +1-888 5 70 45 54
info@mediusa.com
www.mediusa.com

medi Trading (Shanghai) Co. Ltd.
Room 105, Building No. 4,
Lane 500 ShengXia Road
Pudong New District
201210 Shanghai
People's Republic of China
T: +86-21 61761988
F: +86-21 61769607
info@medi.cn

Medi Turk Ortopedi Medikal İc ve Dis Tic. AS
Mustafa Kemal Mah. 2156 Sokak No: 16 /4-5
06510 Çankaya Ankara
Turkey
T: +90 312 435 20 26
F: +90 312 434 17 67
info@medi-turk.com
www.medi-turk.com

MAXIS a.s.
medi group company
Slezská 2127/13
120 00 Prague 2
Czech Republic
T: +420 571 633 510
F: +420 571 616 271
info@maxis-medica.com
www.maxis-medica.com

Tachezy Sanit Srl.
Via Druso 48/a
39100 Bolzano
Italy
T: +39 0471 288 068
F: +39 0471 282 773
info@tachezysanit.com
www.tachezysanit.com

medi Japan K.K.
5-1 Nihonbashi Kabutocho Chuoku
103-0026 Tokyo
Japan
T: +81 3 6778 2590
F: +81 3 5847 7901
info@medi-japan.jp
www.medi-japan.co.jp

Inhalt / Content

ENGLISH / ENGLISH	4
DEUTSCH / GERMAN	7
FRENCH / FRANÇAIS	10
ESPAÑOL / SPANISH	13

ENGLISH / ENGLISH

circaid® reduction kit lobe lifter

Instructions for use

Purpose

Supporting band for application in combination with another circaid compression garments. The garment is designed to provide compression to the leg for patients with venous and lymphatic disorders with the presence of a lobe/lobule. The garment consists of a piece of inelastic material that can be customized to have inelastic bands extending from its center.

Indications

- Lymphedema
- other forms of edema
- Lipedema
- Chronic venous insufficiency
- Varicose veins
- DVT/Thrombosis Prevention
- Post Thrombotic Syndrome
- Venous Stasis Ulcer
- Post Sclerotherapy
- Lipodermatosclerosis
- Dependent oedema

Contraindications

- Severe Peripheral Arterial Disease
- Uncontrolled Congestive Heart Failure
- Septic Phlebitis
- Phlegmasia Cerulea Dolens
- Untreated infection
- Any circumstance where increased venous and lymphatic return is undesirable
- Suspected or known untreated acute DVT

Cautions

- Post Thrombotic Syndrome
- Venous Stasis Ulcer
- Post Sclerotherapy
- Lipodermatosclerosis

Storage instructions

Store in a dry place

Keep out of sunlight

Lifespan

circaid juxtacures has a lifespan of 2 months.

Due to material wear-and-tear the medical efficacy can only be expected for a defined lifespan. This assumes correct handling of the garment (e.g. proper care, application, and removal).

Disposal

Please dispose of the garment in household waste. No special criteria for disposal apply for medical compression garments.

This Package contains

- 1 reduction kit lobe garment
- 8 beige hook tabs
- 6 white hook stays
- 1 booklet instructions for use

Material composition

Nylon 57%

Polyurethane 37%

Elastane 4%

Polyethylene 2%

This garment does not contain latex.

Care instructions

The garment can be machine washed using a gentle cycle and dried on a low heat setting. If doing so, it is recommended that you place the garment in a pillow case to aid in keeping the hook and loop tap lint-free and to protect other clothing from getting damaged.

To extend the life of the garment, hand wash and drip dry.

 Machine wash warm

 Do not bleach

 Tumble dry, low heat

 Do not iron

 Do not dry clean

 Do not wring

Wearing recommendation

The circaid compression garment should never hurt. It should feel firm but comfortable. If experiencing any pain, remove the garment and consult your physician. Initial fitting and customization of the garment should be done under the guidance of a healthcare professional. The garment is customizable; one size fits all. Do not tuck bands. Hook tabs should never touch the skin!

Donning/Customizing Instructions

Step 1:

Before you begin the customization and application process for the reduction kit lobe lifter, the patient must be fitted and wearing his or her reduction kit lower leg and upper leg components, as well as undersleeves (1).

Step 2:

Lay the reduction kit lobe lifter on a flat surface. Cut along line A1 as far as the printed line indicates (2).

Step 3:

Start by attaching the lobe kit along the posterior side of the lobe to the reduction kit upper leg component (3).

Step 4:

Rotate around the patient to the anterior side of the lobe and identify where the center of the lobe approximately is (4).

Step 5:

Now pull the lobe lifter underneath the lobe towards the anterior side and cut along line A2 until you reach the center of the lobe. Start with small cuts and gradually adjust, as not to overcut the kit (5).

Note

If cutting near the patient's limb, be sure to use blunt-ended scissors for their safety.

Step 6:

Lift the lower portion of the lobe lifter underneath the lobe and attach it to the reduction kit upper leg component with two pieces of hook tab (6).

Step 7:

With the lobe now lifted, wrap the upper portion of the lobe lifter around the lobe and attach it firmly with two pieces of hook tab (7).

Step 8:

Once all bands are secured, inspect the garment for any gaps or creases (8a). Adjust the bands if necessary (8b).

Step 9:

If the upper portion of the anterior section is too long, its overall length can be trimmed. Ideally the straps will attach just past where the lobe ends (9).

Note

If desired, the lobe lifter can be further customized by trimming the length of the anterior straps or by creating additional straps. If additional straps are desired for contouring to the lobe, you can cut along lines B1, C1, B2, and C2 if desired. However, it is recommended to have as much of the lobe as possible covered by a solid piece of material to provide maximum support and compression.

Patient Application Instructions

Step 1:

Apply the circaid undersleeve and reduction kit leg components as instructed in the reduction kit leg Instructions for Patients.

Step 2:

In a seated position with your leg supported (ex. on your bed), attach side 1 of the lobe kit to the reduction kit upper leg garment on the backside of the lobe with the hook tabs. Side 1 can be identified by A1, B1, and C1.

Step 3:

Bring the lobe lifter around the front of your leg, pulling up and across the lobe.

Step 4:

Fasten side 2 to the front side of the lobe with the hook tabs, applying firmly with comfortable compression. Side 2 can be identified by A2, B2, C2.

Note

It is recommended to follow your clinician's recommendations for how to apply and how long to wear your lobe lifter.

Doffing instructions

To remove the reduction kit lobe, unfasten the hook tabs from both the front and back sides of the lobe.

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Nylon 57%
Polyurethan 37%
Elastan 4%
Polyethylen 2%

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