medi

protect.Toe spreader / protect.Forefoot cushion

Produkthinweise. Product Information. Informations sur le produit. Indicaciones del producto. Indicações para o produto. Informazioni sul prodotto. Opmerkingen inzake het product. Produktoplysninger. Produktanvisningar. Produktinformasjon. Informace o výrobku. Napomene o proizvodu. Указания касательно изделия. Ürün bilgileri. Informacje dotyczące produktu. Termékinformációk. Інформація про продукт. 产品提示. 產品注意事項. دليل الاستخدام. 製品に関する注意. Indicații de produs. מידע לגבי מוצרים.

medi. I feel better.

Produktthinweise protect.Toe spreader / protect.Forefoot cushion

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Product Information for protect.Toe spreader / protect.Forefoot cushion

Important Notes

- This product is not a medical device under the definition set out in Article 2 No. 1 of Regulation (EU) 2017/745 (MDR).
- This product is to be used exclusively for orthotic treatment of the foot and is only suitable for use on unbroken skin. It may not be worn on open wounds.
- This product is intended for multiple use by one individual patient.
- The stress that the product is placed under due to general wearing, salves, skin moisture and environmental influences can reduce its service life and impair the effectiveness of the product. Therefore, we recommend that your doctor and/or specialist retailer checks the insoles regularly.
- It is not permitted to make any changes to the product.

How to wear the product

- **protect.Toe spreader:** Place directly between the toes with the narrow side facing upwards. The soft material fits the contour of your toes.
- **protect.Forefoot cushion:** The side of the product with the circular texture on it is the side that is in contact with the skin. The smaller side of the cushion should be positioned behind the big toe, and the larger side of the cushion should be located behind the little toe. The loop can be pulled over either the second or third toe, as wished, then push the pad towards the heel.
- If pain, skin irritation and/or discomfort occur whilst wearing the product, please remove the product immediately and contact your doctor or your specialist medical retailer.
- If you have experienced ongoing problems due to wearing this product, please consult your doctor.
- If you know that you are intolerant to the material used in this aid, please discuss this with your doctor before using it.

Transportation and storage conditions

- Store the product in a dry place, out of direct sunlight.
- The product has a shelf life of 42 months. This includes the storage

life and the length of time that it may be used for. The length of time that it may be used for extends between 6 - 12 months, depending on how much and how heavily it is worn.

Instructions for care:

- Wash by hand in lukewarm water using a mild detergent.
- Allow to dry completely.
- Do not dry this product on heating systems or other sources of direct heat.

Disposal

- You can dispose of this product with your household waste.

Material composition

- 100 % medical grade silicone

The date of the last revision of the instructions is stated on the last page of the instructions. Information on the manufacturer, the conformity symbol and, if required, the authorised representatives, is stated on the packaging.

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medi Canada Inc / médi Canada Inc
Medizintechnik GmbH & Co. KG