**medi**

# circaid<sup>®</sup> shelf strap

Gebrauchsanweisung. Instructions for use. Mode d'emploi. Instrucciones de uso. Instruções para aplicação. Istruzioni per l'uso. Gebruiksaanwijzing. Bruksanvisning. دليل الاستخدام. Инструкция по использованию. Instrukcja zakładowania. הוראות שימוש. 使用说明.

**medi. I feel better.**

**circaid® juxtafit shelf strap**

**circaid reduction kit shelf strap**

## Inhalt / Content

DEUTSCH / GERMAN	3
ENGLISH / ENGLISH	4
FRANCAIS / FRENCH	6
ESPAÑOL / SPANISH	7
PORTUGUÊS / PORTUGUESE	9
ITALIANO / ITALIAN	10
NEDERLANDS / DUTCH	12
SVENSKA / SWEDISH	13
عربي / Arabic	16
РУССКИЙ / RUSSIAN	17
POLSKI / POLISH	18
עברית / Hebrew	21
中文 / CHINESE	22

DEUTSCH / GERMAN

# circaid® shelf strap

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# circaid® shelf strap

### **Intended purpose**

Supporting band for application in combination with another circaid compression garments.

### **Intended users and patient target groups**

Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device, provided that they have received briefing on this from healthcare professionals.

Patient target group: Healthcare professionals should provide care to adults and children, applying the available information on the measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their own responsibility.

### **Material composition**

circaid reduction kit shelf strap:  
55% Nylon  
41% Polyurethane  
4% Elastane

circaid juxtafit premium shelf strap:

45% Nylon  
50% Polyurethane  
5% Elastane

### Care instructions

We recommend using a laundry bag.

- 
-  Machine wash warm - gentle cycle
  -  Do not bleach
  -  Tumble dry, low heat
  -  Do not iron
  -  Do not dry clean
  -  Do not wring
- 

### Storage instructions

Keep the product in a dry place and do not expose to direct sunlight.

### Lifespan

6 months - due to material wear-and-tear the medical efficacy can only be expected for a defined lifespan. This assumes correct handling of the garment (e.g. proper care, application, and removal).

### Disposal

The product can be disposed of in the domestic waste.

### Donning the circaid reduction kit shelf strap:

Step 1: Apply the appropriate circaid compression system as directed.

Step 2: If using the circaid reduction kit shelf strap, trim the strap to the desired lengths or keep as one long band, and

secure hook tabs to the strap ends.

Step 3: If there is an overhang of oedematous skin and additional support or coverage is desired, use the shelf strap to wrap up underneath the area in need, pulling up and around the limb until firm and comfortable compression is achieved and secure to the circaid compression system of choice.

Step 4: If a gap between the bands occurs and cannot be corrected by adjusting the bands then cover the gap with the shelf strap pulling to firm and comfortable compression and securing the two hook tabs to the circaid compression system of choice.

### Removing the circaid shelf strap:

Step 1: Detach the two hook tabs and remove the shelf strap.

*Note: Do not tightly roll the hook tabs. This will cause the tabs to curl and to no longer lay flat while in use.*

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).

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### **Important notes**

This medical device **MD** is made only for multiple uses on a single patient . If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Only wear the product on unbroken skin or over injured skin that has been medically treated, not directly on injured or damaged skin, and only after previous consultation with a medical professional.

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**medi UK Ltd.**