**medi GmbH & Co. KG**  
**Medicusstraße 1**  
**95448 Bayreuth**  
**Germany**  
**T +49 921 912-0**  
**F +49 921 912-783**  
**ortho@medi.de**  
**www.medi.de**

**medi Australia Pty Ltd**  
**83 Fennell Street**  
**North Parramatta NSW 2151**  
**Australia**  
**T +61-2 9890 8696**  
**F +61-2 9890 8439**  
**sales@mediaustralia.com.au**  
**www.mediaustralia.com.au**

**medi Austria GmbH**  
**Adamgasse 16/7**  
**6020 Innsbruck**  
**Austria**  
**T +43 512 57 95 15**  
**F +43 512 57 95 15 45**  
**vertrieb@medi-austria.at**  
**www.medi-austria.at**

**medi Belgium NV**  
**Staatsbaan 77/0099**  
**3945 Ham**  
**Belgium**  
**T: + 32-11 24 25 60**  
**F: +32-11 24 25 64**  
**info@medibelgium.be**  
**www.medibelgium.be**

**medi Canada Inc / médi Canada Inc**  
**104-1375 Lionel-Boulet,**  
**Varennes, Québec,**  
**QC Canada J3X 1P7**  
**T +1 450-583-3317 / +1 800 361 3153**  
**F +1 888-583-6827**  
**service@medicanada.ca**  
**www.medicanada.ca**

**MAXIS a.s.,**  
**medi group company**  
**Slezská 2127/13**  
**120 00 Prague 2**  
**Czech Republic**  
**T: +420 571 633 510**  
**F: +420 571 616 271**  
**info@maxis-medica.com**  
**www.maxis-medica.com**

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**medi Trading (Shanghai) Co. Ltd.**  
**Room 1703, T1, CES WEST BUND CENTER,**  
**No. 277, Long Lan Road**  
**Xuhui District**  
**200232 Shanghai**  
**People's Republic of China**  
**T: +86-21 50582319**  
**F: +86-21 50582319**

**UK Responsible Person**  
**medi UK Ltd.**  
**Plough Lane**  
**Hereford HR4 0EL**  
**Great Britain**  
**T +44-1432 37 35 00**  
**F +44-1432 37 35 10**  
**enquiries@mediuk.co.uk**  
**www.mediuk.co.uk**

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## Important notes

This medical device **MD** is made only for multiple uses on a single patient. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If you experience excessive pain or any discomfort whilst wearing, remove the product and immediately contact your doctor or medical supply store. Only wear the product on unbroken skin or over injured skin that has been medically treated, not directly on injured or damaged skin, and only after previous consultation with a medical professional.

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# Lumbamed<sup>®</sup> plus E<sup>+</sup>motion<sup>®</sup>

## Intended purpose

Lumbamed plus E<sup>+</sup>motion is a brace for stabilising  the lumbar spine using a back pad.

## Indications

All indications which require stabilisation of the lumbar spine by redistributing pressure using a back pad, such as:

- Lower back pain (lumbago)
- Postural muscle imbalance in the lumbar spine

## Contraindications

None known at present.

## Risks / Side Effects

If the aids are too tight, this can cause local pressure sores or constriction of blood vessels or nerves. Therefore you should consult the doctor treating you before you use it if the following circumstances apply to you:

- Condition or injury of the skin in the area of application, particularly with infectious indications (excessive heat, swelling or reddening)
- Paraesthesia and circulatory disorders (e.g. as seen in diabetes, varicose veins)
- Lymph drainage disorders – equally, swelling of soft tissue adjacent to the area of application which is not related to a condition

When wearing snug-fitting aids, this can result in localised skin inflammation or irritation that is due to mechanical irritation of the skin (particularly in conjunction with sweating) or due to the composition of the material.

## Intended users and patient target groups

Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device, provided that they have received briefing on this from healthcare professionals.

Patient target group: Healthcare professionals should provide care to adults and children, applying the available information on the measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their own responsibility.

## Note on wearing

Supports are particularly beneficial during physical exercise. The supports is designed to fit comfortably and may be worn all day. It should be removed during prolonged periods of inactivity (e.g. sitting for a long time or a long car drive, sleeping).

The support is also suitable for use in water. After use, please rinse out the product thoroughly.

## Putting on / Taking off

- Pull the textile cover over the massage pad and fix in the middle on the central, black hook and loop area between the two stays according to the instructions of your doctor or a specialist (Fig. 1 + 2). The pad should be fastened so that its lower end rests just above the fold between the buttocks when worn.
- First, slide your left hand into the inside hand loop on the left side of the fastening. Wrap the bandage around your body and then slide your right hand into the outer hand loop on the right fastening side (Fig. 3+4).
- Pull the two fastening sides over each

other and then secure them as centrally as possible using the hook-and-loop fastening. The bandage has been put on correctly if the bottom edge of the fastener is located just above the pubic bone (Fig. 5+6).

- The small finger tab on the edge of the right side of the fastener will help you to undo the bandage again (Fig. 7).

Please always wear the Lumbamed plus on the skin and under your clothing. This maximises the Clima Comfort Effect.

---

### Care instructions

Hook and loop fasteners must be closed for washing. Do not wash the pad with the support. It can be cleaned with a damp cloth. We recommend using a laundry bag. Fabric softener, fats, oils and ointments can attack the material and the Clima Comfort Effect can be impaired. Soap residues can cause skin irritation and material wear.

- You can wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

---

### Storage instructions

Keep the product in a dry place and do not expose to direct sunlight.

---

### Material composition

Support: Polyamide, polyester, elastane, wool (merino)

Pad: Polyurethane, polyethylene

### Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

### Disposal

The product can be disposed of in the domestic waste.

### Your medi Team

Wishes you a rapid recovery!

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).

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