medi GmbH & Co. KG
 Medicusstraße 1
 95448 Bayreuth
 Germany
 T +49 921 912-0
 F +49 921 912-780
 ortho@medi.de
 www.medi.de

medi Australia Pty Ltd
 Unit 4/13-21 Hallmark Street
 Pendle Hill NSW 2145
 Australia
 T +61 2 9890 8696
 F +61 2 9890 8439
 sales@medi.au
 www.medi.australia.com.au

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
 T +43 512 57 95 15
 F +43 512 57 95 15 45
 vertrieb@medi-austria.at
 www.medi-austria.at

medi Belgium NV
 Staatsbaan 77/0099
 3945 Ham
 Belgium
 T: +32-11 24 25 60
 F: +32-11 24 25 64
 info@medibelgium.be
 www.medibelgium.be

medi Canada Inc / médi Canada Inc
 104-1375 Lionel-Boulet,
 Varennes, Québec,
 QC Canada J3X 1P7
 T +1 450-583-3317 / +1 800 361 3153
 F +1 888-583-6827
 service@medicanada.ca
 www.medicanada.ca

MAXIS a.s.,
 medi group company
 Slezská 2127/13
 120 00 Prague 2
 Czech Republic
 T: +420 571 633 510
 F: +420 571 616 271
 info@maxis-medica.com
 www.maxis-medica.com

medi Danmark ApS
 Vejlegardsvej 59
 2665 Vallensbaek Strand
 Denmark
 T +45-70 25 56 10
 kundeservice@sw.dk
 www.meditandmark.dk

medi Bayreuth Espana SL
 C/Canigo 2-6 bajos
 Hospitalet de Llobregat
 08901 Barcelona
 Spain
 T +34-932 60 04 00
 F +34-932 60 23 14
 medi@mediespana.com
 www.mediespana.com

medi France
 Z.I. Charles de Gaulle
 25, rue Henri Farman
 93297 Tremblay en France Cedex
 France
 T +33-1 48 61 76 10
 F +33-1 49 63 33 05
 infos@medi-france.com
 www.medi-france.com

medi Hungary Kft.
 Bokor u. 21.
 1037 Budapest
 Hungary
 T +36 1 371-0090
 F +36 1 371-0091
 info@medi.hu
 www.medi.hu

medi Japan K.K.
 5-1 Nihonbashi Kabutocho Chuoku
 103-0026 Tokyo
 Japan
 T: +81 3 6778 2590
 F: +81 3 5847 7901
 info@medi-japan.jp
 www.medi-japan.co.jp

medi Nederland BV
 Heusing 5
 4817 ZB Breda
 The Netherlands
 T +31-76 57 22 555
 F +31-76 57 22 565
 info@medi.nl
 www.medi.nl

medi Polska Sp. z o.o.
 ul. Łąbiedzka 22
 44-121 Gliwice
 Poland
 T: +48-32 230 60 21
 F: +48-32 202 87 56
 info@medi-polska.pl
 www.medi-polska.pl

medi Bayreuth Unipessoal, Lda
 Rua do Centro Cultural, no. 43
 1700-106 Lisbon
 Portugal
 T +351-21 843 71 60
 F +351-21 847 08 33
 medi.portugal@medibayreuth.pt
 www.medi.pt

UK Responsible Person
 medi UK Ltd.
 Plough Lane
 Hereford HR4 0EL
 Great Britain
 T +44-1432 37 35 00
 F +44-1432 37 35 10
 enquiries@mediuk.co.uk
 www.mediuk.co.uk

medi

Lumbamed® sacro

Gebrauchsanweisung. Instructions for use. Mode d'emploi. Instrucciones de uso. Instruções para aplicação. Istruzioni per l'uso. Gebruiksaanwijzing. Bruksanvisning. Návod k použití. Uputa za uporabu. Инструкция по использованию. Kullanna kilavuzu. Instrukcja zakładowa. Használati útmutató. Инструкция з використання. Navodila za uporabo. Návod na použitie. دليل الاستخدام. 使用说明. Bruksanvisning. הוראות שימוש. Uzvilkšanas instrukcija. Užsimovimo instrukcija.

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Important notes

This medical device **MD** is made only for multiple uses on a single patient. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If you experience excessive pain or any discomfort whilst wearing, remove the product and immediately contact your doctor or medical supply store. Only wear the product on unbroken skin or over injured skin that has been medically treated, not directly on injured or damaged skin, and only after previous consultation with a medical professional.

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Lumbamed[®] sacro

Intended purpose

Lumbamed sacro is a brace for stabilisation  of the pelvis.

Indications

All indications which require stabilisation of the pelvis, such as:

- Sacroiliac joint dysfunction
- Sacroiliac joint instability
- Muscle pain and tendon disease in the pelvic region
- Pelvic ring instability
- For recurring muscle and tendon disease (rectus abdominis muscle, piriformis muscle)

Contraindications

None known at present.

Risks / Side Effects

If the aids are too tight, this can cause local pressure sores or constriction of blood vessels or nerves. Therefore you should consult the doctor treating you before you use it if the following circumstances apply to you:

- Condition or injury of the skin in the area of application, particularly with infectious indications (excessive heat, swelling or reddening)
- Paraesthesia and circulatory disorders (e.g. as seen in diabetes, varicose veins)
- Lymph drainage disorders – equally, swelling of soft tissue adjacent to the area of application which is not related to a condition

When wearing snug-fitting aids, this can result in localised skin inflammation or irritation that is due to mechanical irritation of the skin (particularly in conjunction with sweating) or due to the composition of the material.

Intended users and patient target groups

Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device, provided that they have received briefing on this from healthcare professionals.

Patient target group: Healthcare professionals should provide care to adults and children, applying the available information on the measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their own responsibility.

Note on wearing

Supports have the greatest effect during physical exercise. In principal, the support can be worn all day. The special cut in the area of the groin is decisive for the correct and comfortable fit of the support. However, this depends on how comfortable the support is to wear. The support should be taken off for longer breaks (e.g. sitting for a long time or a long car drive, sleeping).

The support is also suitable for use in water. After use, please rinse out the product thoroughly.

Fitting instructions

- Before you put the brace on, loosen the straps so that the brace can be put on easily. Release the belts' fasteners and fix them loosely at the back of the belt, on the velour (fig. 1).
- Fix the pads in the middle and/or according to the doctor's or specialist's instructions on the dark grey hook and loop fastener. If no other advice has been given, the pads should be fixed with the narrow end pointing downwards (fig. 2).
- Use the "magenta-coloured edge"

(bottom border) as a guide when putting on the brace.

- First, slide your left hand into the inside hand loop on the left side of the fastening (fig. 3).
- Wrap the brace around your pelvis and slide your right hand into the outer hand loop on the right side of the fastening (fig. 4+5).
- Pull both fastening sides together in the middle so that they overlap as evenly as possible (fig. 6).
- Now first release the fastening straps' fasteners and pull the straps to the front simultaneously (fig. 7).
- You can adjust the tightness of the brace yourself according to your needs and level of pain. The brace is positioned correctly when the bottom fastening edge is just above the pubic bone (fig. 8).
- To check the position of the brace, lift your thigh up slightly. The bottom edge of the fastening strap should be level with your groin (fig. 9).
- You can remove the brace and just wear the outer, non-elastic component with the fastening strap system. For this, please loosen the hook-and-loop fastening on both sides of the fastening interior side and remove the elastic knitted fabric.
- You can also place the pads in the centre of the belt here or position them according to your doctor's or a specialist's instructions, if required.
- Then put the brace on as described above.

Our recommendation: Wear the brace directly over underwear, or a very thin layer of clothing, so that the massage pads can have an optimal effect.

If you also wish to wear this brace at night, please observe the additional

instructions for use for the Fastening Extension, you can order separately.

There is no data on the use of Lumbamed sacro by pregnant women. Therefore, only doctors may decide whether this brace is appropriate, and doctors must be consulted for clarification in individual cases.

Instructions for care

The pads should be removed before washing. If necessary, the pads can be cleaned with a dampened cloth. Please fasten all hook and loop fastenings before washing. We recommend that you use a laundry bag. Fabric softeners, fats, oils, lotions, ointments and soap residues can affect the material, hinder the Clima Comfort effect and cause skin irritations.

- Wash the product by hand, preferably with mediclean soap, or in a delicate wash cycle at 30°C with mild detergent and without fabric conditioner.
- Do not bleach.
- Allow to air-dry.
- Do not iron.
- Do not dry-clean.

Storage instructions

Keep the product in a dry place and do not expose to direct sunlight.

Material composition

Band: Polyamide, polyester, elastane, viscose

Pads: Polyurethane

Liability

The manufacturer's liability will become

void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in these instructions for use.

Disposal

Dispose of properly after use.

**Your medi Team**

wishes you a speedy recovery!

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).

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